

Hearing configurations: Own voice




How do you calibrate your own voice?

Nuance Audio allows you to calibrate your own voice so that you can decide how clearly you'd like to hear it for your own comfort.




How do you calibrate your own voice?

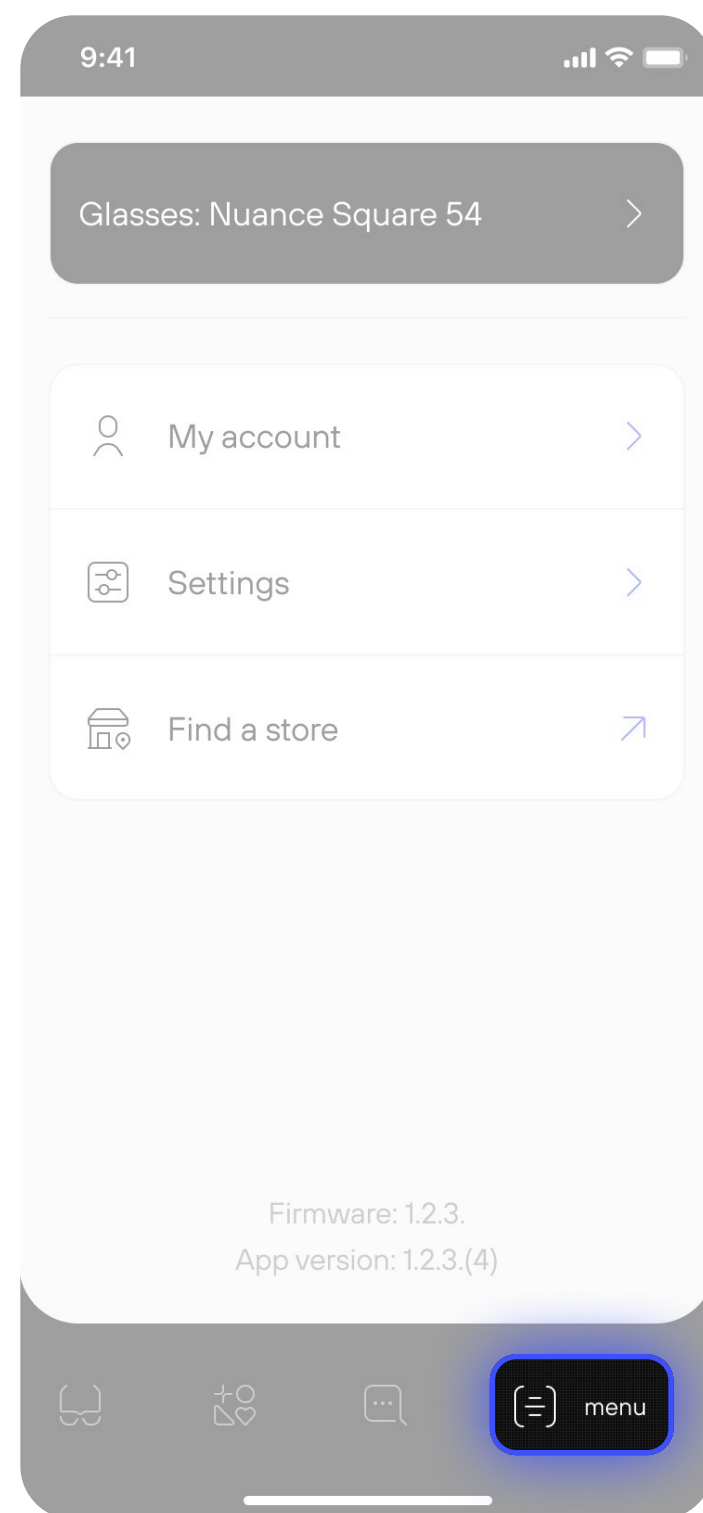


Within the Nuance App's dashboard, tap on **Menu** (bottom-right corner in the navigation bar).

In the Menu tap on **Settings**

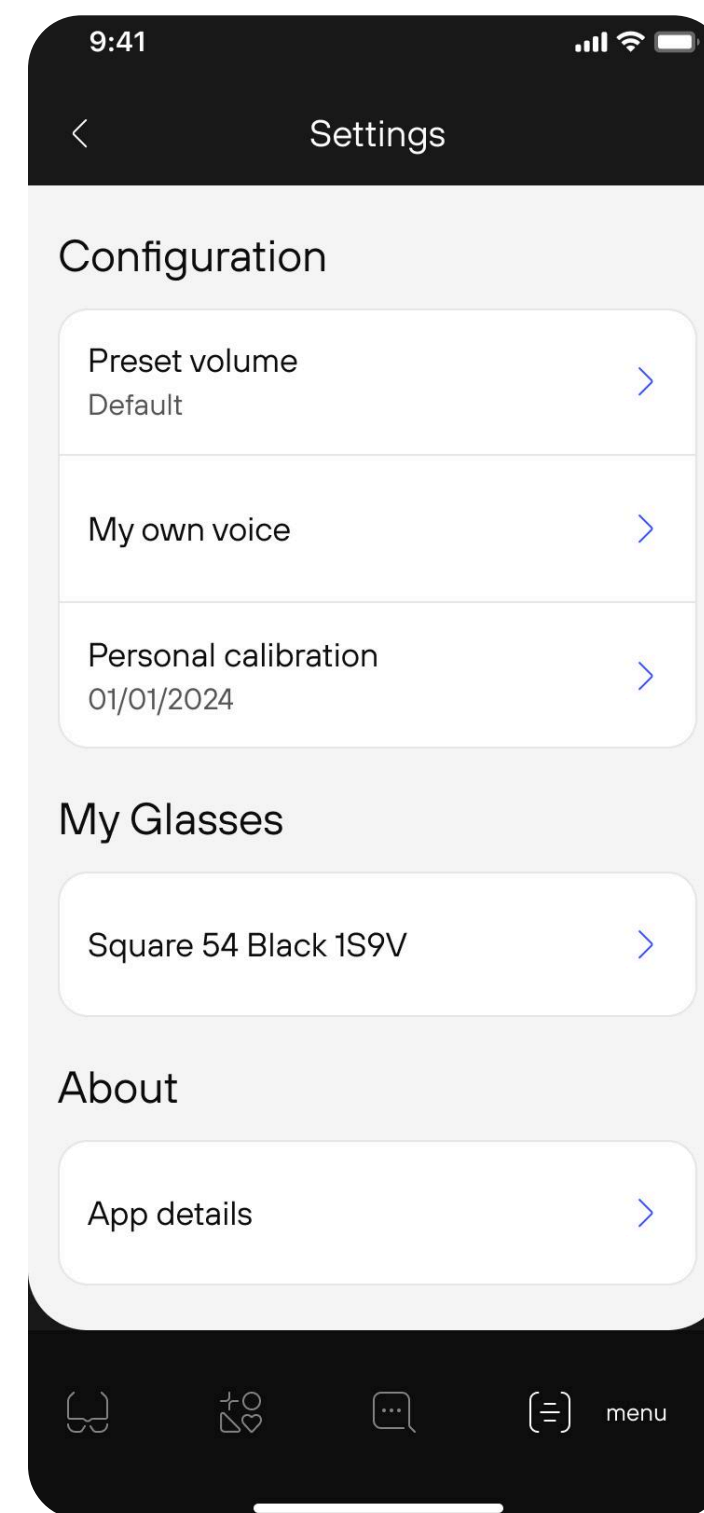

In Settings, select **My own voice**

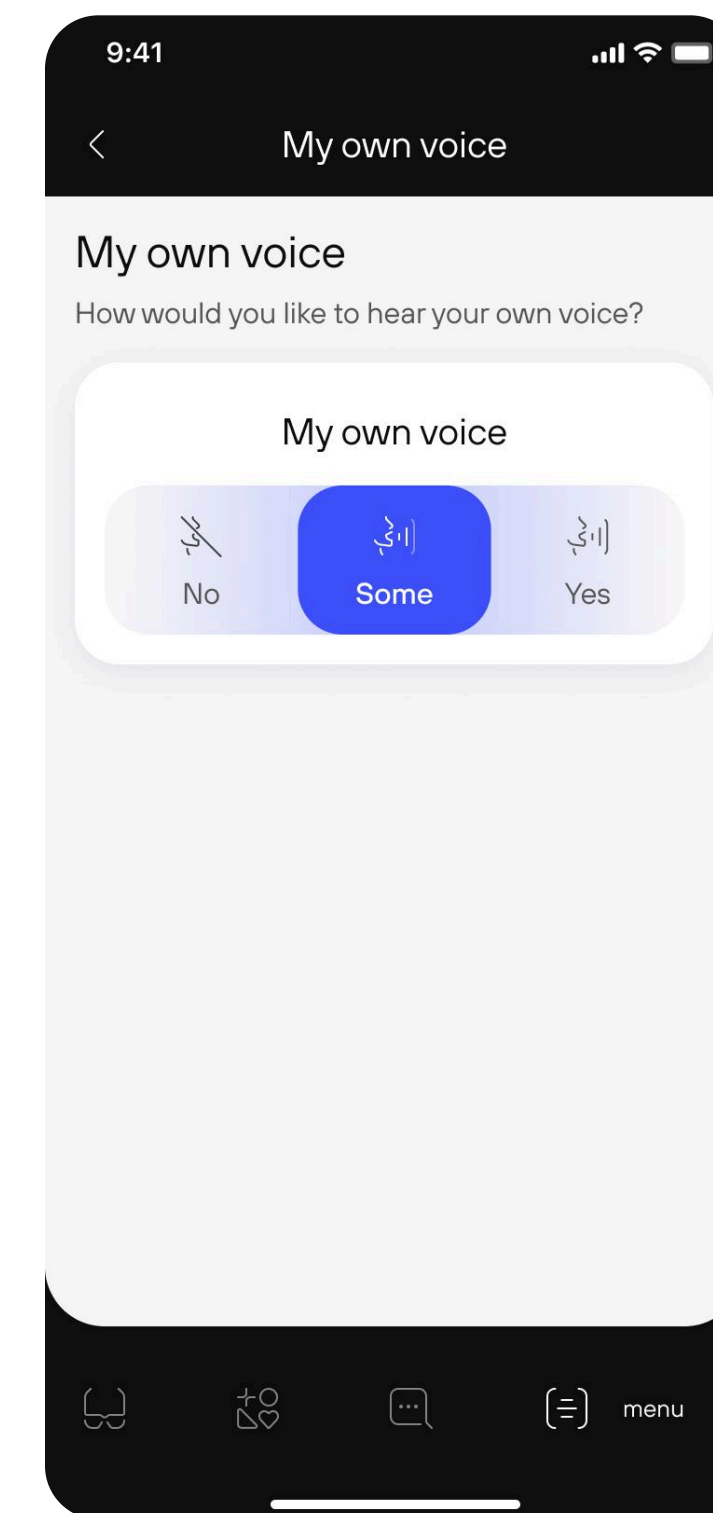

Now you can freely **adjust your amplification levels** to what suits you and your environment best.

Notes

Please consider that setting the **Personal Calibration** is also relevant for optimizing your overall experience, ensuring the most natural sound and clarity for your own voice, as well as for other audio settings.



For the most updated Instructions for Use
please follow this link

[↗ View Instruction for Use](#)