



Instructions For Use - Safety & Warranty



• • nuance audio

Instructions For Use - Safety & Warranty

Nuance Audio™ Remote Control

Nuance Audio™ Remote Control is referred to within this Instructions For Use as the "Product". The Remote Control is an optional accessory that is intended to assist the Nuance Audio™ hearing aid functionality in terms of controlling limited settings of the hearing aid. The Remote Control is intended to be used only in conjunction with the Nuance Audio™ Glasses, to wirelessly control the hearing aid volume and modes. The Remote Control works via Bluetooth-paired with the Glasses, and it includes one coin cell battery. The coin cell battery (CR2450) is customer replaceable. The Nuance Audio™ Hearing Aid is a SaMD OTC air conduction hearing aid feature software-only medical device. The Remote Control is to be used as an accessory to the Nuance Audio™ Glasses. Please refer to the User Guide for images and usage explanation of the Remote Control.

Restrictions to combinations The Remote Control works only with Nuance Audio™ Glasses, meaning that the Product isn't compatible with other devices.

Read instructions. Read and follow all instructions included with the Remote Control before using it.

Cautions.

- When adjusting the volume, turn it up gradually until you reach a comfortable listening level.
- Do not use the Remote Control if any part of it is damaged or overheating. If this happens, stop using the Remote Control immediately, then please visit us at www. nuanceaudio.com to get helpful service and contact information.
- 3. Do not make unauthorized alterations to this Product.

HEALTH & SAFETY INFORMATION

WARNING To reduce the risk of personal injury, discomfort, or property damage, please read and follow the health and safety information below before using the Remote Control.

Intended uses & users

Intended use. The Remote Control accessory is intended to assist the air conduction hearing aid functionality in terms of controlling settings on the hearing aid. The Remote Control is used to wirelessly control the Nuance Audio™ Glasses' volume and modes. The two possible hearing aid modes are:

- Frontal: Frontal mode enhances sound coming from the front, making it easier to focus
 on conversations with someone directly in front.
- · All-around: All-around mode amplifies sounds and speech from all directions.

Generally the Remote Control works BT-paired with the Glasses, and it includes a coin cell battery. The Remote Control is intended to be used only in conjunction with the Nuance Audio™ Glasses and is intended only for use by individuals 18 years old and older.

For ages 18 and up. The Remote Control is intended to be used only in conjunction with the Nuance Audio™ Hearing Aid and is intended only for use by individuals 18 years old and older. Do not allow anyone under 18 to use the Remote Control. Children can choke or suffocate in case of swallowing of this Product. Keep this Product away from children.

Interference with personal medical devices.

This Product contains components that emit radio waves, which could affect the operation of medical devices such as but not limited to pacemakers or other implantable devices, or insulin pumps, consult your doctor or the medical device manufacturer before using the Product. Turn off the electronic functionalities of the Product if you notice any interference with your medical device.

Interference. Check before using the Remote Control in areas where electronics or wireless devices are restricted. This may cause interference.

Interactions & environments. Pay attention. Interacting with the Remote Control may distract you or increase the time it takes to react to hazards. Always pay attention to your surroundings, including signage and obstacles.

Driving. Do not use the Remote Control while driving.

Limitations of Use. Do not use the Remote Control in environments with strong electric or magnetic fields. The effective usage distance of the Remote Control is less than 25 meters. The Remote Control expected lifetime is 5 years from manufacturing date.

Flying. Check applicable aviation security restrictions on personal electronics before flying with the Remote Control. Use the Remote Control in compliance with all the applicable local laws, rules, and regulations.

Intended handling & maintenance

Handle with care & avoid damage The Remote Control is sensitive electronic equipment, it contains sensitive electronic hardware and should be handled with responsible care. Do not heat or bend, as it may become damaged and even catch fire. Do not drop, strike, or shake the Remote Control excessively. Do not use the Remote Control if it is damaged. Do not attempt to open, disassemble, or tamper with the Remote Control. Do not excose to the sun.

Operating conditions:

14°F (-10°C) to 113°F (+45°C); 101.3kPa to 61.9kPa; humidity between 20% and 60%

under 95°F (+35°C) for 1000 hours; maximum altitude of 13.000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C).

Transport and Storage: -40°F (-40°C) to 185°F (+85°C); humidity between 20% and 90%; Remote Control shall remain functional after recovery from non-operational exposure to altitude of 40,000 ft (18.8kPa) for 12 hours at both 14°F (-10°C) and 113°F (+45°C).

Cleaning and maintenance. To clean the Remote Control use a dampened cloth and neutral soap avoiding all types of sprays, solvents, chemicals or cleaning solutions containing alcohol or ammonia. Do not use dirty or abrasive cloths which could damage the Remote Control. If the Product becomes excessively wet, there is a risk of damaging it, thoroughly dry it, store them in a container with desiccants or in a drying case.

WARNING Potentially explosive atmospheres. Do not store or carry flammable liquids, gases, or explosive materials in the same compartment as the Remote Control. Turn off your Remote Control, and do not remove or install batteries when you are in an area with potentially explosive atmospheres.

Water & liquids. Do not expose to or submerge in water. Extended exposure to water may damage the Remote Control. The Remote Control is not designed to resist submersion or extended exposure to water or other liquids. In case of water exposure, dry the Remote Control thoroughly with dry cloth.

Do not use if broken or damaged. Periodically inspect the Remote Control for damage or signs of wear. Do not use it if any part is damaged, cracked, or if any internal component is exposed. Do not try to repair the Remote Control yourself, instead contact support at www.nuanceaudio.com.

Charging, electrical & heat

Overheating. Keep the Remote Control away from fire and excessive heat (e.g., from storage in direct sunlight, near candles, or in a hot car) to avoid burns or damage to the Remote Control. The Remote Control shouldn't overheat during normal use. If it does, stop using it immediately and contact support. Keep the Remote Control away from fire and heat sources.

Battery. The Remote Control contains a coin battery that is user replaceable. The battery may present a fire or chemical burn hazard if misused. Do not leave in the car under direct sunlight or expose to excessive heat or incinerate. Dispose of the battery in line with instructions and local requirements. Remove and immediately recycle or dispose of used batteries according to local regulations and keep away from children. Do NOT dispose of batteries in household trash or incinerate. Remove and immediately recycle or dispose of batteries from equipment not used for an extended period of time according to local regulations.

Do not mix old and new batteries, different brands or types of batteries, such as alkaline, carbon-zinc, or rechargeable batteries. Expected battery lifetime is at least 6 months under standard use conditions from the moment of pulling out the battery protective tab. For battery replacement procedure, please follow the indications on the User Guide accompanying the Remote Control and use a pin or small pen tip or toothpick to open remote LID. Be cautious when replacing the battery to avoid damaging the structure of the Remote Control. Ensure the batteries are installed correctly according to polarity (+ and -). Always completely secure the battery compartment. If the battery compartment does not close securely, stop using the product, remove the batteries, and keep them away from children.

Non-rechargeable batteries are not to be recharged. Do not force discharge, recharge, disassemble, heat above 100 °C (212 °F) or incinerate. Doing so may result in injury

due to venting, leakage, or explosion resulting in chemical burns.

Battery related hazards are fire, explosion and burn. Do not short circuit, crush, incinerate the battery, or expose contents to water. Always keep your batteries away from children to prevent swallowing. Do not ingest battery.

Compatible batteries are lithium 3V CR2450 coin cell batteries.

Even used batteries may cause severe injury or death. Call a local poison control center for treatment information.

On device button
On device buttons
Yes, BT LE
No
2402-2480 MHz
Free space max EIRP is -3.22 dBm
GFSK
2MHz
2MHz

Number of channels	40
TECHNICAL SPECIFICATIONS	
Battery	Lithium 3V CR2450 coin cell battery
Lifetime	5 years
Shelf life	5 years
TRANSPORT AND STORAGE CONDITIONS	
Temperature	From -40°F (-40°C) to +185°F (+85°C)
Humidity	From 20% to high humidity 90% and temperature 113°F (+45°C) for 288 hours
Altitude & atmospheric pressure	Altitude of 40,000 ft (18,8kPa) for 12 hours at both 14°F (-10°C) and 113°F (+45°C)
OPERATING CONDITIONS	
Temperature	From 14°F (-10°C) to 113°F (+45°C)
Humidity	From 20% to humidity 60% under 95°F (+35°C) for 1000 hours
Altitude & atmospheric pressure	Altitude of 13.000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C); 101.3kPa to 61.9kPa

FCC compliance statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that

may cause undesired operation.

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- · Reorient or relocate the receiving antenna.
- · Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio technician for help.

Note: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. No part of this work may be reproduced, modified, distributed or otherwise used without prior written permission. Keep the Product away from the womb of pregnant women. Keep the Product away from the lower abdomen of adolescents. Use the Product in good reception conditions to reduce the amount of electromagnetic radiation received.

FCC Radiation Exposure Statement

This equipment complies with FCC radiation exposure limits set forth for an

uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Disposal Information The Nuance Audio™ Remote Control and the coin battery should not be discarded as household waste and should be delivered to an appropriate collection facility for recycling. For more info on proper disposal, please visit www.nuanceaudio.com.

Safety certifications for the Remote Controls' battery

This article describes the international safety standards that the Remote Control's battery meet. All the Remote Controls' batteries are tested, certified, and in compliance with these international safety standards:

- United Nations (UN) Transport Regulations UN38.3: Covers battery safety during air transport.
- Regulation (EU) 2023/1542
- · UL1642 Standard for Lithium Batteries

WARNING The battery contains no mercury. Do not puncture or incinerate, do not squeeze or soak in water.

▲ WARNING

- · INGESTION HAZARD: This product contains a button cell or coin battery.
- DEATH or serious injury can occur if ingested.
- A swallowed button cell or coin battery can cause Internal Chemical Burns in as little as 2 hours.
- KEEP new and used batteries OUT OF REACH of CHILDREN.
- Seek immediate medical attention if a battery is suspected to be swallowed or inserted inside any part of the body.



LUXOTTICA, LIMITED WARRANTY This Warranty is provided by Luxottica Group S.p.A., Piazzale Cadorna 3, 20123 Milano, Italy. This Warranty covers defects and malfunctions in the new Luxottica product(s) it accompanies (the "Product"). The Warranty continues for one (1) year from the date of purchase of the Product (the "Limited Warranty Period"). However, if you purchase the Product from within the EU or EFTA, the Limited Warranty Period shall be two (2) years.

FOR THE US ONLY, ANY IMPLIED WARRANTIES THAT CANNOT BE DISCLAIMED ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTY PERIOD AS SET OUT IN THIS LIMITED CONSUMER WARRANTY (THE "WARRANTY"). Some countries, states, provinces, and territories (including the EU, EFTA and UK) do not allow the exclusion or limitation of incidental or consequential damages, the exclusion or limitation of certain damages or types of loss or allow limitations on how long an implied warranty or condition may last, so the limitations or exclusions described in this Warranty may not apply to you.

For products purchased in California: This hearing aid is warranted to be specifically fit for the particular needs of you, the buyer. If the hearing aid is not initially fit for your particular needs, it may be returned to the seller within 45 days of the initial date of delivery to you. If you return the hearing aid, the seller will either adjust or replace the hearing aid or promptly refund the total amount paid. This warranty does not affect the protections and remedies you have under other laws. We warrant that the Product will, under normal and intended use, function in accordance with the technical specifications or accompanying product documentation (the "Warranted Functionality") during the Limited Warranty Period. For the avoidance of doubt, any software or services that are required to achieve the Warranted Functionality may be updated, modified, or limited, so long as it continues

to maintain (or exceed) the Warranted Functionality. This Warranty is not available to products that were purchased from any source other than Luxottica or an authorized dealer. This Warranty is valid only with proof of purchase that clearly shows the purchase date. If you send us a Product without valid proof of purchase, then we reserve the right to return the Product to you at your cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up. If your Product is defective or malfunctioning, we will either repair or replace that Product, or update software or services, so that the Product performs substantially in accordance with the Warranted Functionality. In some cases, we may instead refund the purchase price paid for the Product, Your remedy under this Warranty will be the repair, replacement or refund of the Product unless it is impossible or disproportionate for us to do so. This Warranty gives you specific legal rights, and you may also have other rights under consumer protection laws or regulations in the jurisdiction of purchase or, if different, in your country of residence. Such rights vary from state to state and country to country. The benefits conferred by this Warranty are in addition to and do not affect any rights and remedies conveyed by such

consumer protection laws and regulations (including, without limitation, national laws implementing Directive 1999/44/EC and Directive 2019/771). You are entitled to remedies free of charge in the event that your goods do not conform with the requirements set out under the laws in your jurisdiction. The other remedies available to you according to the laws in your jurisdiction are not affected by this Warranty. For full information on your rights according to the laws in your jurisdiction, please see below and/or contact your local consumer rights organization/citizens advice bureau. This Warranty applies only to those countries to which Luxottica ships and supports. Please visit us at www.nuanceaudio.com for information about the countries Luxottica supports and to get helpful service and contact information. The scope of technical support offered by Luxottica consists of helping to diagnose and resolve problems with defects in the Product, Please note that when contacting Luxottica via telephone, long distance charges may apply,

The consumer has a period of two (2) years from the date of delivery of the goods to obtain the implementation of the legal warranty in the event of any lack of conformity. During this period, the consumer is only required to establish the existence of the lack of conformity and not the date on which it became apparent. Where the contract for the sale of goods provides for the supply of digital content or a digital service on a continuous basis for a period of time of over two (2) years, the legal warranty shall apply to that digital content or digital service throughout this period. During this period, the consumer is only required to establish the existence of the lack of conformity affecting the digital content or service and not the date on which it became apparent. The legal warranty entails an obligation on the seller's part, where applicable, to provide any updates necessary to maintain the goods in conformity. The legal warranty shall give the consumer the right to repair or replace the goods within thirty (30) days of his or her request. free of charge and without any major inconvenience to him or her. If the goods are repaired under the

legal warranty, the consumer shall benefit from a six (6) -month extension of the initial warranty. If the consumer asks for the good to be repaired, but the seller requires it to be replaced, the legal warranty of conformity shall be renewed for a period of two (2) years from the date on which the good is replaced. The consumer may obtain a reduction in the price of the goods while keeping the goods or a full refund by terminating the contract and in exchange of the goods, if: 1° The professional refuses to repair or replace the goods: 2° The goods are repaired or replaced after a downtime period of thirty days; 3° The repair or replacement of the goods causes major inconvenience to the consumer, in particular where the consumer definitively bears the cost of taking back or removing the non conforming goods, or if he or she bears the cost of installing the repaired or replaced goods; 4° The lack of conformity of the goods persists after the seller's unsuccessful attempt to bring them into conformity. The consumer is also entitled to a reduction in the price of the goods or to termination

of the contract where the lack of conformity is so

significant that it justifies the reduction in price or termination of the contract being immediate. The consumer is then not required to request repair or replacement of the goods beforehand. The consumer shall not be entitled to terminate the sale if the lack of conformity is minor. Any period of immobilization of the goods for the purpose of repair or replacement shall suspend the warranty that was running until the goods are delivered in good condition. The rights mentioned above result from the application of Articles L. 217-1 to L 217-32 of the French Consumer Code The professional who hinders the implementation of the legal warranty of conformity in bad faith shall be liable to a civil fine of up to 300,000 euros, which may be increased by 10% of the average annual turnover (Article L. 241-5 of the French Consumer Code). The consumer also benefits from the legal warranty for

hidden defects under Articles 1641 to 1649 of the

French Civil Code, for a period of two (2) years from

the discovery of the defect. This warranty entitles the

consumer to a reduction in price if the goods are kept

or to a full refund in return for the return of the goods.

The limitation of liability clause will not apply to French consumers in case of breach of any of its obligation by Luxottica.

For Italian consumers This Warranty is a conventional warranty offered by Luxottica Group S.p.A. according to Article 135-auinquies of the Legislative Decree of 6 September 2005, no. 206 ("Consumer Code"). Regardless of the Warranty offered by Luxottica Group S.p.A., the consumer has the statutory rights set forth by Article 128-135-septies of the Consumer Code for two (2) years as from delivery of the Product, Therefore. this Warranty does not affect consumers' statutory rights concerning the legal warranty of conformity. For additional information about your statutory rights under Article 128-135-septies of the Consumer Code, please visit www.nuanceaudio.com **REGISTRATION** Product registration is not required as a condition to coverage under this Warranty, but some

Luxottica products require periodic connection to an online account to ensure full functionality.

HOW TO MAKE A WARRANTY CLAIM If you have a problem with your Product, please visit us at www. nuanceaudio.com to get helpful service and contact

information, and to obtain warranty service. Unless

specifically stated elsewhere in this Warranty and except as provided by applicable state law, to make a warranty claim, you need to return your Product to Luxottica or to the authorized dealer from which the product was purchased, together with your proof of purchase. If your warranty claim is deemed to be valid, and we find a defect or malfunction covered by this Warranty, we will replace or repair the Product so it provides the functionality warranted. Subject to your rights under any local laws, if we determine that a Product should be replaced, the replacement may be a new, refurbished, or a re-manufactured Product. The Limited Warranty Period is extended by a duration equal to the time during which we had the Product in our possession for performance of the Warranty, as described above. Save for your rights and remedies available under your local laws, the provision of a repaired or replacement product does not restart or otherwise extend the Limited Warranty Period. Should it be found that none of the listed means are reasonable to correct a defect or malfunction, then we may refund to you the price you paid to purchase the Product. We may not return the original Product to you. If your warranty claim is not deemed to be valid, then we reserve the right to return the Product to you at your cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up before it is disposed of. Except to the extent that the law requires otherwise, any expenses incurred by you in returning the Product for warranty assessment will be borne by Luxottica using a pre-paid returns label which will be provided to you to enable you to return your Product for warranty assessment free of charge, or other methods listed at www.nuanceaudio. com. If the returned Product is deemed (at Luxottica's discretion) to be eligible for a valid warranty claim, then Luxottica will automatically incur the costs of any further freight charges required to return the repaired or replacement product to you. If the warranty claim is deemed to be invalid and we do not find a defect or malfunction covered by this Warranty, we will contact you to see if you want us to conduct repairs at your cost and, in any event, you will be charged for the outbound freight to return your Product based on Luxottica's standard shipping rates.

LIMITATIONS AND EXCLUSIONS THIS WARRANTY DOES NOT COVER, AND LUXOTTICA IS NOT RESPONSIBLE FOR: DELIVERY OR INSTALLATION, OR LABOR CHARGES FOR SETUP OF THE PRODUCT AND/OR ADJUSTMENT OF CUSTOMER CONTROLS ON THE PRODUCT, DAMAGES CAUSED BY MISUSE, ABUSE, ACCIDENTS, FIRE, THEFT, DISAPPEARANCE, MISPLACEMENT, FLUCTUATIONS AND POWER SURGES. CONNECTIONS TO IMPROPER VOLTAGE OR INCORRECT ELECTRICAL LINE VOLTAGE, VIRUSES, MALWARE, RECKLESS, WILLFUL, OR INTENTIONAL CONDUCT OR NEGLECT, DAMAGES CAUSED BY SERVICING THE PRODUCT WHICH HAS NOT BEEN PRE- AUTHORISED BY LUXOTTICAL DAMAGES CAUSED BY USAGE THAT IS NOT IN ACCORDANCE WITH THE PRODUCT INSTRUCTIONS/DOCUMENTATION. DAMAGES CAUSED BY FAILURE TO FOLLOW THE PRODUCT INSTRUCTIONS/ DOCUMENTATION OR FAILURE TO PERFORM CLEANING OR PREVENTIVE MAINTENANCE DAMAGES CAUSED BY THE COMBINATION OF THE PRODUCT WITH OTHER NON-LUXOTTICA BRANDED. PRODUCTS, ACCESSORIES, PARTS OR COMPONENTS OR USE OF PRODUCTS. EQUIPMENT, SYSTEMS, UTILITIES, SERVICES, SOFTWARE, PARTS SUPPLIES. ACCESSORIES, APPLICATIONS, INSTALLATIONS, REPAIRS, EXTERNAL WIRING OR CONNECTORS NOT SUPPLIED OR AUTHORISED BY LUXOTTICA THAT DAMAGE THE PRODUCT, SIGNAL ISSUES, RECEPTION PROBLEMS AND DISTORTION

RELATED TO NOISE, ECHO, INTERFERENCE OR OTHER SIGNAL TRANSMISSION AND DELIVERY PROBLEMS. ANY EQUIPMENT OR COMPONENTS THAT WERE NOT INCLUDED IN, OR WITH YOUR PRODUCT AS ORIGINALLY SOLD, NORMAL WEAR AND TEAR, DAMAGE CAUSED AS A RESULT OF IMPROPER TRANSPORTATION OR PACKING/PACKAGING WHEN RETURNING THE PRODUCT TO LUXOTTICA OR A COLLECTION DEALER DAMAGES CAUSED BY MODIFICATION OR ADAPTATION THAT MAY BE REQUIRED TO ENABLE A PRODUCT TO OPERATE IN ANY COUNTRY OTHER THAN THE COUNTRY FOR WHICH IT WAS DESIGNED. MANUFACTURED. APPROVED AND/OR AUTHORISED, OR REPAIR OF THE PRODUCT WHICH RESULTS IN DAMAGE AS A RESULT OF THESE MODIFICATIONS, DAMAGE TO PRODUCTS WHERE THE ANTI-COUNTERFEITING CODE (OR EQUIVALENT) HAS BEEN REMOVED, ERASED, DEFACED, ALTERED OR MADE ILLEGIBLE. This Warranty does not include any specific guarantees that the Product will be error-free, or regarding uptime or continued availability of data security features of software or online accounts, that any software, firmware or online sites will function uninterrupted or error-free. Except in the US and to the extent prohibited by any applicable law in any other country, this Warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover).

IMPLIED WARRANTIES EXCEPT IN THE EU, EFTA AND UK AND EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW IN ANY OTHER COUNTRY, ANY IMPLIED WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NON-INFRINGEMENT IS DISCLAIMED. IF YOU ARE A CONSUMER SUBJECT TO THE QUEBEC CONSUMER PROTECTION ACT. THE WARRANTY

DISCLAIMER CONTAINED IN THIS SECTION DOES NOT LIMIT YOUR RIGHTS AND REMEDIES UNDER SUCH ACT, INCLUDING THE RIGHT TO MAKE A CLAIM UNDER ANY OF THE STATUTORY WARRANTIES PROVIDED UNDER SECTIONS 34 TO 54 OF SUCH ACT.

LIMITATION OF LIABILITY EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW, LUXOTTICA SHALL NOT, UNDER THIS WARRANTY, BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES FOR BREACH OF ANY EXPRESS OR IMPLIED WARRANTY. EXCEPT TO THE EXTENT PROHIBITED BY LAW, LUXOTTICA'S MAXIMUM LIABILITY FOR ANY CLAIM ARISING OUT OF OR RELATED TO THIS WARRANTY IS LIMITED TO THE PURCHASE PRICE PAID BY YOU FOR THE PRODUCT PLUS ANY INTEREST ALLOWED BY LAW REGARDLESS OF THE FORM OF CLAIM.

TO THE EXTENT PERMITTED BY LAW, LUXOTTICA IS NOT LIABLE FOR EVENTS BEYOND ITS CONTROL, SUCH AS ACTS OF GOD, VIRUSES, PROPERTY DAMAGE, LOSS OF USE, INTERRUPTION OF BUSINESS, LOST PROFITS, LOST DATA OR OTHER CONSEQUENTIAL, PUNITIVE OR SPECIAL DAMAGES, HOWSOEVER CAUSED, FOR BREACH OF THIS WARRANTY, CONTRACT OR TORT (INCLUDING NEGLIGENCE). This Warranty does not affect your legal rights under any applicable state and national law governing the sale of consumer goods.



Nuance Audio™ is a registered trademark of Essilor International SAS. FCC ID: 2AYOA-4006



Luxottica Group S.p.A.

Piazzale Cadorna 3, 20123 Milan, Italy



Made in China

Fabriqué en Chine



www.nuanceaudio.com



For the latest and up-to-date Instructions For Use - Safety & Warranty, please refer to the following website page: www.nuanceaudio.com/en-us/c/support/manual-guide.







