


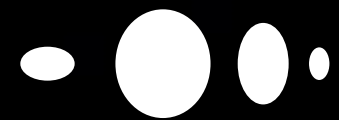


Getting started: account creation, pairing, calibration



Unboxing



Unboxing

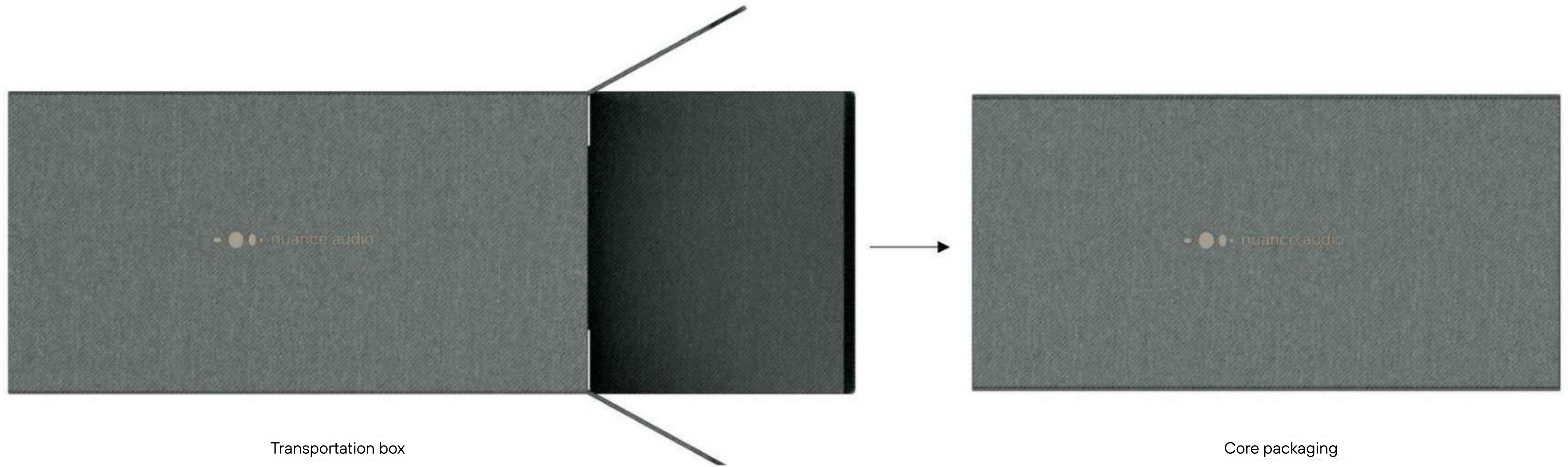




Unboxing



Glasses



Charging pad

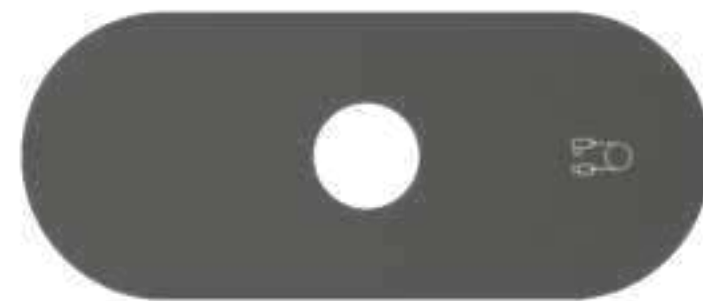

Cable wrap



Folded case



Envelope



User guide



Safety & warranty



Cleaning cloth envelope + cleaning cloth




Activate and set up your Nuance Audio Glasses



Download app & sign up



To activate and personalize your Nuance Audio Glasses, **download the Nuance Audio App** from iOS App Store or Google Play Store, which contains the Instructions for Use.

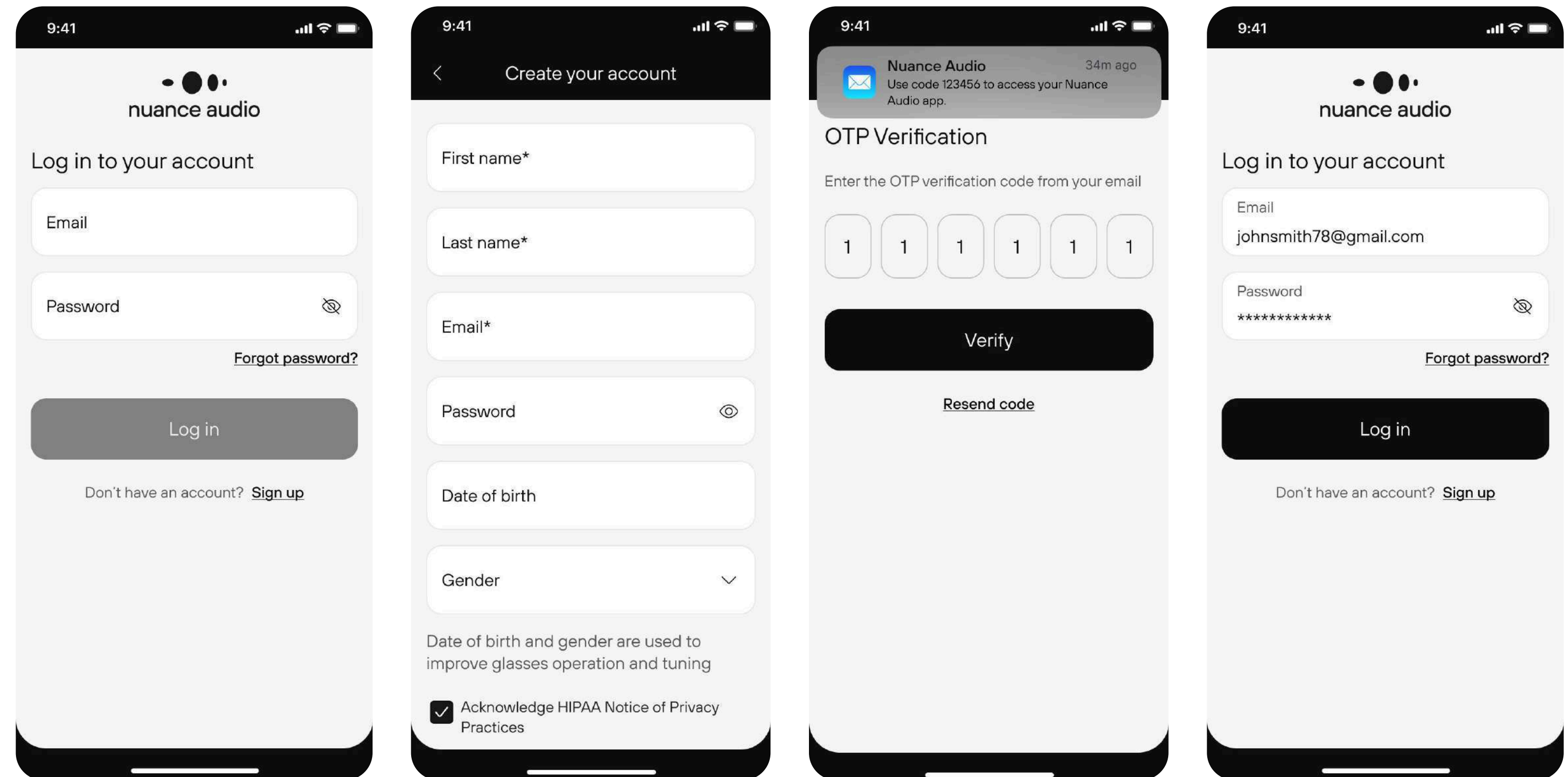


Open the Nuance Audio App and sign up or log in. Follow the step-by-step instructions provided on-screen to get started.



Glasses-phone pairing



Start pairing
Press **Start**

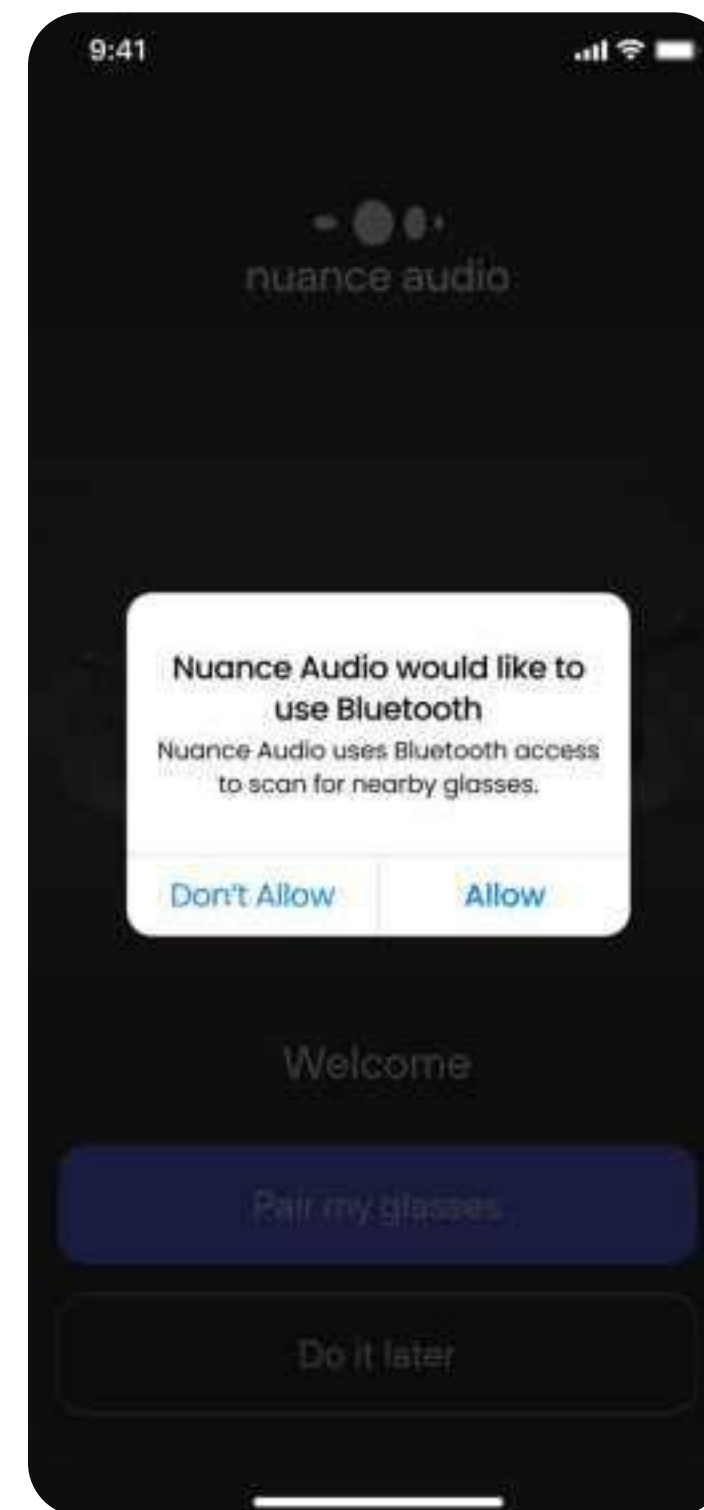

Allow Bluetooth permission.
Press **Allow**




Enter pairing mode by **pressing & holding the multifunction button** until the blue LED blinks.







Glasses-phone pairing

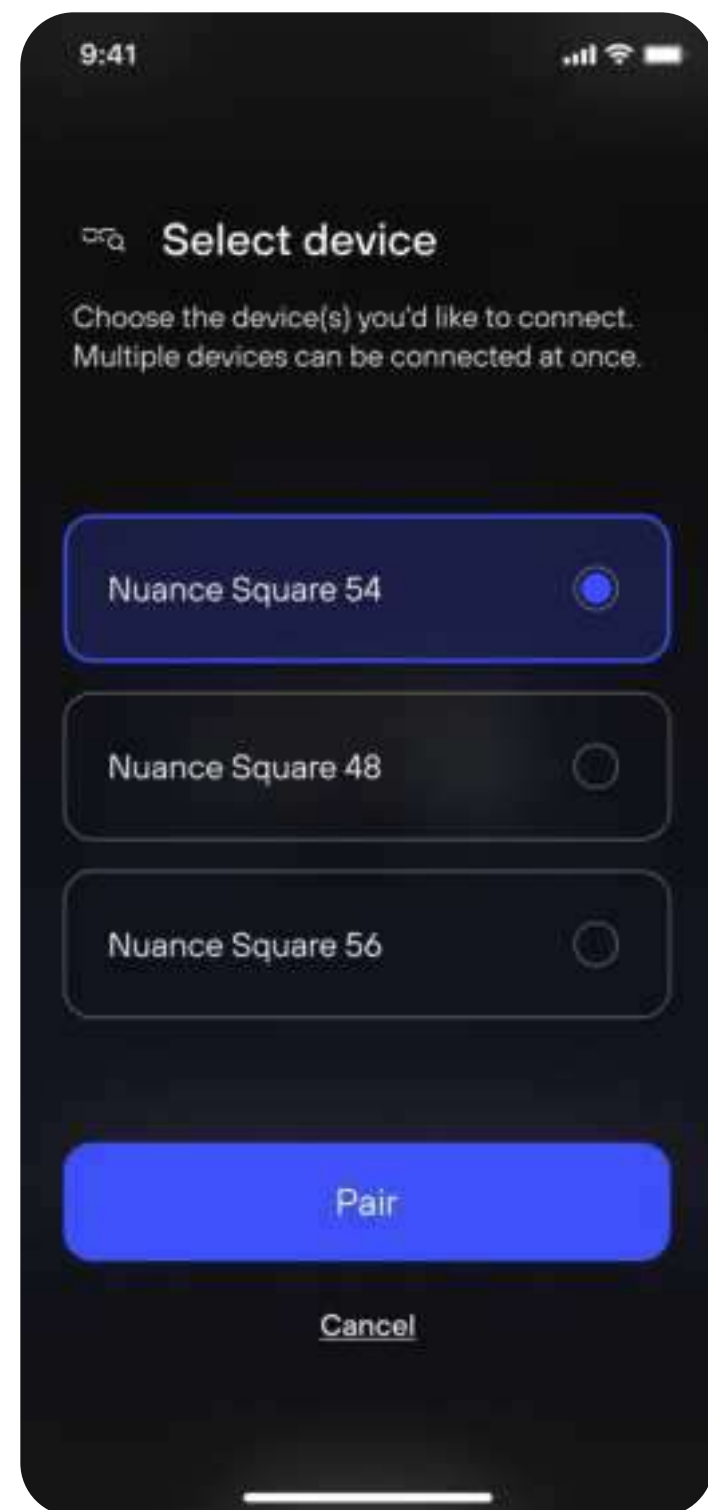

Select Nuance Audio Glasses from the list. Press **Pair**

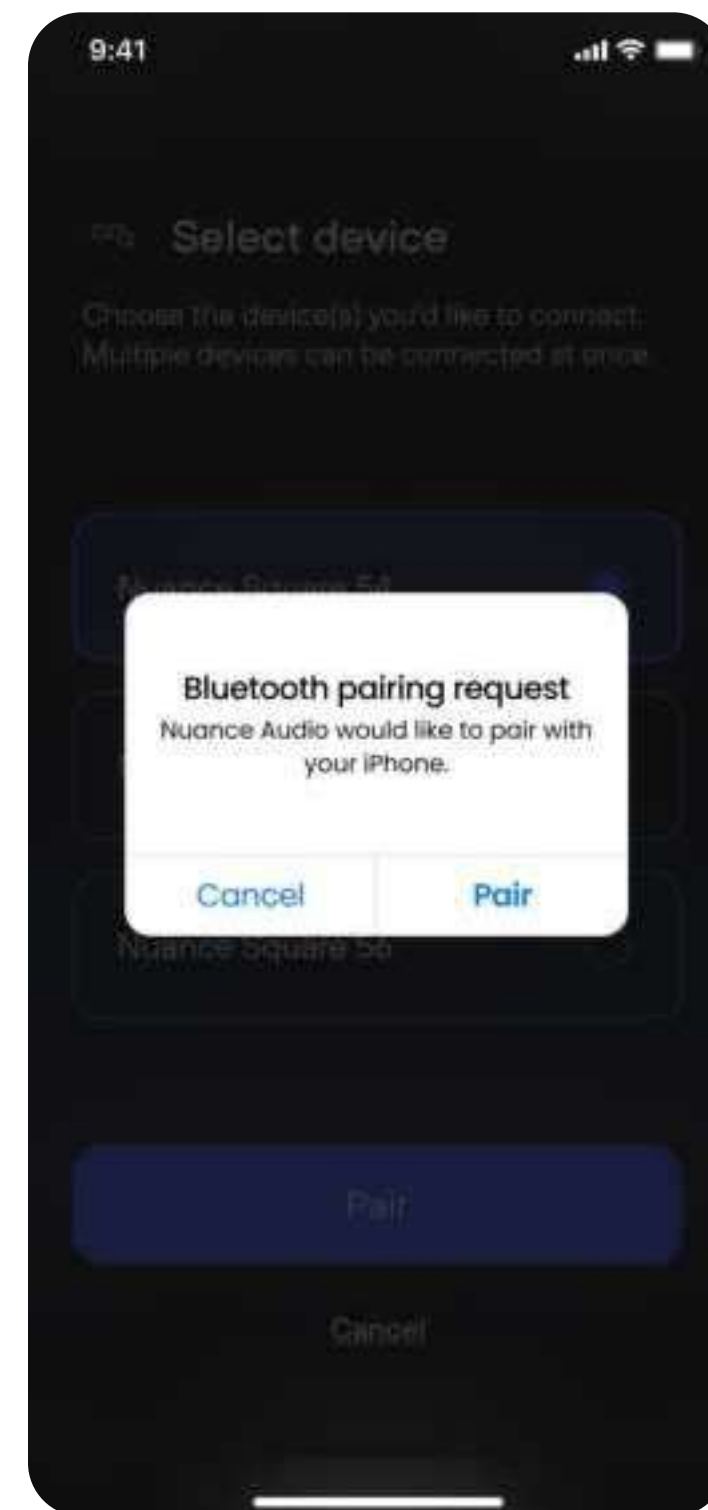

Accept the request to pair with your iPhone. Press **Pair**

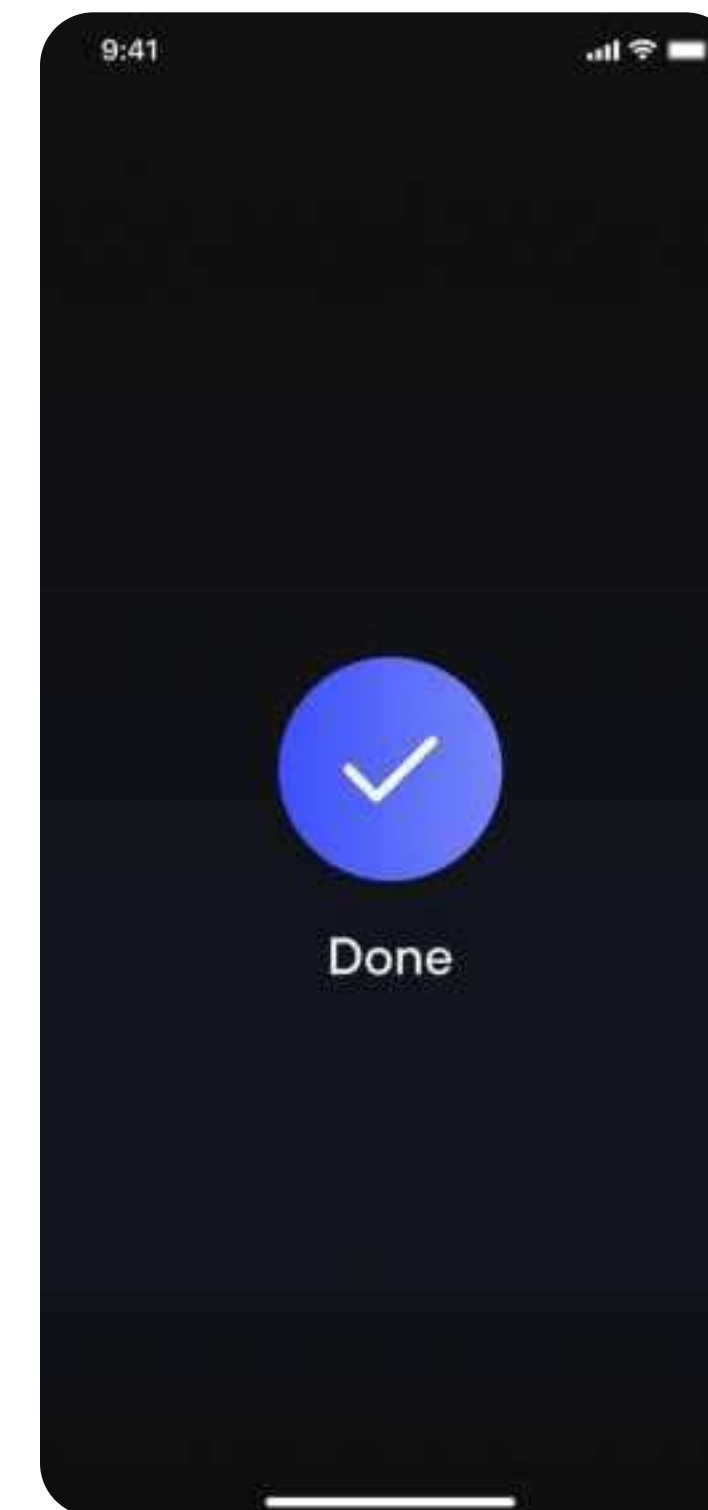

Pairing complete.



How to make the most of your Calibration?

Calibration takes just one minute but helps you get the best from your Nuance Audio Glasses.

If you own multiple pairs, calibrate each one individually.

A few tips to optimize

Personal Calibration:

- Find a quiet spot in your home, avoiding large or open spaces.
- Stay away from areas with many or large windows.
- Avoid rooms with high ceilings where background noise can be unpredictable.
- Position yourself away from corners and avoid facing mirrors.
- Ideally, stay as close to the center of the room as possible for optimal sound quality.
- Final tip: Your bedroom is often the best option.

A few tips to optimize

Voice Calibration:

- Speak loudly but without shouting.
- Project your voice as if speaking to someone across the room.
- Minimize pauses as much as possible.



Personal calibration



Calibrating will optimize your Glasses' audio performance to your head shape including:


- Acoustic feedback reduction
- Own voice reduction

To start calibrating, find a silent room.


Press **Start calibration**

Notes

The calibration feature can be found in the settings. We recommend to perform calibration every 6 months.

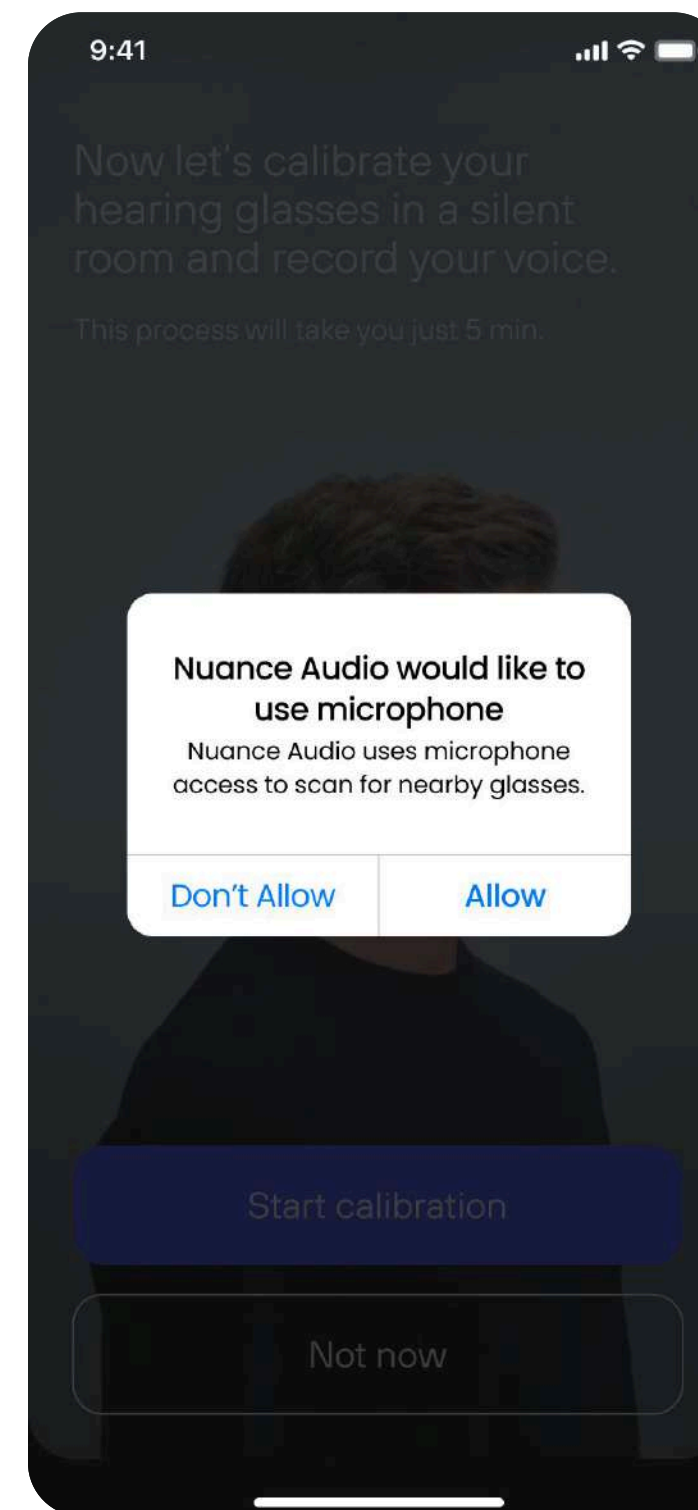


Allow your phone's microphone to measure and validate sounds. Press **Allow**

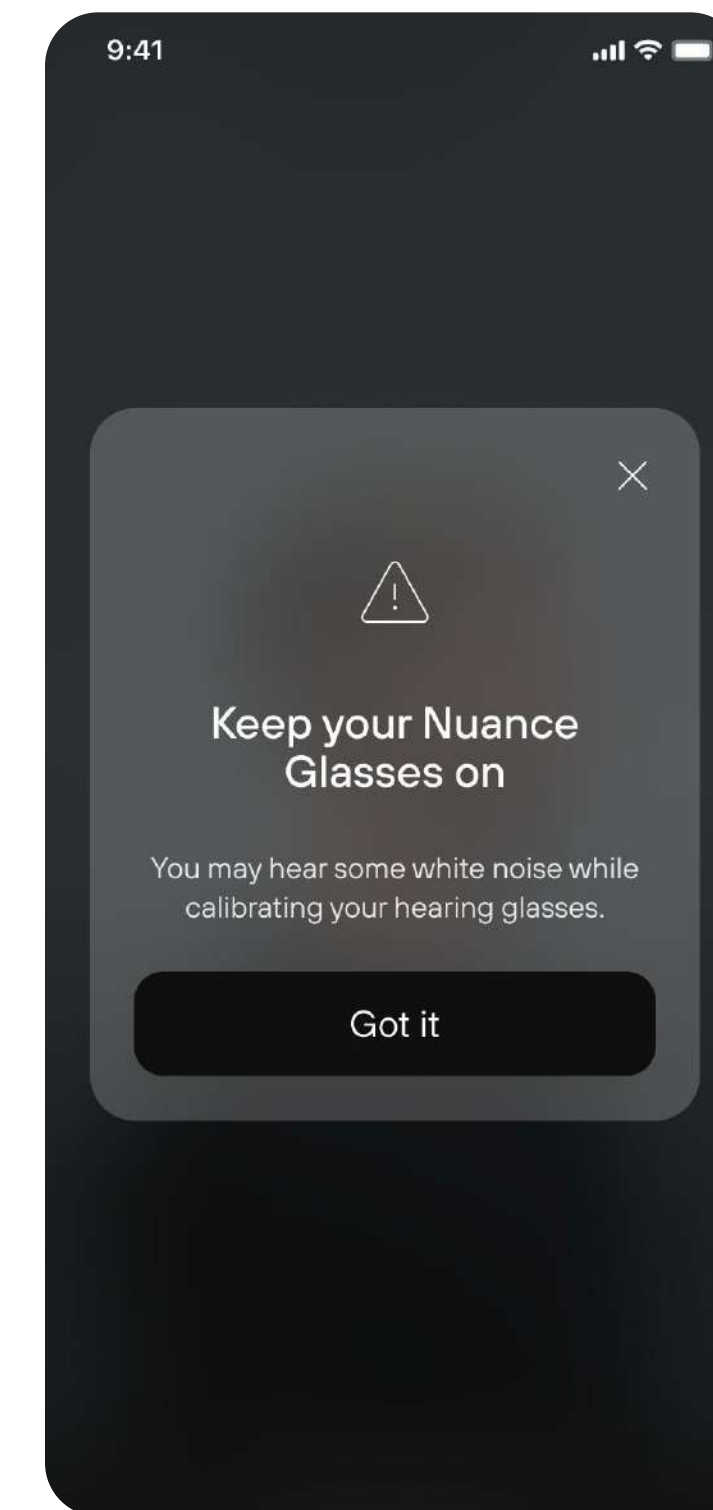


A pop-up will remind you to **keep your Nuance Audio Glasses on** during the entire process.




Personal calibration

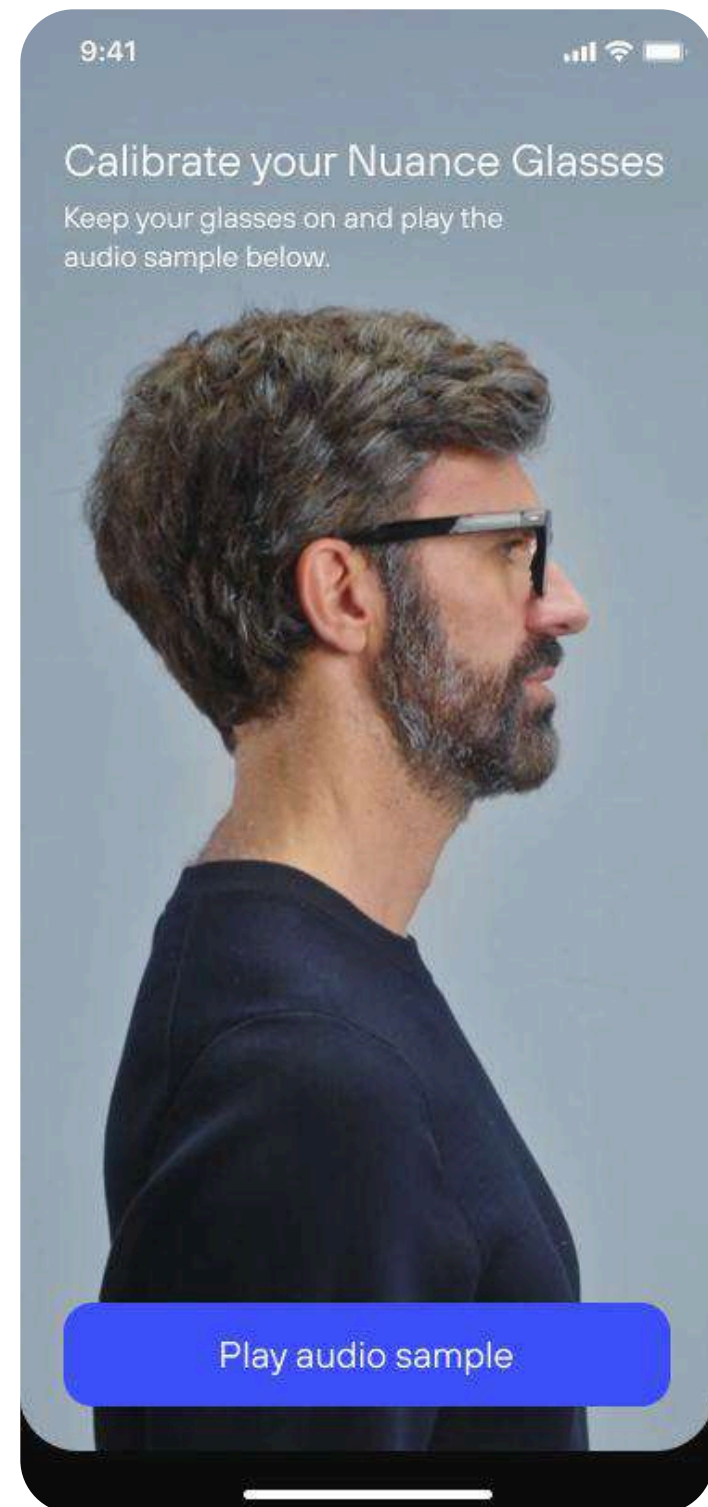

Step 1.1
Environmental noise measurement.
Keep your glasses on and tap **Play audio sample**

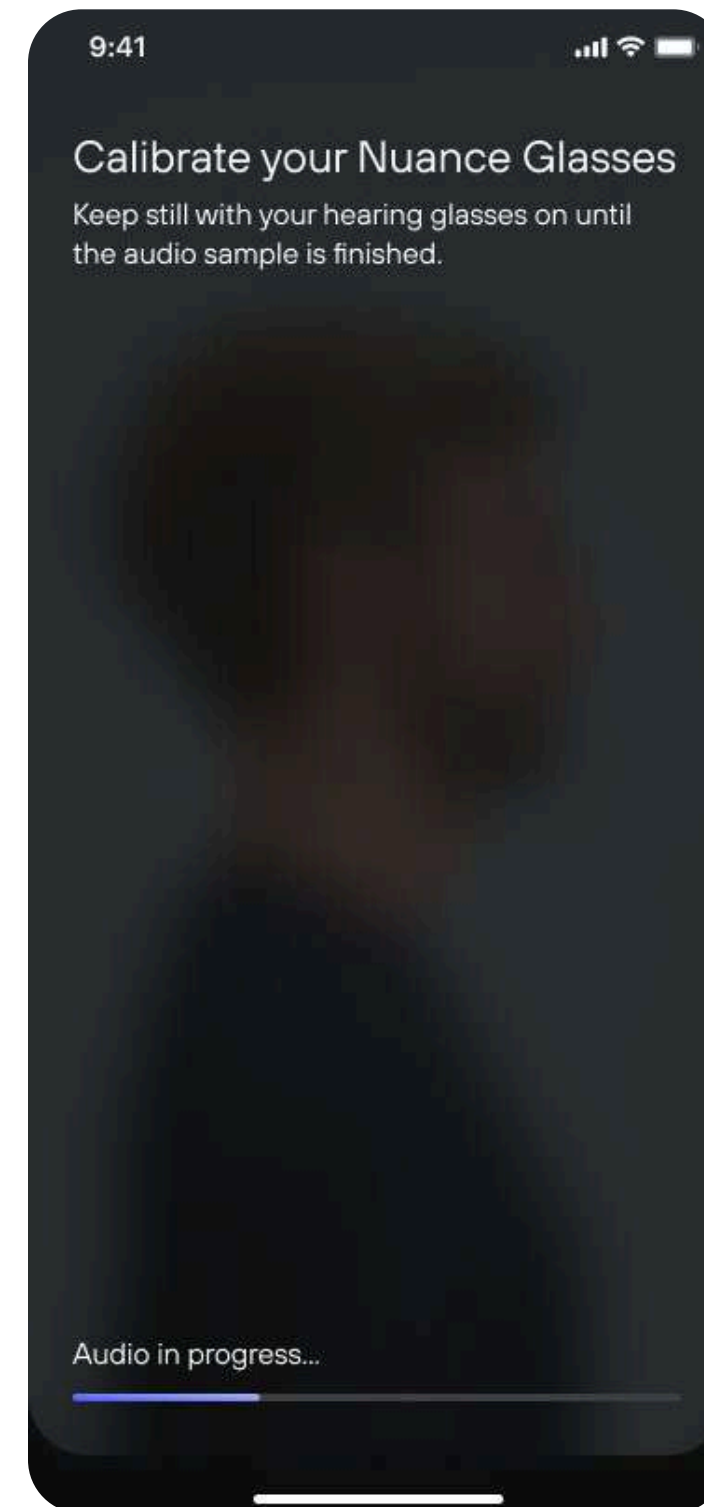


Step 1.2
Stay still and don't take off your glasses, **wait until the end of the audio sample.**

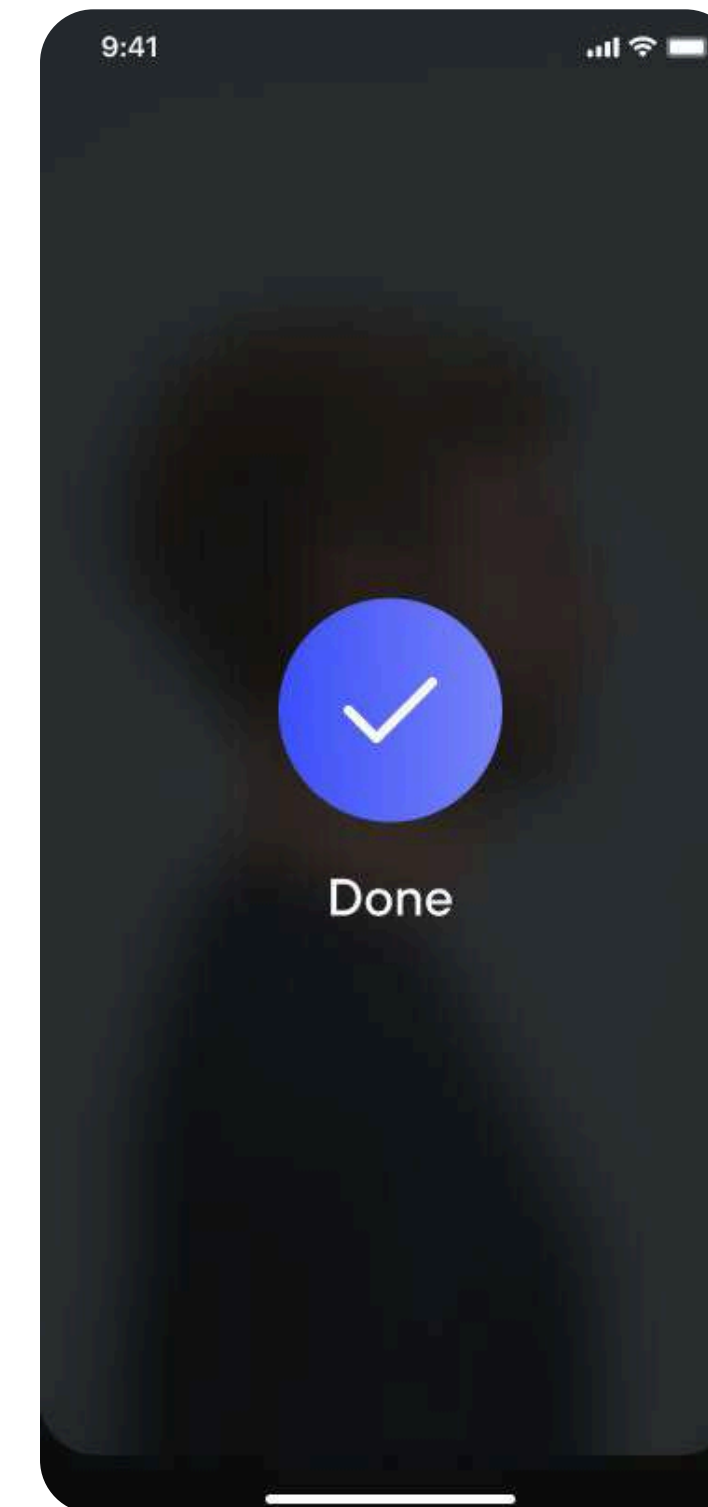


Step 1.3
Environmental noise measurement completed.




Voice calibration

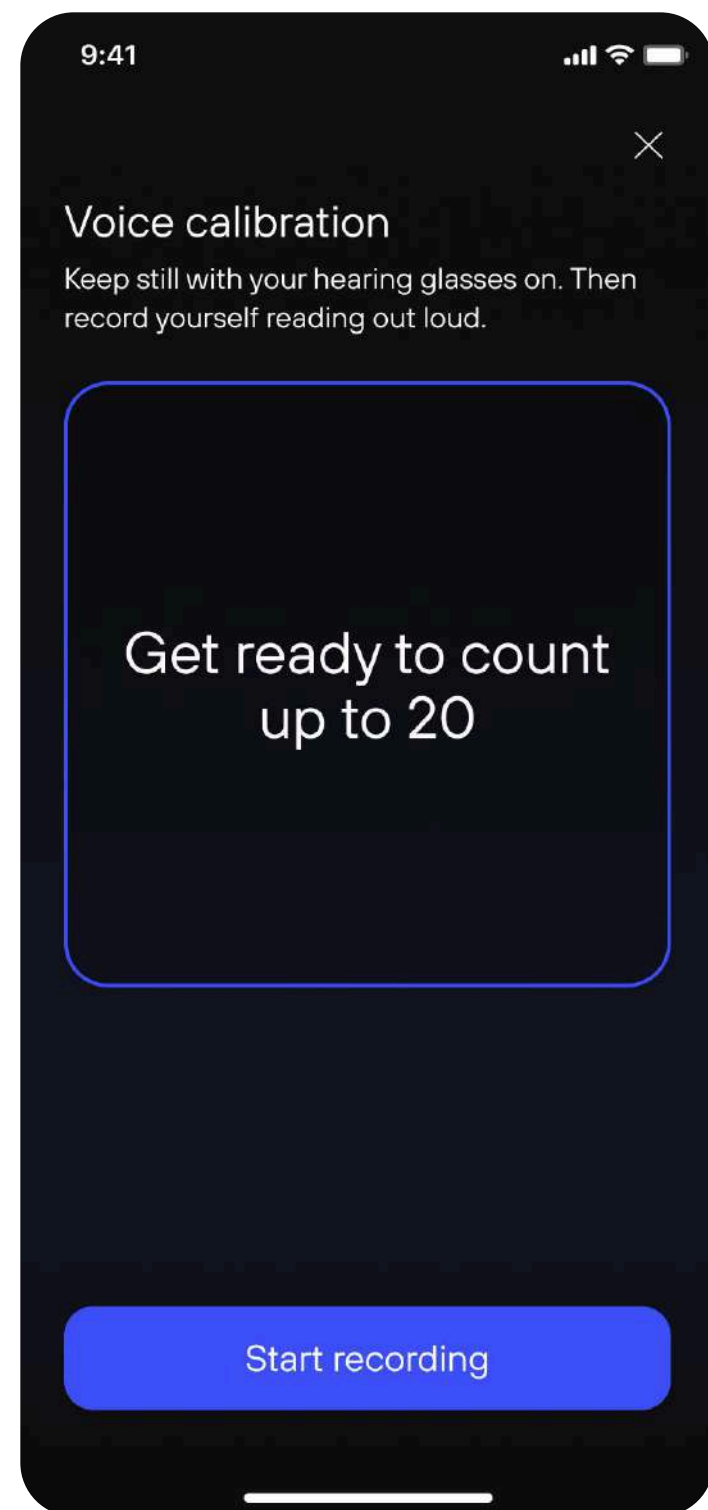

Step 2.1
Own voice calibration.
Stay still and don't take off
your glasses
Press **Start recording**

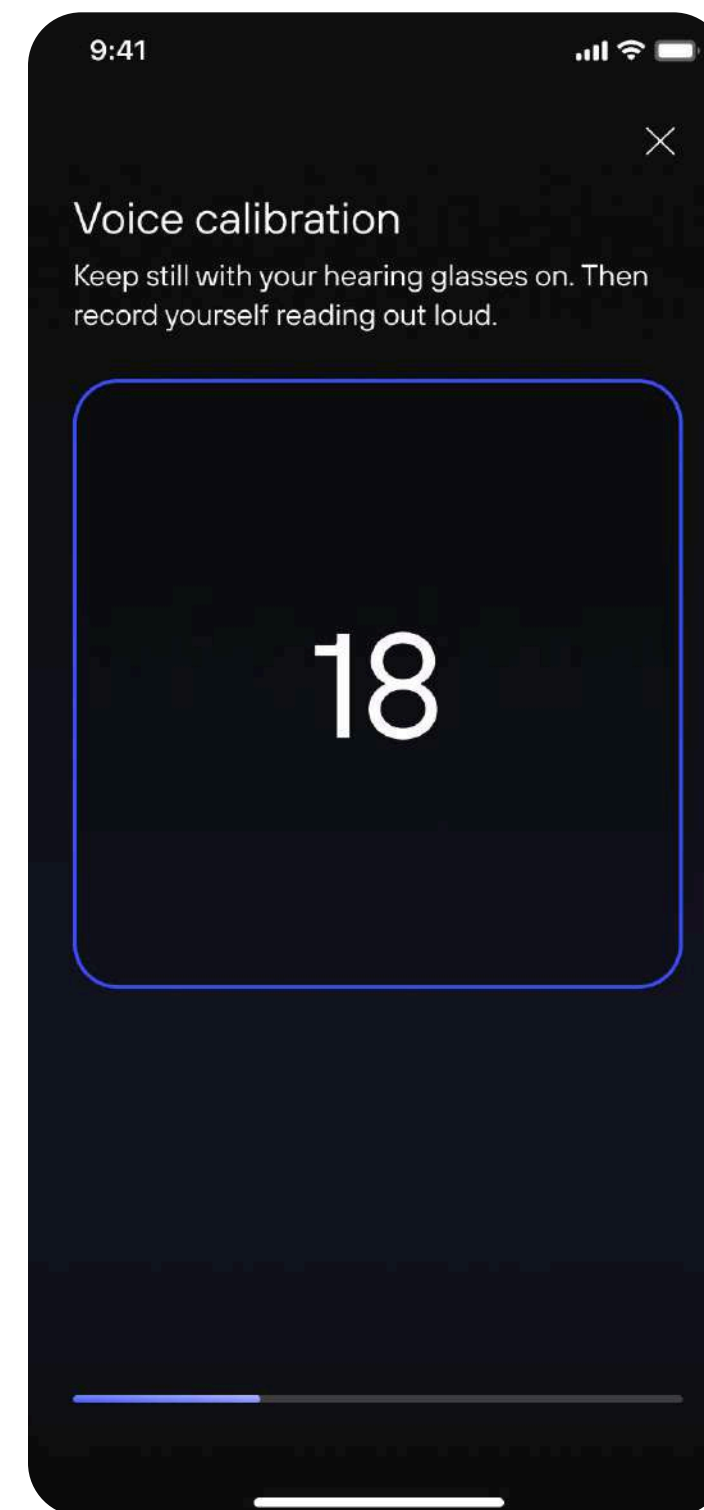

Step 2.2
**Count out loud until the
progress bar reaches
the end.**

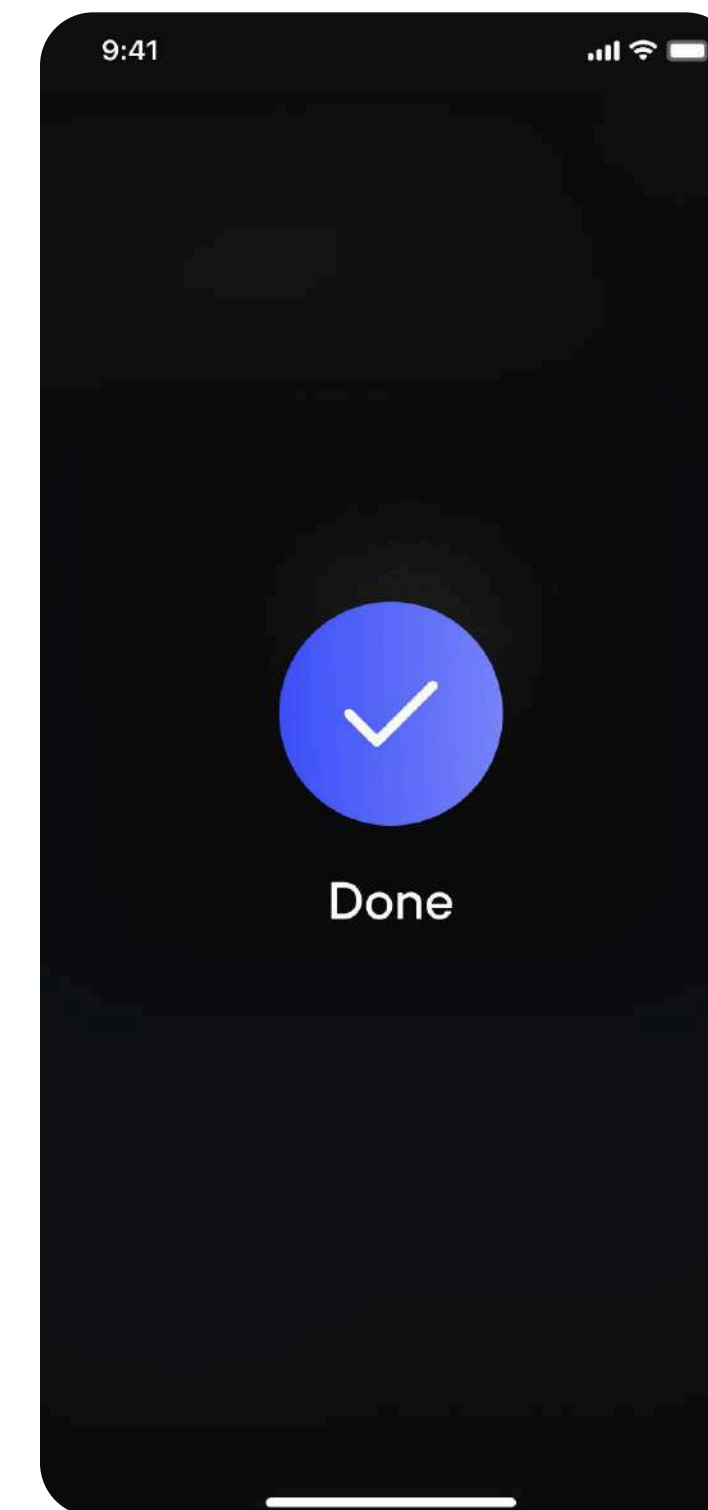

Step 2.2
Personal calibration
successfully completed.



For the most updated Instructions for Use
please follow this link

[↗ View Instruction for Use](#)