


Getting started: account creation, pairing, calibration



Unboxing



Unboxing

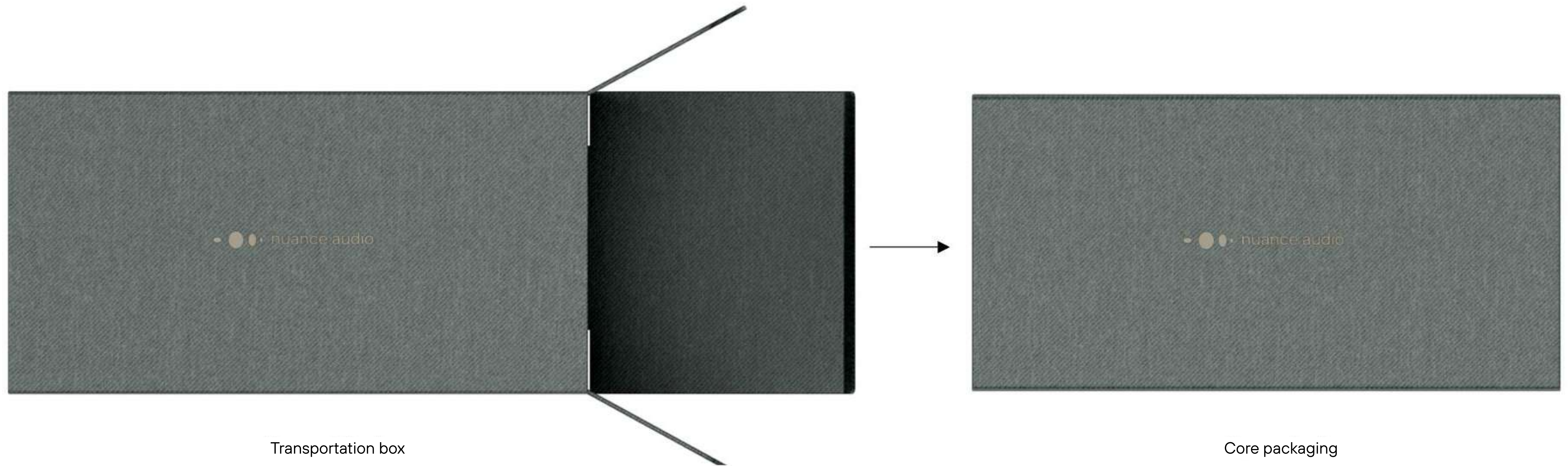




Unboxing



Glasses




Charging pad

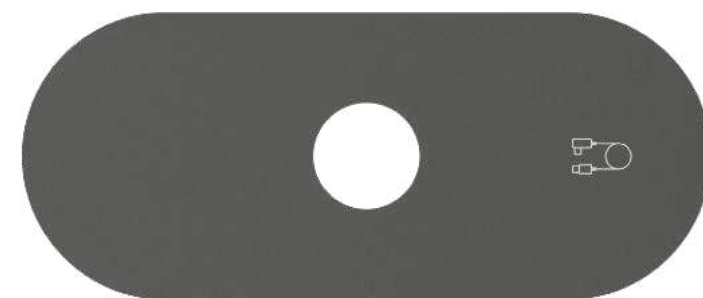

Cable wrap



Folded case

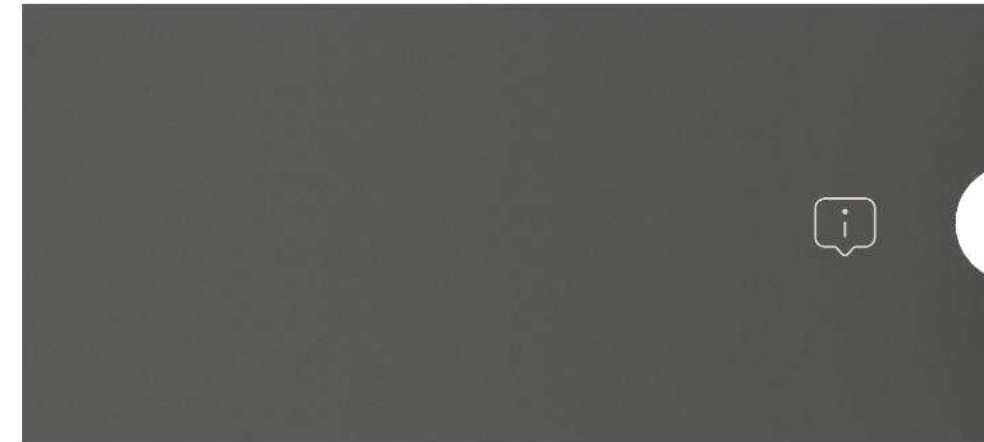


Envelope




User guide



Safety & warranty

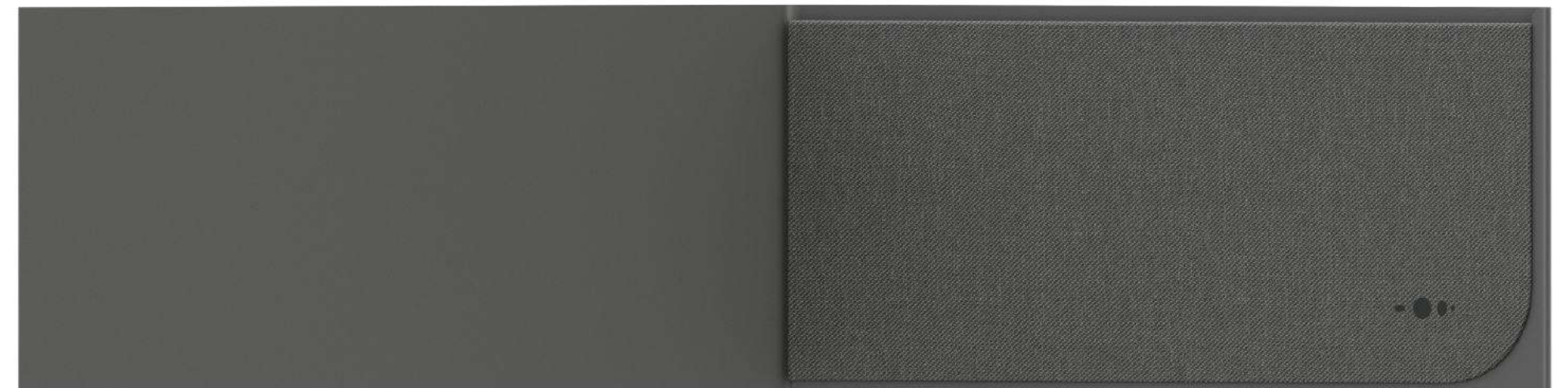

Cleaning cloth envelope + cleaning cloth

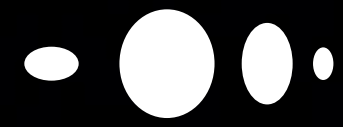


Activate and set up your Nuance Audio Glasses



Find the Nuance Audio App

For easy use of your Nuance Audio Glasses, our intuitive mobile app allows you change audio modes, control volume, and reduce background noise – all from your phone.

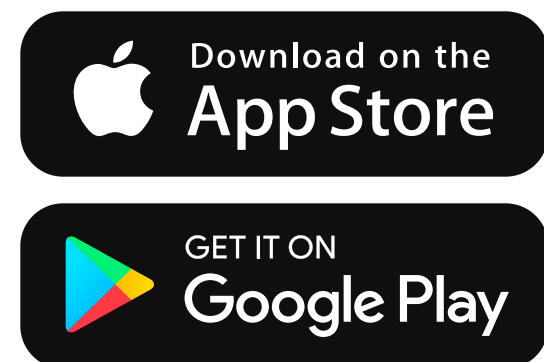


Option A.

Scan the QR Code with your phone and [Download](#)




Notes

You can find the QR Code also on the core box

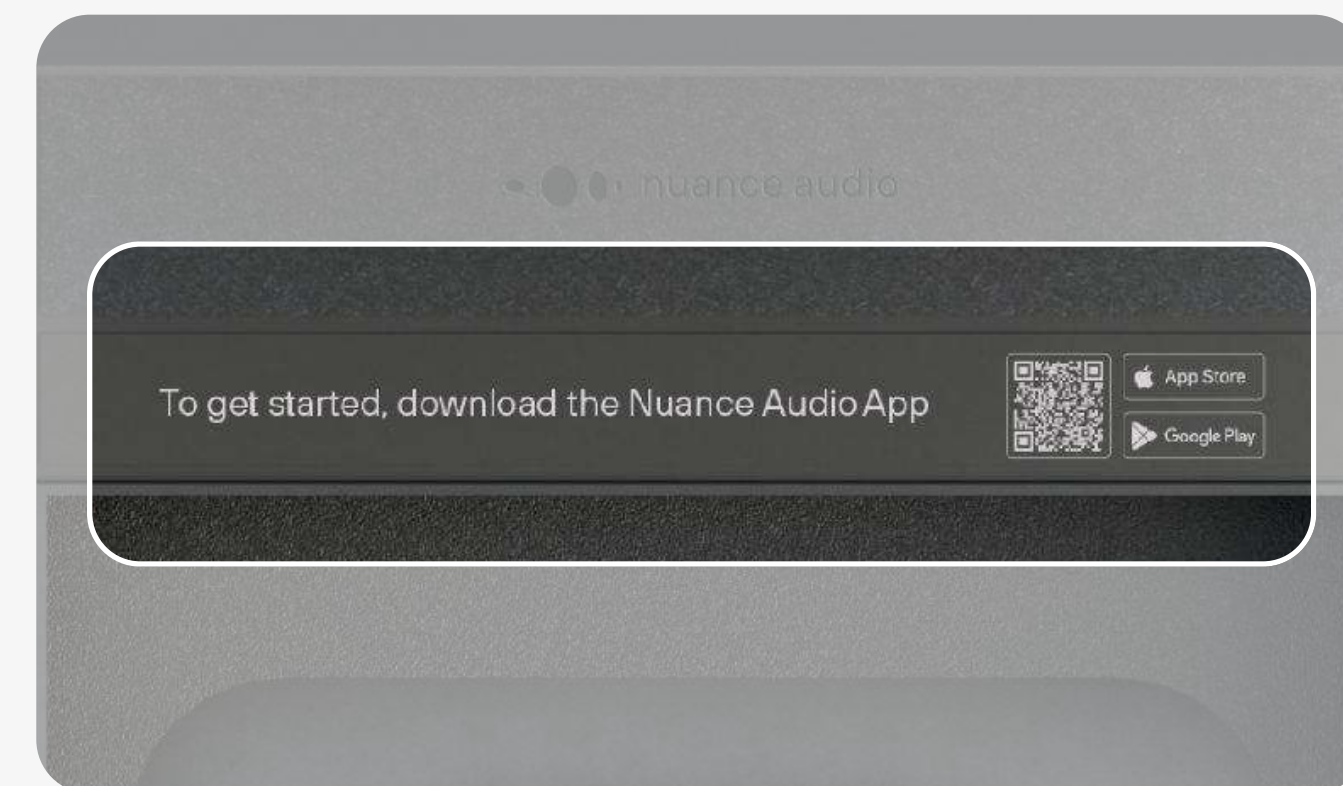

Option B.


Search for “Nuance Audio” in the App Store or Google Play Store and [Download](#)

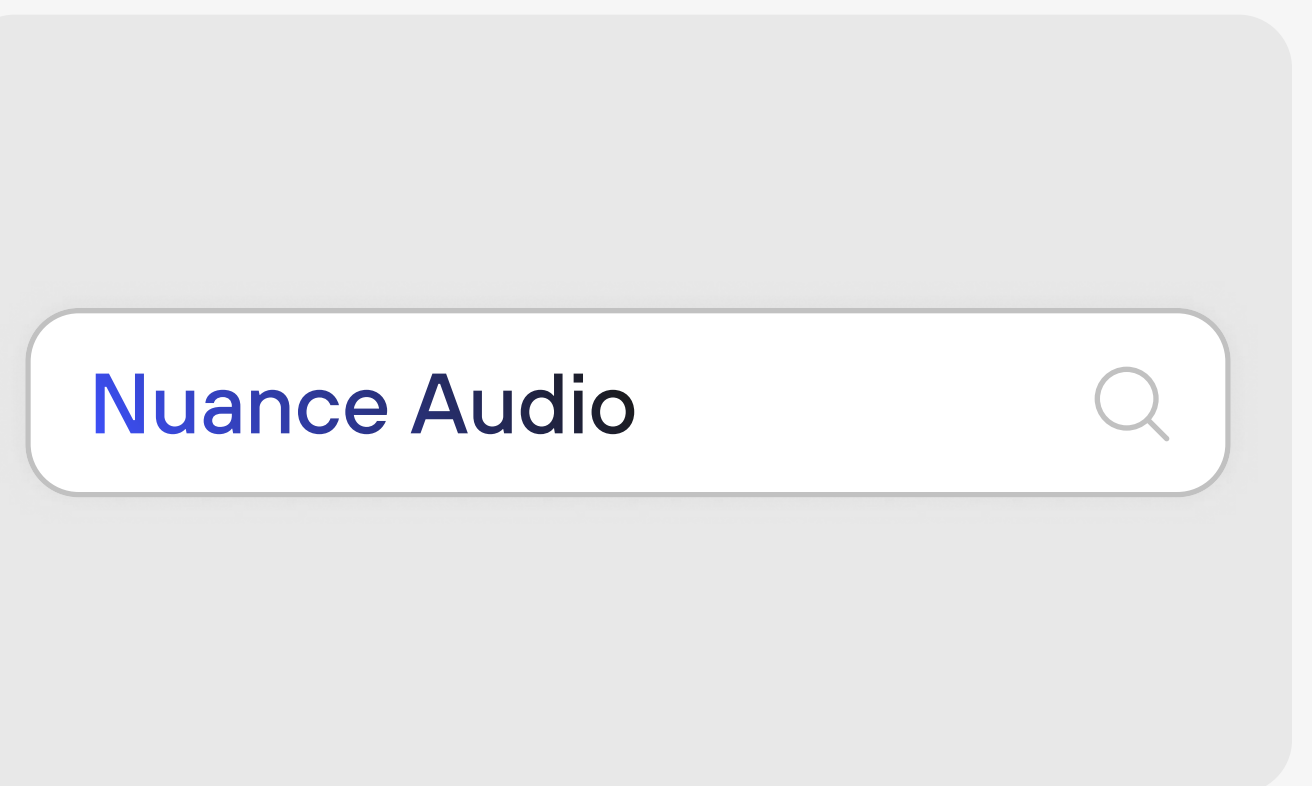




Download from your device's app store

To activate and personalize your Nuance Audio Glasses, **download the Nuance Audio App** from the iOS App Store or Google Play Store.

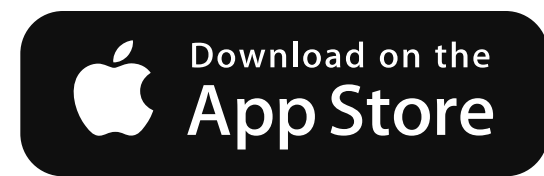


Apple App Store

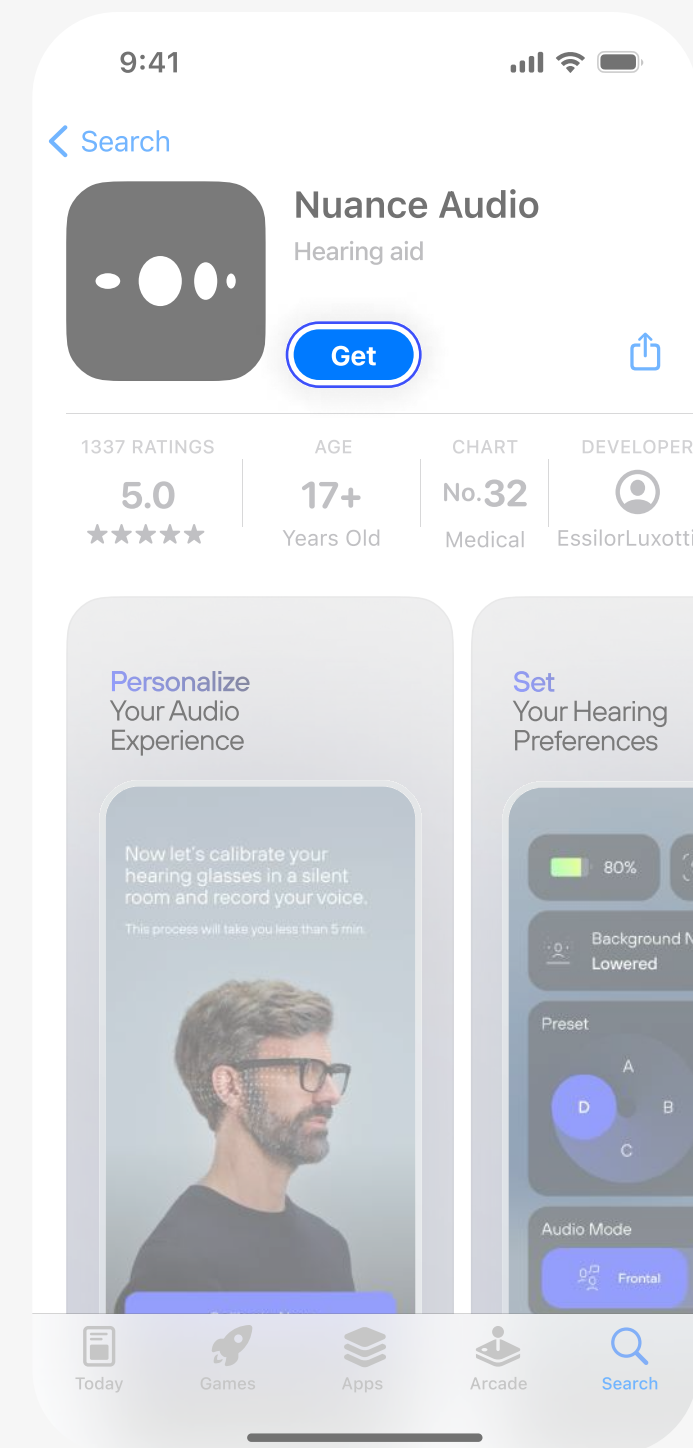

Search for “Nuance Audio” in the iOS App Store and tap **Get**

the Get button will change to Open. You can launch the app from the App Store or find it on your home screen.

Google Play Store

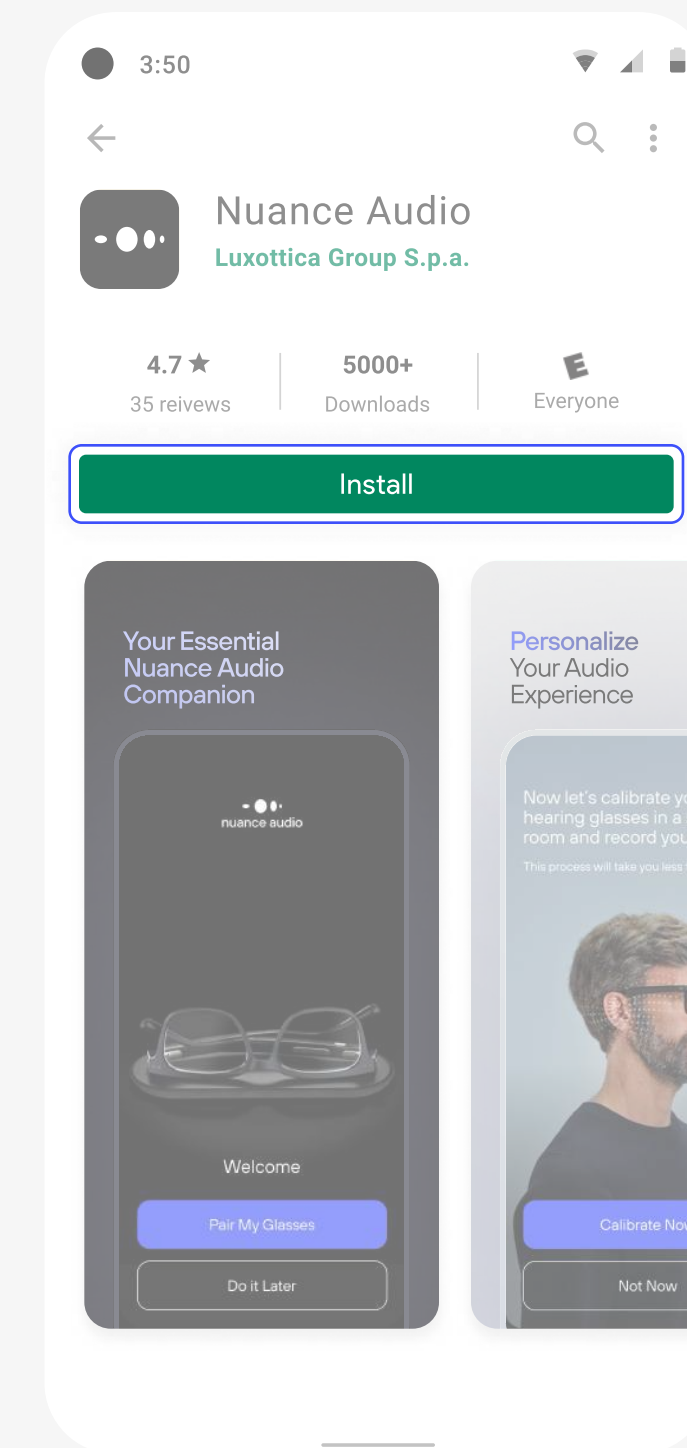

Search for “Nuance Audio” in the Google Play Store and tap **Install**

the Get button will change to Open. You can launch the app from the Store or find it on your home screen.



Sign up

Once you have downloaded the App, you can proceed with the Sign Up.

nuance audio

Log in to your account

Email

Password

[Forgot password?](#)

Log in

Don't have an account? [Sign up](#)

< Create your account

First name*

Last name*

Email*

Confirm Email*

Password

Date of birth

Gender

Date of birth and gender are used to

Nuance Audio

34m ago

Use code 123456 to access your Nuance Audio app.

OTP verification

Enter the OTP verification code from your email

111111

Verify

[Resend code](#)

nuance audio

Log in to your account

Email

johnsmith78@gmail.com

Password

[Forgot password?](#)

Log in


Don't have an account? [Sign up](#)



Glasses-phone pairing



Start pairing.
Press **Start**

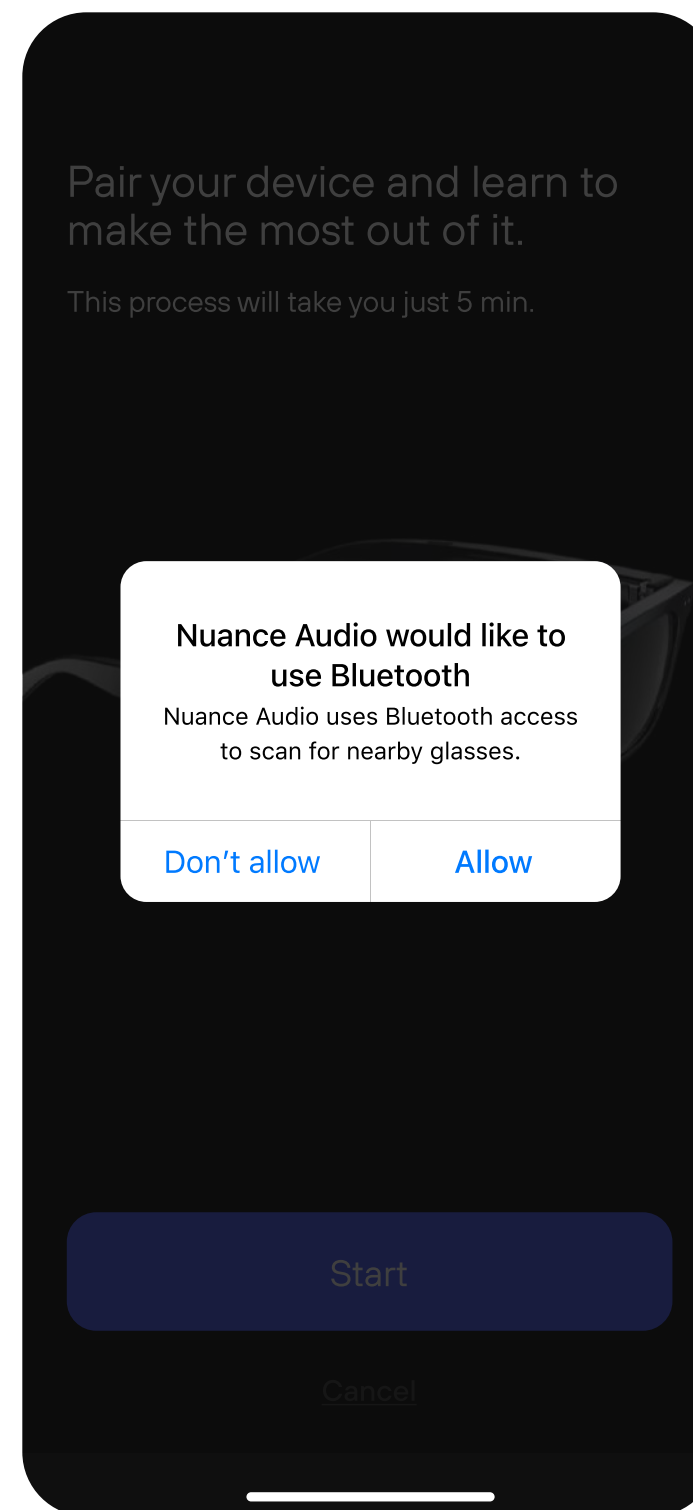

Allow Bluetooth
permission.
Press **Allow**




Enter pairing mode by
**pressing & holding the
multifunction button**
until the blue LED blinks.







Glasses-phone pairing

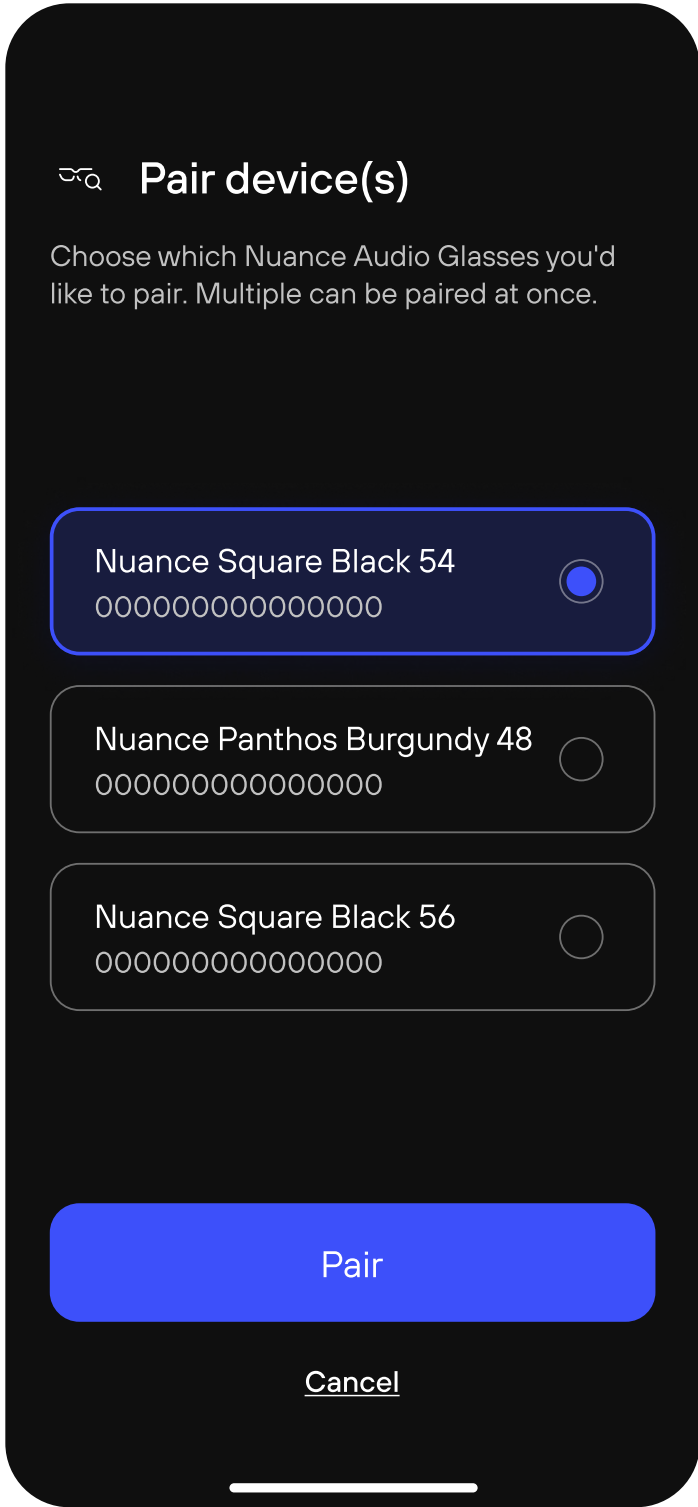

Select Nuance Audio Glasses from the list. Press **Pair**

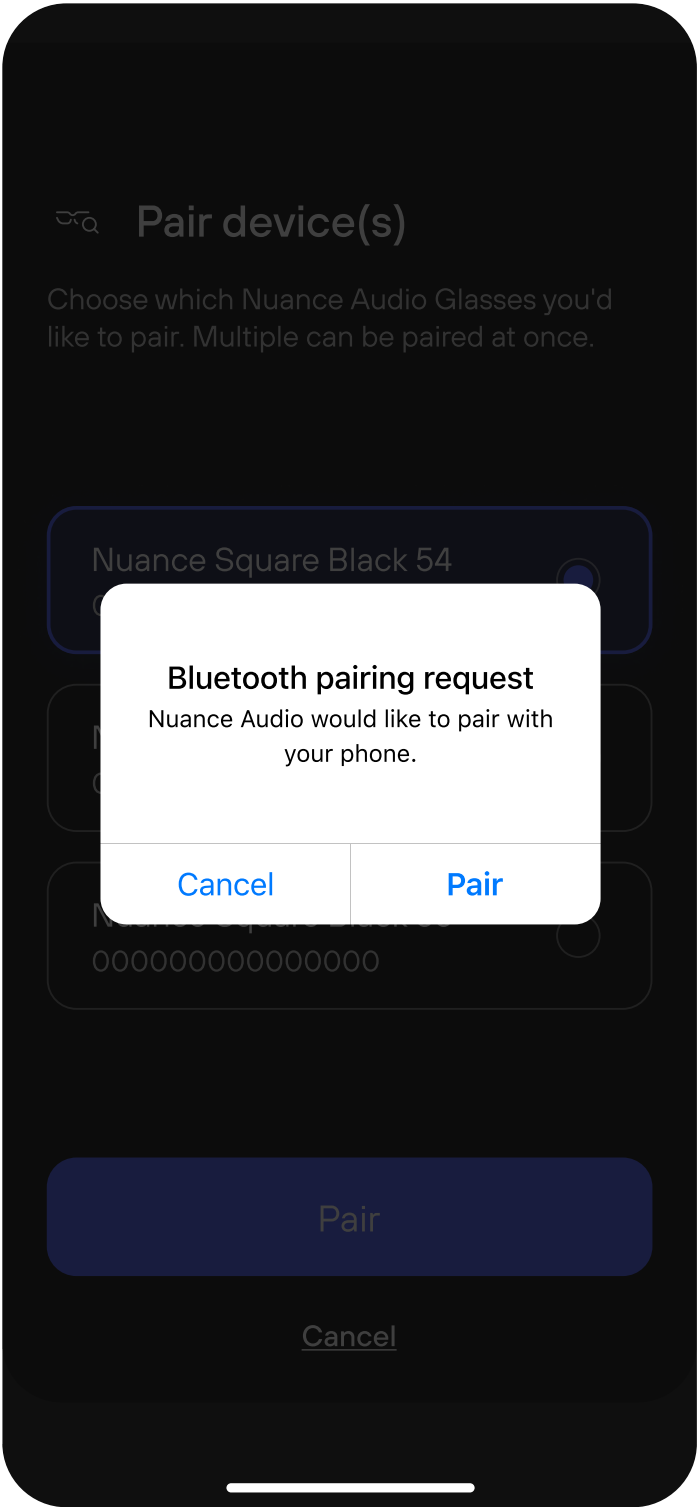

Accept the request to pair with your iPhone. Press **Pair**

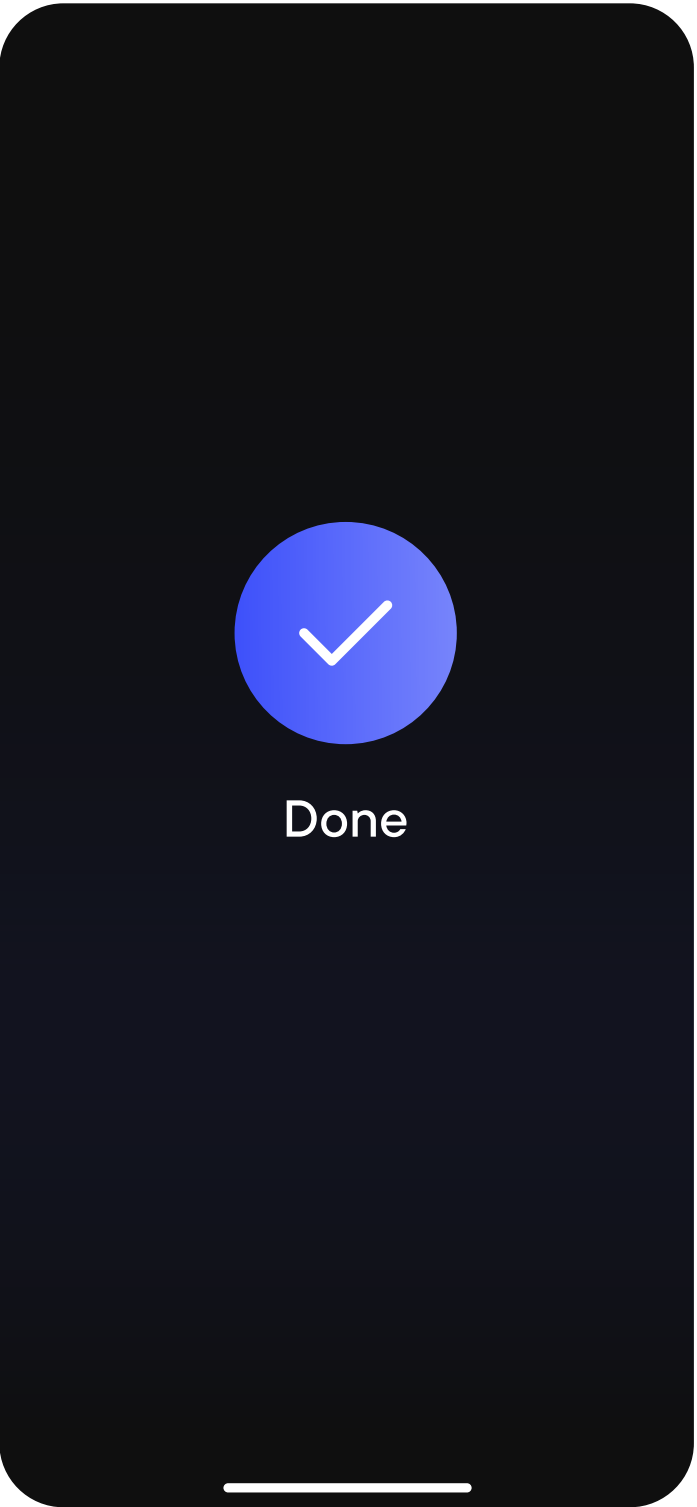

Pairing complete.



How to make the most of your Calibration?

Calibration takes just one minute but helps you get the best from your Nuance Audio Glasses.

If you own multiple pairs, calibrate each one individually.

A few tips to optimize Personal Calibration:

- Find a silent spot in your home, avoiding large or open spaces.
- Stay away from areas with many or large windows.
- Avoid rooms with high ceilings where background noise can be unpredictable.
- Position yourself away from corners and avoid facing mirrors.
- Ideally, stay as close to the center of the room as possible for optimal sound quality.
- Final tip: Your bedroom is often the best option.

A few tips to optimize Voice Calibration:

- Speak loudly but without shouting.
- Project your voice as if speaking to someone across the room.
- Minimize pauses as much as possible.



Personal calibration



Calibrating will optimize your Glasses' audio performance to your head shape including:


- Acoustic feedback reduction
- Own voice reduction

To start calibrating, find a silent room.


Press **Start calibration**

Notes

The calibration feature can be found in the settings. We recommend to perform calibration every 6 months.

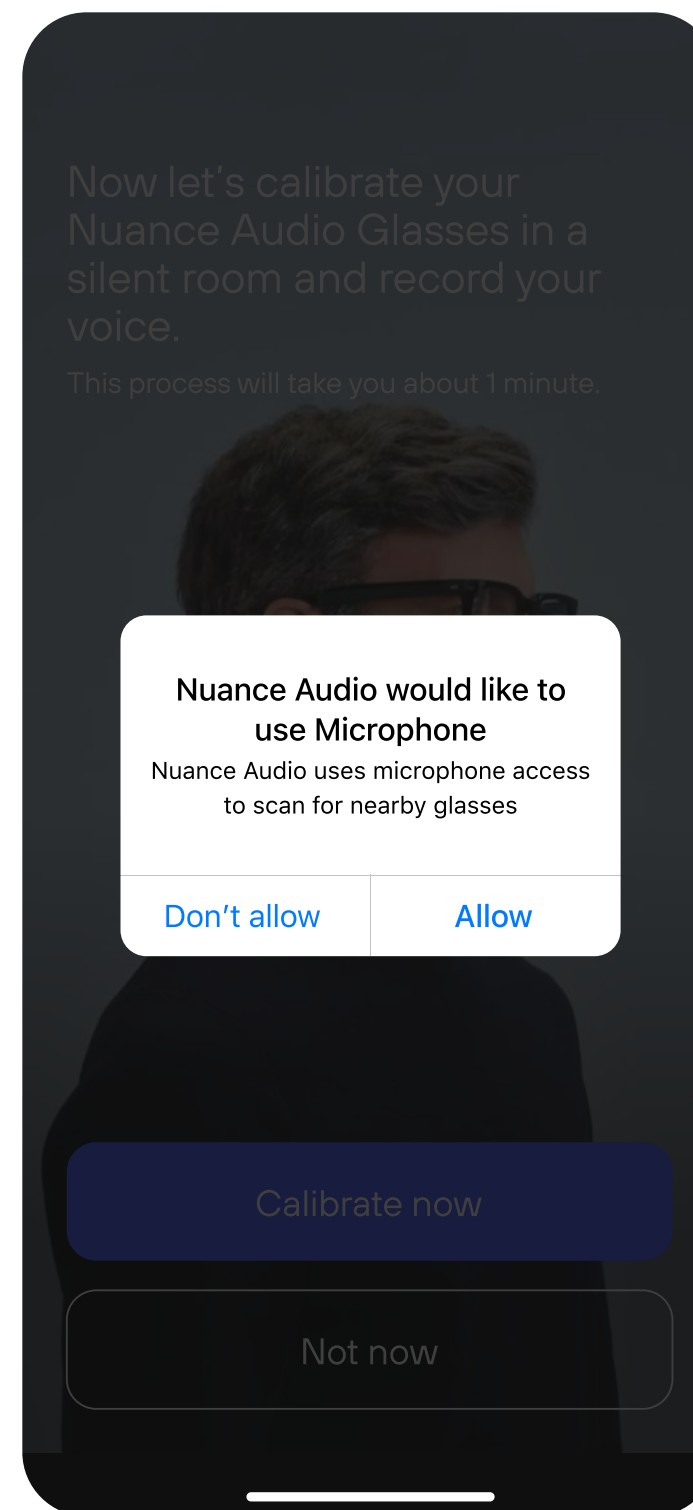

Allow your phone's microphone to measure and validate sounds. Press **Allow**

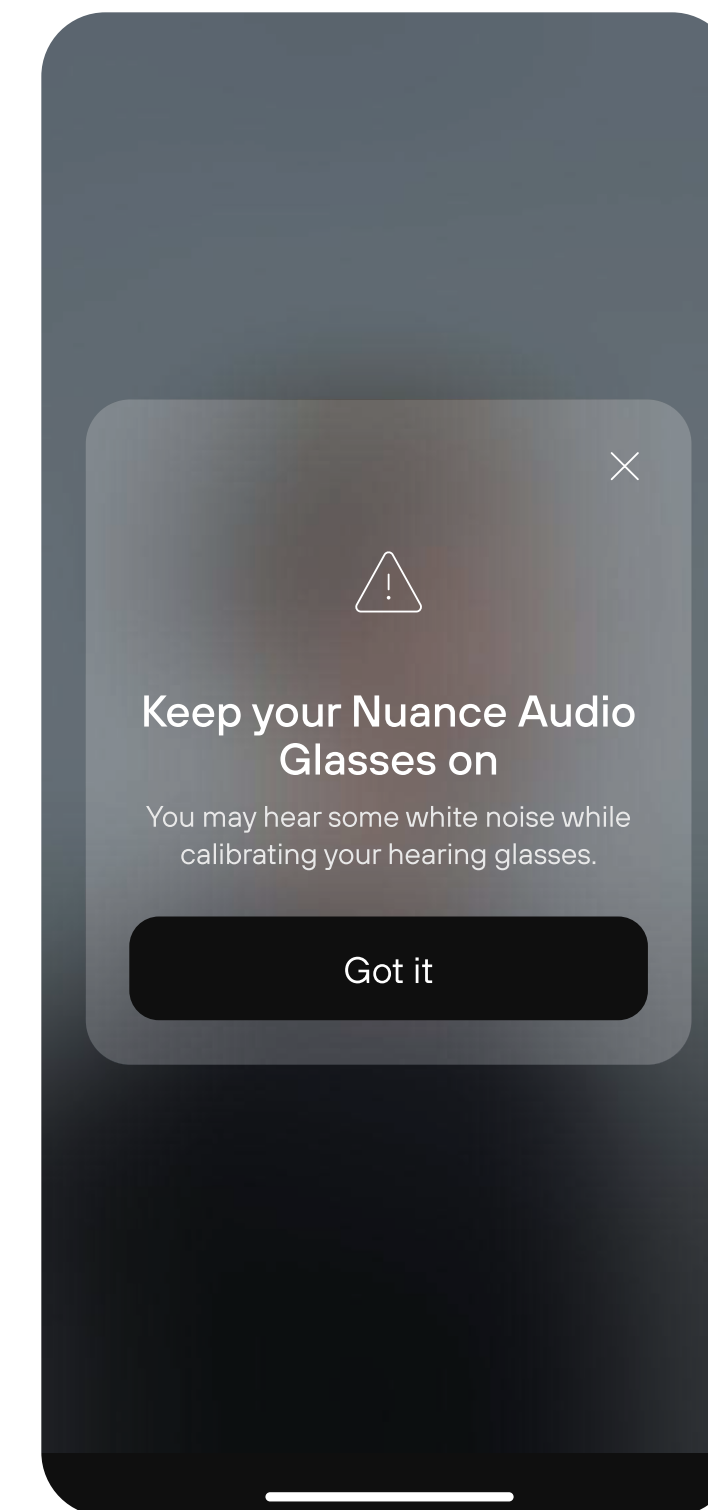


A pop-up will remind you to **keep your Nuance Audio Glasses on** during the entire process.




Personal calibration



Step 1.1
Environmental noise
measurement.
Keep your glasses on and
tap **Play audio sample**

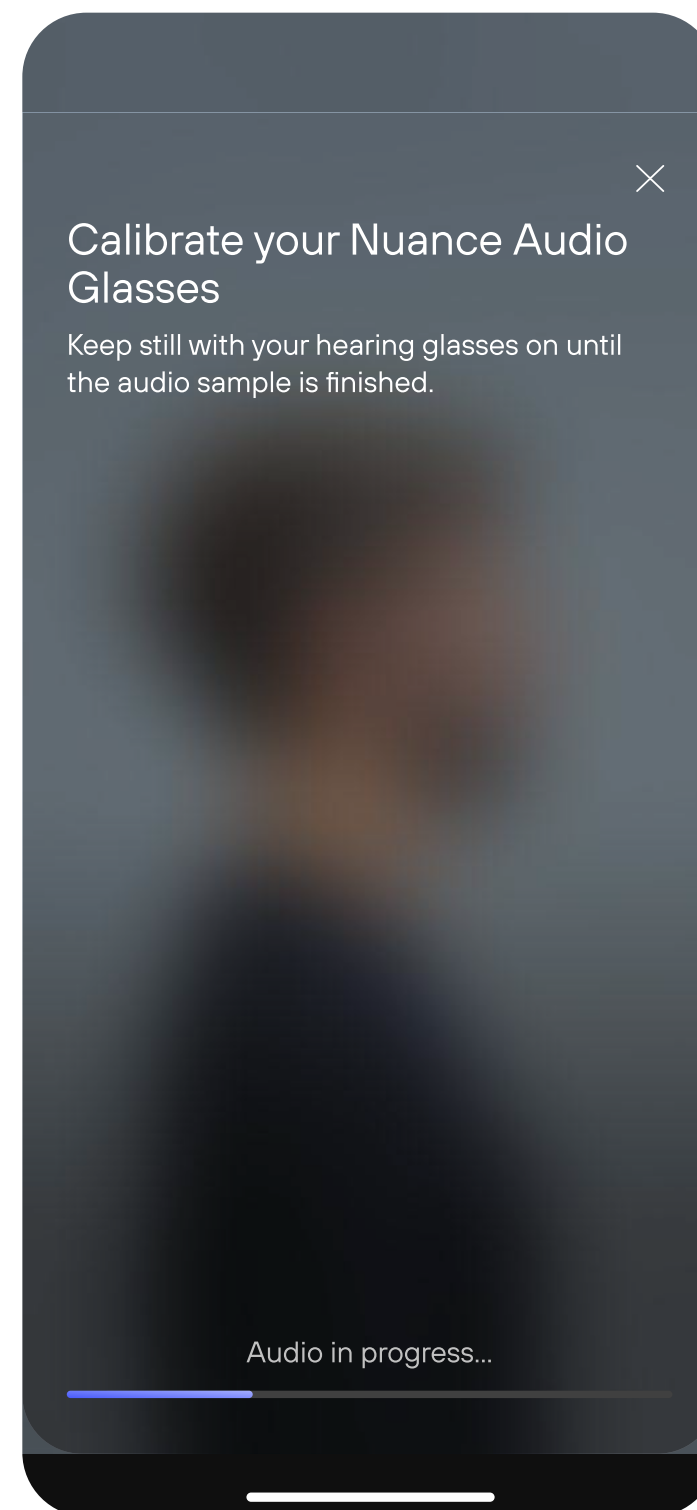


Step 1.2
Stay still and don't take off
your glasses, **wait until
the end of the audio
sample.**

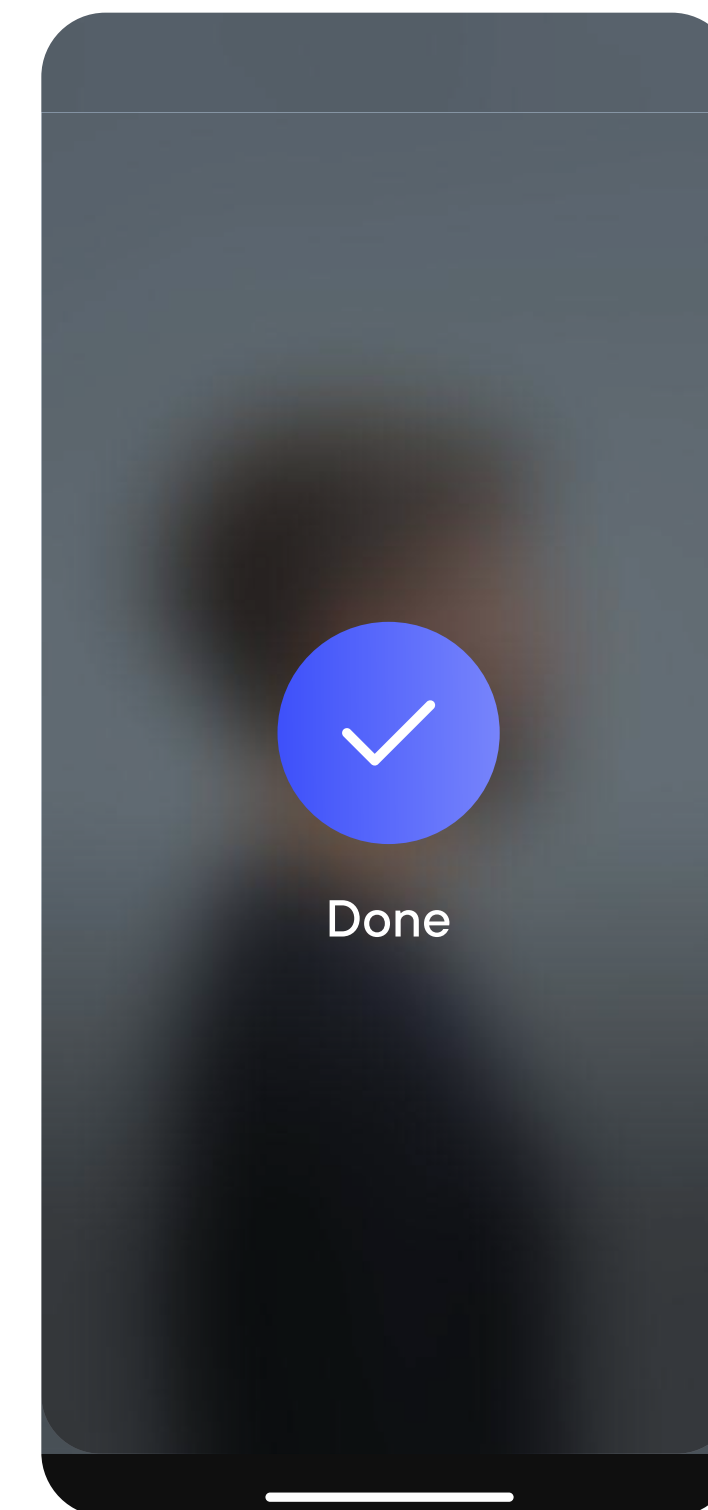


Step 1.3
Environmental noise
measurement completed.




Voice calibration

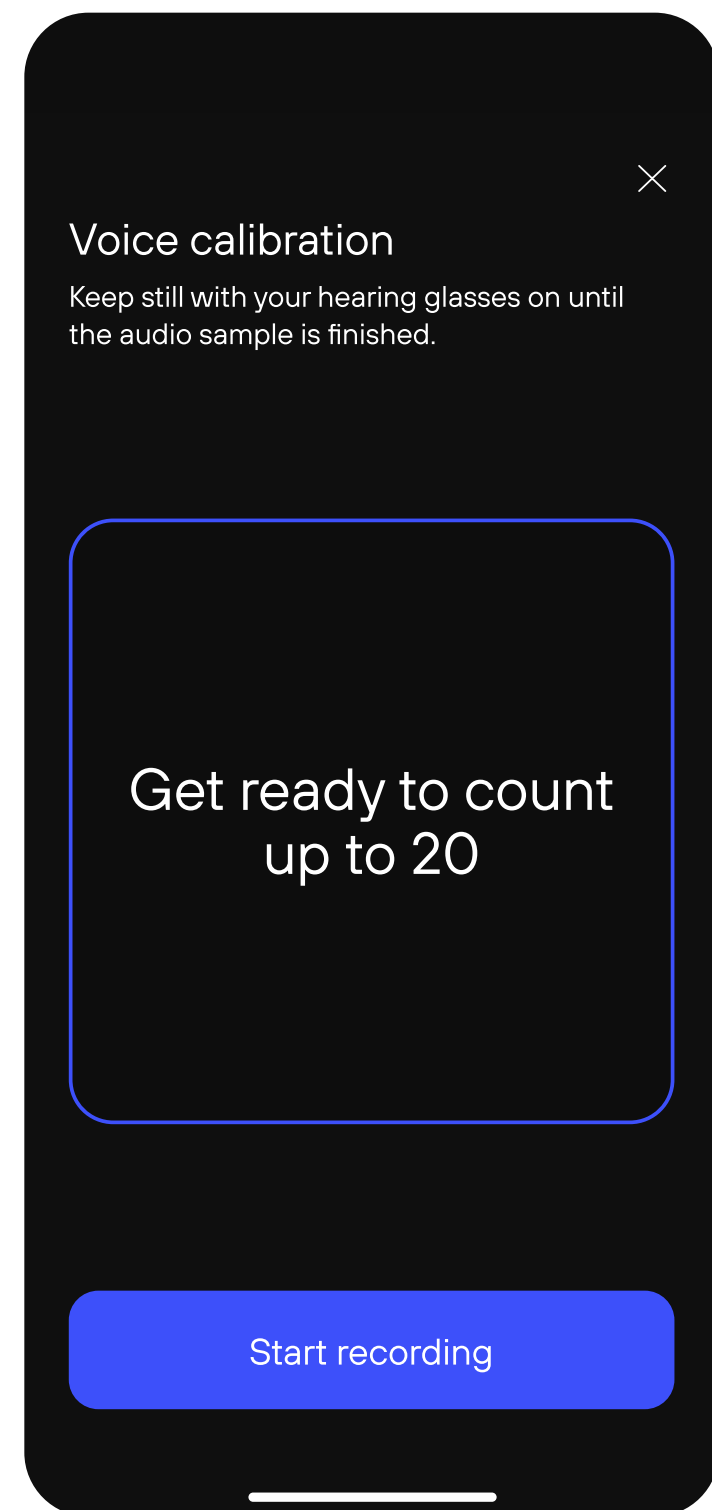

Step 2.1
Own voice calibration.
Stay still and don't take off
your glasses
Press **Start recording**

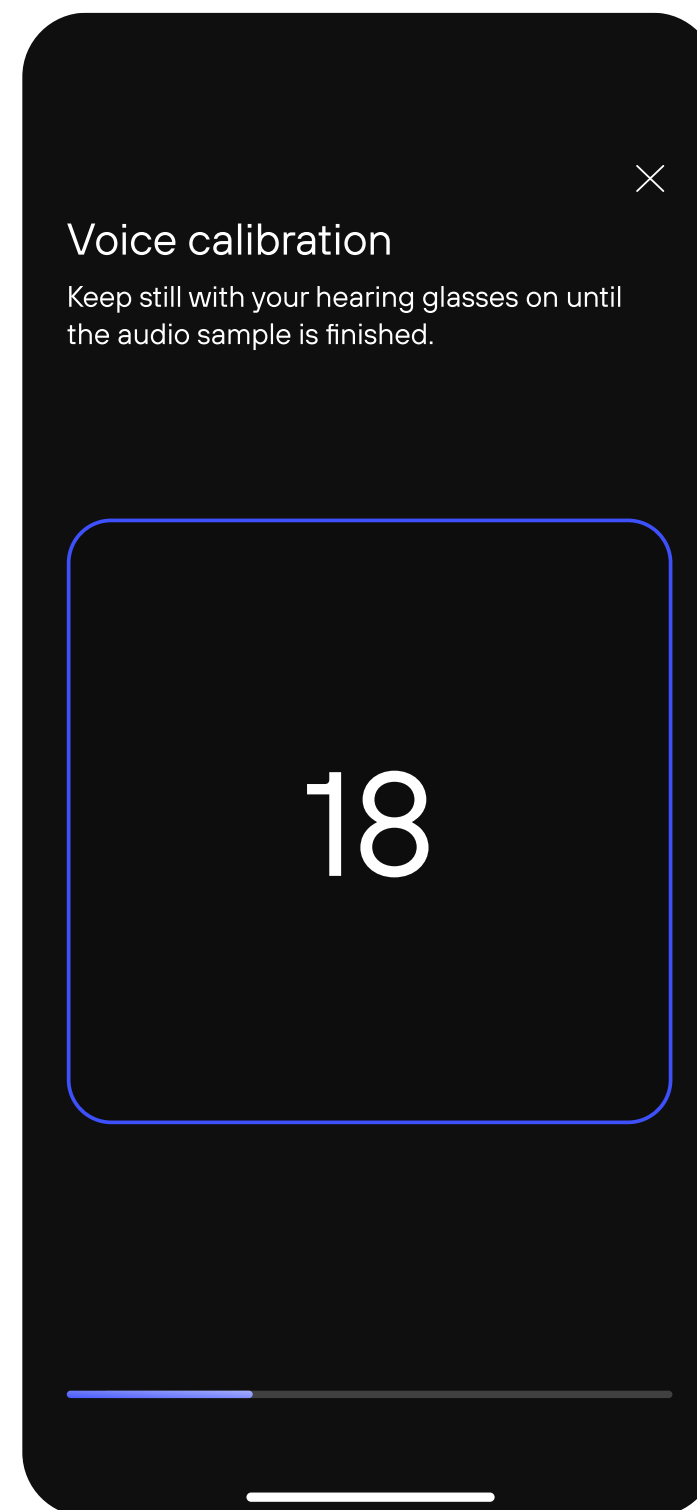

Step 2.2
**Count out loud until the
progress bar reaches
the end.**

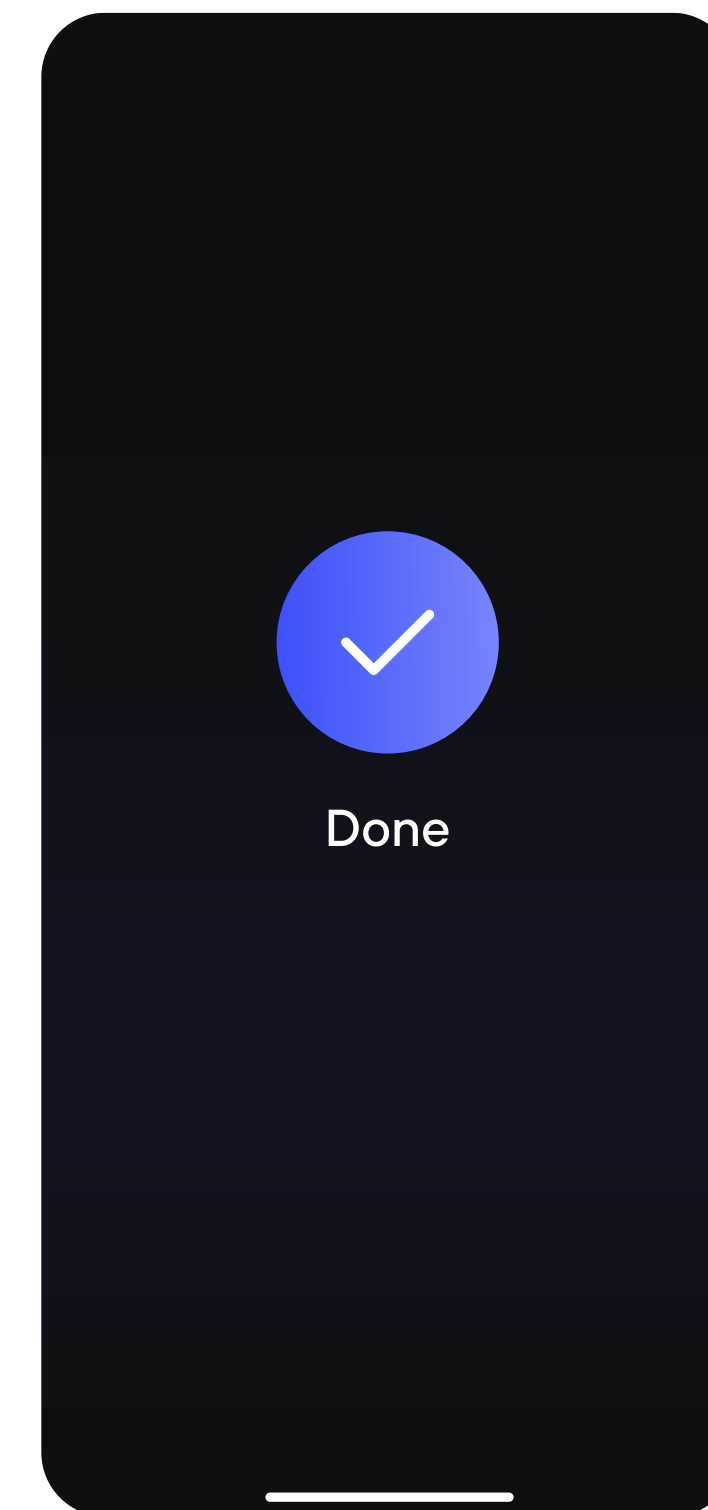

Step 2.2
Personal calibration
successfully completed.



For the most updated Instructions for Use
please follow this link

[↗ View Instruction for Use](#)