

# Hearing configurations: Audio mode



## What is audio mode?

Audio Mode lets you choose where to focus your hearing whether it's all around you or directly in front of you.

Choose between these two options:

### **Frontal Mode**

Enhances sounds in front of you, making it easier to focus on conversations with someone directly, especially in noisy environments.



### **All-around Mode**

Amplifies sounds and speech from all directions, which is ideal for situations like walking or driving where you need to be aware of your surroundings.






## How to adjust audio mode?

You can switch between modes in several ways: on your Glasses, with the Remote Control or on the Nuance App.


### On Nuance Audio Glasses

**Double-tap the area** on the right temple. An audio prompt will confirm your input.

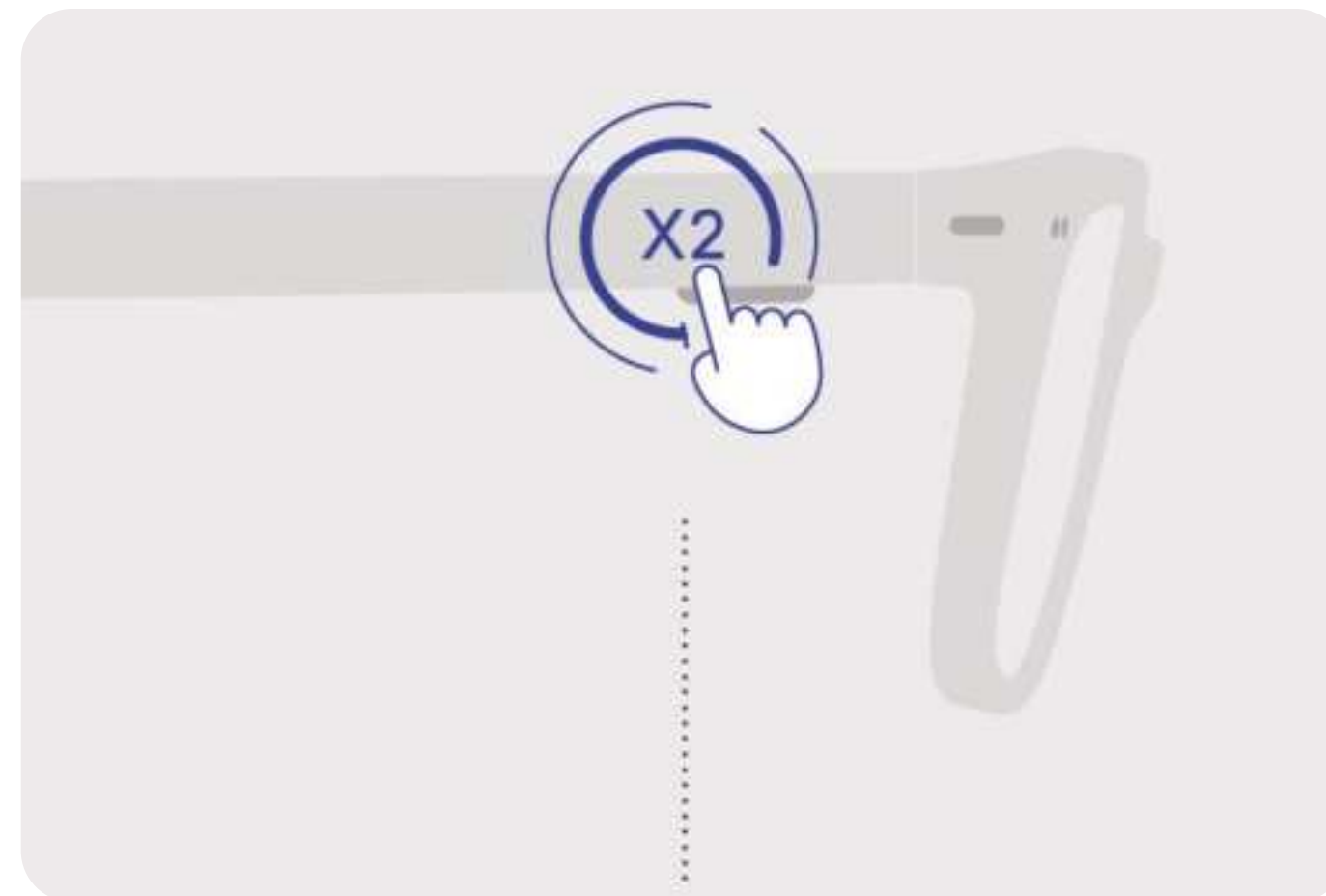

### With Remote Control


Press the **Audio Mode button**.

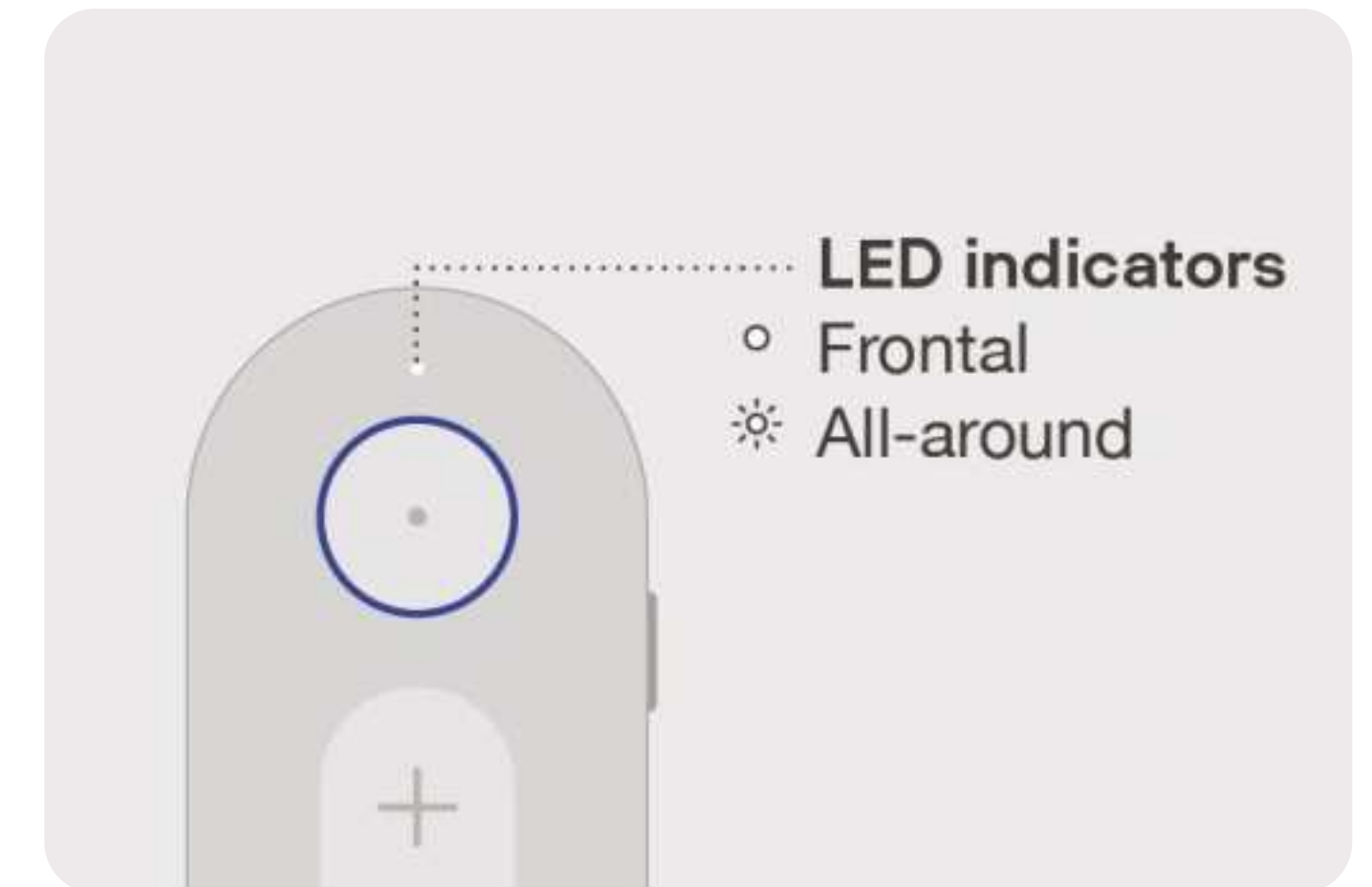



## How to adjust audio mode?

**With Nuance Audio App**  
Slide or tap on the **Audio mode toggle** available on the app's home page.

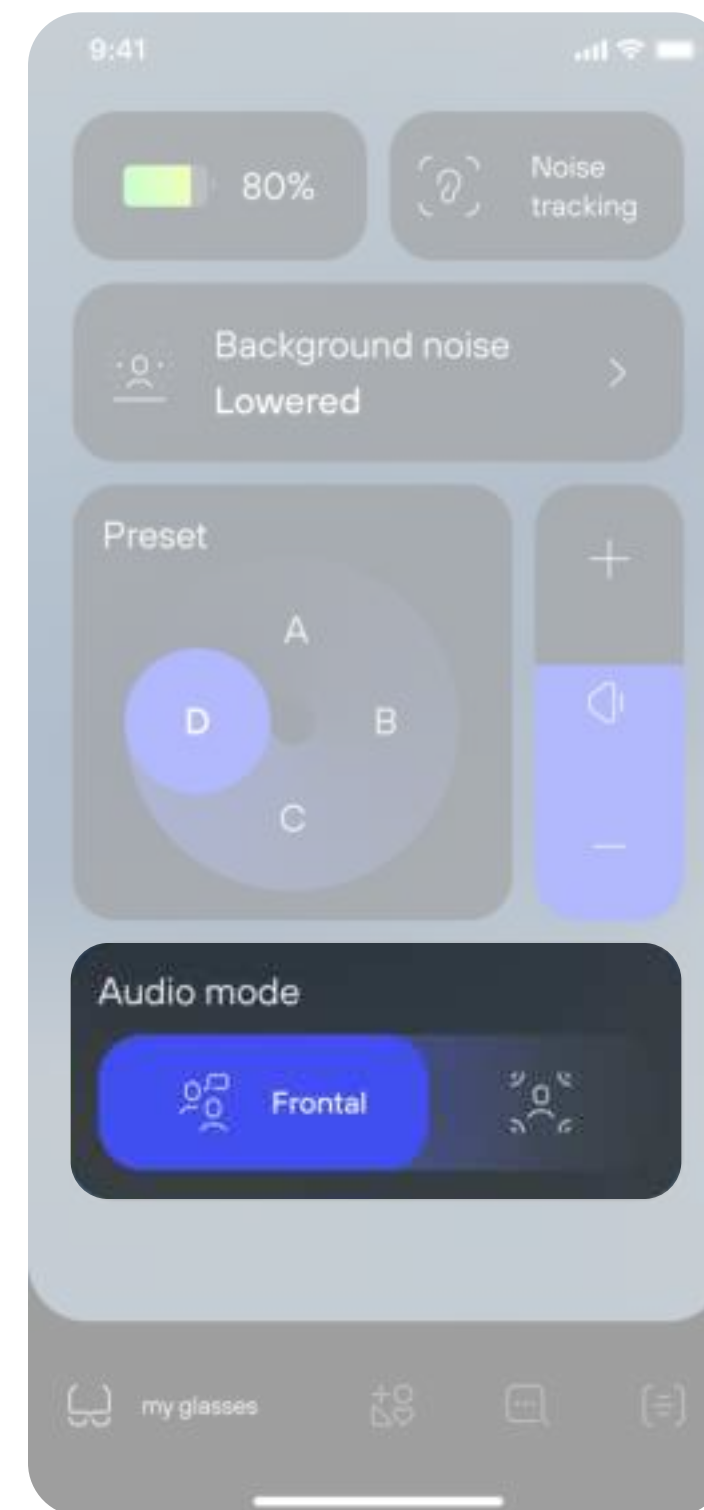

### Frontal Mode


This mode enhances sounds in front of you, making it easier to focus on conversations with someone directly.





## How to adjust audio mode?

**With Nuance Audio App**  
Slide or tap on the **Audio mode toggle** available on the app's home page.

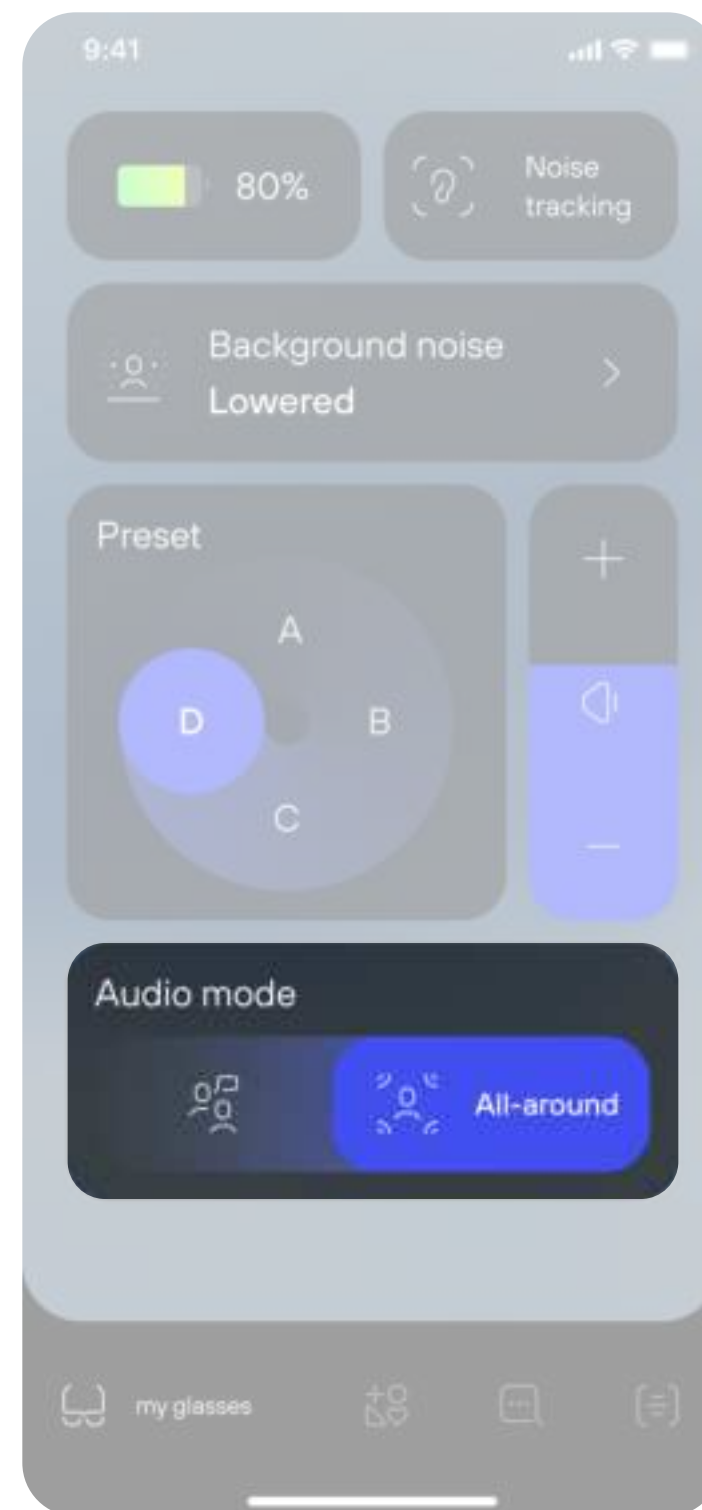

### All-around Mode

This mode amplifies sounds and speech from all directions, which is ideal for situations like walking or driving where you need to be aware of your surroundings.





For the most updated Instructions for Use  
please follow this link

[↗ View Instruction for Use](#)