


# How to charge



## Ready to charge your Nuance Audio Glasses?


### Step 1

First connect your Charging Pad using a standard USB-C charger or USB-C power source (minimum 5V) to connect the supplied charging pad and USB-C cable.

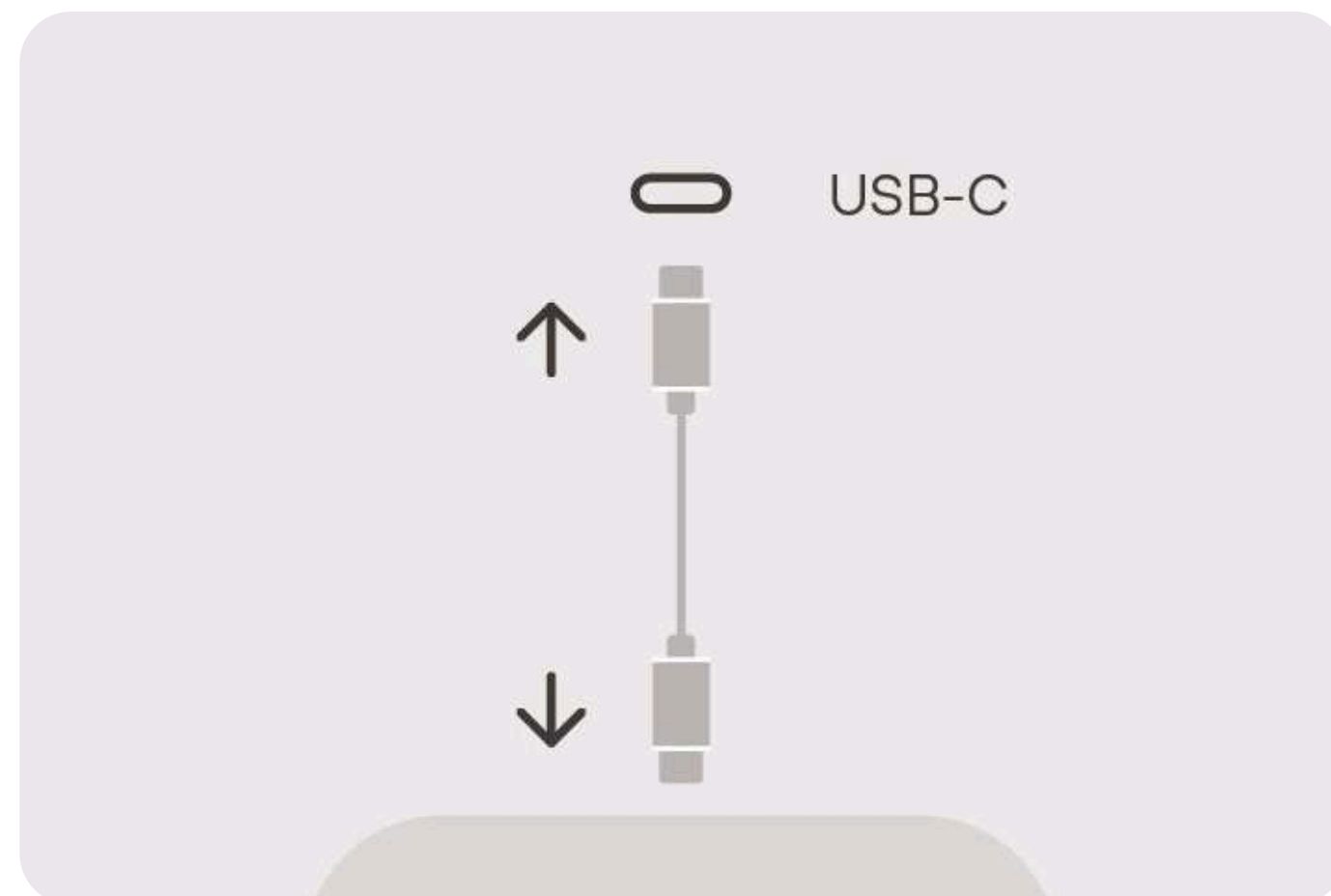

### Step 2

Then, fold your Nuance Audio Glasses and place them directly on the Charging Pad exactly as shown. The magnets inside the Charging Pad will help position the glasses correctly.

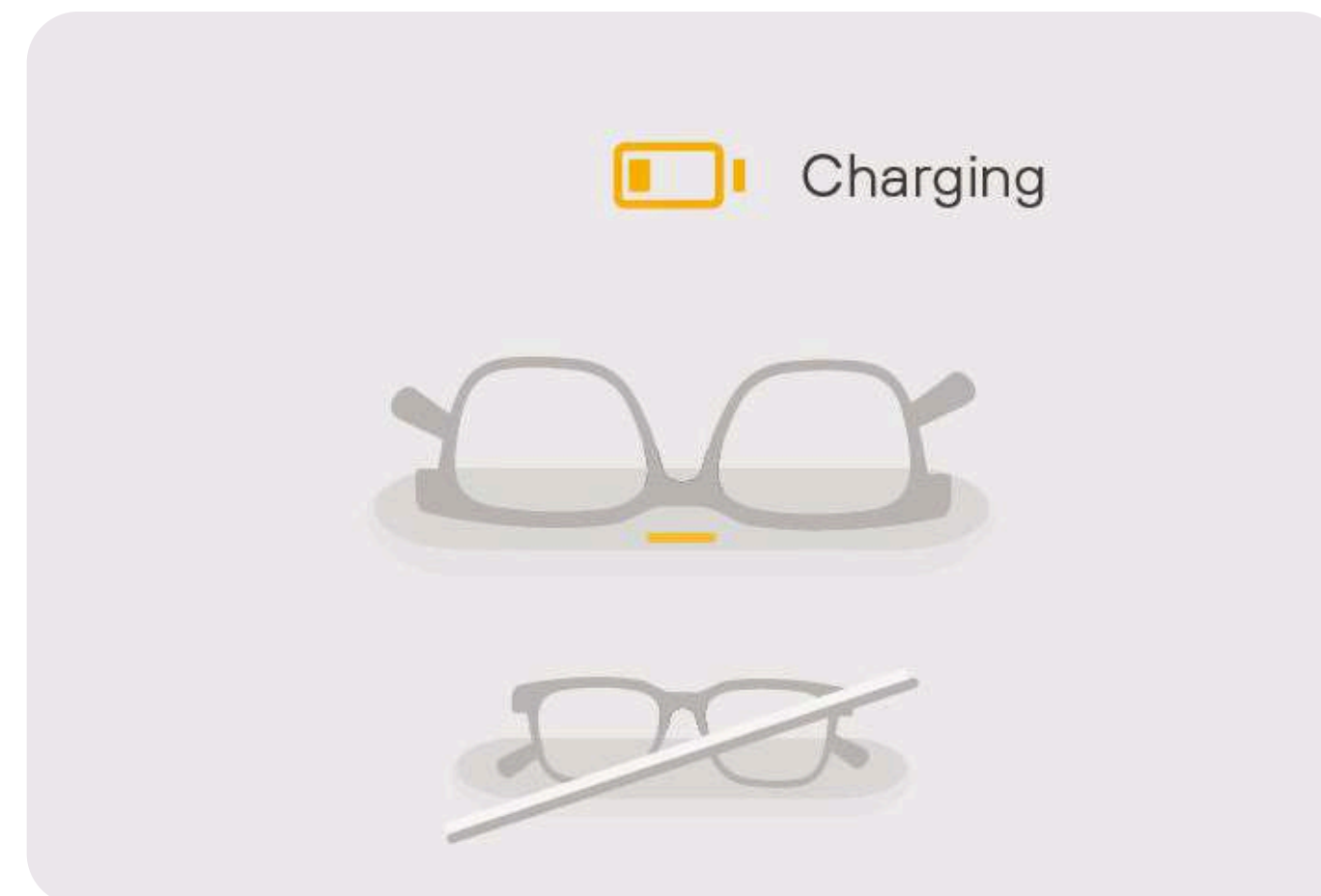





## How to know when your glasses are charged?

### Orange LED

Charging is in progress.

### Green LED

Glasses are fully charged (at least 96%).

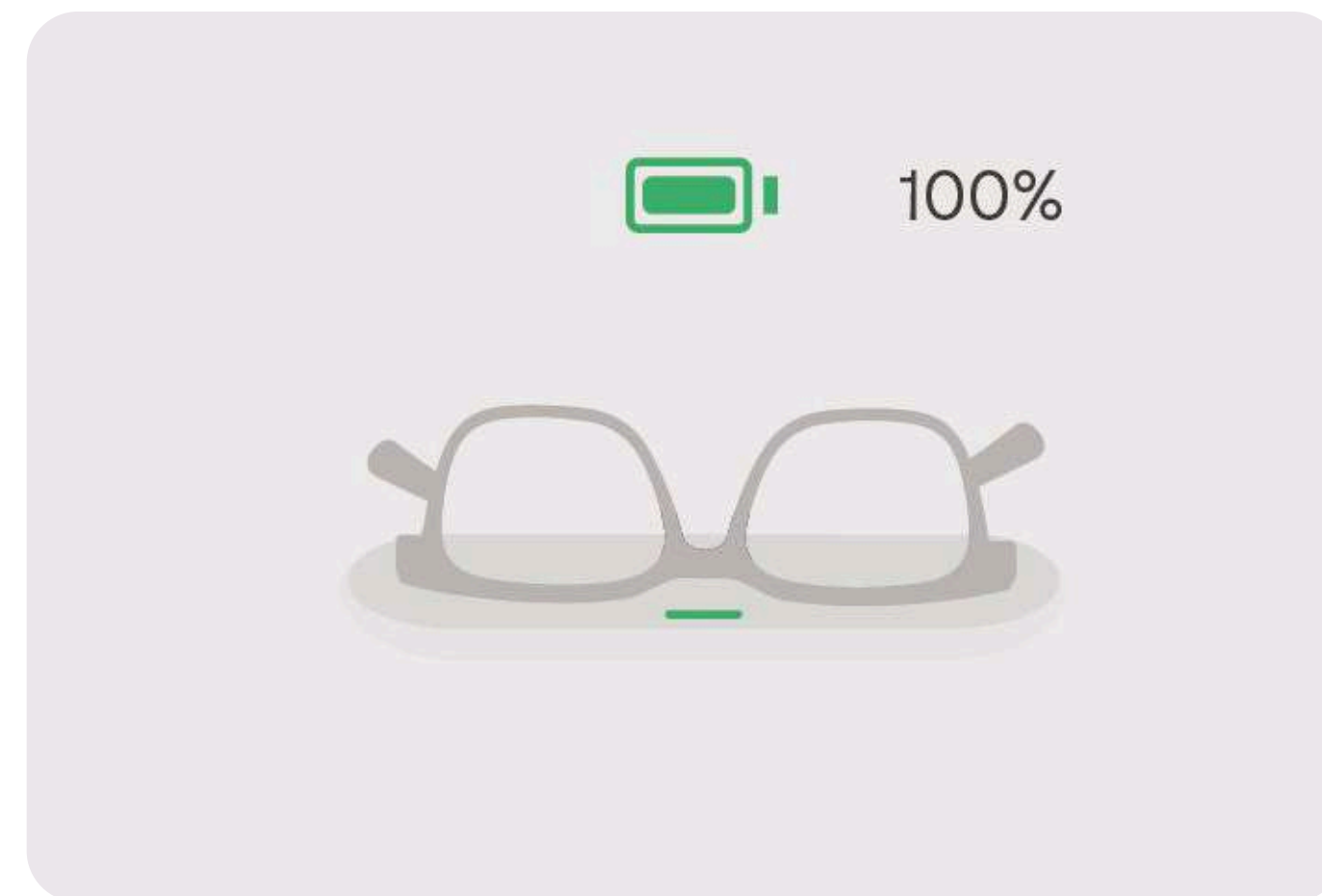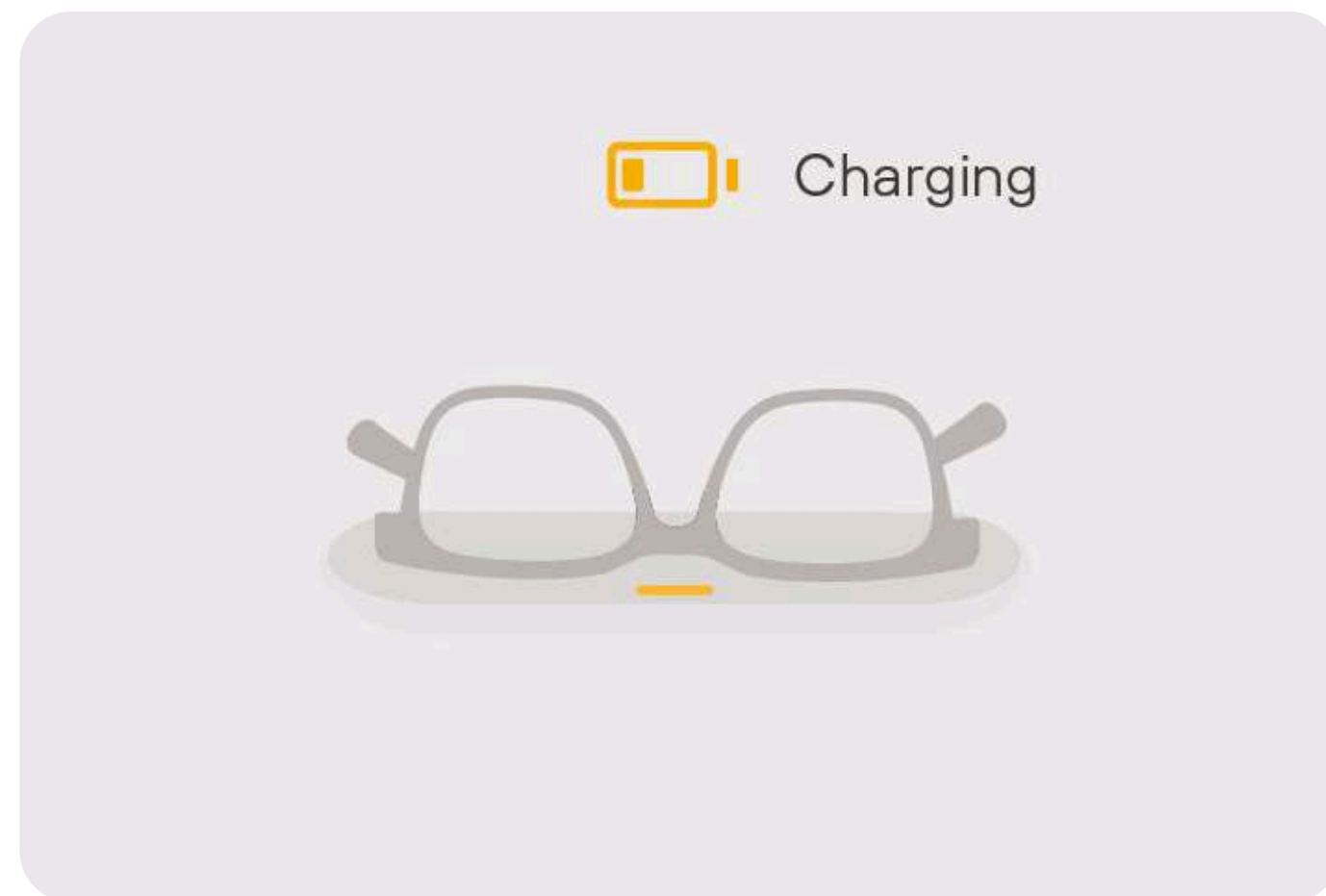




## Battery level indications

When you pick up your glasses, both the charging pad LED and the frame LED will indicate the battery level for a few seconds.

### Red LED

Battery level is lower than 20%.

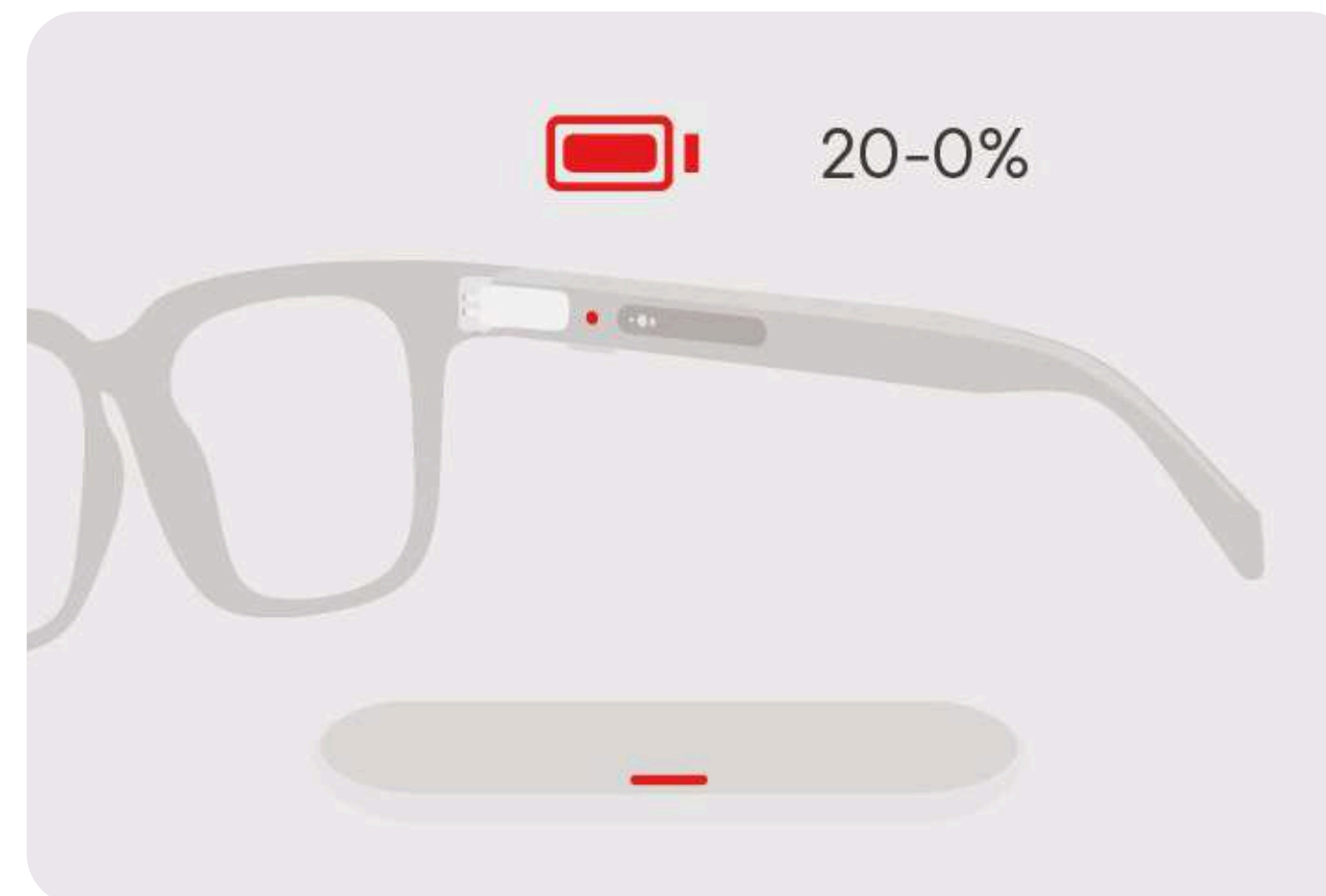

### Green LED

Battery level is higher than 20%.







For the most updated Instructions for Use  
please follow this link

[↗ View Instruction for Use](#)