

Hearing configurations: Audio mode



What is audio mode?

Audio Mode lets you choose where to focus your hearing whether it's all directly in front of you or around you.

Choose between these two options:

Frontal Mode

Enhances sounds in front of you, making it easier to focus on conversations with someone directly, especially in noisy environments.



All-around Mode

Amplifies sounds and speech from all directions, which is ideal for situations like walking or driving where you need to be aware of your surroundings.






How to adjust audio mode?

You can switch between modes in several ways: on your Glasses, with the Remote Plus or on the Nuance Audio App.


On Nuance Audio Plus

Double-tap the area on the right temple. An audio prompt will confirm your input. This is only possible if shortcuts are enabled.

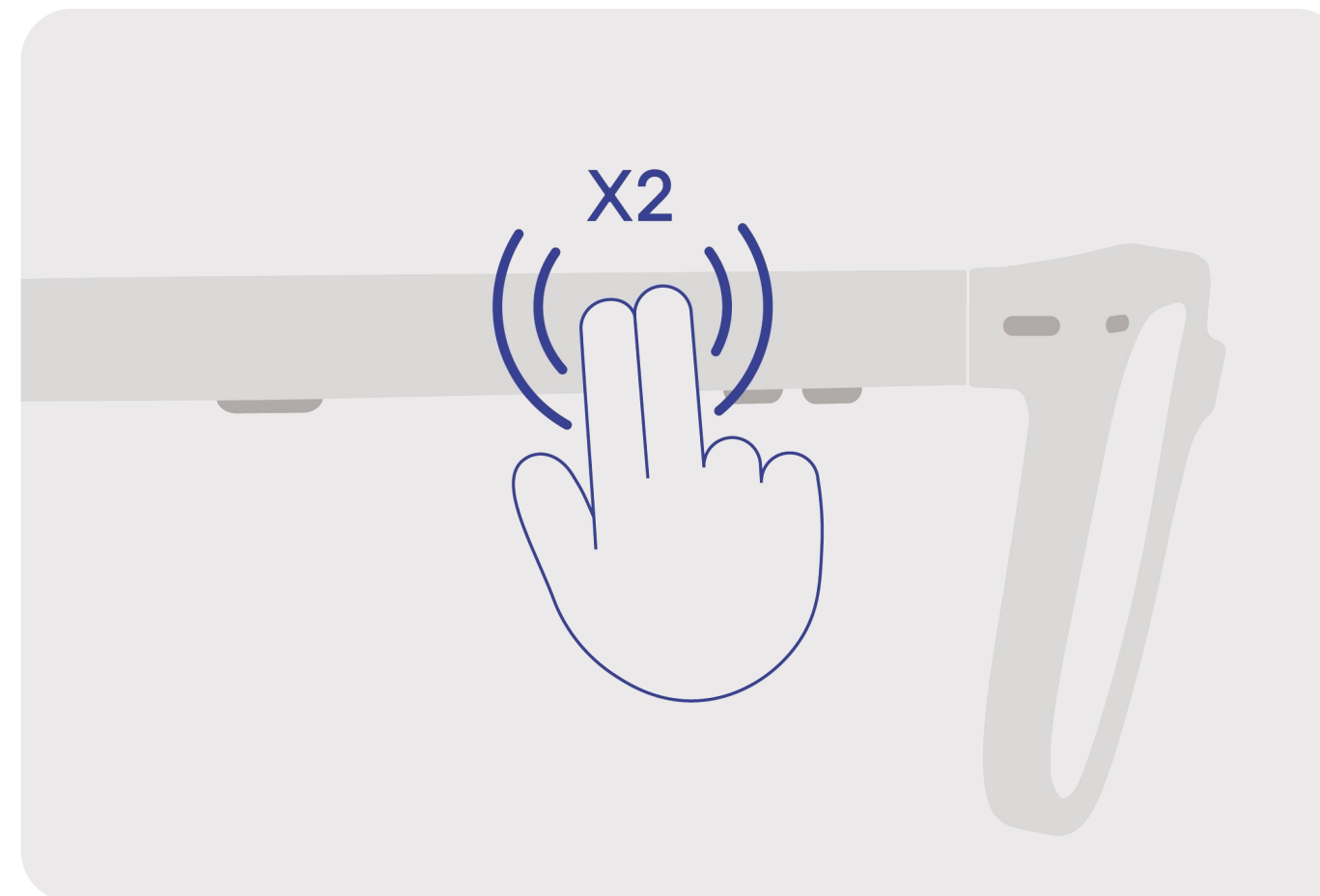

With Remote Control

Press the **Audio Mode button**.

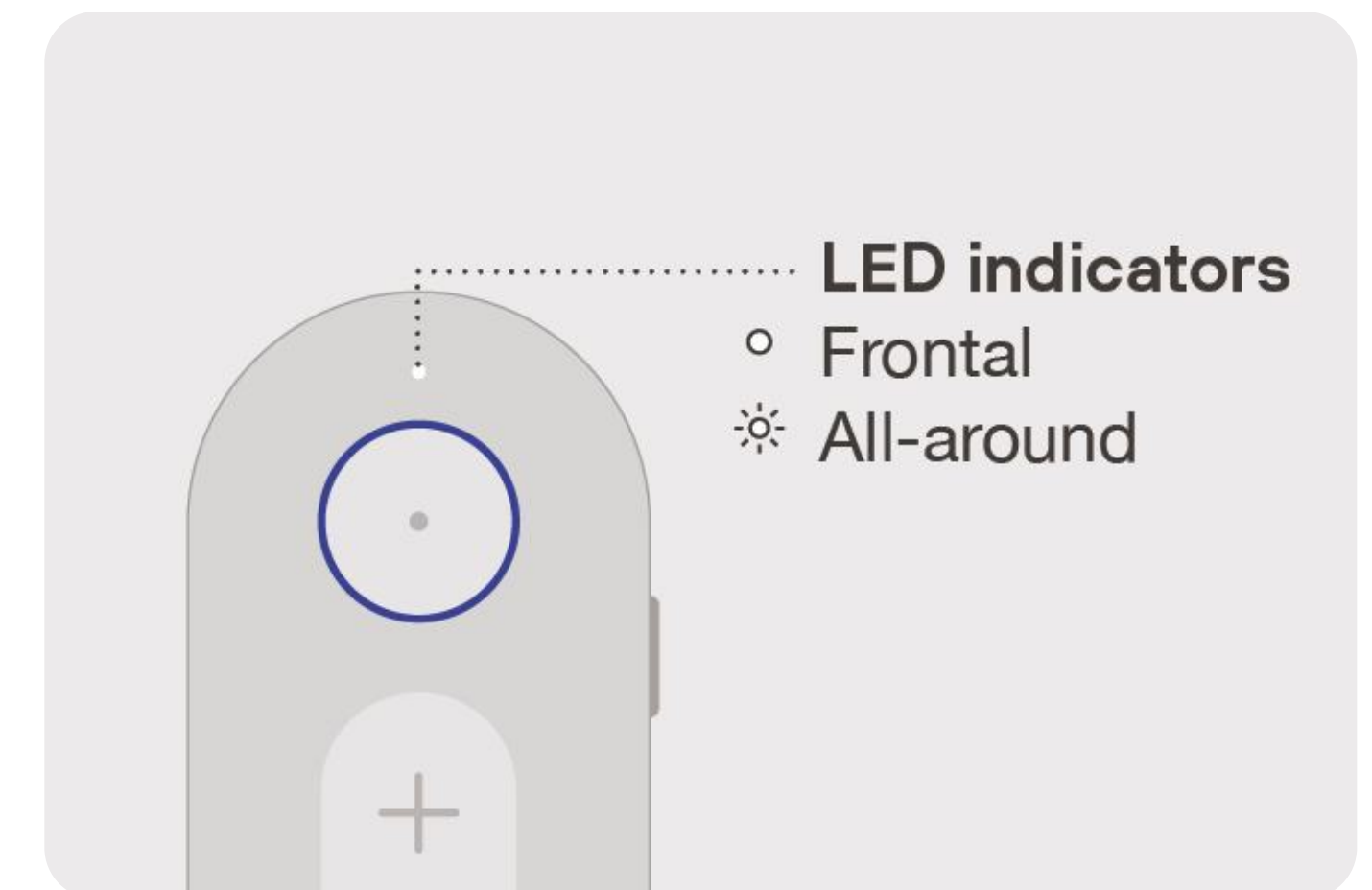




How to adjust audio mode?

With Nuance Audio App

Slide or tap on the **Audio Mode toggle** available on the App's home page.

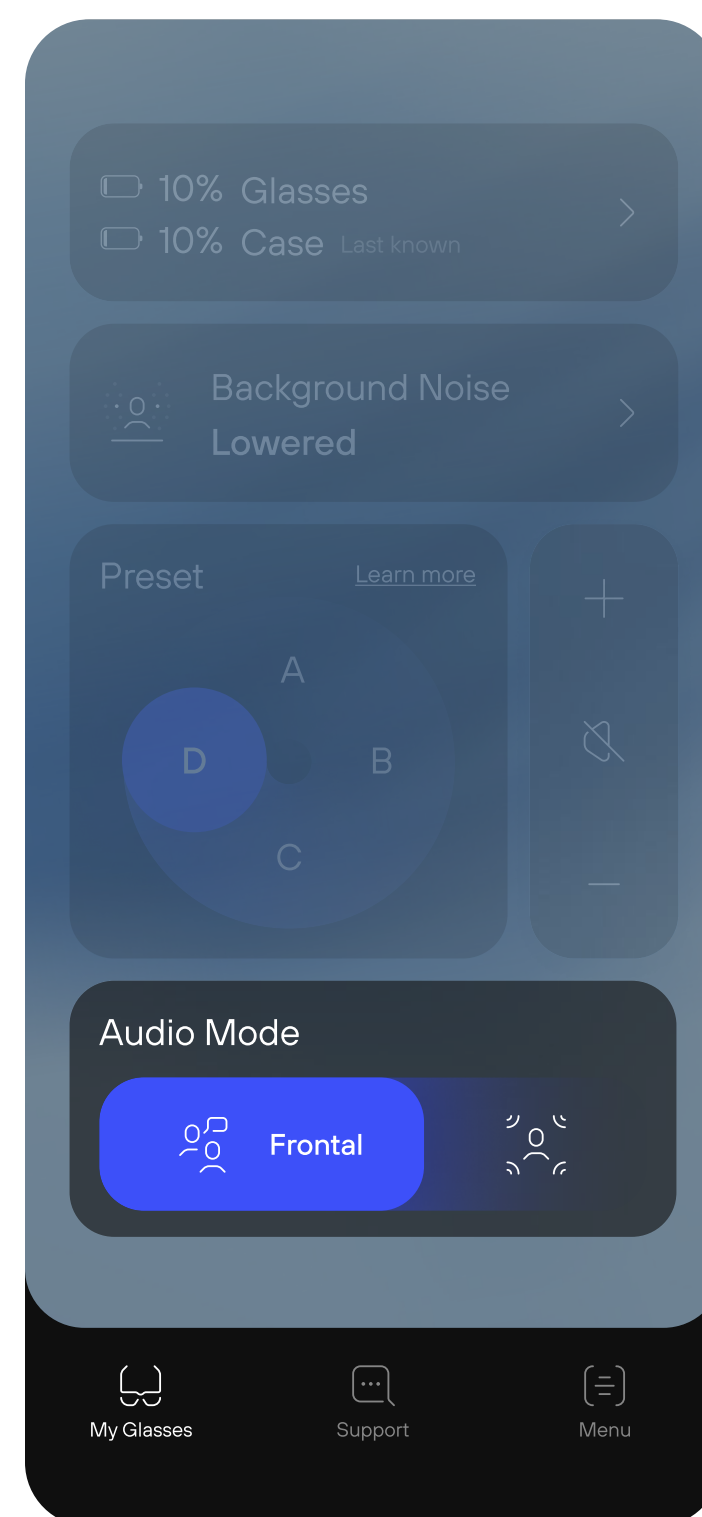

Frontal Mode


This mode enhances sounds in front of you, making it easier to focus on conversations with someone directly.





How to adjust audio mode?

With Nuance Audio App
Slide or tap on the **Audio Mode toggle** available on the App's home page.

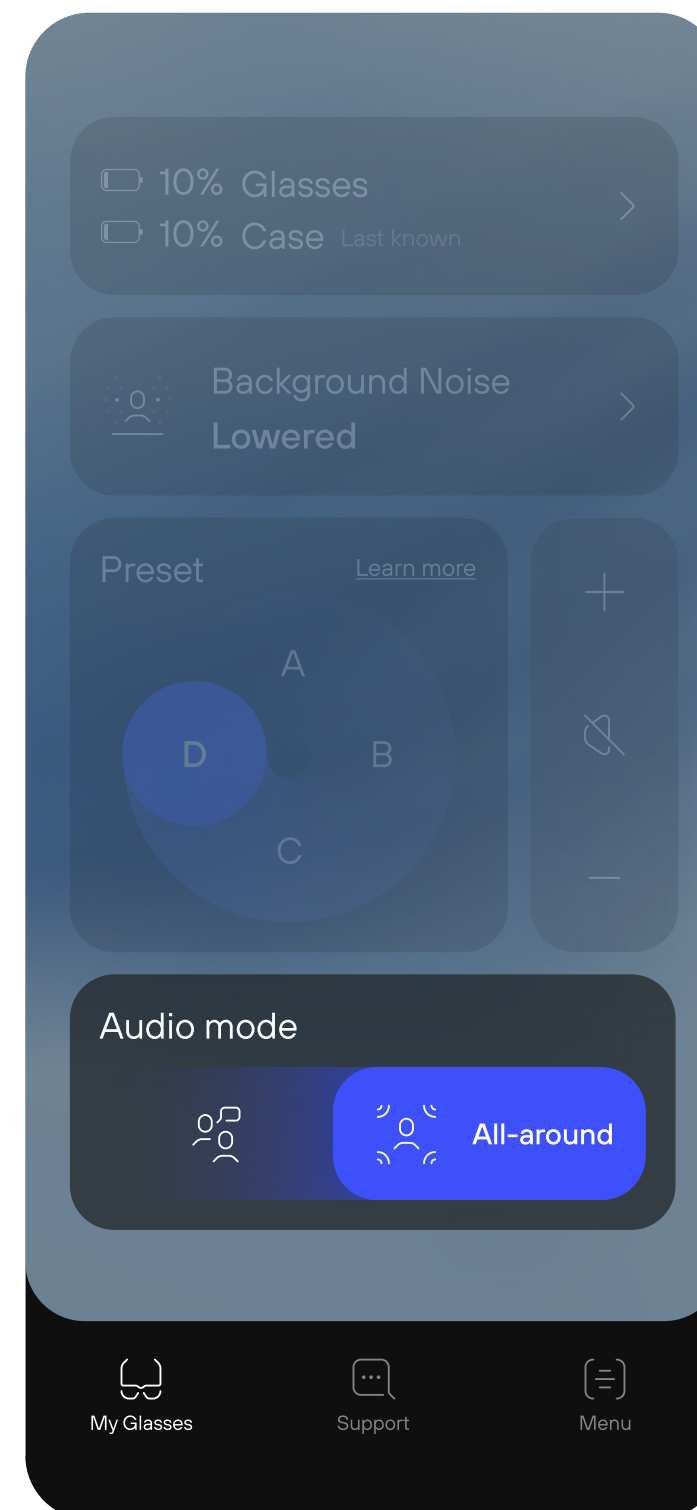

All-around Mode

This mode amplifies sounds and speech from all directions, which is ideal for situations like walking or driving where you need to be aware of your surroundings.





For the most updated Instructions
for Use please follow this link

[↗ View instruction for use](#)