

Hearing configurations: Eyewear volume




How to adjust eyewear volume?

When it comes to your Nuance Audio Plus, you can always adjust the volume levels in several ways.


With Nuance Audio Plus

To adjust the volume, there will be two buttons on the right temple: button "+" for **volume up**, button "-" for **volume down**.

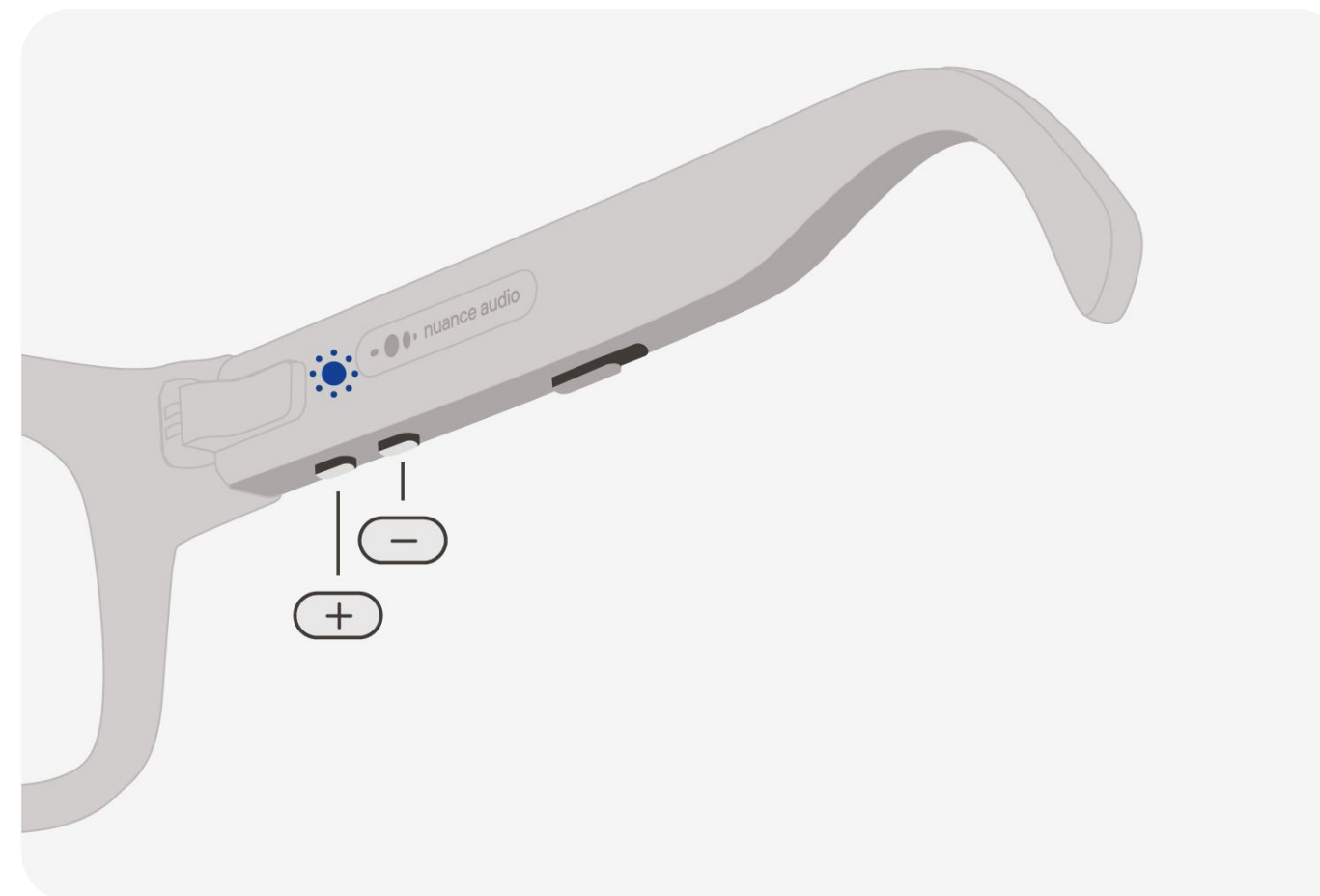

With Remote Plus

Use the "+" and "-" buttons to adjust volume.

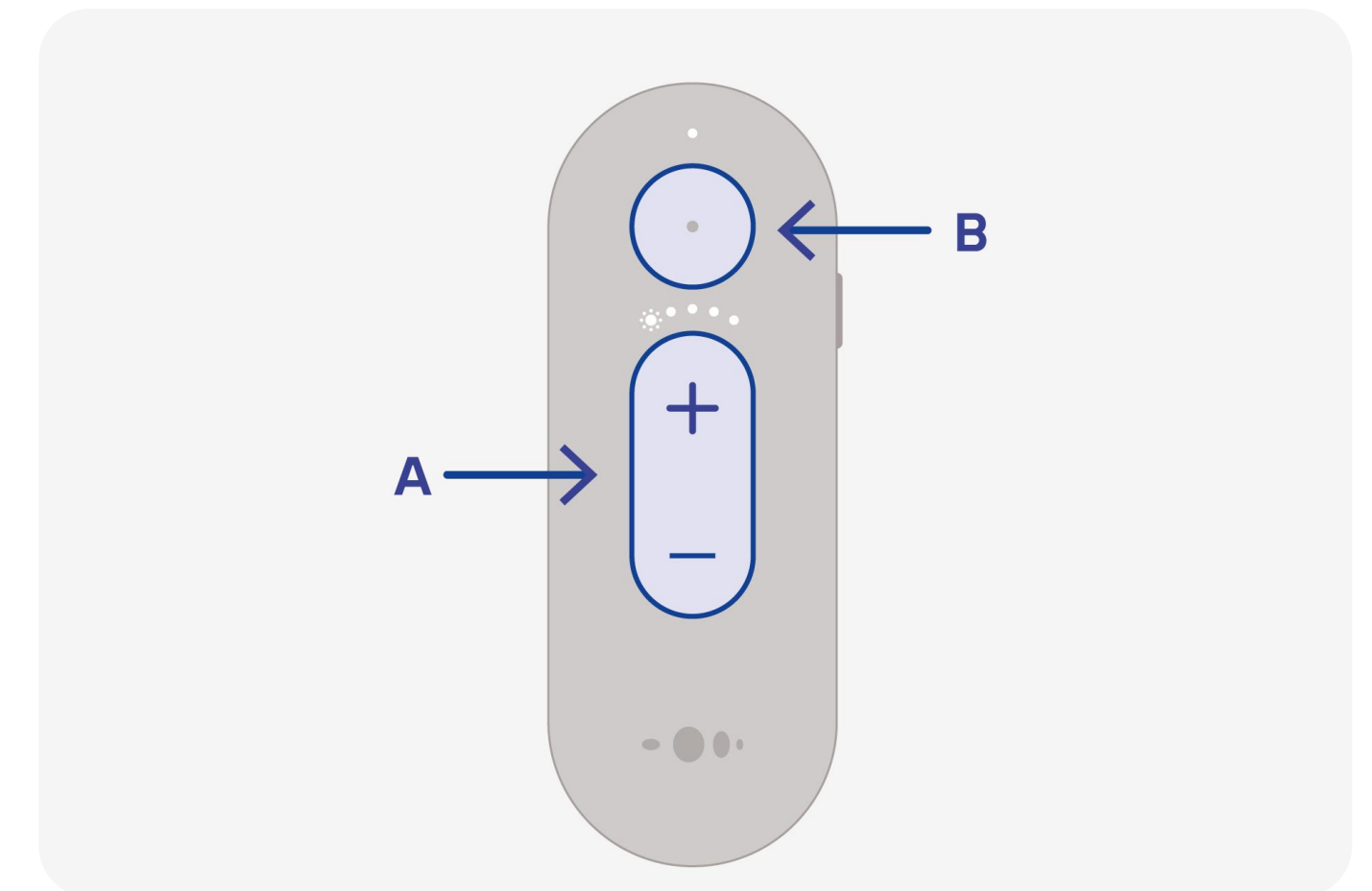




How to adjust eyewear volume?

With Nuance Audio App


Another way is to use the Volume Slider on your Nuance Audio App's home screen to adjust the volume to your preference.

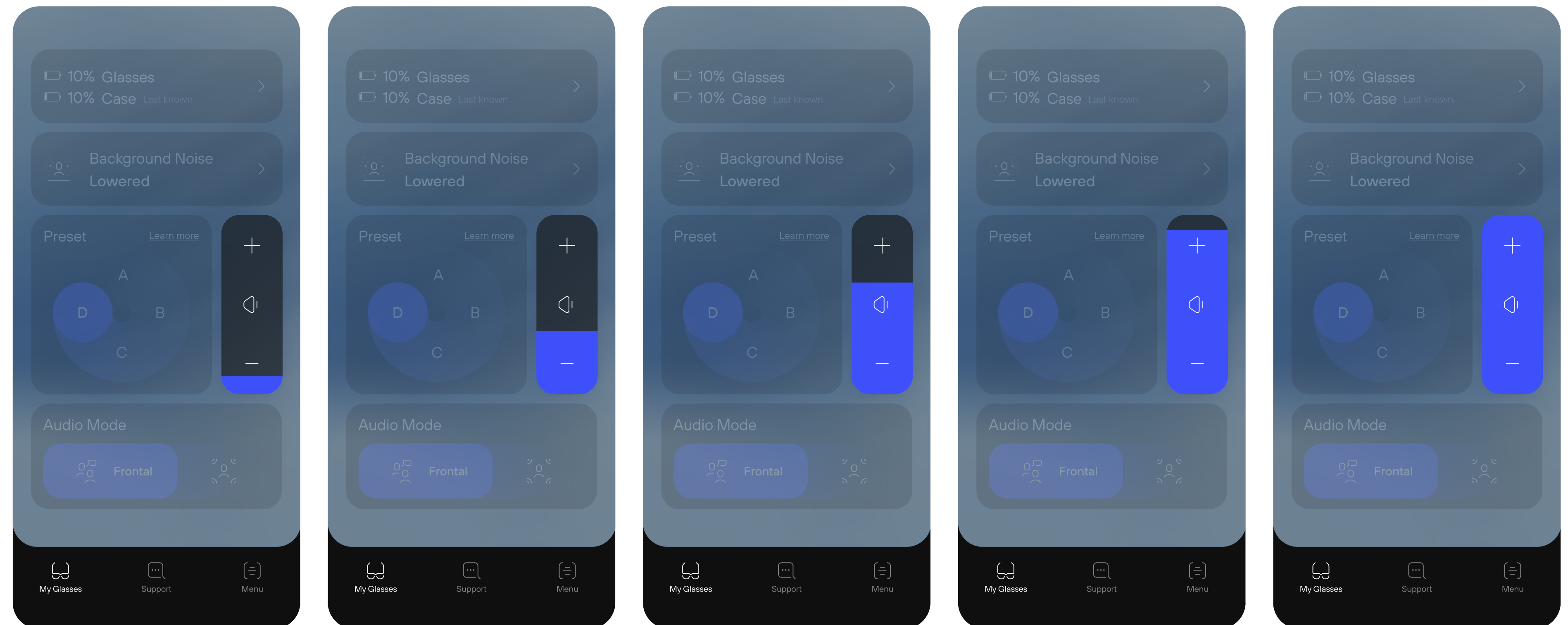



How to adjust eyewear volume?

With Nuance Audio App
Additionally, in the Settings section of the App, you can conveniently select and set your three preferred volume ranges for easy access.

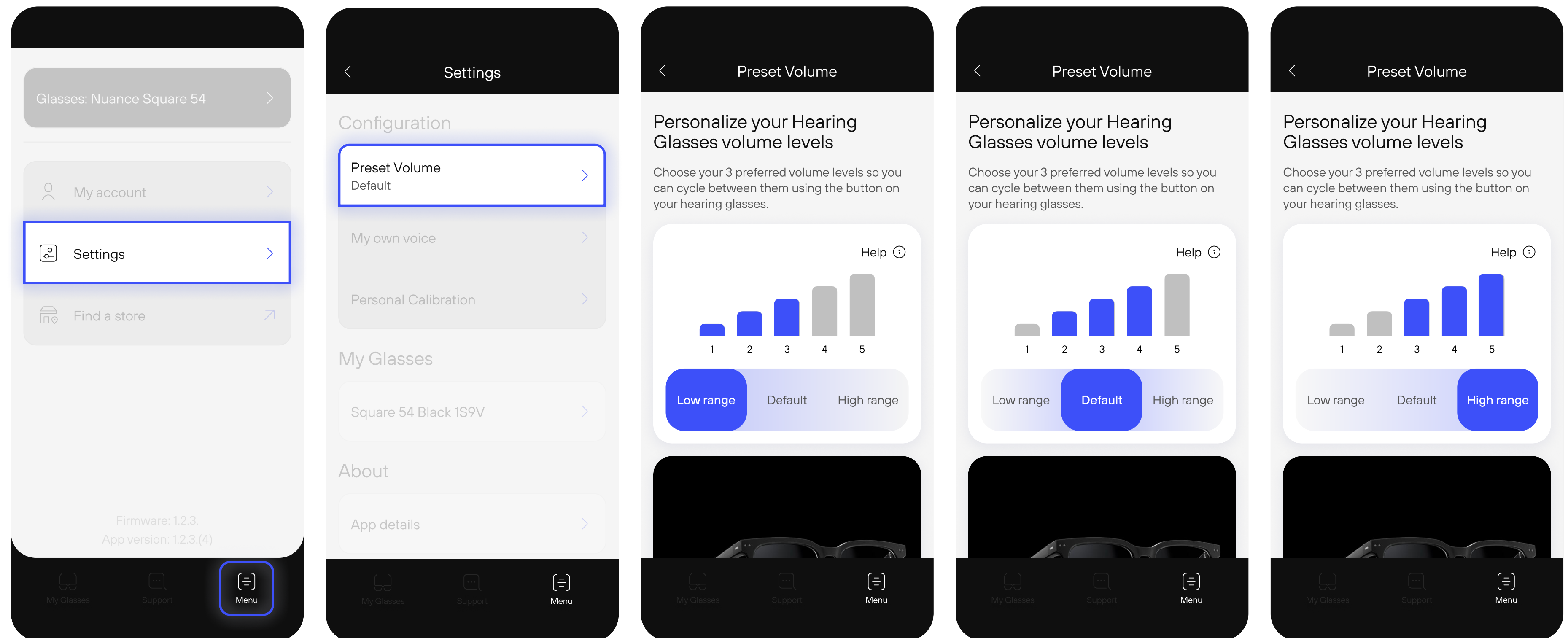



For the most updated Instructions
for Use please follow this link

[↗ View instruction for use](#)