


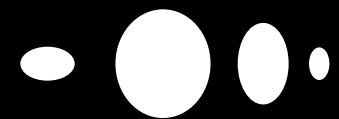


Getting started: Account creation, pairing, calibration



Unboxing



Unboxing

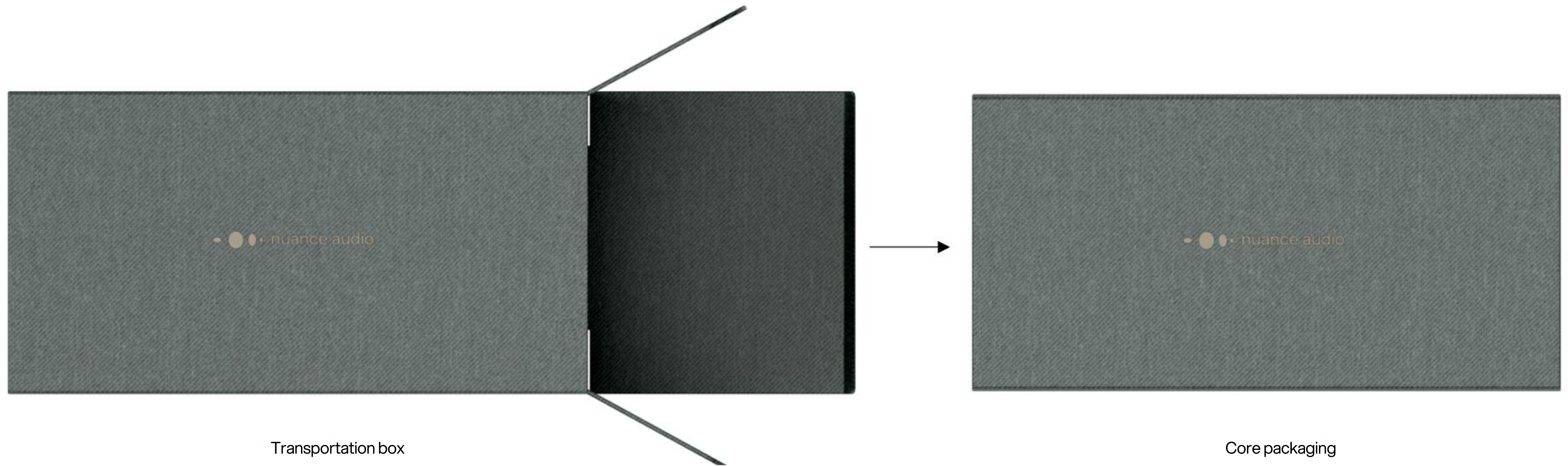




Unboxing



Package




Charging Pad & literature drawer

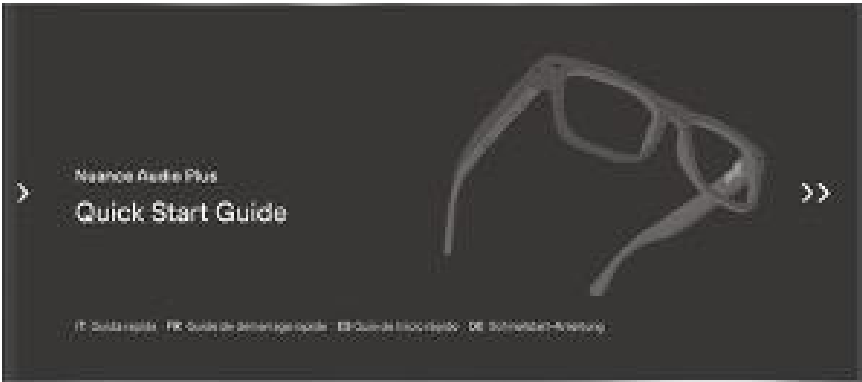

QSG




IFU

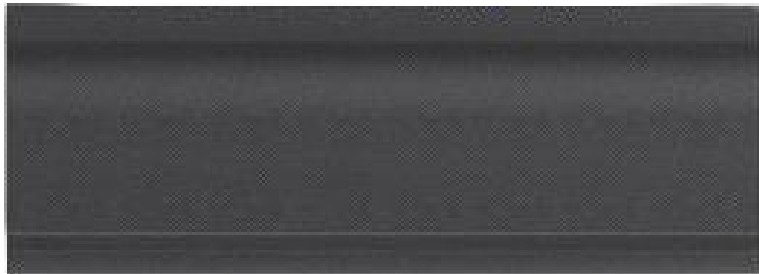


Folded Case




Charging Pad

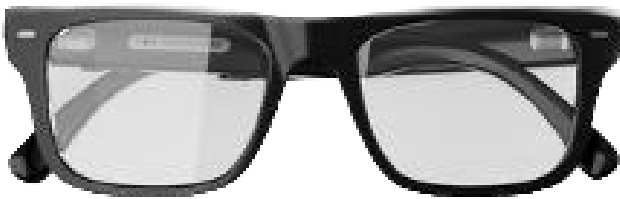

Nuance Audio Plus

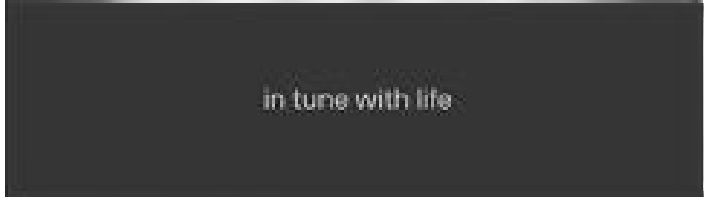

Cleaning Cloth

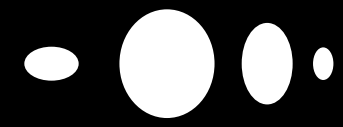


Activate and set up your Nuance Audio Plus




Find the Nuance Audio App

For easy use of your Nuance Audio Plus, our intuitive mobile App allows you to change presets, control volume, and reduce background noise – all from your phone.

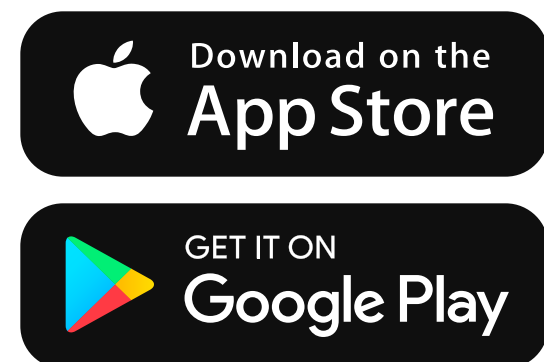


Option A.

Scan the QR Code with your phone and [Download](#).

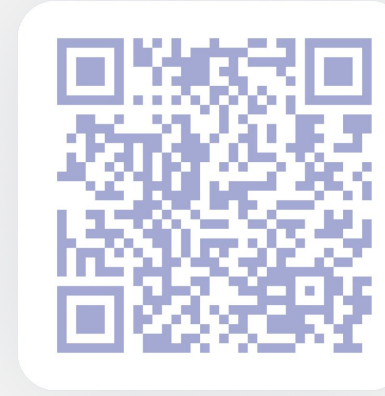


Notes

You can find the QR Code also on the core box

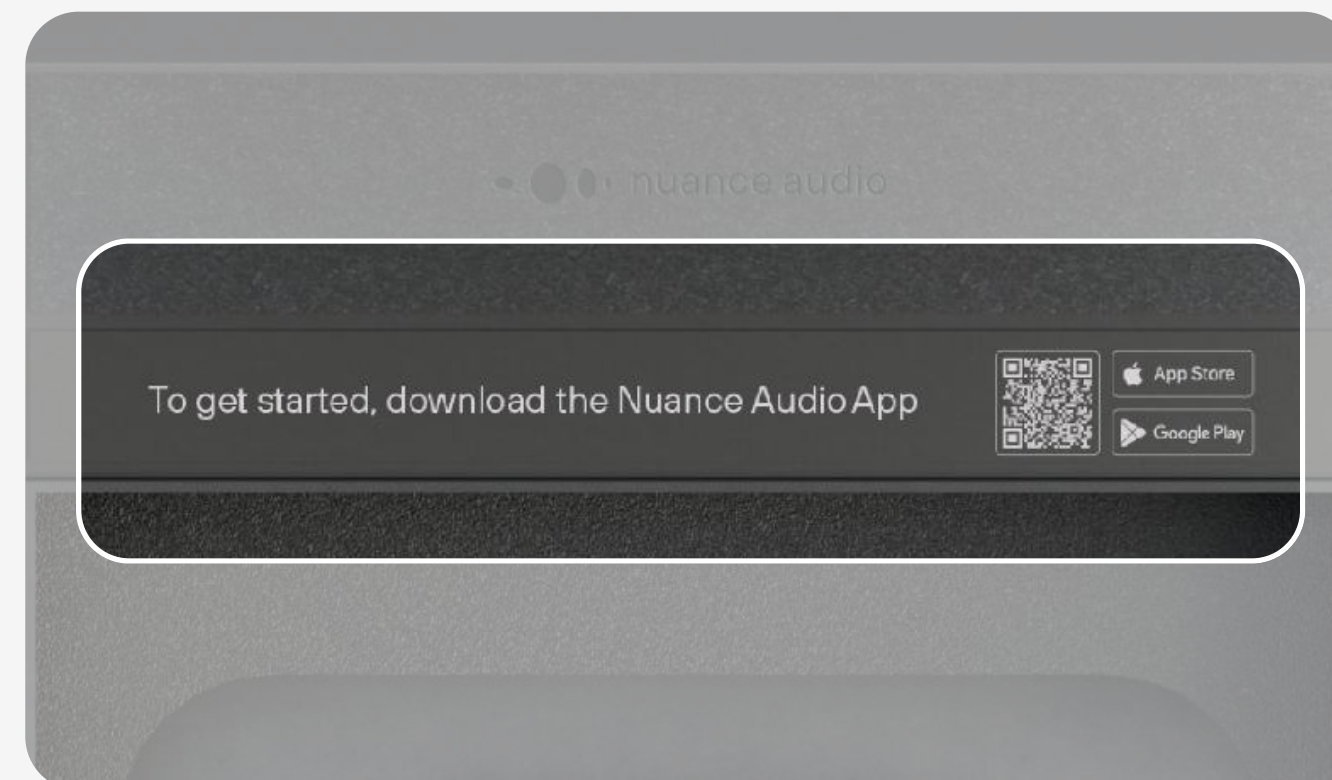

Option B.


Search for “Nuance Audio” in the App Store or Google Play Store and [Download](#).

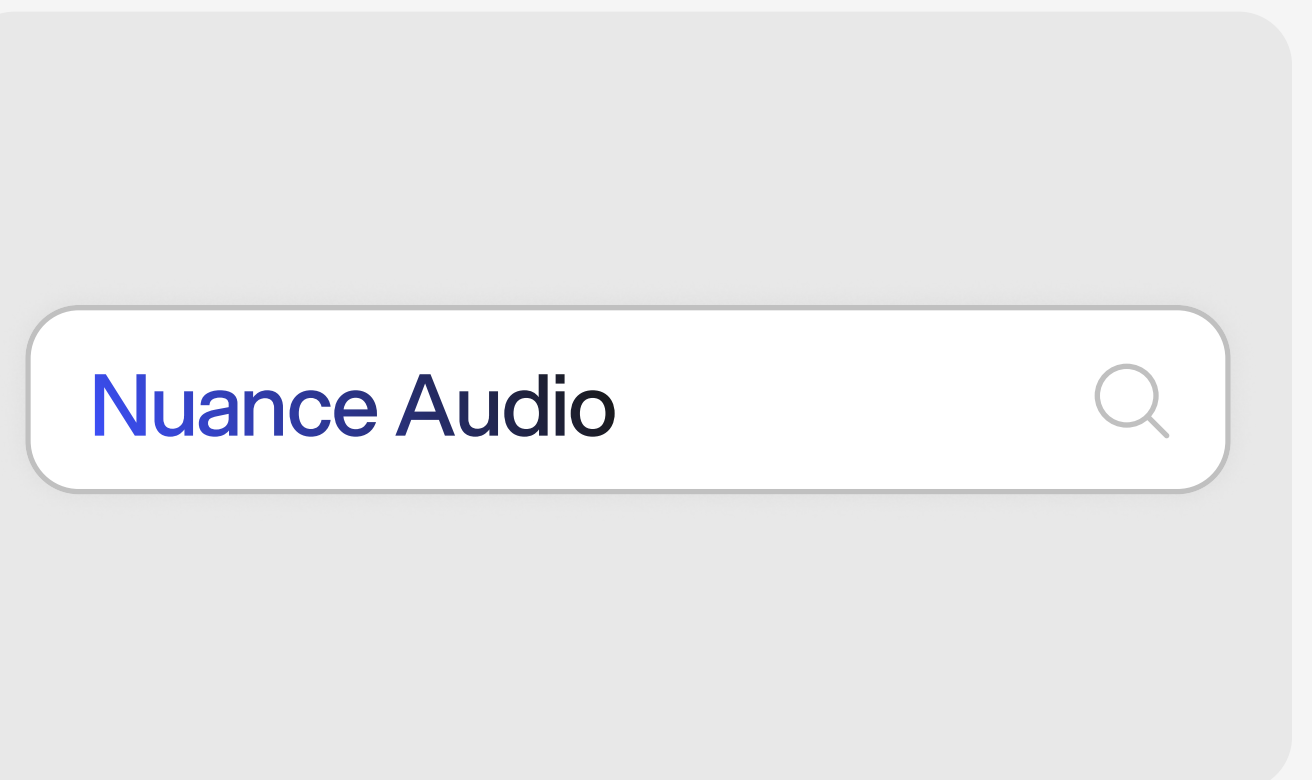




Download from your device's App store

To activate and personalize your Nuance Audio Plus, **download the Nuance Audio App** from the iOS App Store or Google Play Store.

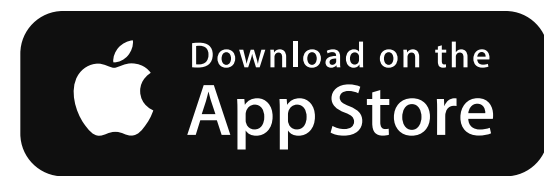


Apple App Store

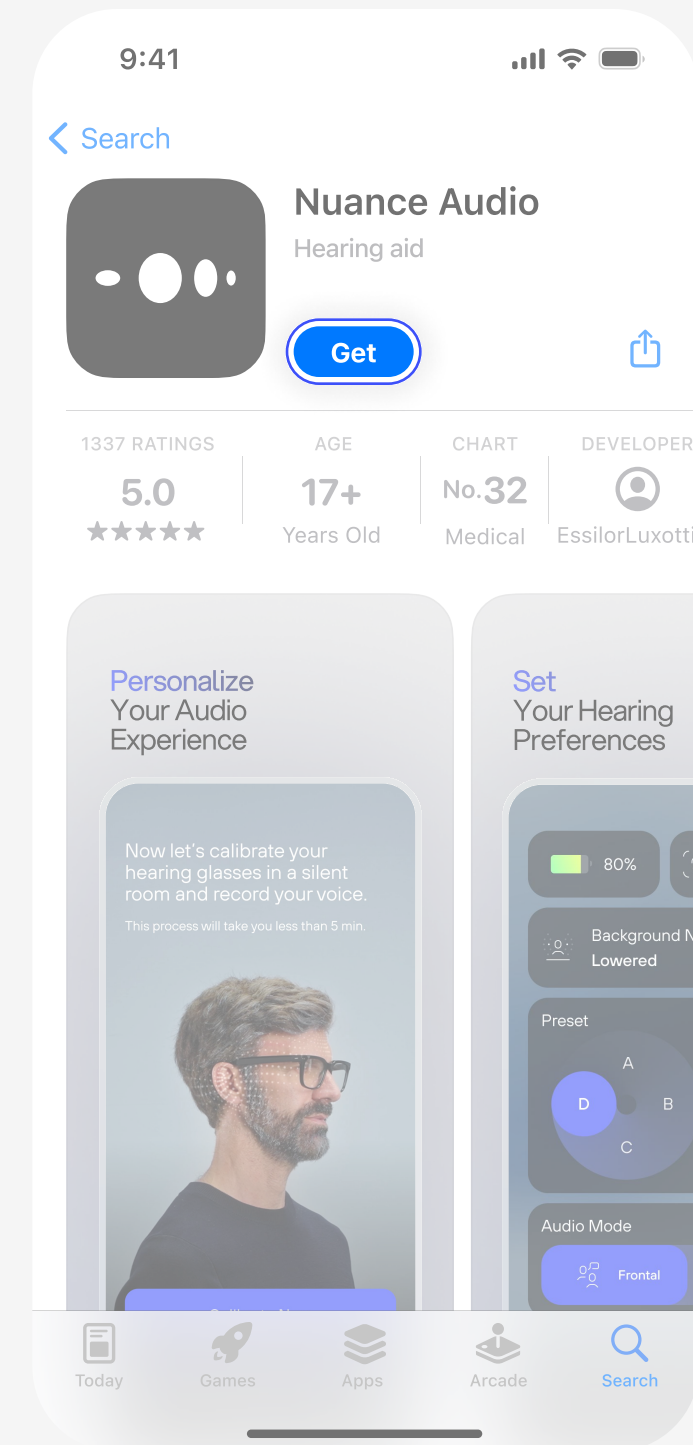

Search for “Nuance Audio” in the iOS App Store and tap **Get**.

The Get button will change to Open. You can launch the App from the App Store or find it on your home screen.

Google Play Store

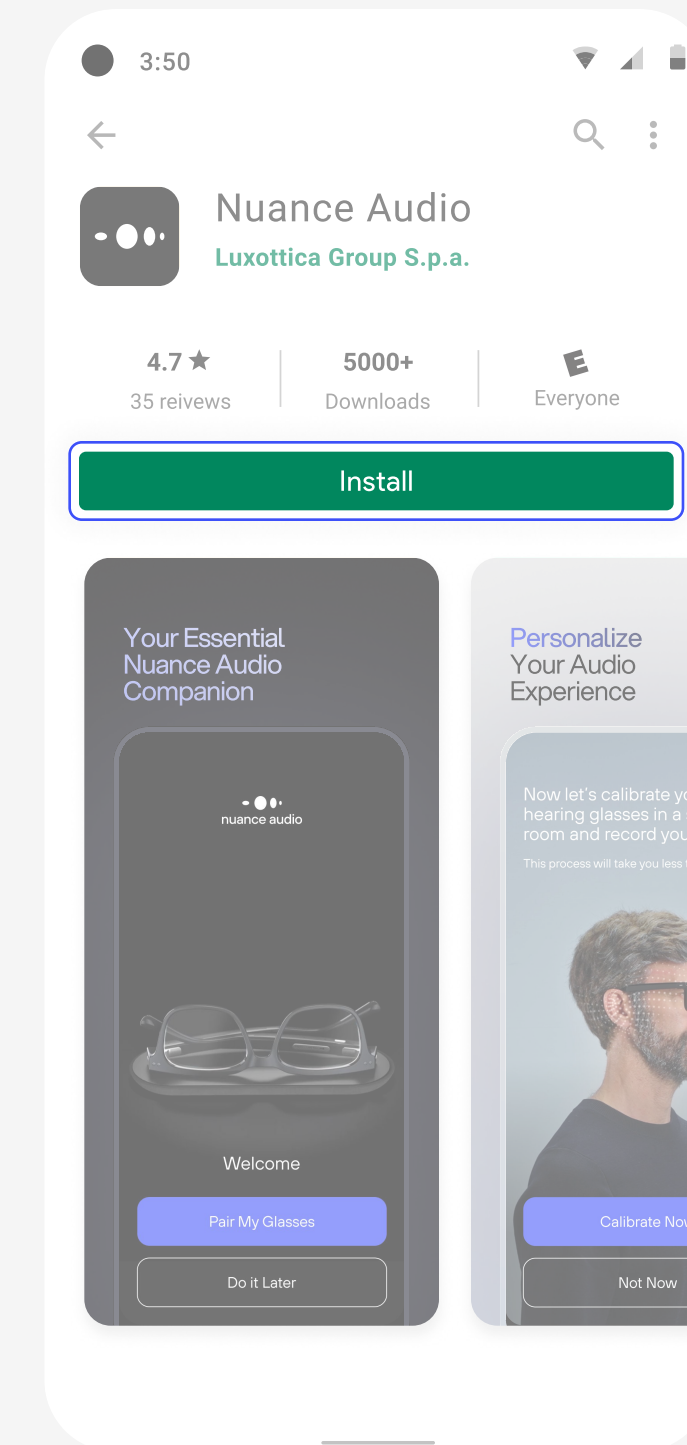

Search for “Nuance Audio” in the Google Play Store and tap **Install**.

The Get button will change to Open. You can launch the App from the Play Store or find it on your home screen.



Sign Up

Once you have downloaded the App, you can proceed with the Sign Up.

nuance audio

Log in to your account

Email

Password

[Forgot password?](#)

Log In

Don't have an account? [Signup](#)

< Create your account

First name*

Last name*

Email

Confirm Email*

Password

Date of birth

Gender

Date of birth and gender are used to

Nuance Audio

34m ago

Use code 123456 to access your Nuance Audio app.

OTP verification

Enter the OTP verification code from your email

111111

Verify

[Resend code](#)

nuance audio

Log in to your account

Email

johnsmith78@gmail.com

Password

[Forgot password?](#)


Log in

Don't have an account? [Sign up](#)




Sign up with phone number using OTP


If you chose to sign up with your phone number, you'll receive a OTP (one-time password) via SMS.

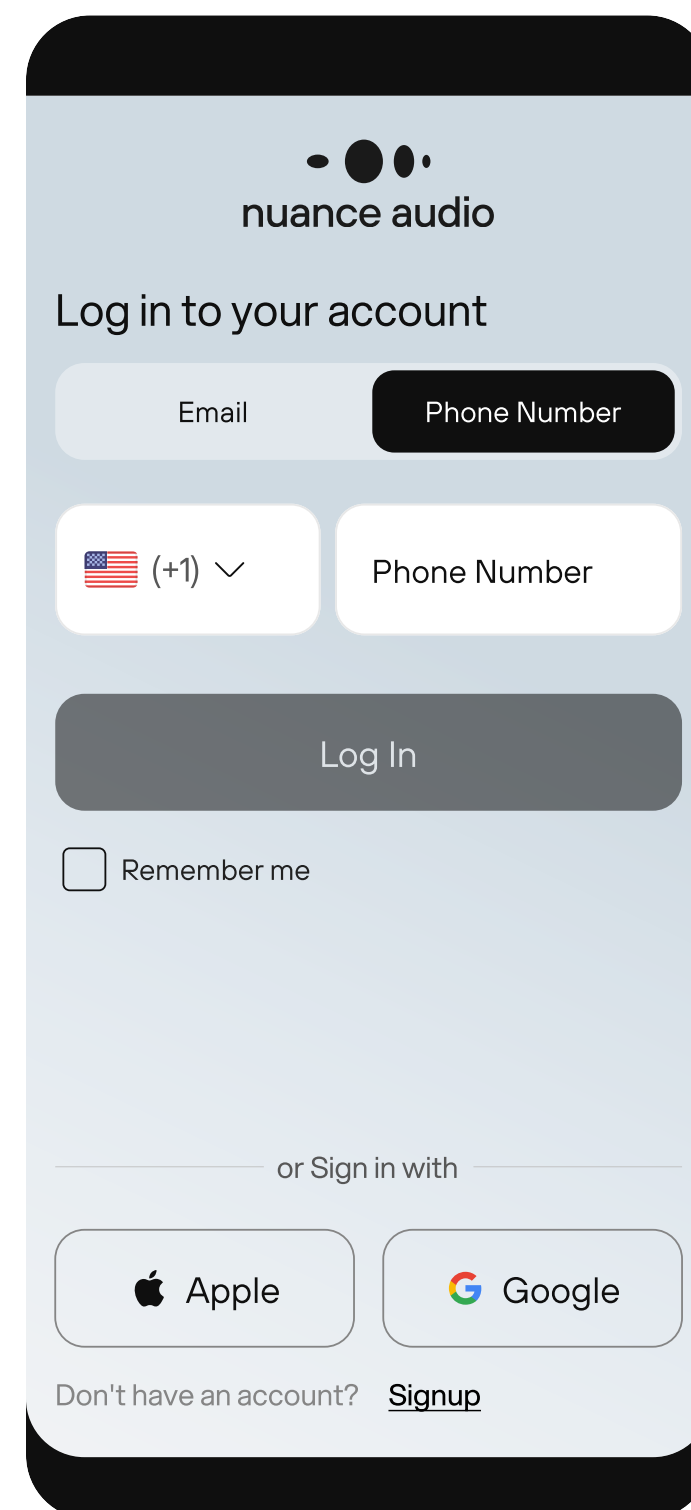


The first screen shows the Nuance Audio login interface. At the top is the Nuance Audio logo. Below it is the text "Log in to your account". There are two tabs: "Email" and "Phone Number", with "Phone Number" selected. Below the tabs is a phone number input field with a dropdown menu showing the country code (+1). Below the input field is a "Log In" button. At the bottom, there is a "Remember me" checkbox and a link to "Signup" for users who don't have an account.

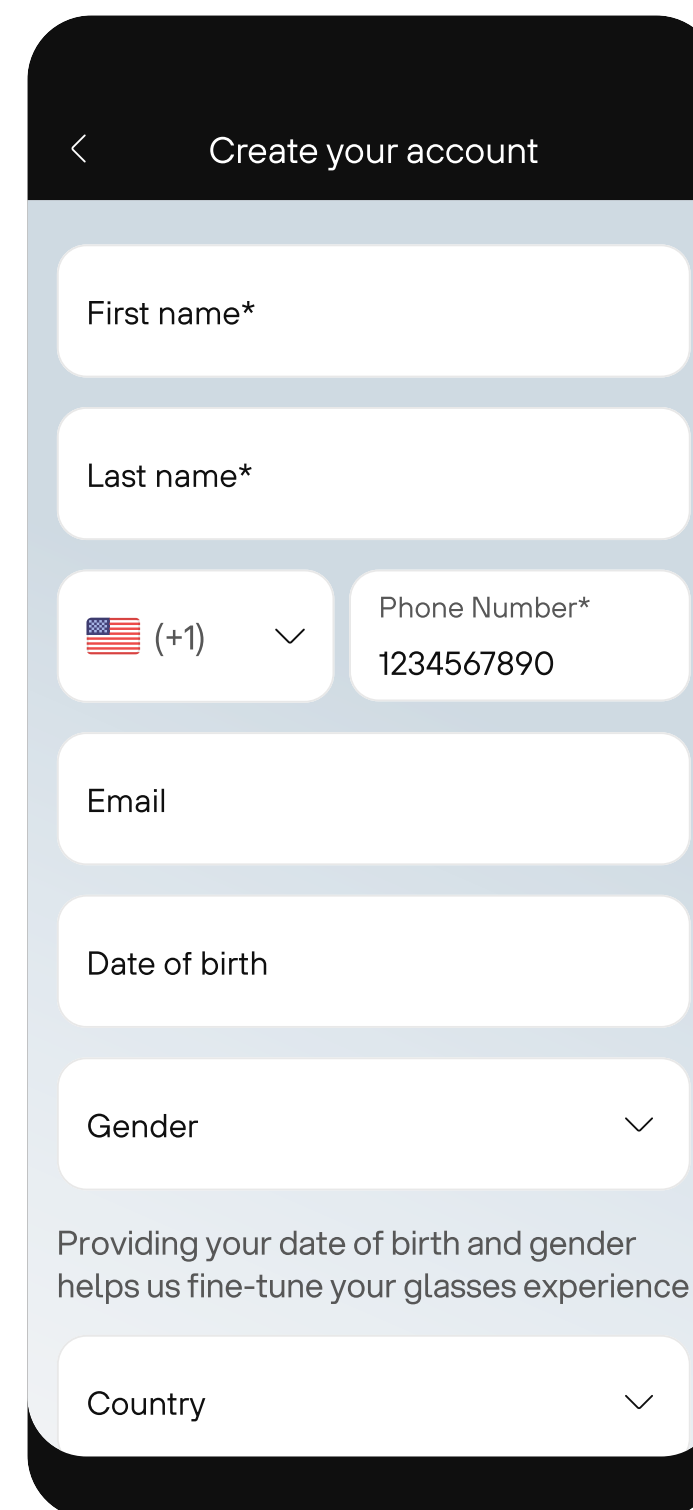

The second screen shows the "Create your account" form. It includes fields for "First name*", "Last name*", "Email", "Date of birth", "Gender", and "Country". The "Phone Number*" field is pre-filled with "+1 1234567890". Below the form, there is a note: "Providing your date of birth and gender helps us fine-tune your glasses experience".

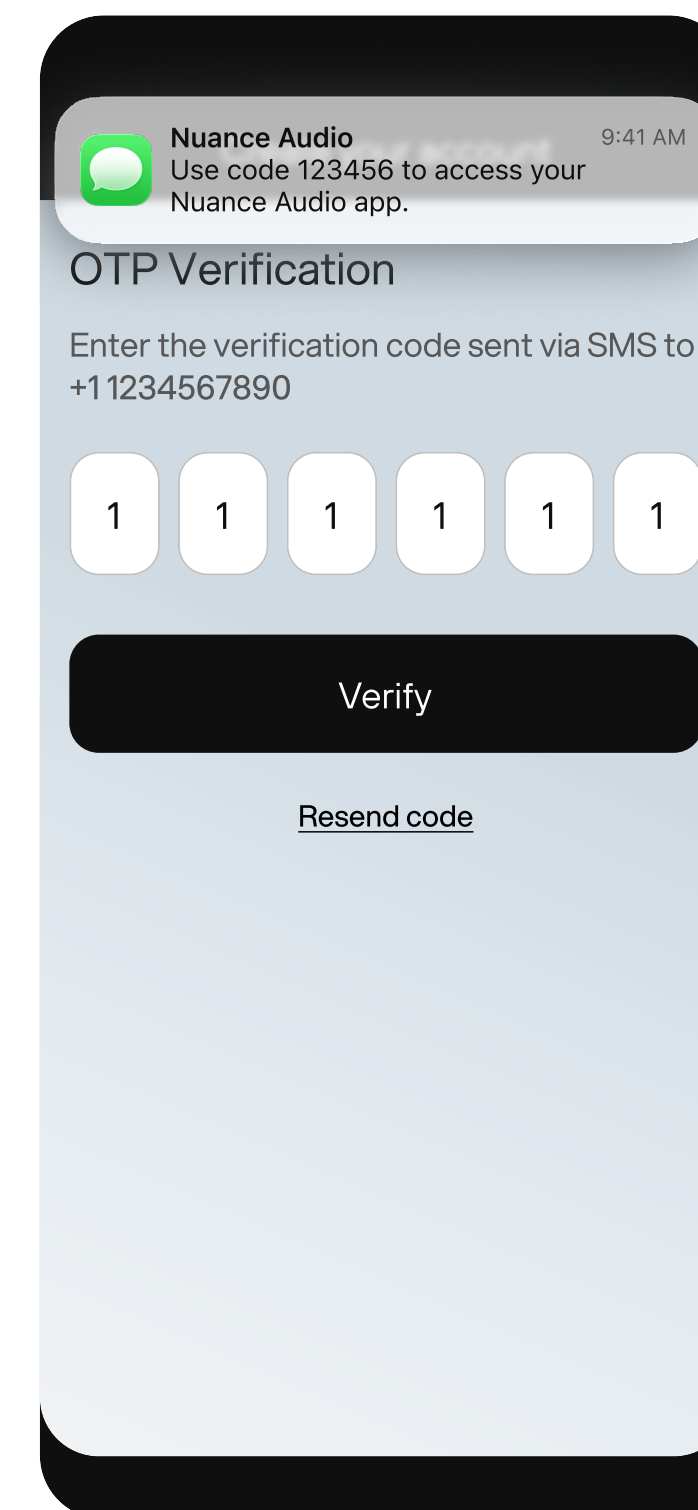

The third screen shows the "OTP Verification" screen. At the top, there is a notification from "Nuance Audio" stating "Use code 123456 to access your Nuance Audio app." with a timestamp of 9:41 AM. Below the notification, the text "Enter the verification code sent via SMS to +1 1234567890" is displayed. There are six input fields for the verification code, each containing the digit "1". Below the input fields is a "Verify" button and a link to "Resend code".

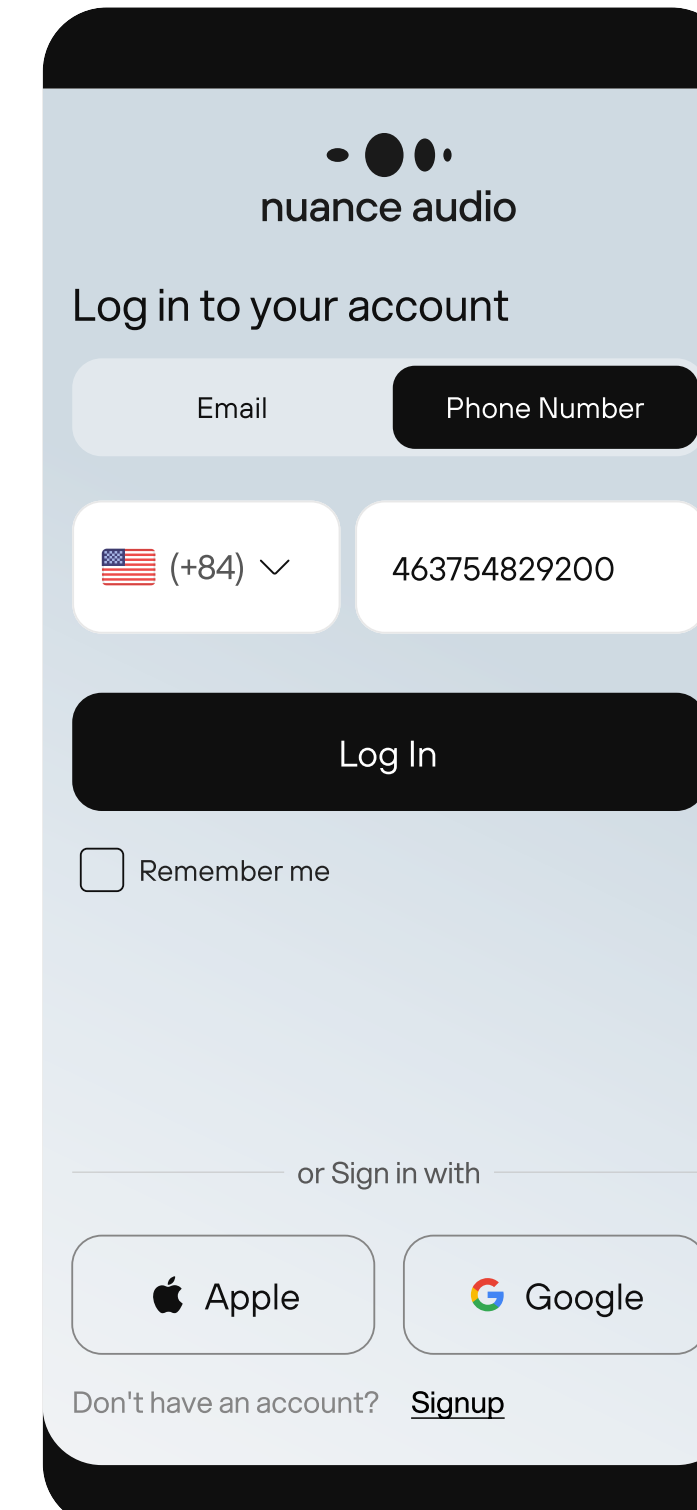


The fourth screen shows the Nuance Audio login interface after successful OTP verification. It is identical to the first screen, but the "Phone Number" tab is selected, and the phone number input field is pre-filled with "+84 463754829200".



Phone pairing: first device



Enter pairing mode by
moving **"Slide On" button
forward** until the blue LED
blinks.

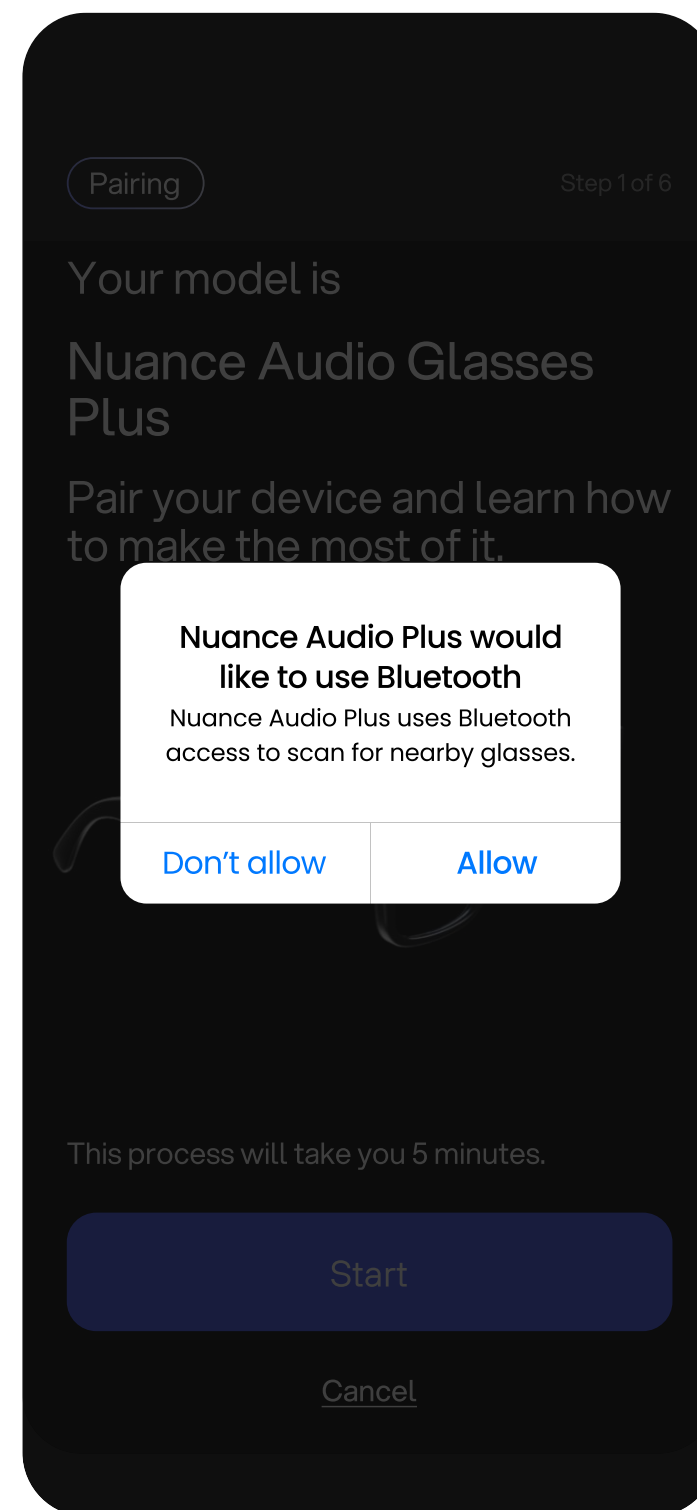

Allow Bluetooth
permission.
Press **Allow**.




Start pairing.
Press **Start**.




Phone pairing: first device



Select Nuance Audio Plus
Press **Pair**.

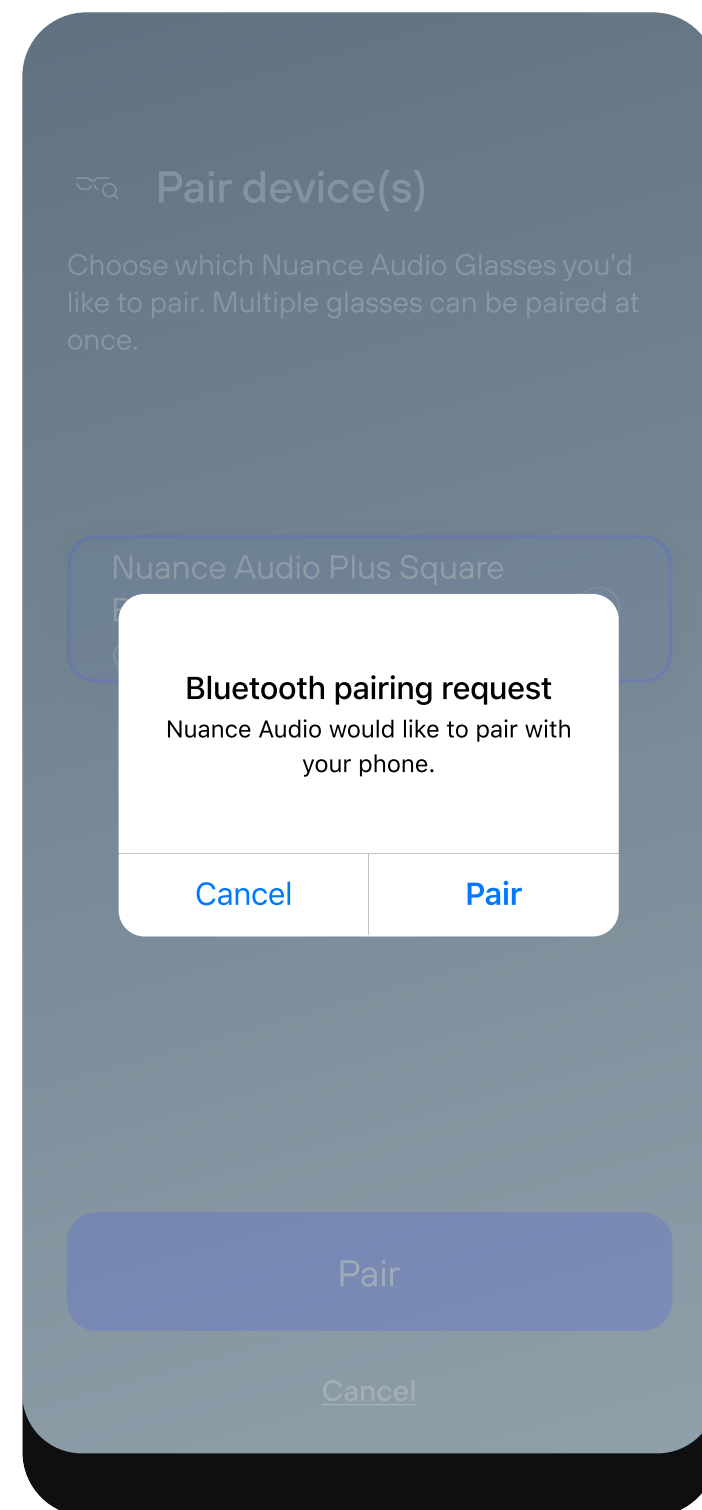


Allow Bluetooth Pairing
Request.
Press **Pair**.

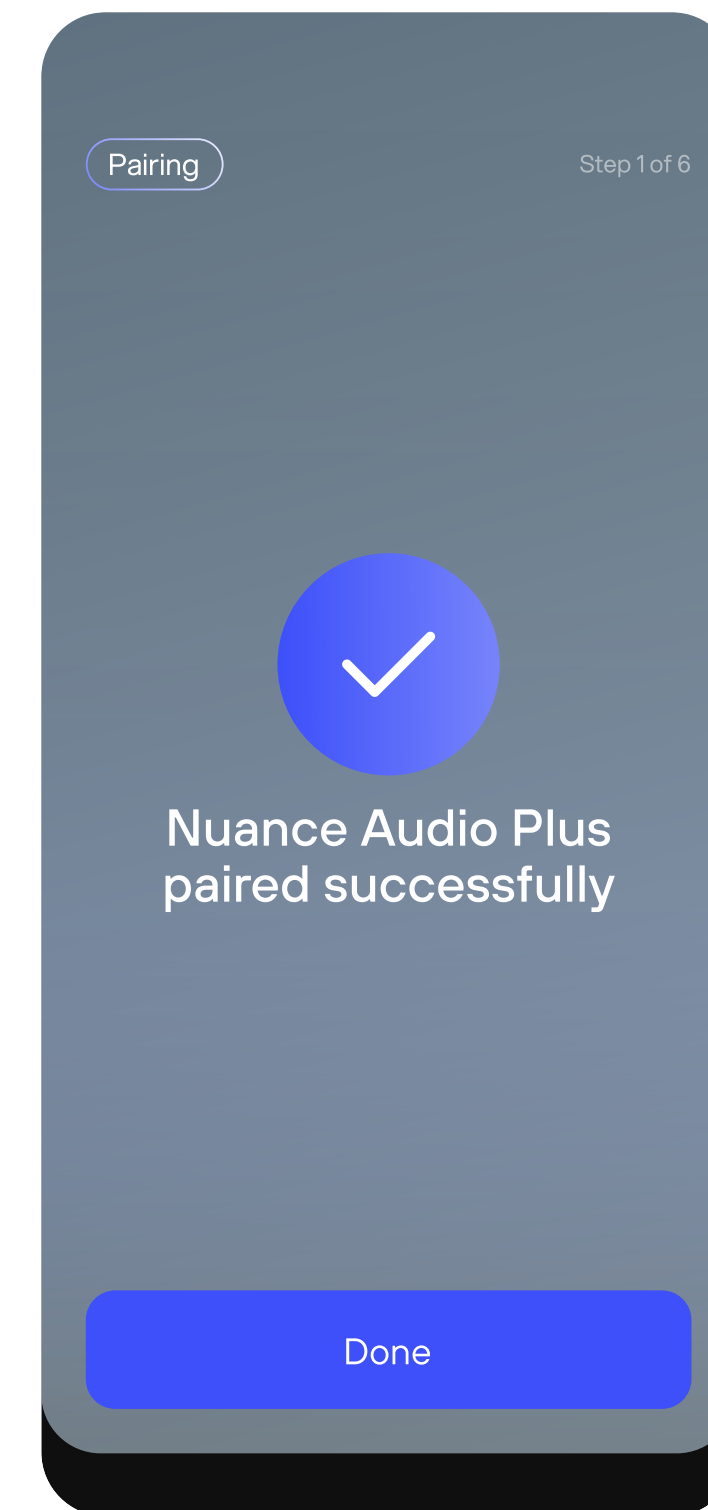

Pairing complete.



Pairing a second device on the same account

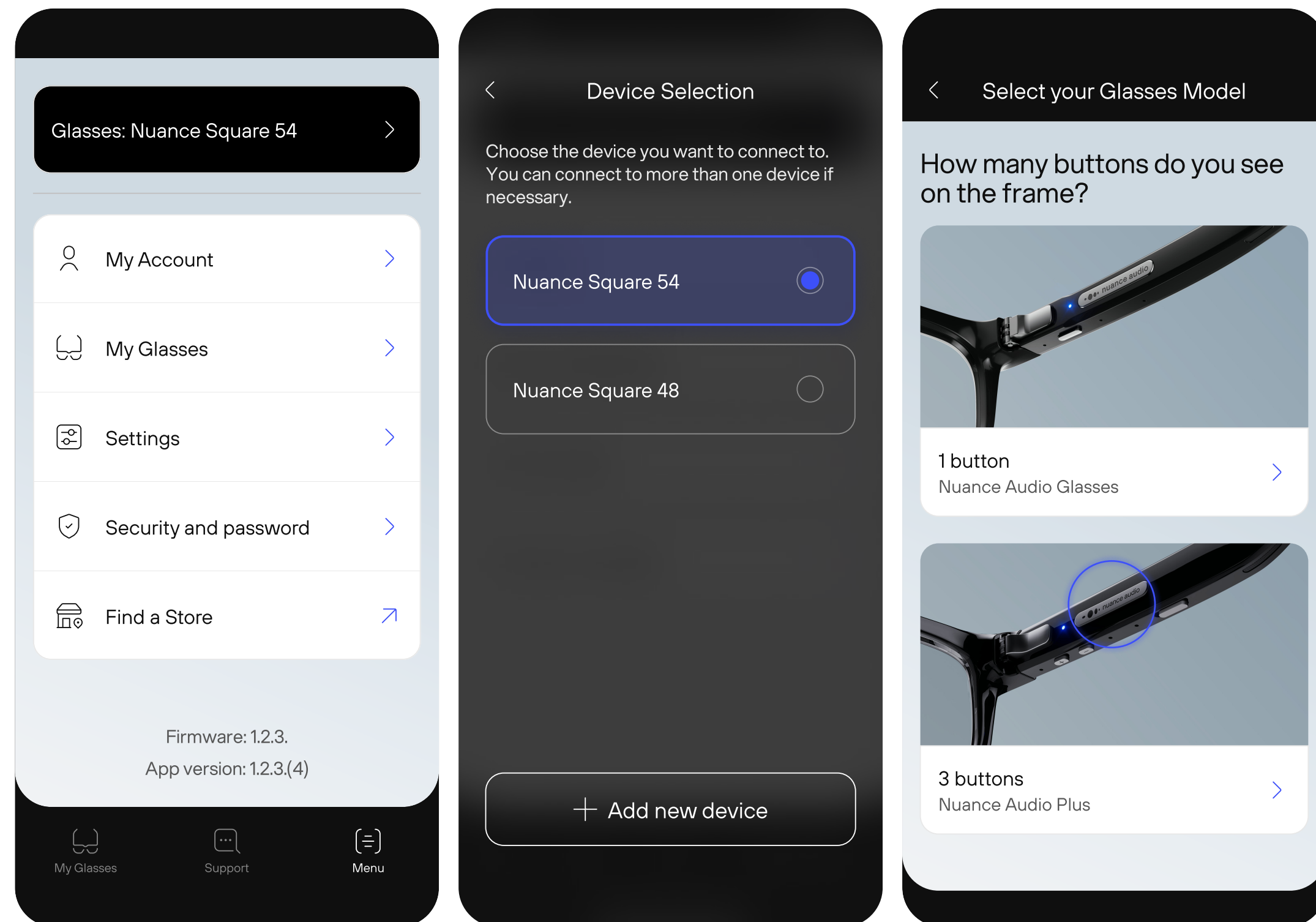


Step 0.1
Go to **Menu**. Tap the “Glasses” icon in the top banner to add a new pair.

Step 0.2
Select the Glasses you want to connect to.
Press “Add New Device.”


Step 0.3
Choose the model of your Glasses.

Step 0.4
To start pairing, slide on the slider button, press “+” or “-” and **hold for 8 seconds** until the blue LED blinks.

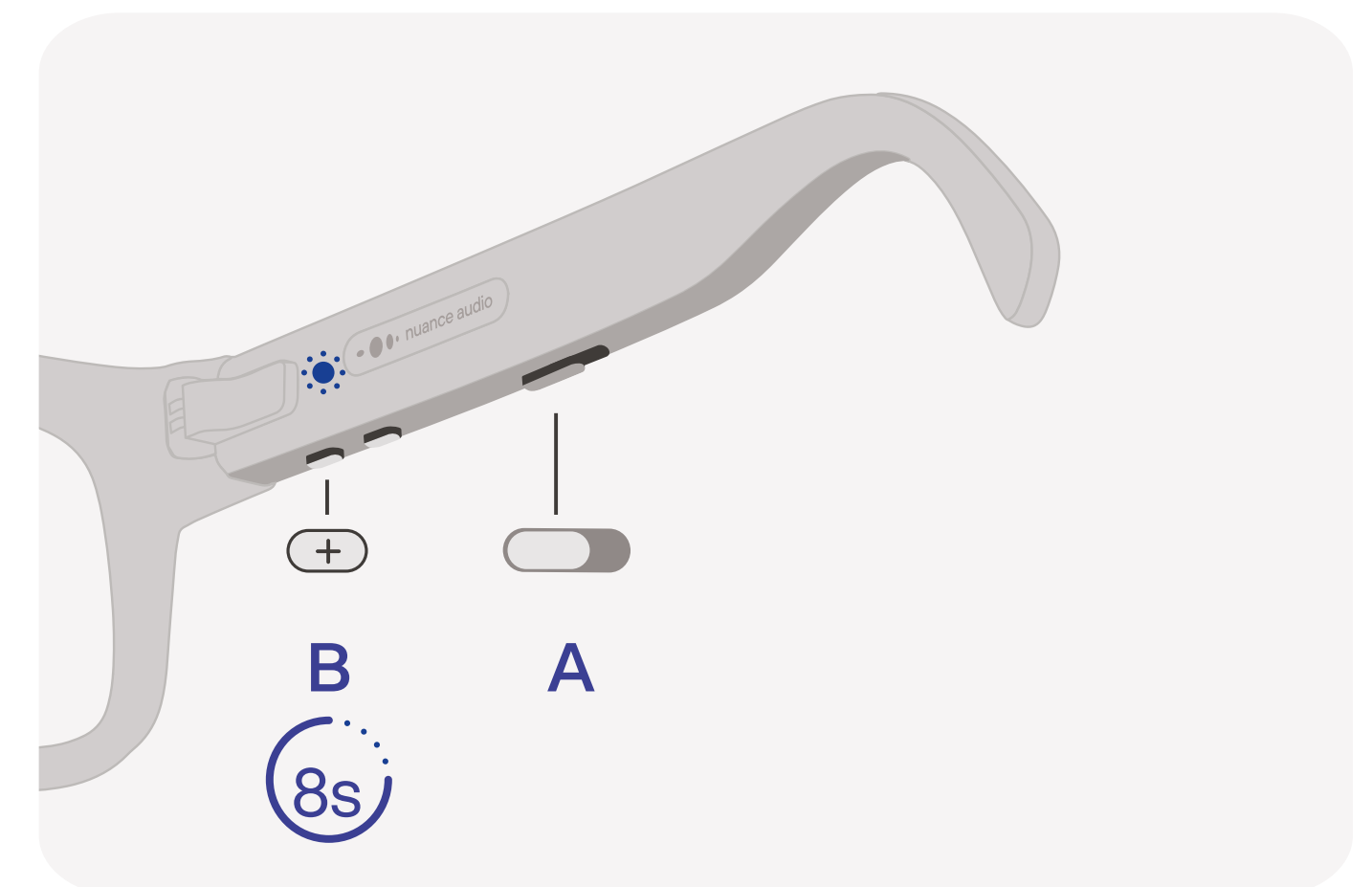





Pairing a second device on the same account

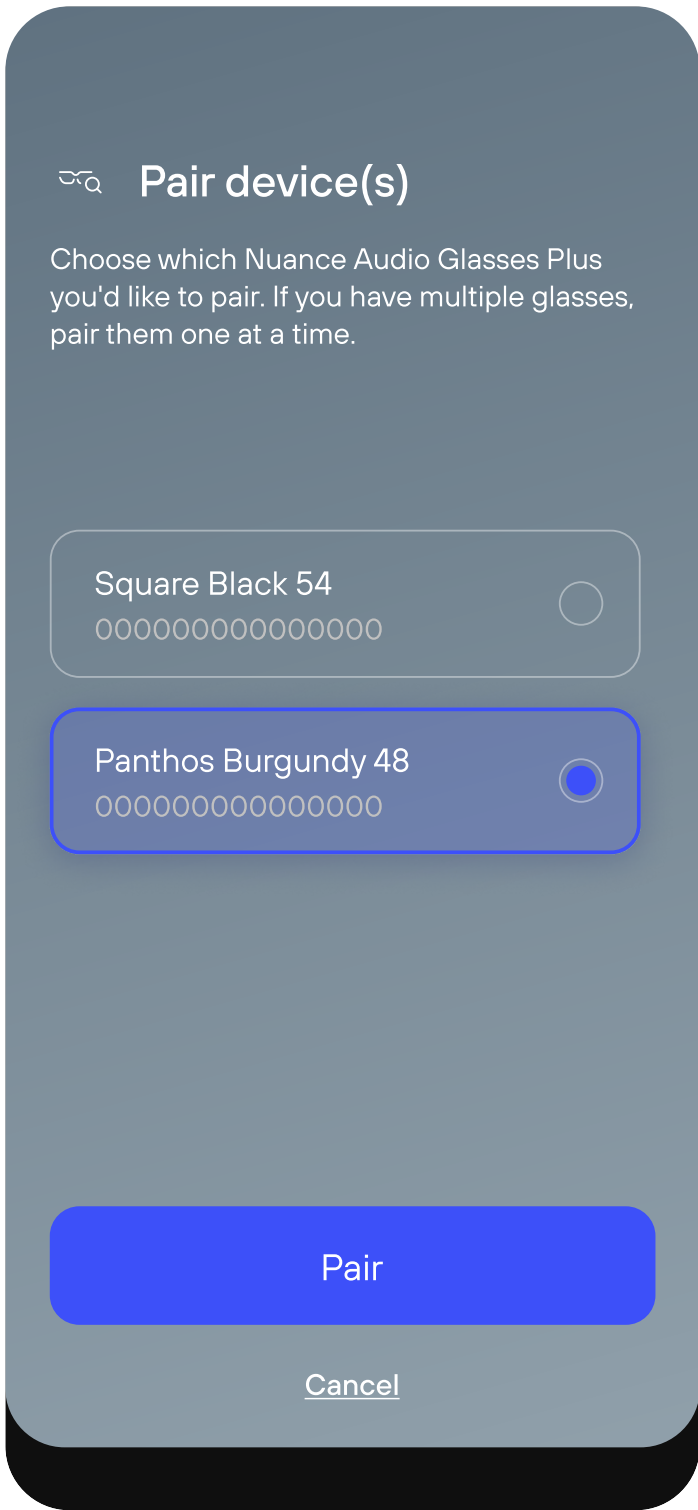

Select Nuance Audio Plus from the list.
Press Pair.

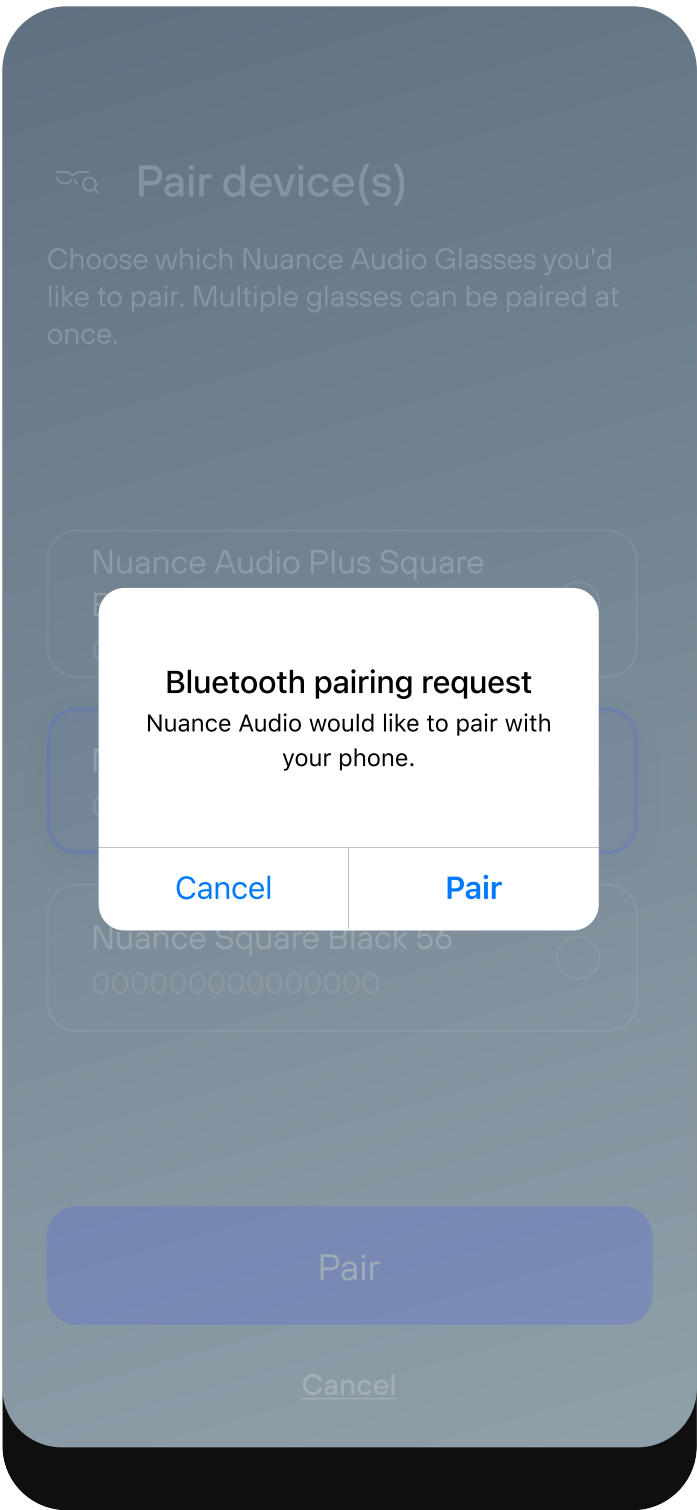

Allow Bluetooth Pairing Request.
Press Pair.

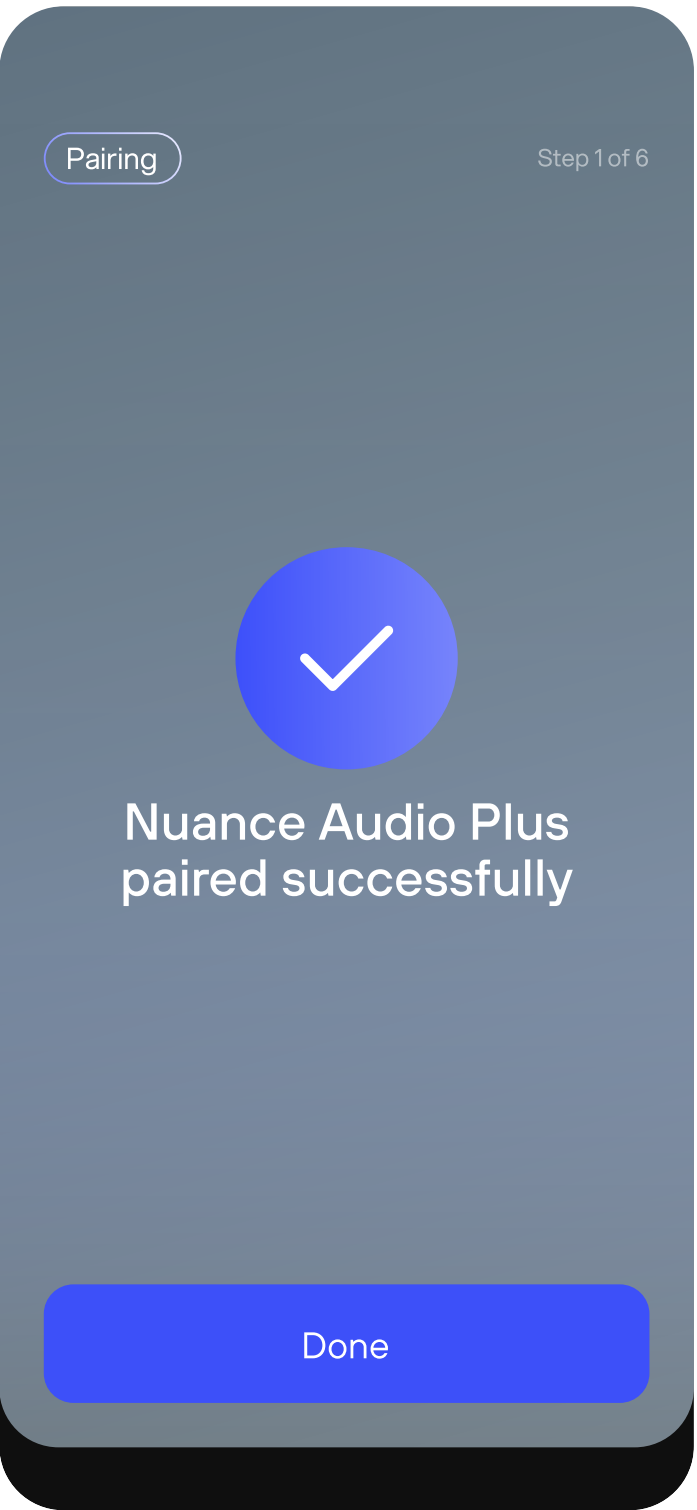

Pairing complete.



What is personalized calibration?

Calibrate your Nuance Audio Plus so that they are personalized and fit your needs.

Personalized calibration optimizes the sound based on your unique head shape and voice.

Here are two areas it focuses on:

- **Clearer sound:** it adapts sounds to reduce feedback and noise, so you hear more clearly.
- **Natural voice feedback:** registers your voice to make your conversations sound more natural.





How to make the most of your Calibration?

Calibration takes just one minute but helps you get the best from your Nuance Audio Plus.

If you own multiple pairs, calibrate each one individually.

A few tips to optimize Personal Calibration:


- Find a silent spot in your home, avoiding large or open spaces.
- Stay away from areas with many or large windows.
- Avoid rooms with high ceilings where background noise can be unpredictable.
- Position yourself away from corners and avoid facing mirrors.
- Ideally, stay as close to the center of the room as possible for optimal sound quality.
- Final tip: Your bedroom is often the best option.

A few tips to optimize Voice Calibration:

- Speak loudly but without shouting.
- Project your voice as if speaking to someone across the room.
- Minimize pauses as much as possible.
- If you own multiple pairs, calibrate each one individually.




How do you calibrate your Glasses?

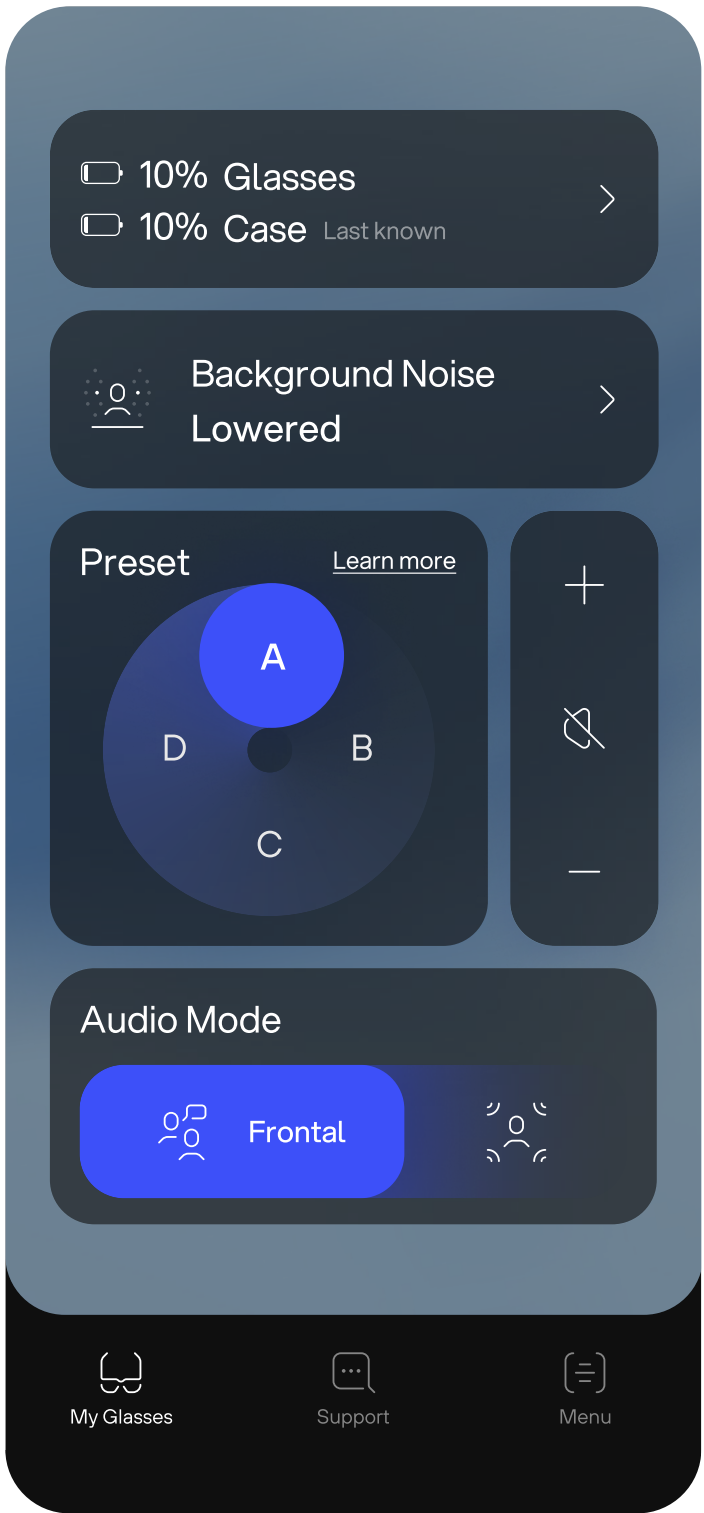


Within the Nuance App’s dashboard, tap on (bottom-right corner in the navigation bar).

Notes
A blue indicator will notify you and guide you to the calibration screen

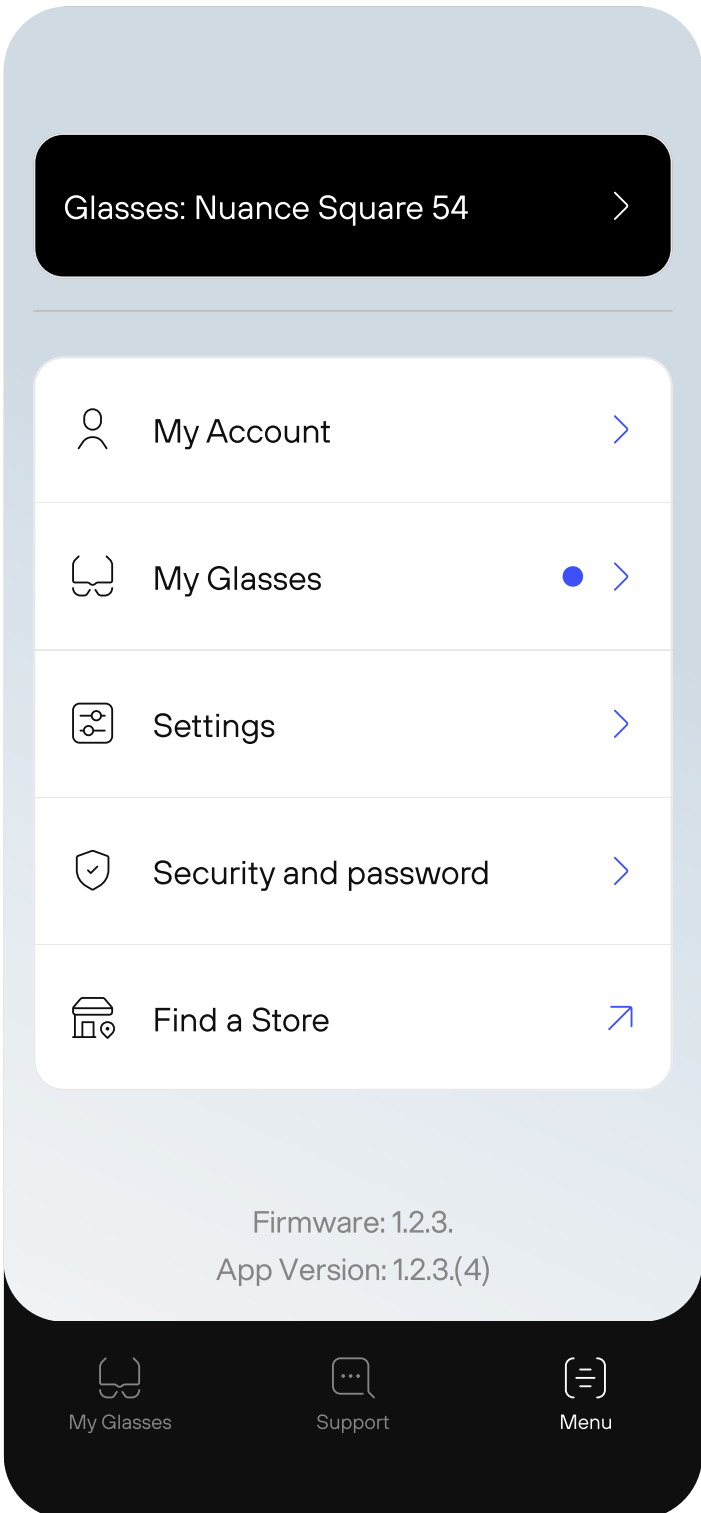

Tap on **Settings**.

Notes
A blue indicator will notify you and guide you to the calibration screen

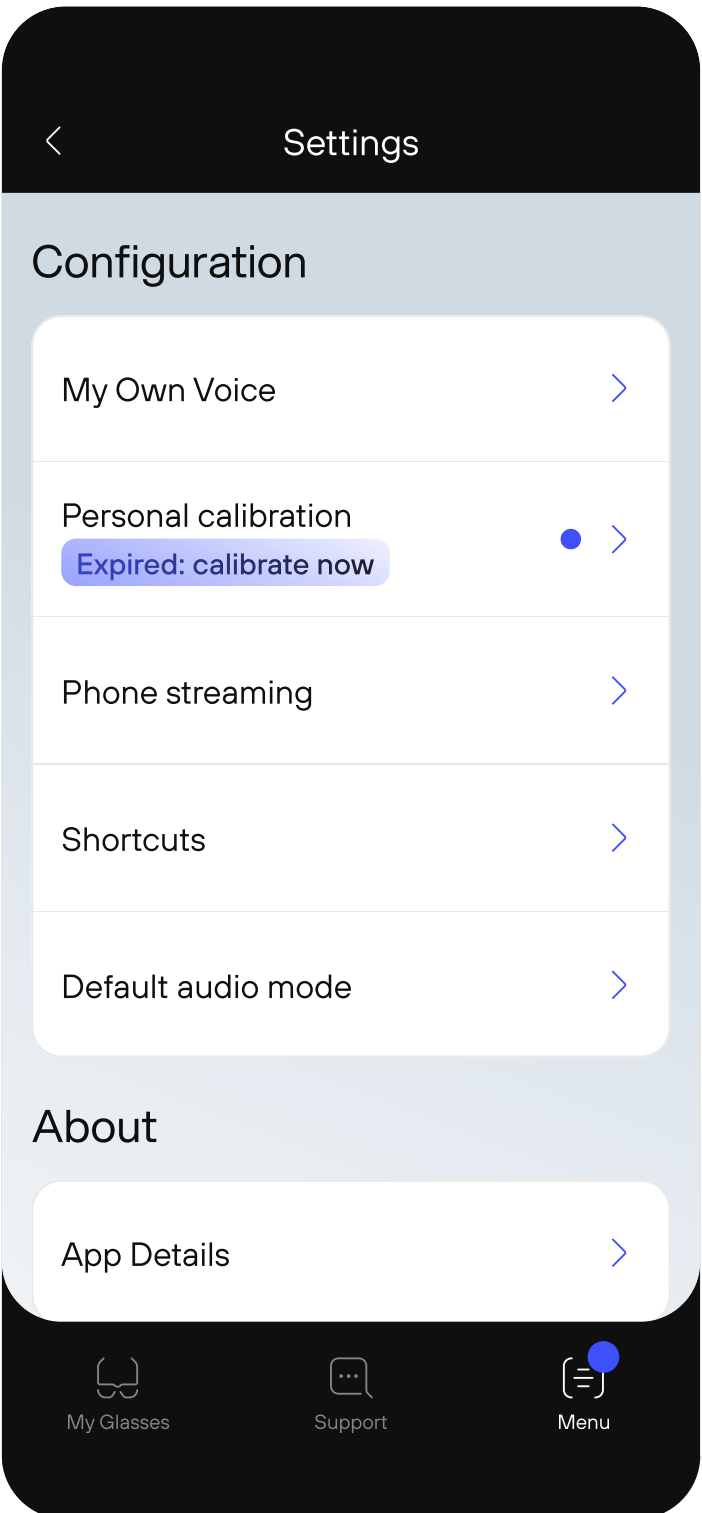

To start calibration, tap on **Personal calibration**.


Notes
A blue indicator and a dedicated label will notify you to calibrate to update your audio preferences.




How do you calibrate your Glasses?




To start calibration, find a quiet room and then press **Start calibration.**

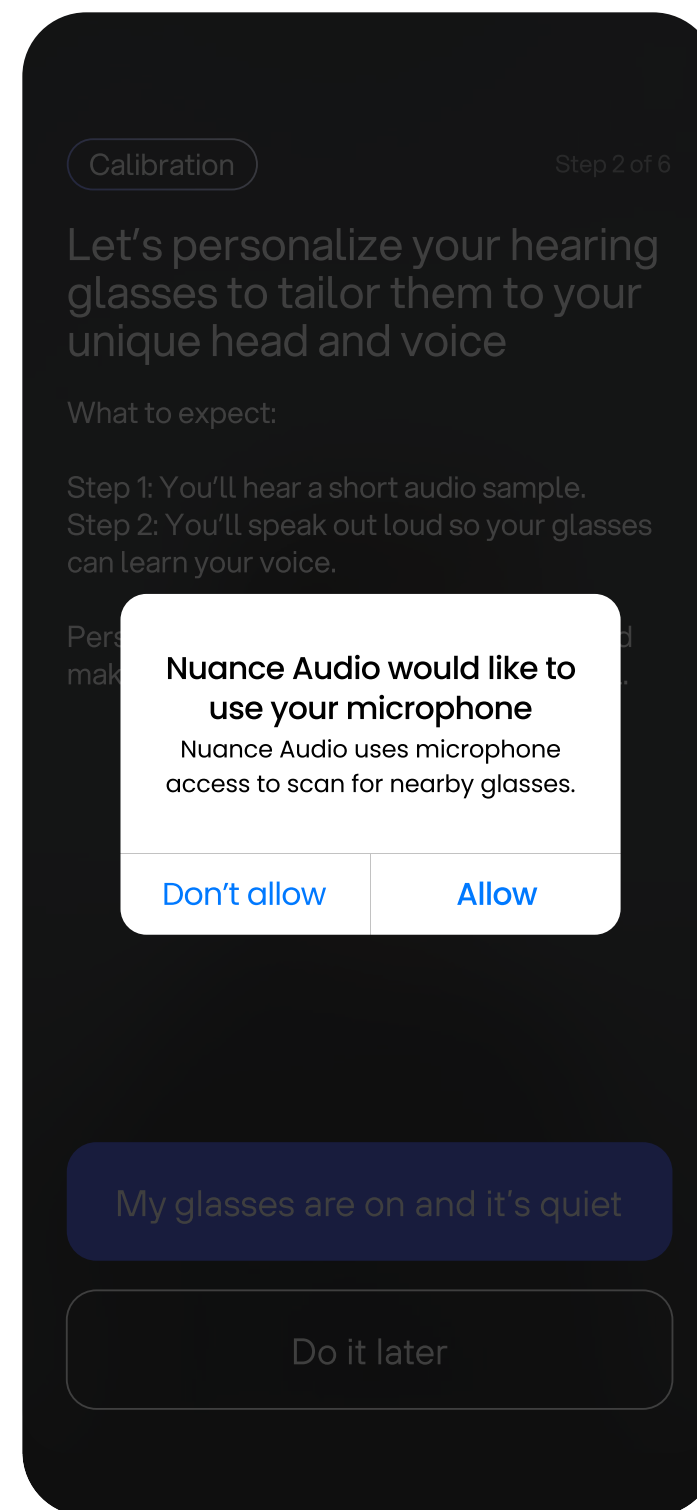

You'll need to allow your phone's microphone to measure and analyze sounds. Press **Allow.**

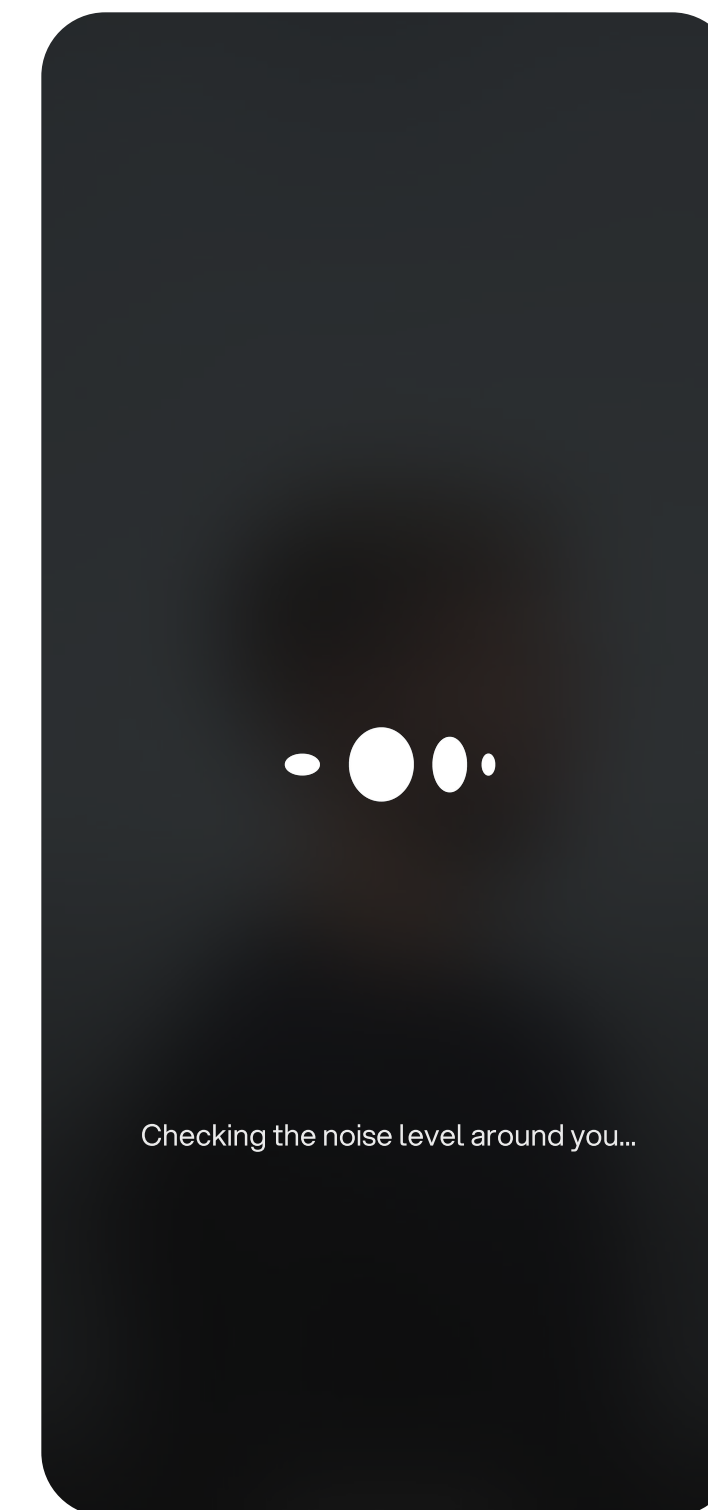


Pairing completed. A pop-up will remind you to keep your Nuance Audio Plus on during the entire process.

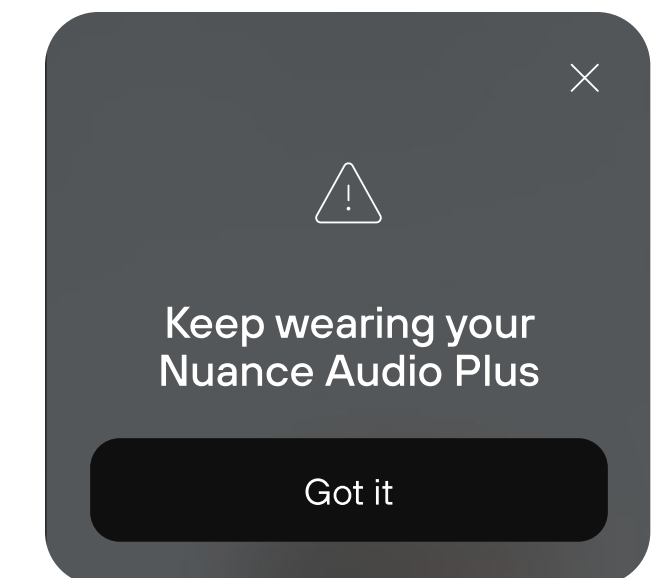





How do you calibrate your Glasses?

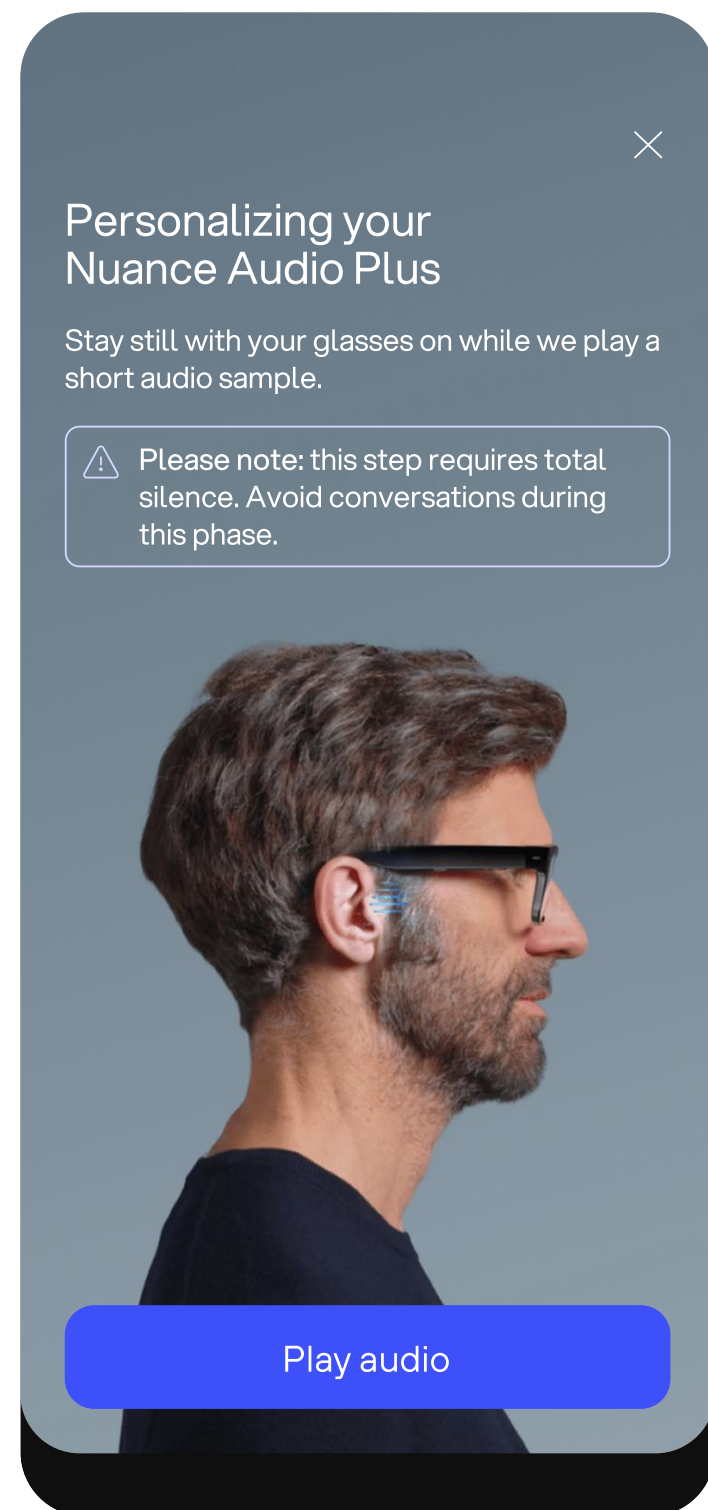

Step 1.1
Environmental noise
measurement.
Keep your Glasses on and
tap **Play audio**.

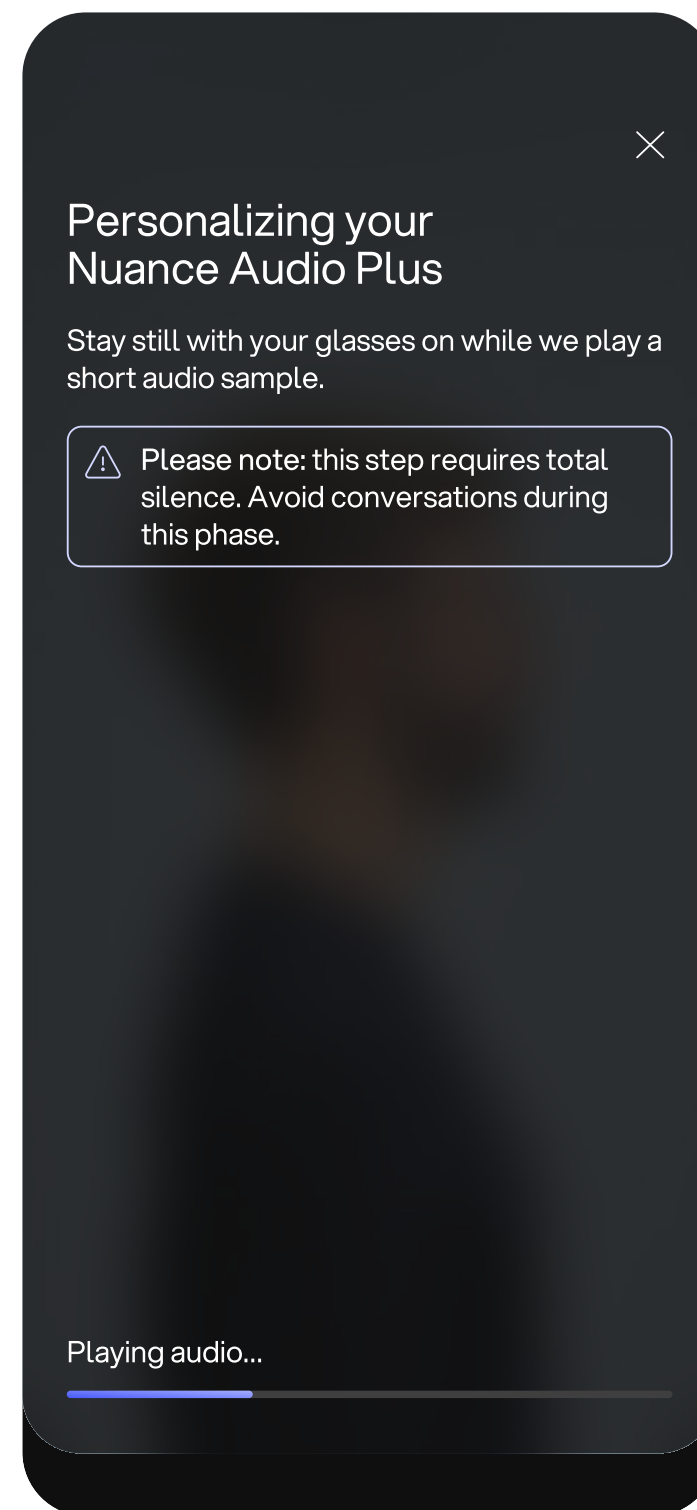


Step 1.2
Stay still and don't take off
your Glasses, **wait until
the end of the audio
sample.**

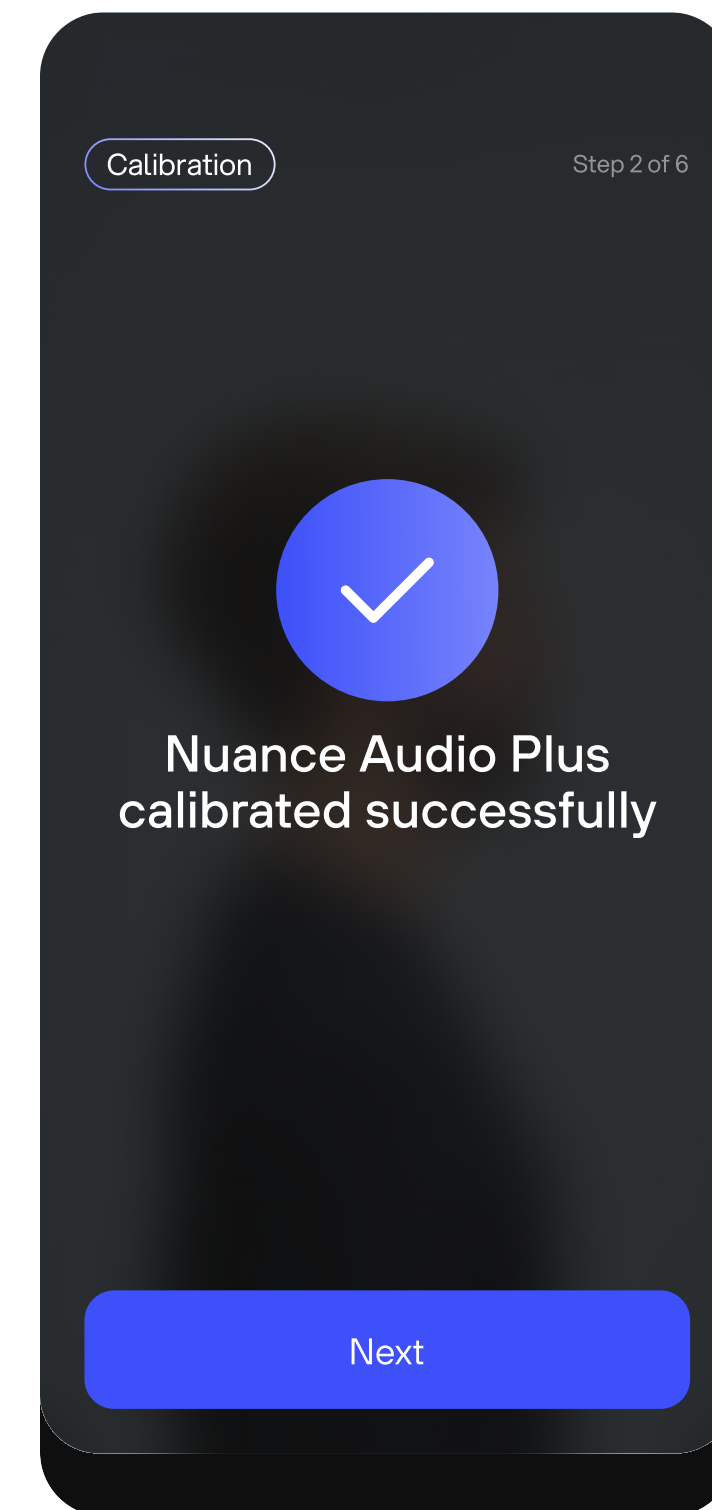


Step 1.3
Environmental noise
measurement completed.




How do you calibrate your Glasses?

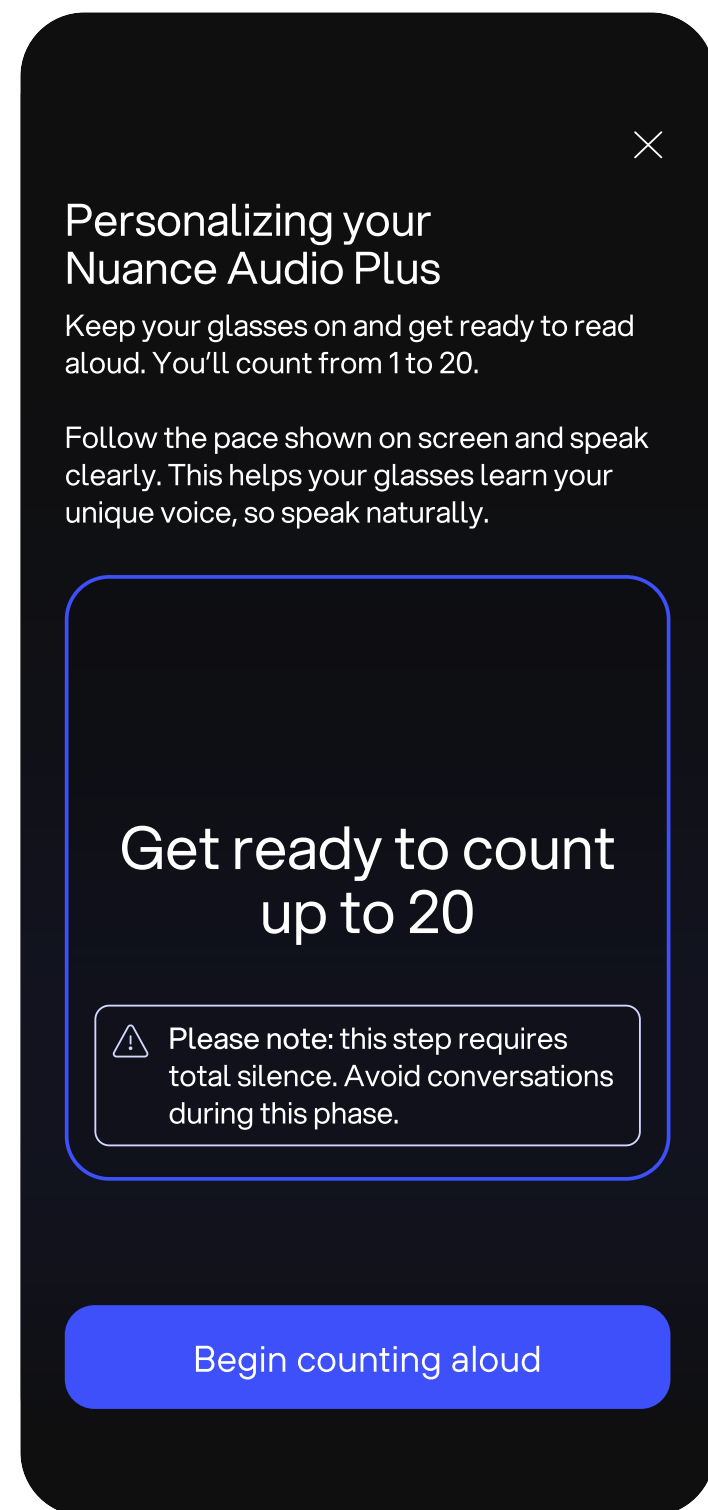

Step 2.1
Own voice calibration.
Stay still and don't take off
your Glasses.
Press **Start recording**.

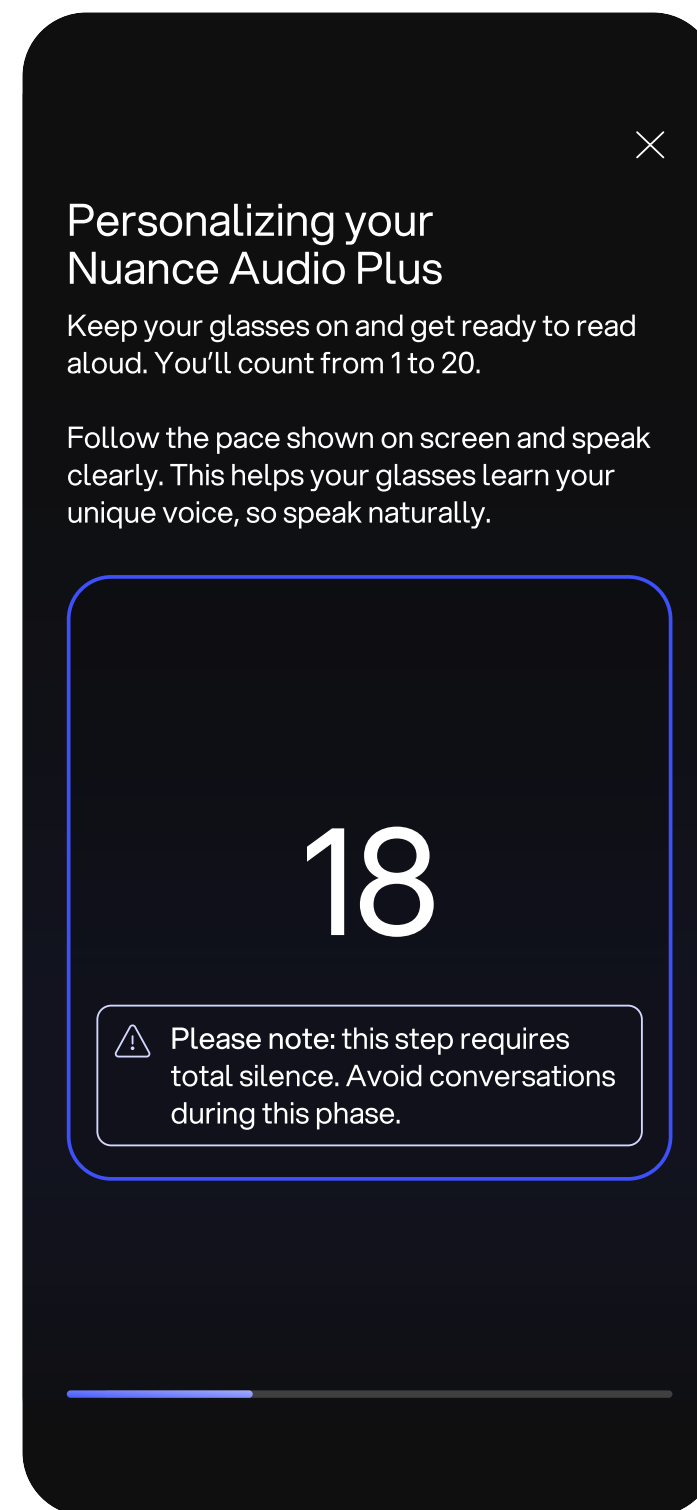

Step 2.2
**Count out loud until the
progress bar reaches
the end.**

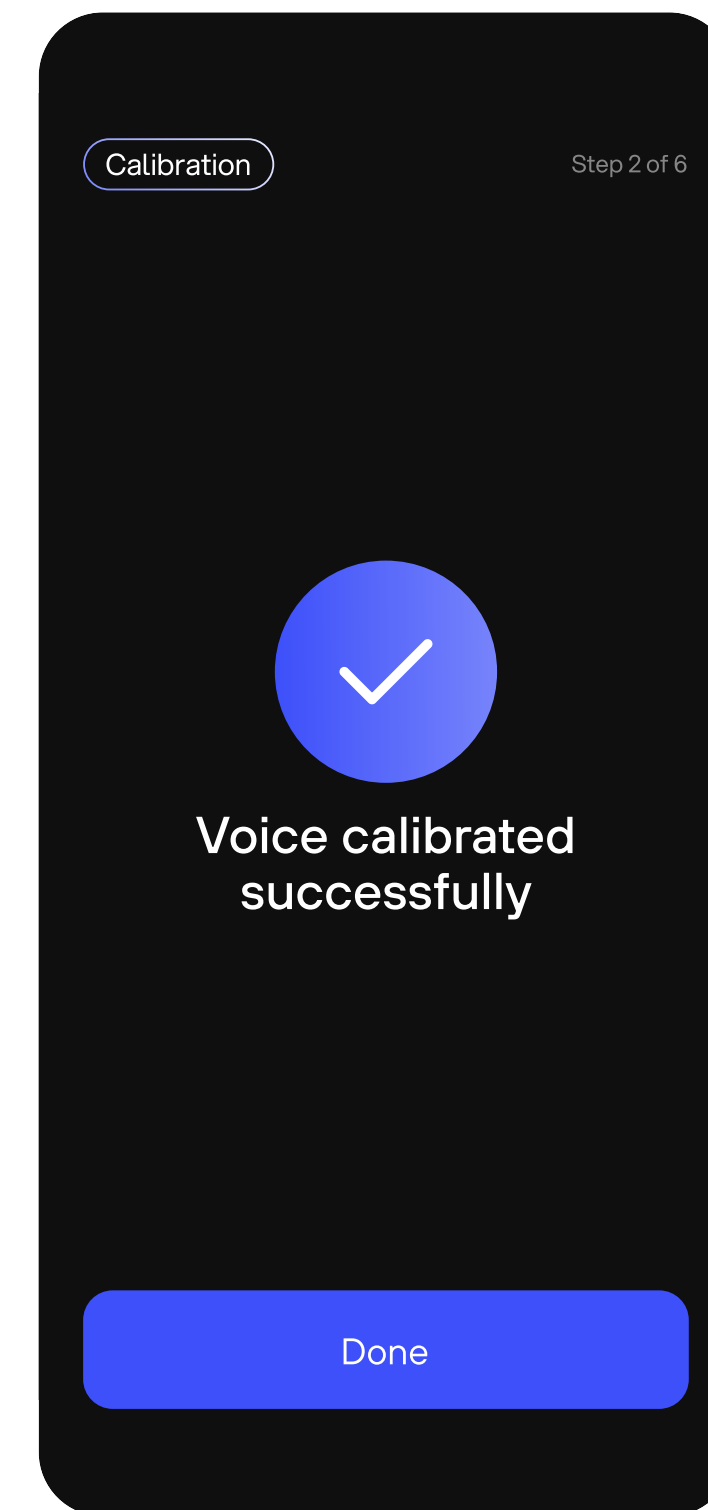

Step 2.3
Personal calibration
successfully completed.



For the most updated Instructions
for Use please follow this link

[↗ View instruction for use](#)