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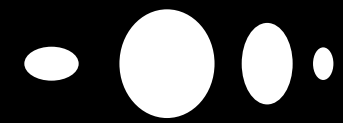
Mobile app login screen for Nuance Audio. The header shows the Nuance Audio logo. Below it, the text "Inicia sesión en tu cuenta" is displayed. There are two input fields: "Correo electrónico" and "Número de teléfono". The "Número de teléfono" field is highlighted with a black background. Below the input fields, there is a dropdown menu for the country code, currently showing "+1" with a US flag. A large black button labeled "Iniciar sesión" is positioned below the input fields. Below the button, there is a "Recuérdame" checkbox. At the bottom, there is a section for social login with "o Inicia sesión con" followed by "Apple" and "Google" buttons. At the very bottom, there is a link "¿No tienes una cuenta? Registrarse".

Mobile app registration screen for Nuance Audio. The header shows a back arrow and the text "Crea una cuenta". Below it, there are several input fields: "Nombre*", "Apellidos*", "Número de teléfono*" (with a dropdown for country code showing "+1" and a US flag), "Correo electrónico", "Fecha de nacimiento", "Género" (with a dropdown arrow), and "País" (with a dropdown arrow). A note at the bottom states: "Si nos indicas tu fecha de nacimiento y sexo, podremos perfeccionar tu experiencia con las gafas."

Mobile app OTP verification screen for Nuance Audio. The header shows a back arrow and the text "Verificación OTP". Below it, there is a message: "Introduce el código de verificación enviado por SMS a +11234567890". There are six input fields for the OTP code, each containing the digit "1". A large black button labeled "Verificar" is positioned below the input fields. Below the button, there is a link "Reenviar el código".

Mobile app login screen for Nuance Audio. The header shows the Nuance Audio logo. Below it, the text "Inicia sesión en tu cuenta" is displayed. There are two input fields: "Correo electrónico" and "Número de teléfono". The "Número de teléfono" field is highlighted with a black background and contains the pre-filled number "463754829200". Below the input fields, there is a dropdown menu for the country code, currently showing "+84" with a US flag. A large black button labeled "Iniciar sesión" is positioned below the input fields. Below the button, there is a "Recuérdame" checkbox. At the bottom, there is a section for social login with "o Inicia sesión con" followed by "Apple" and "Google" buttons. At the very bottom, there is a link "¿No tienes una cuenta? Registrarse".

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