




Nuance Audio Remote Plus




Remote Plus guided onboarding Setup Flow

The Remote Plus is used to activate and set up your eyewear. It guides you through onboarding, lets you choose your preferred language, and helps you through a quick personal calibration so everything feels just right for you.

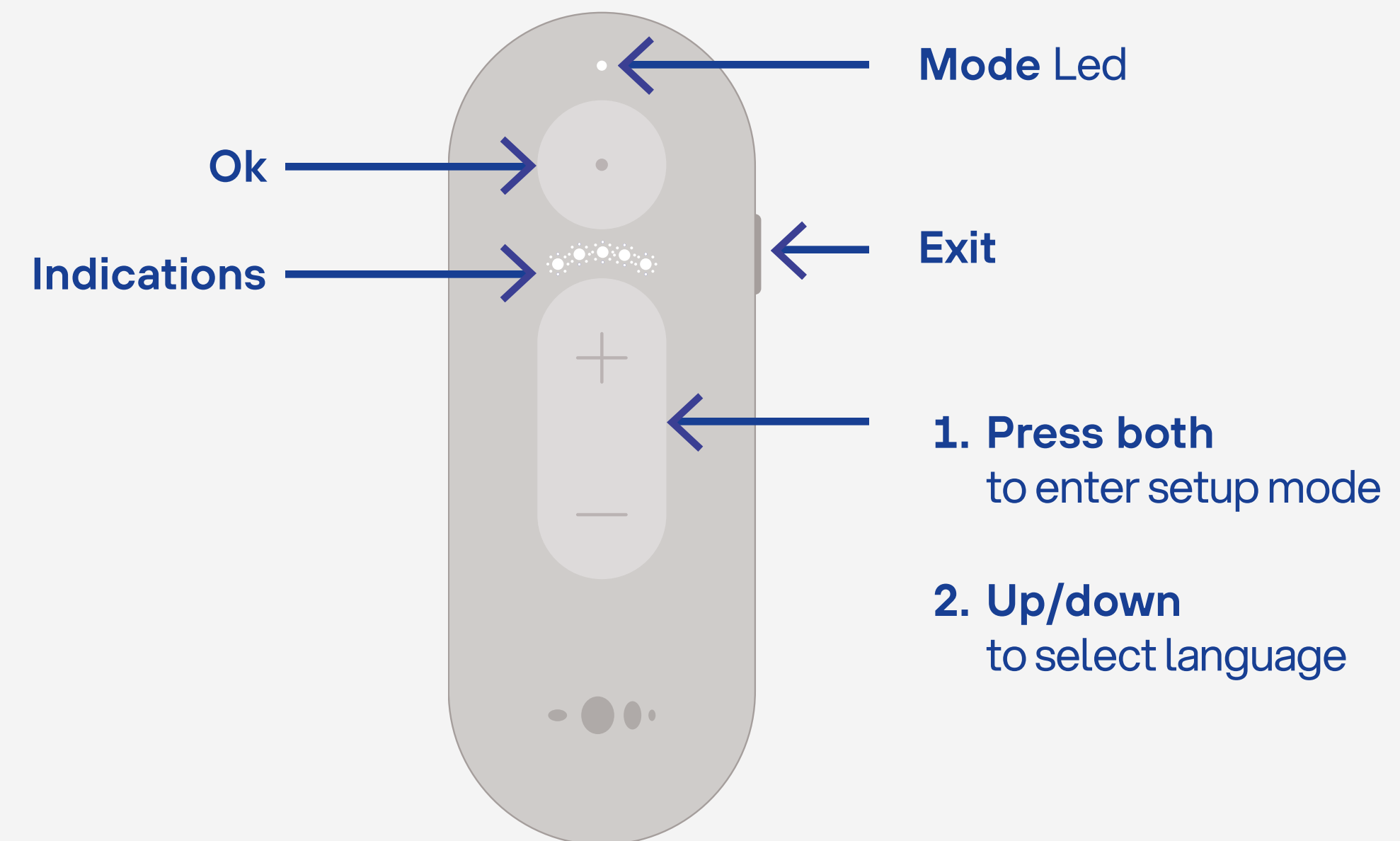




Remote Plus guided onboarding Setup Flow

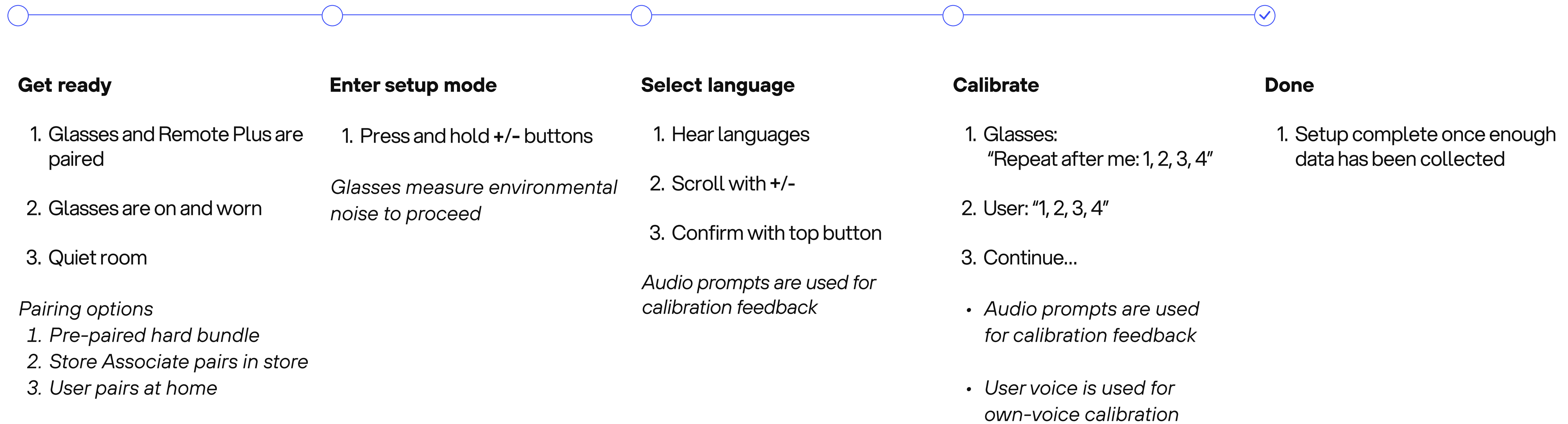




Remote Plus user interface

Frame Pairing and Set Up

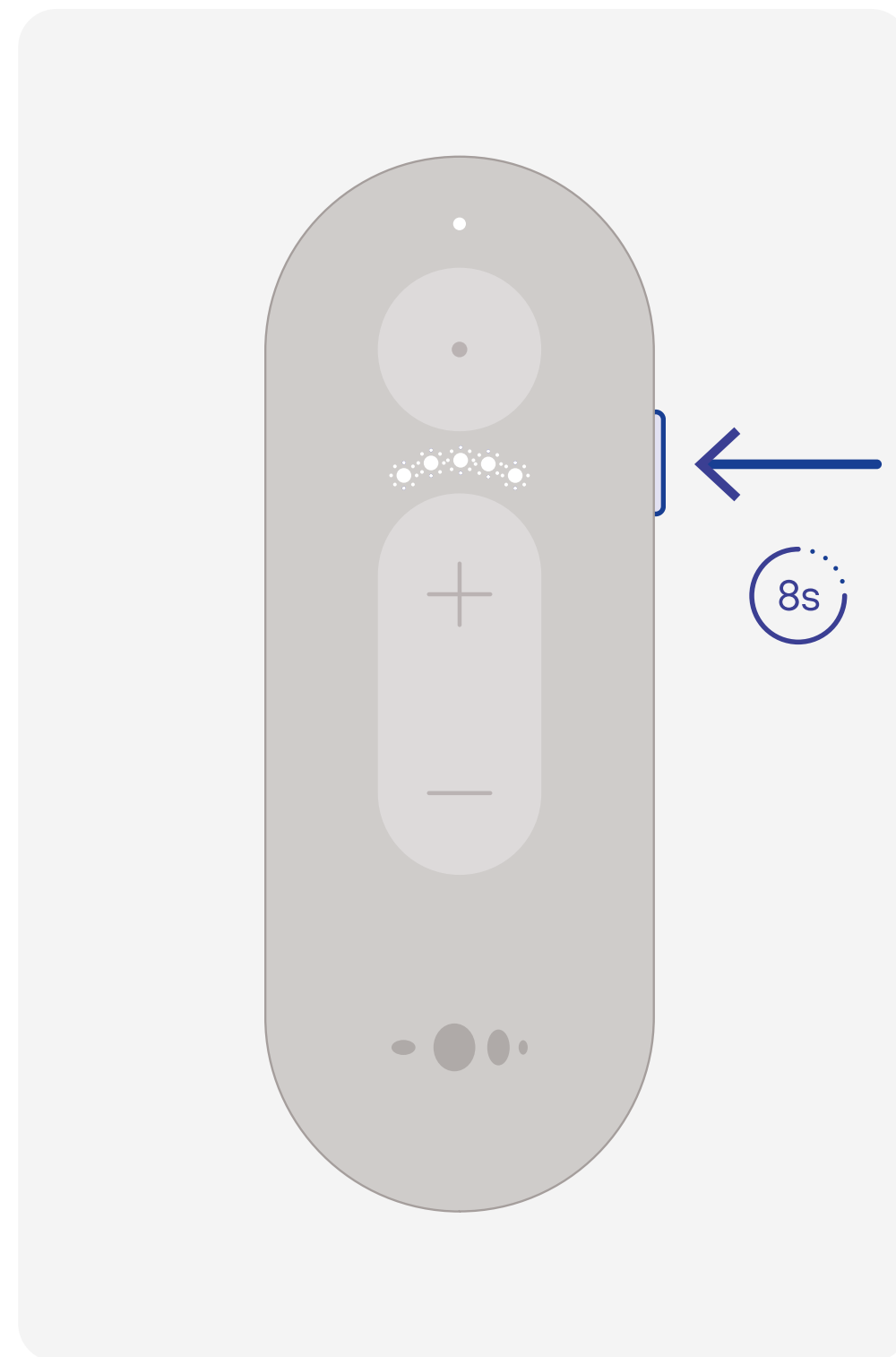

Pairing

Remove the battery tab from the Remote before proceeding. When the Glasses are on, press and hold the "+" or "-" volume button on the frame until blue LED blinks. Then, **press side button** and hold for 8 seconds.

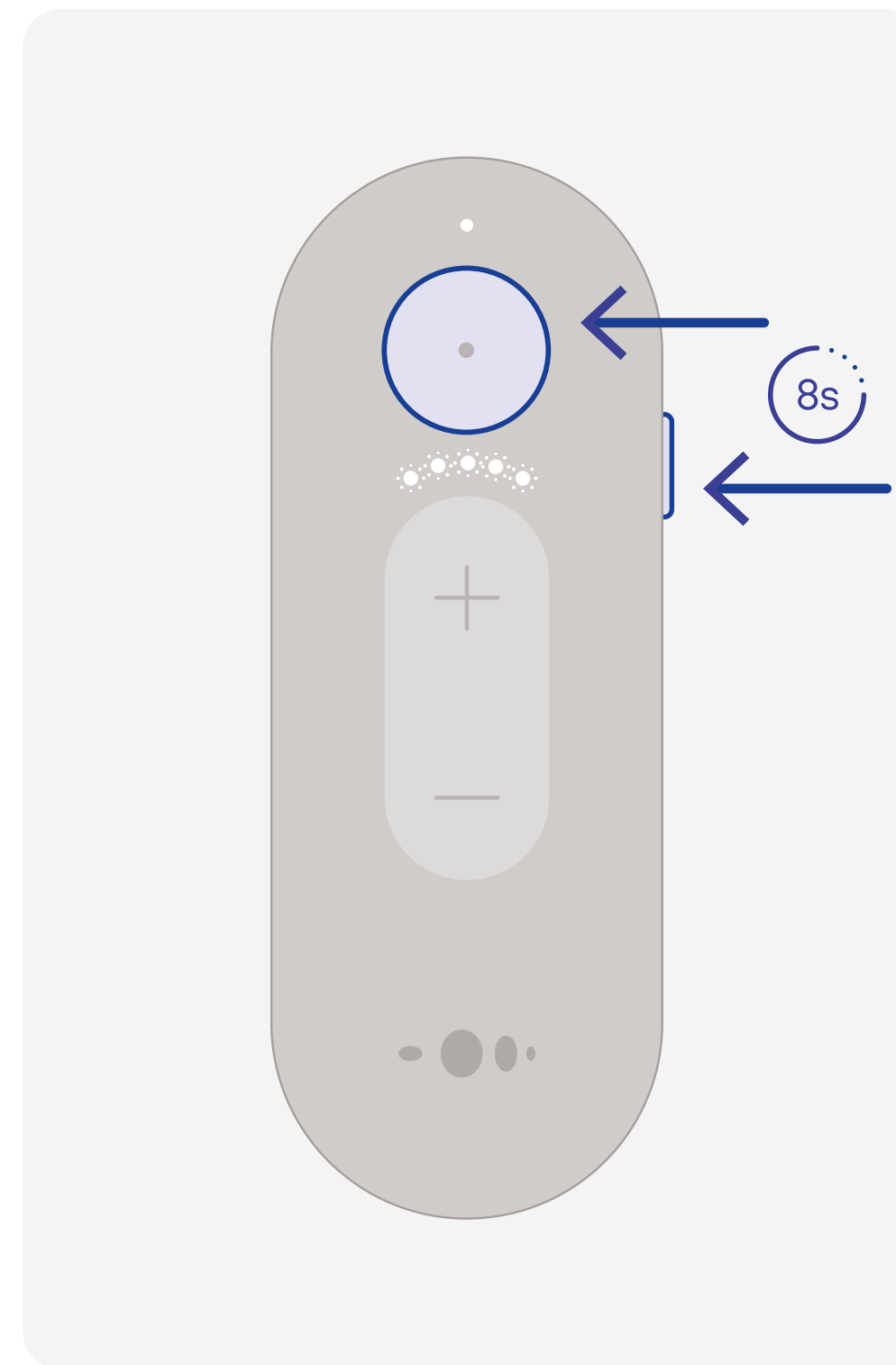


Factory Reset

Press top button and side button. Hold both for 8 seconds.




Remote Plus user interface


Frame Pairing and Set Up

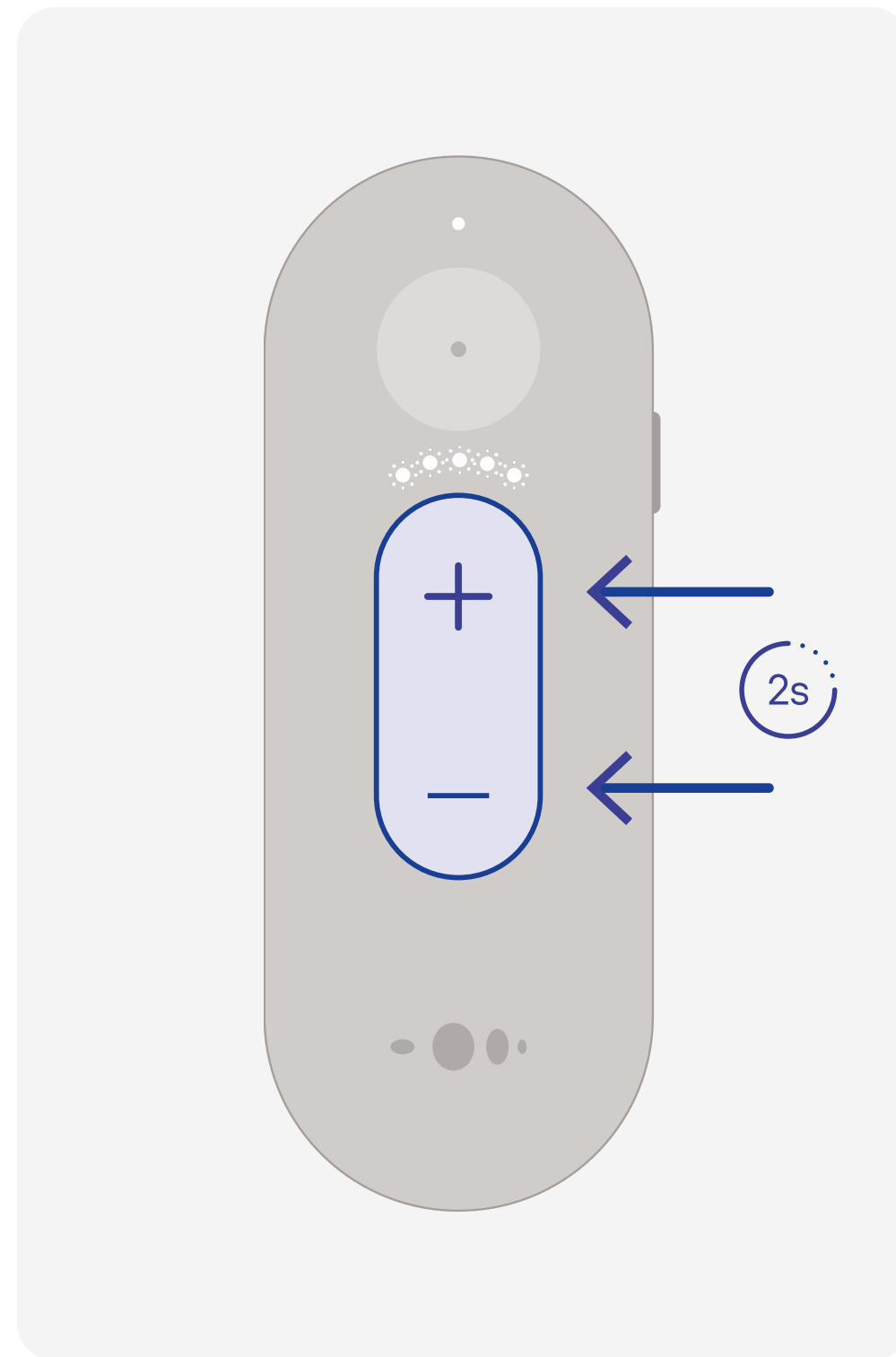

Enter setup mode
Press and hold both
(+) and (-) for 2
seconds.

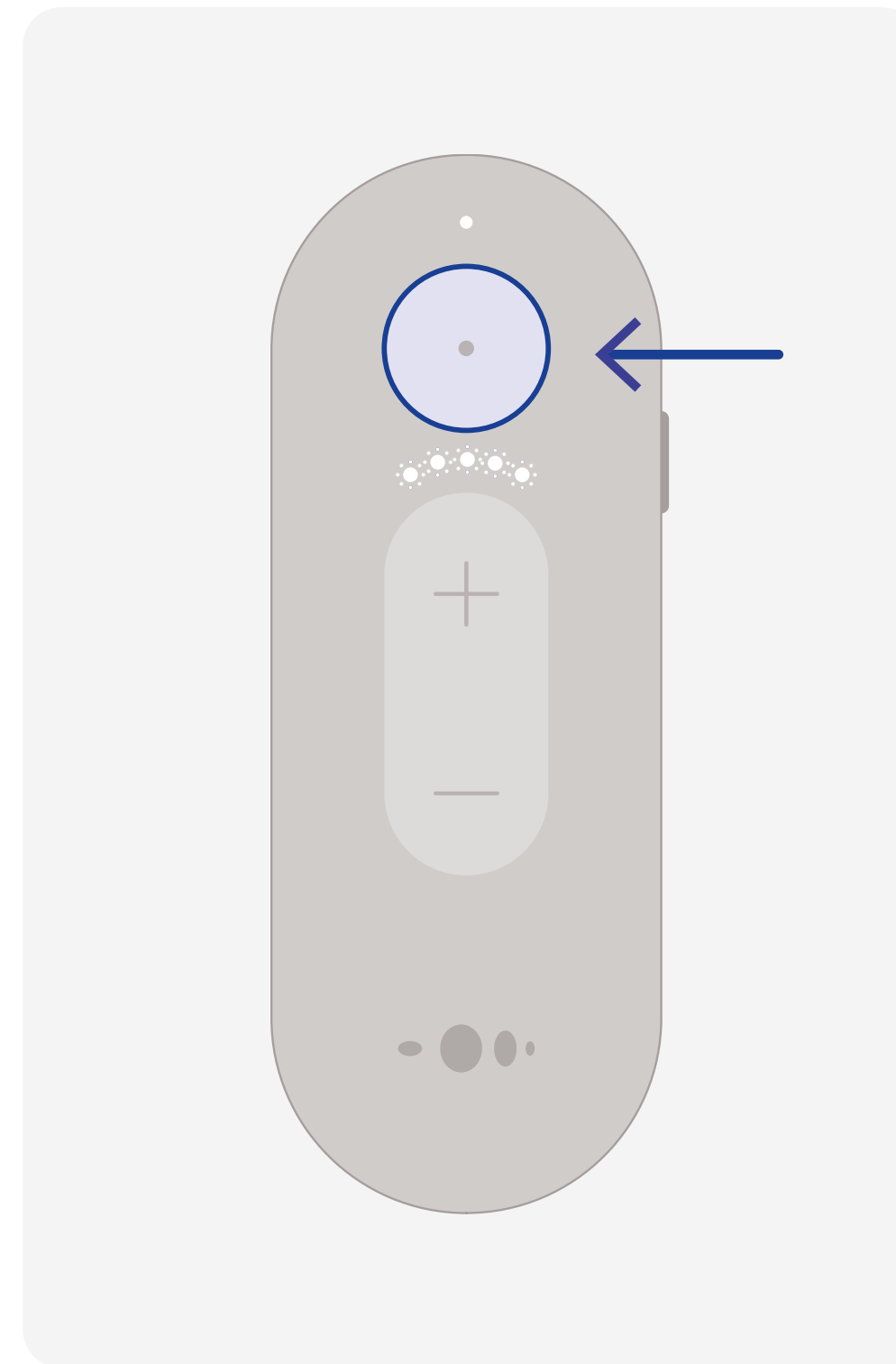

OK
Press top button

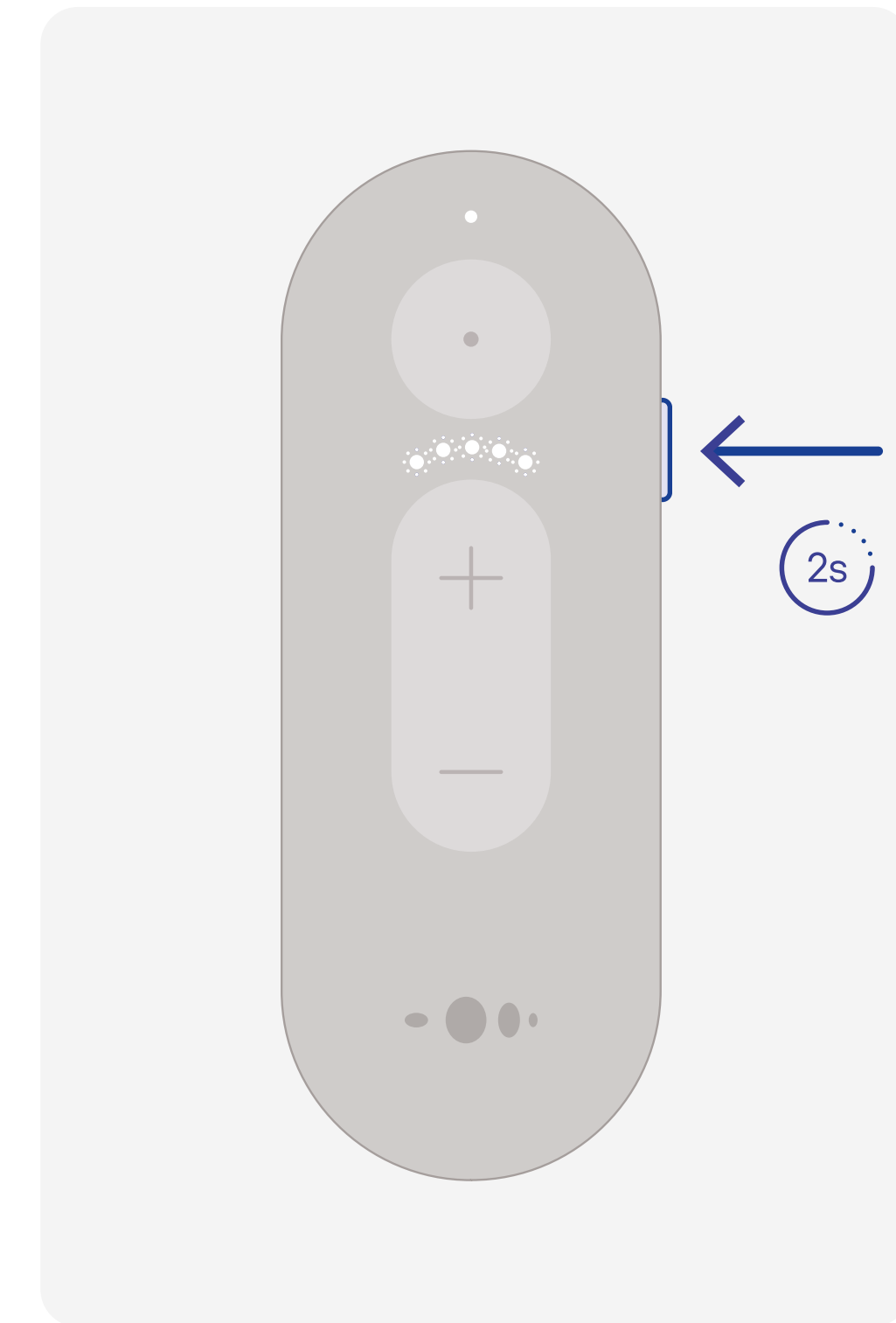


Exit setup
Press side button
and hold for 2
seconds.




Remote Plus user interface

Frame Setup

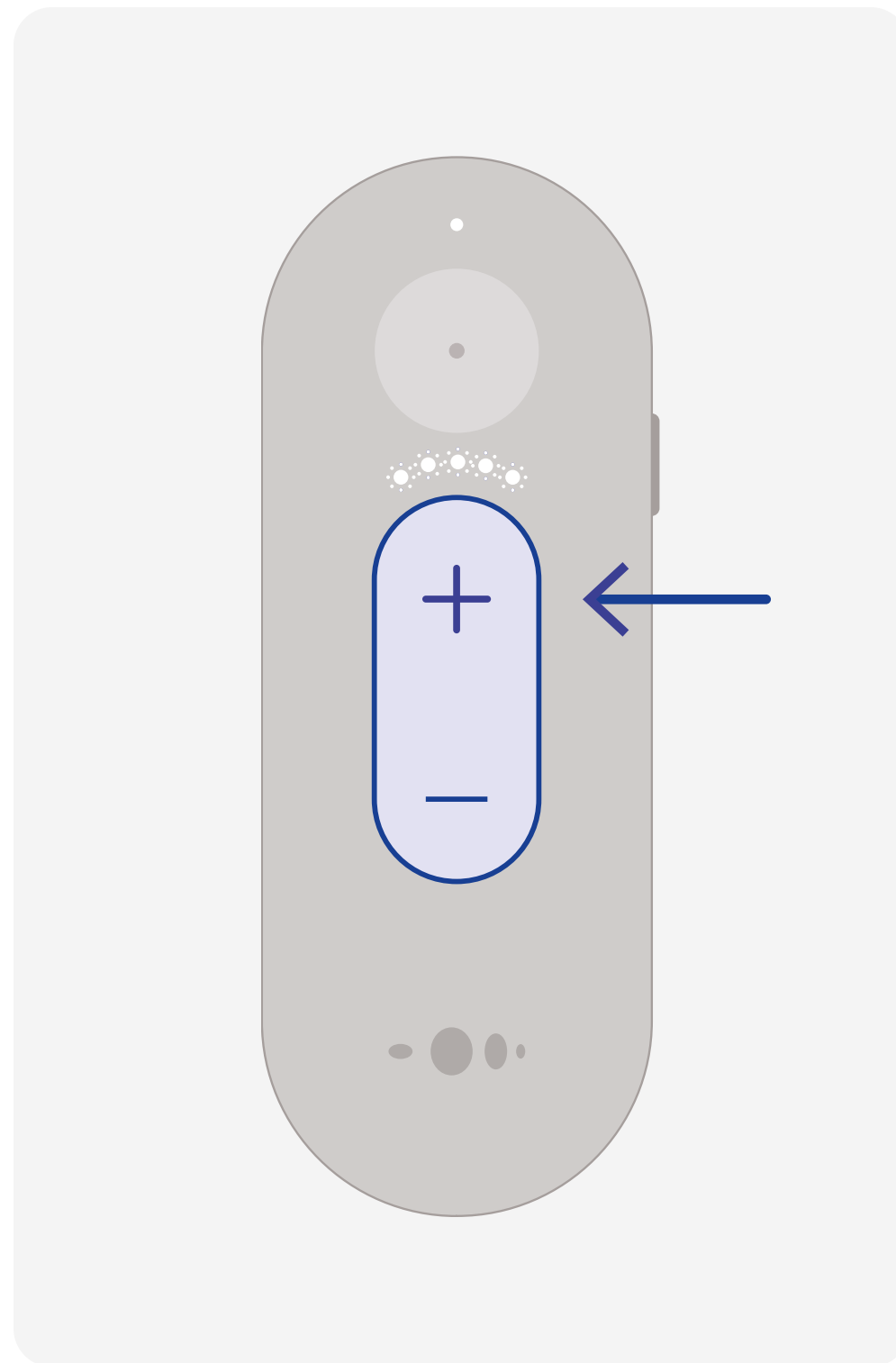

Next language
Press (+)

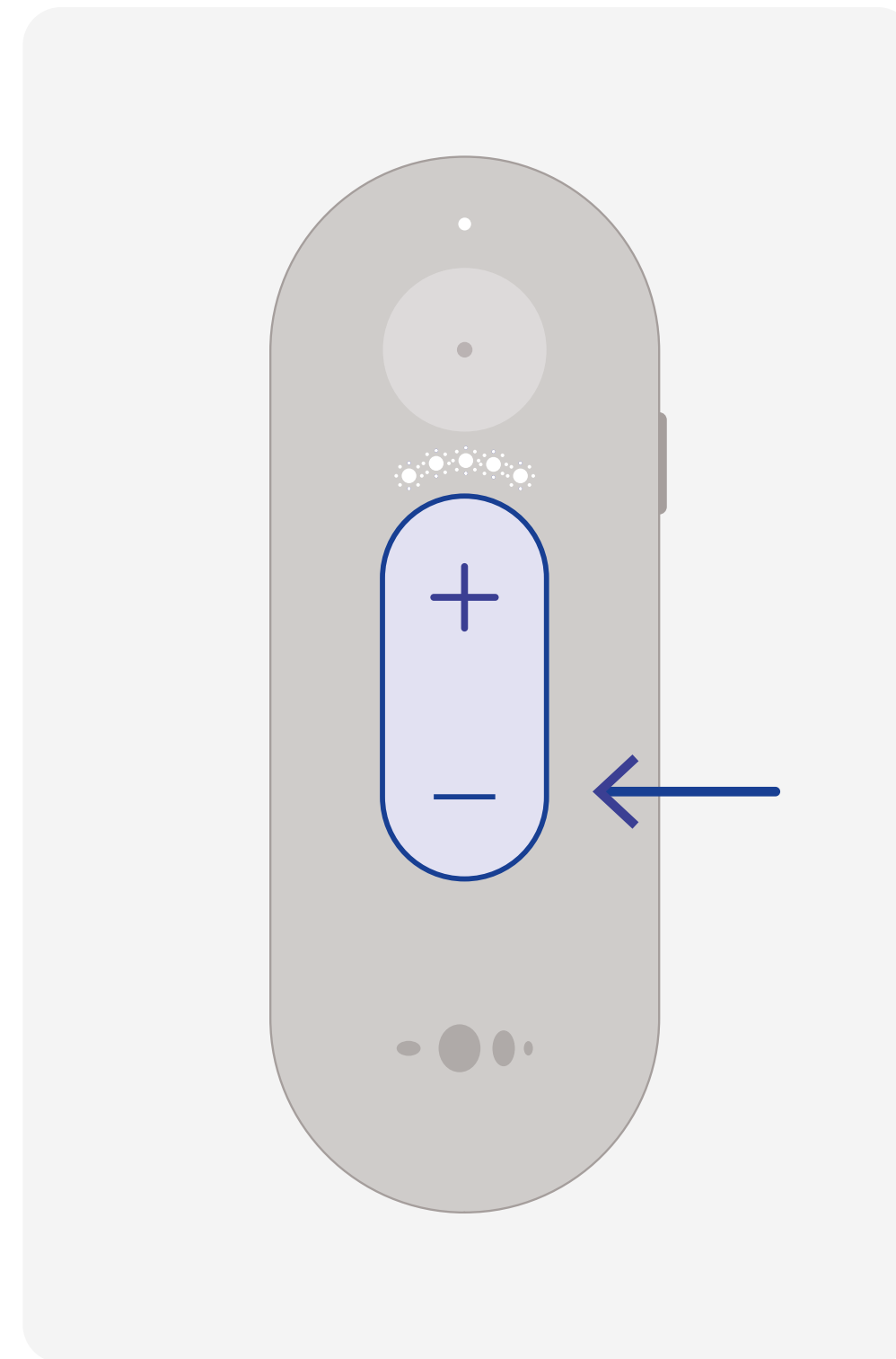


Previous language
Press (-)




Remote Plus interface

Hearing

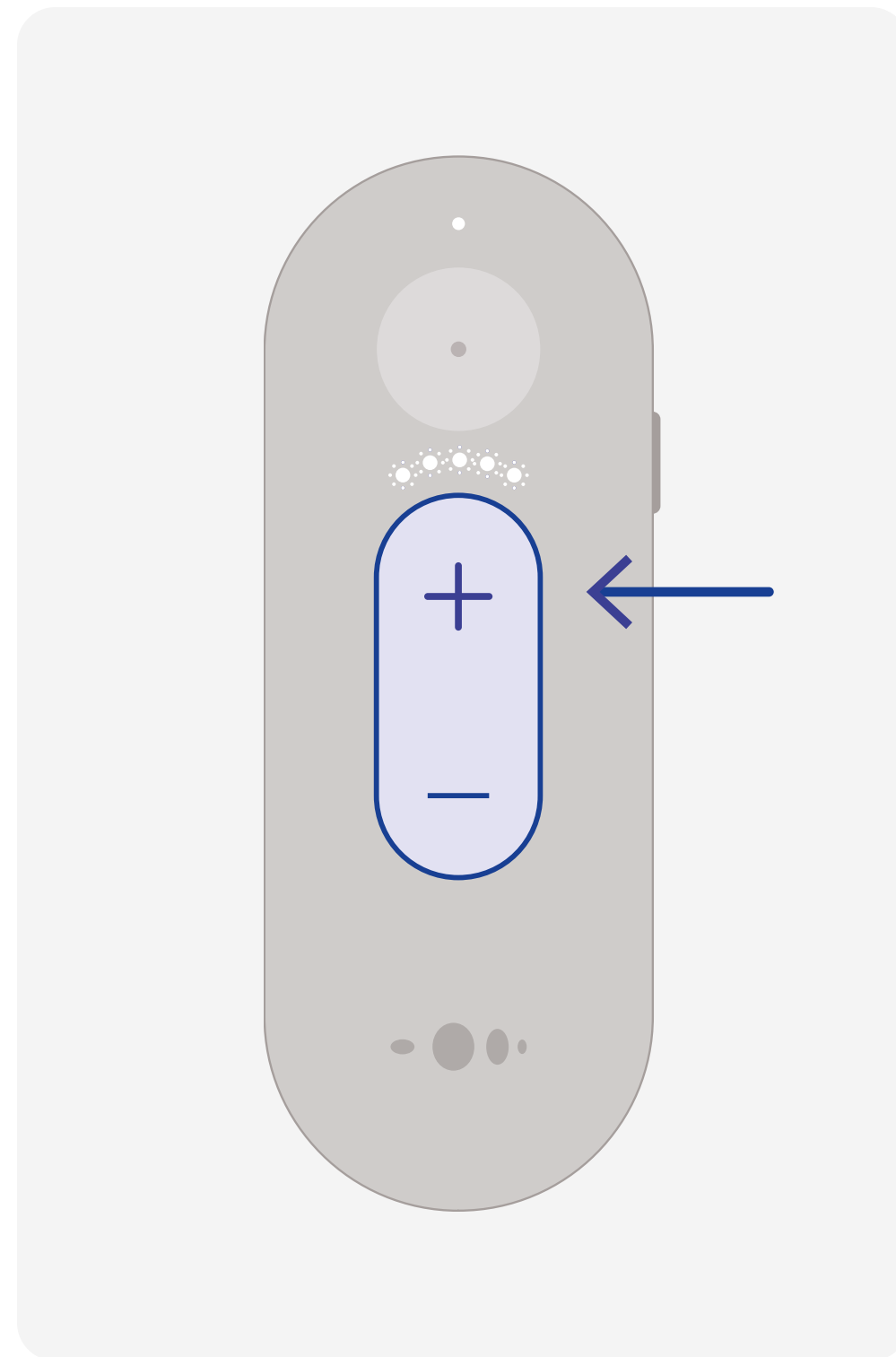

Volume up
Press (+)

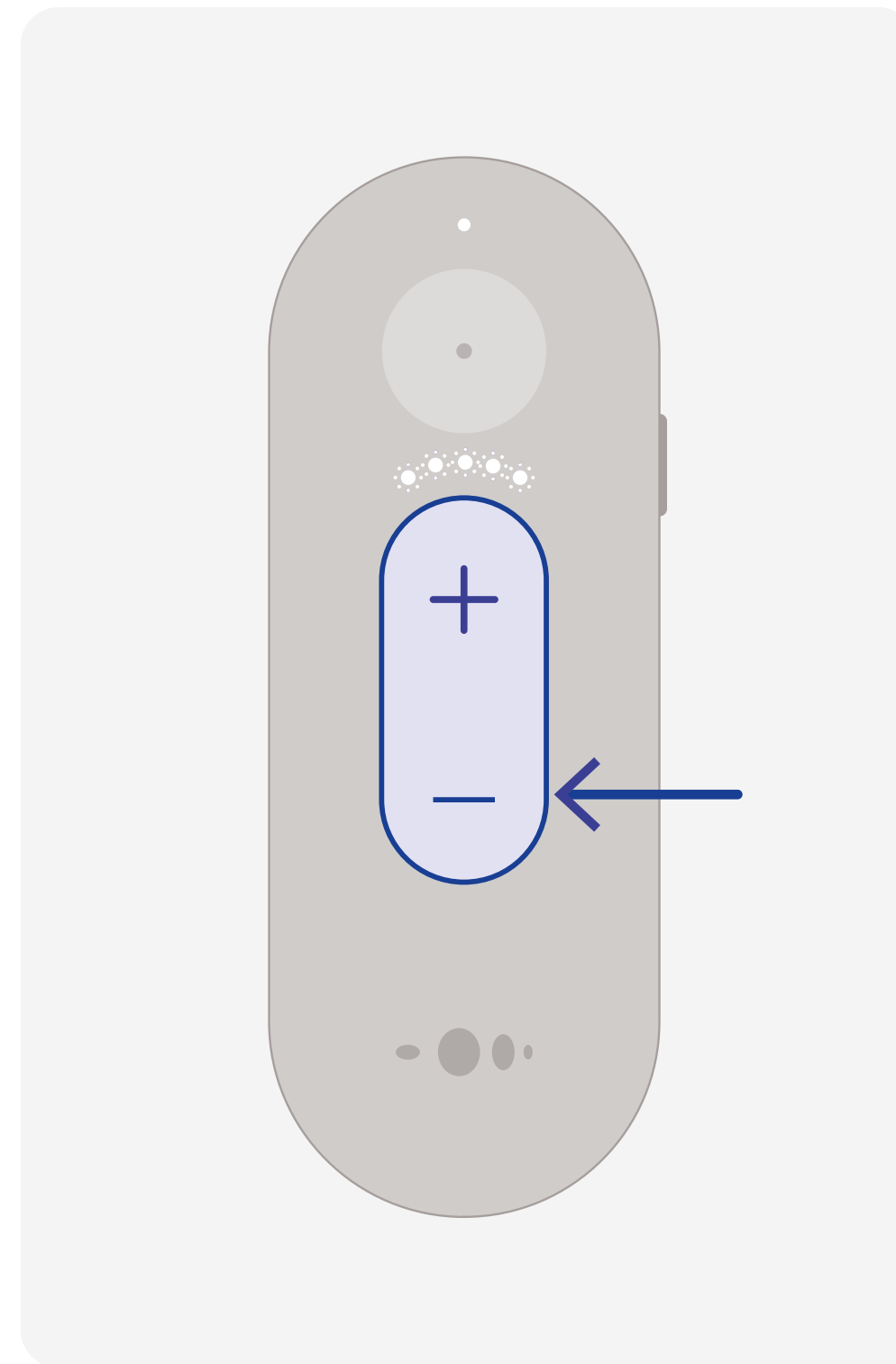


Volume down
Press (-)




Remote Plus user interface

Hearing

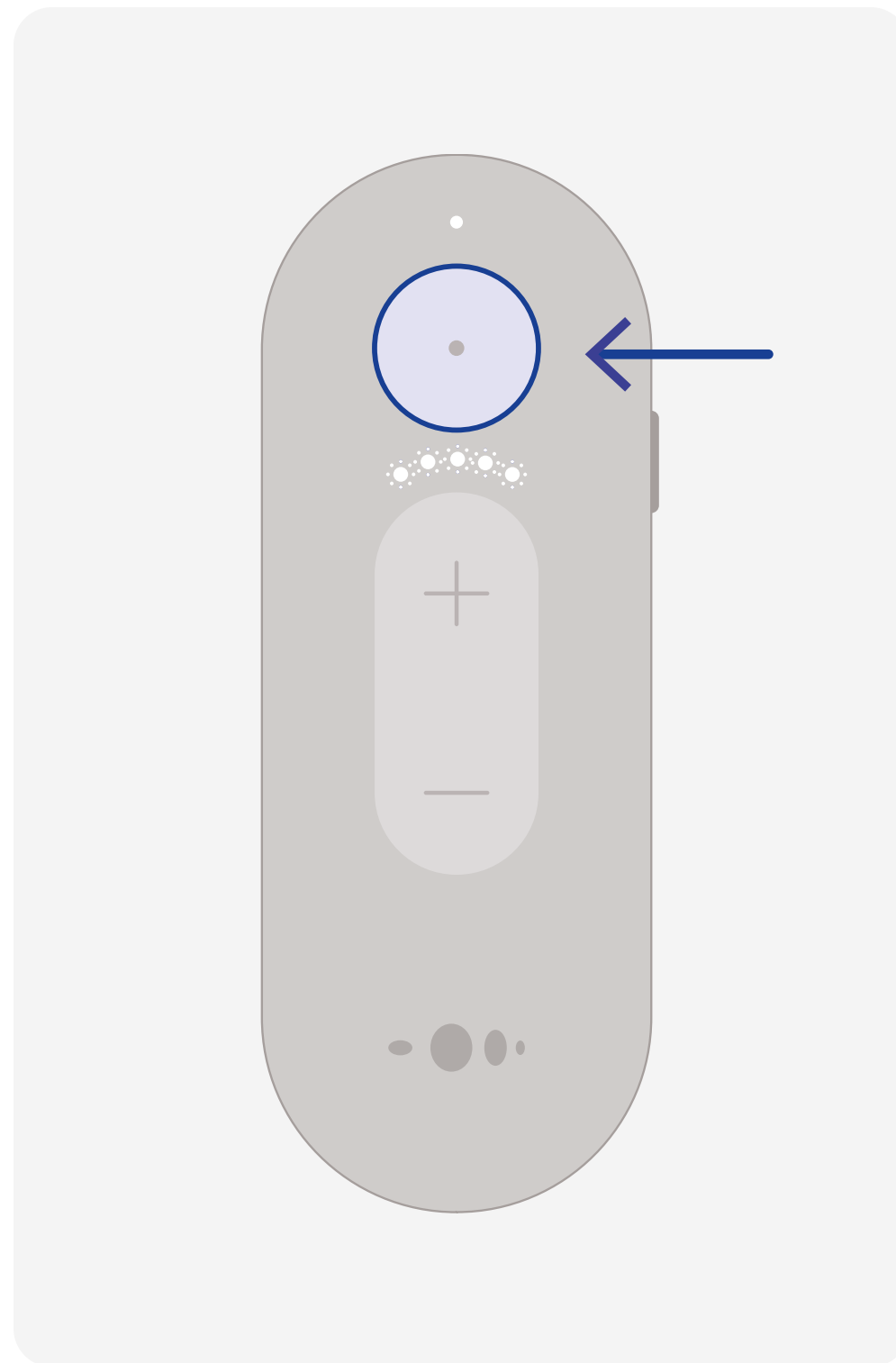

Switch audio mode
Press top button

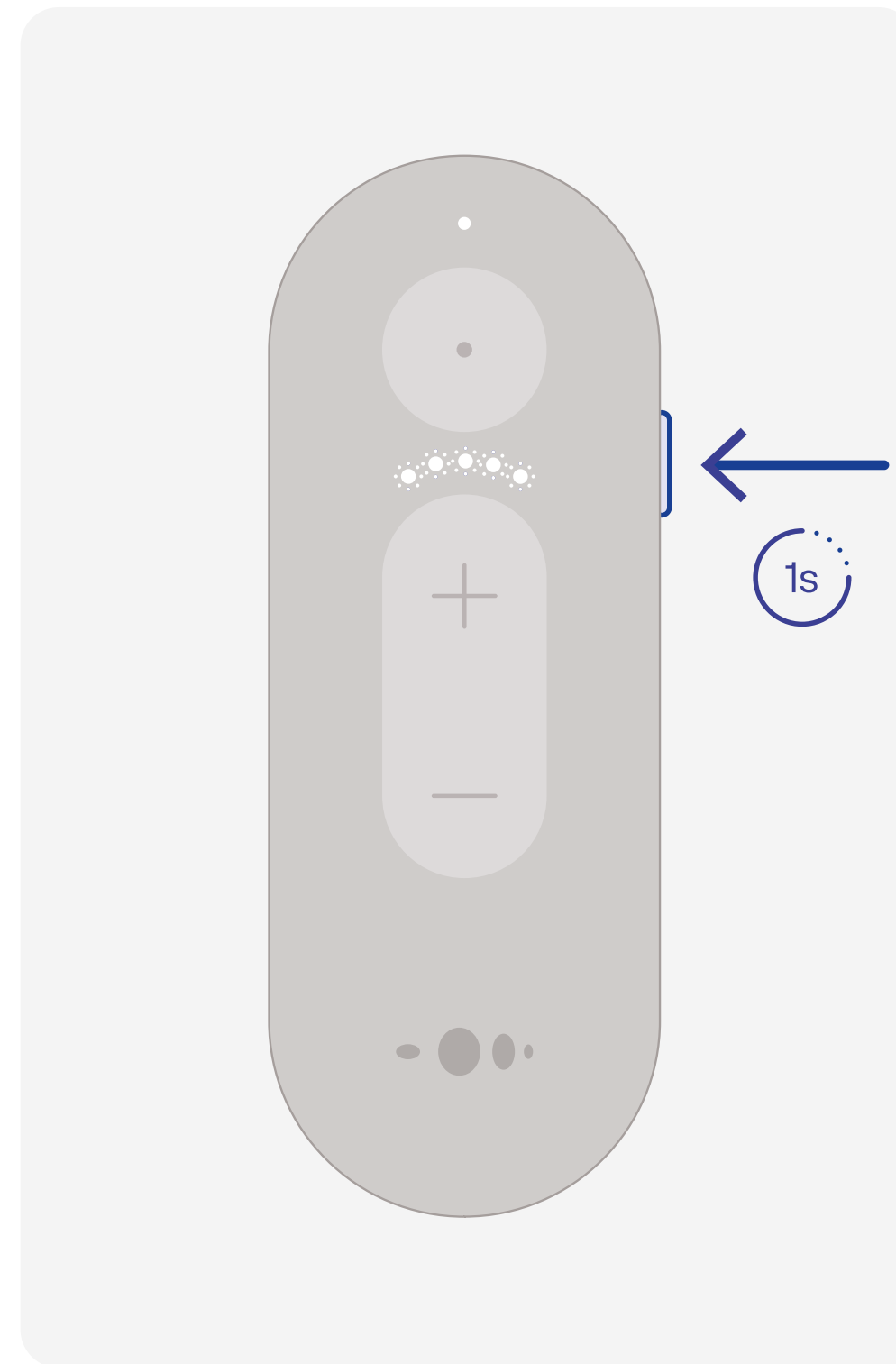


Cycle preset up
Press side-button and hold for 1 second.




Remote Plus user interface

Call

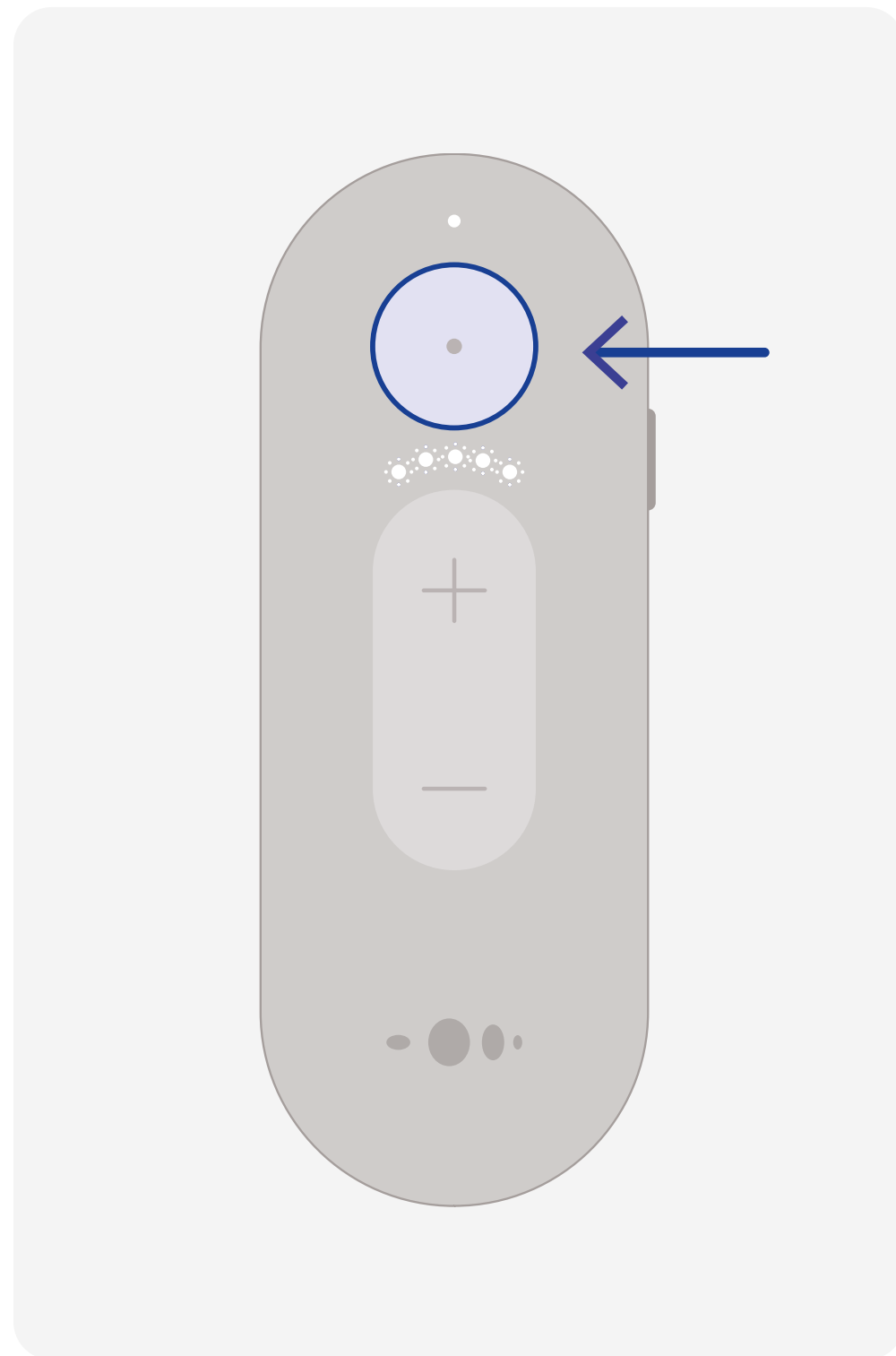

Answer call
Press top button

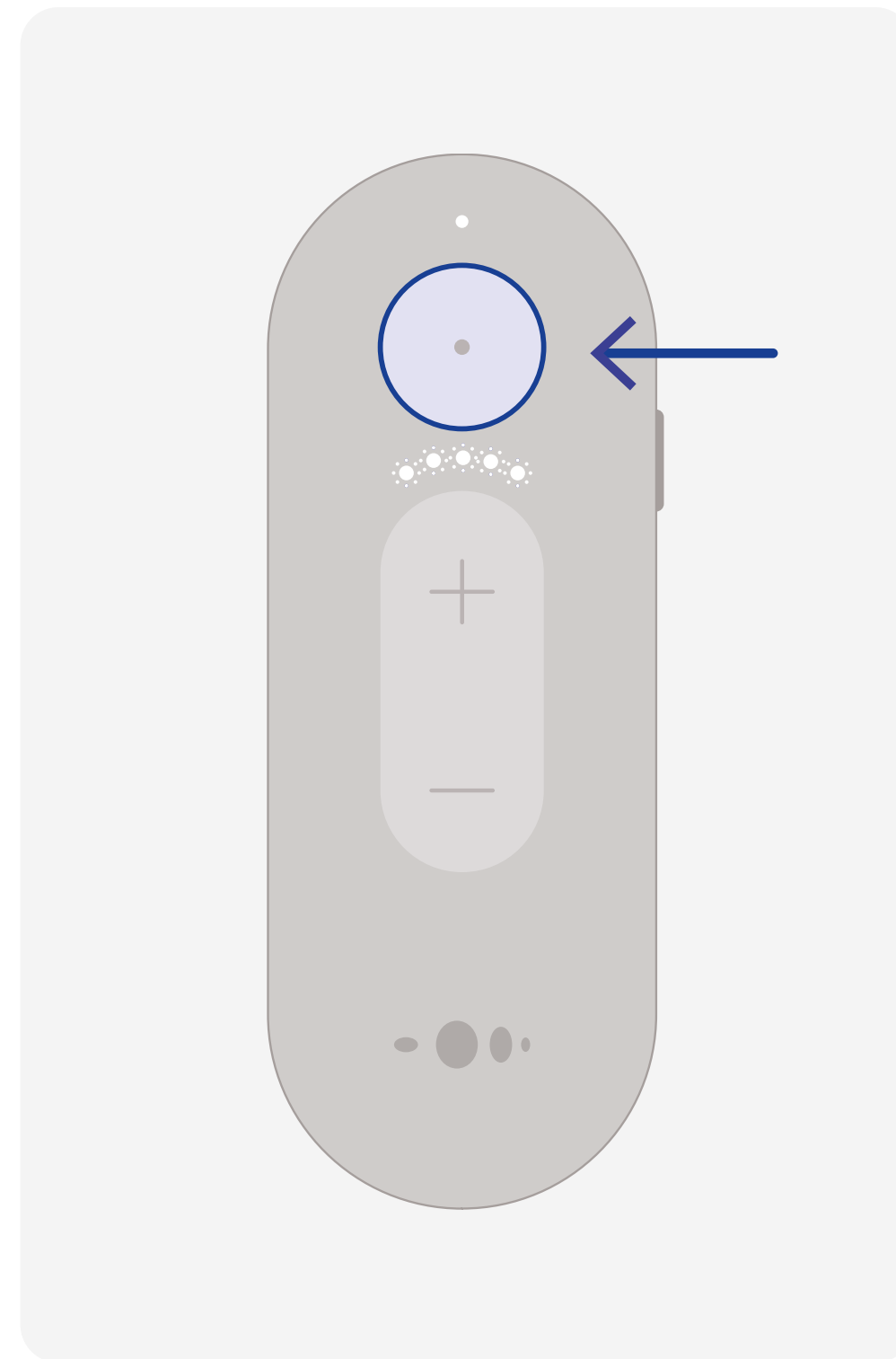

End call
Press top button



Need more support?

Watch the Remote Plus Onboarding video



For the most updated Instructions
for Use please follow this link

[↗ View instruction for use](#)