



Phone calls management



Phone calls management

With Nuance Audio Plus, you can take phone and app-based calls directly from your Glasses, without holding your phone.

Supported calls include standard mobile calls and VoIP apps such as WhatsApp, Telegram, and FaceTime.


Phone Calls are currently available only for Apple iPhones.






Step 1 — iOS Setup (required)


To set up phone calls on iPhone (pt.1)

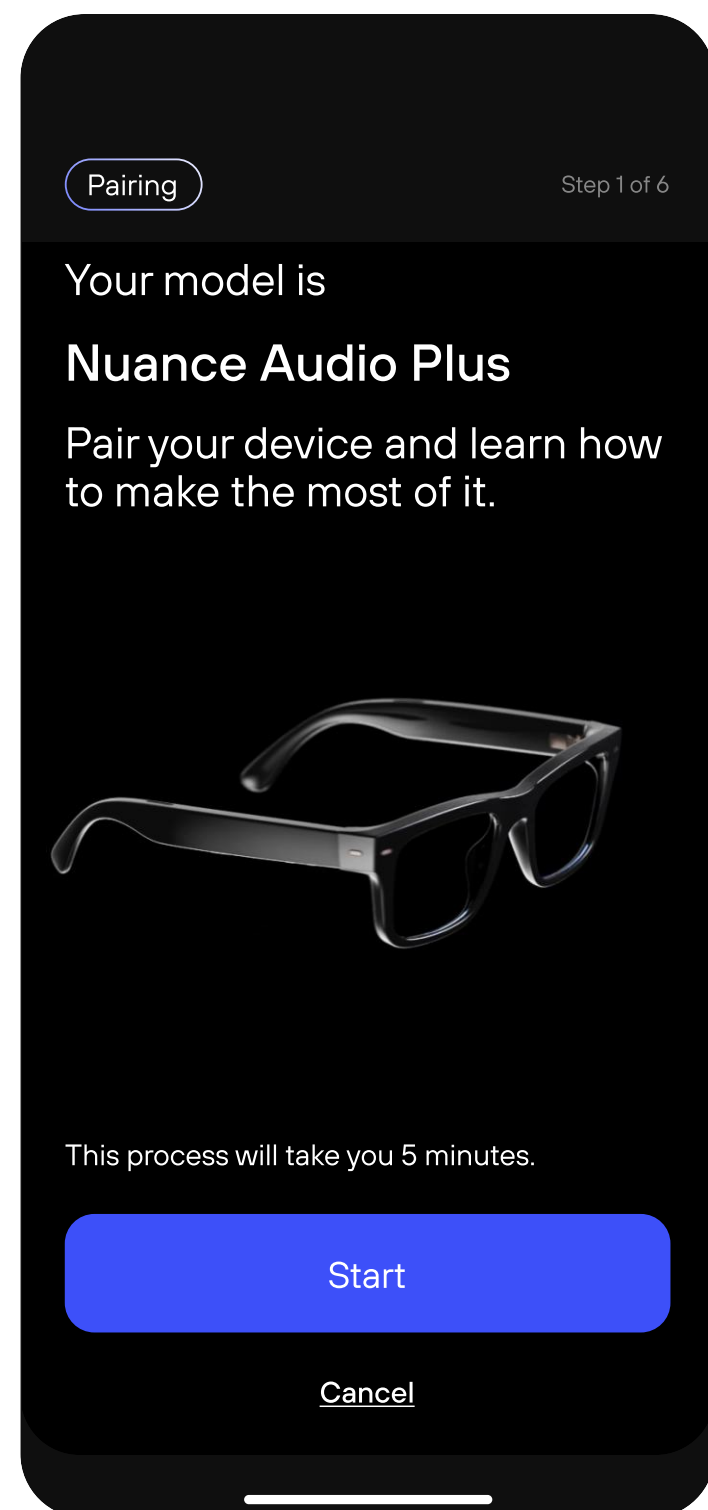

Pair your Nuance Audio Plus with the App.

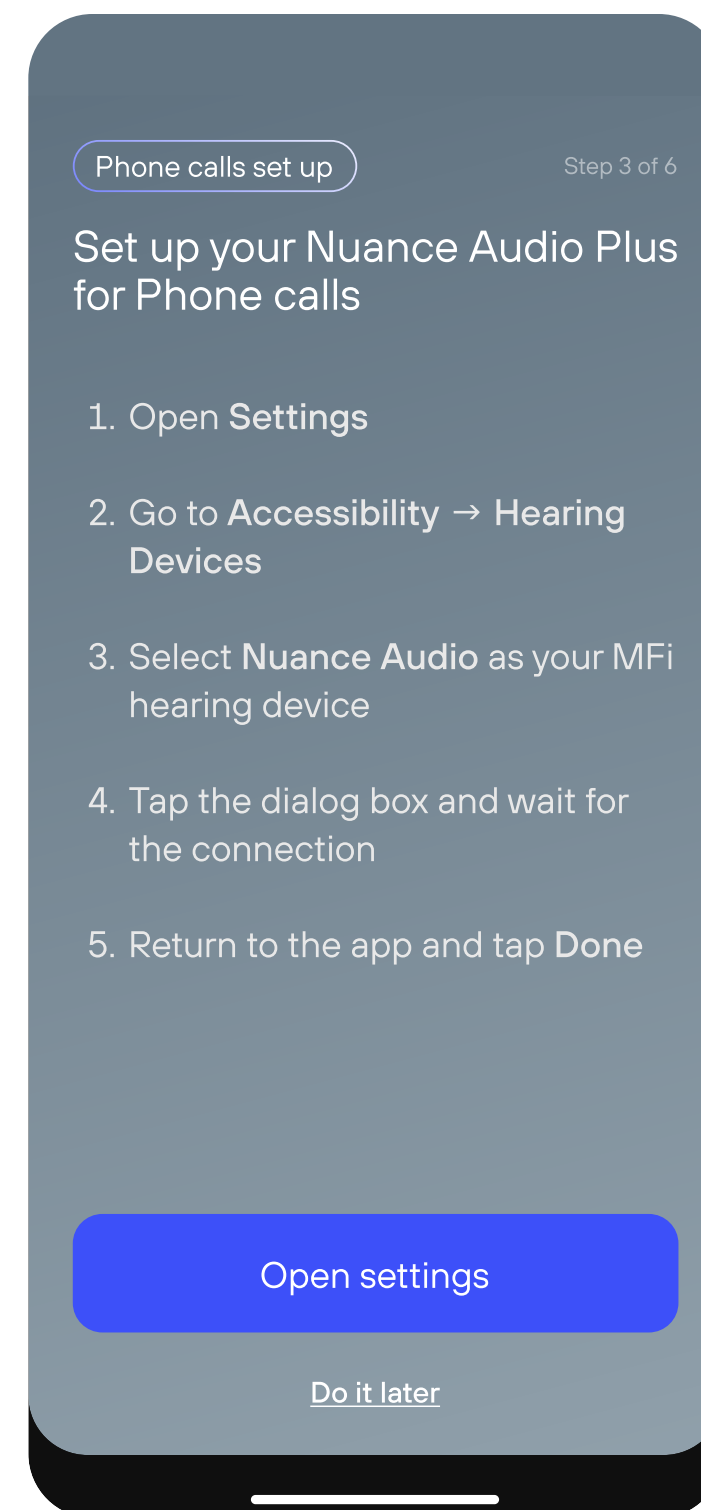

Open **Settings**.

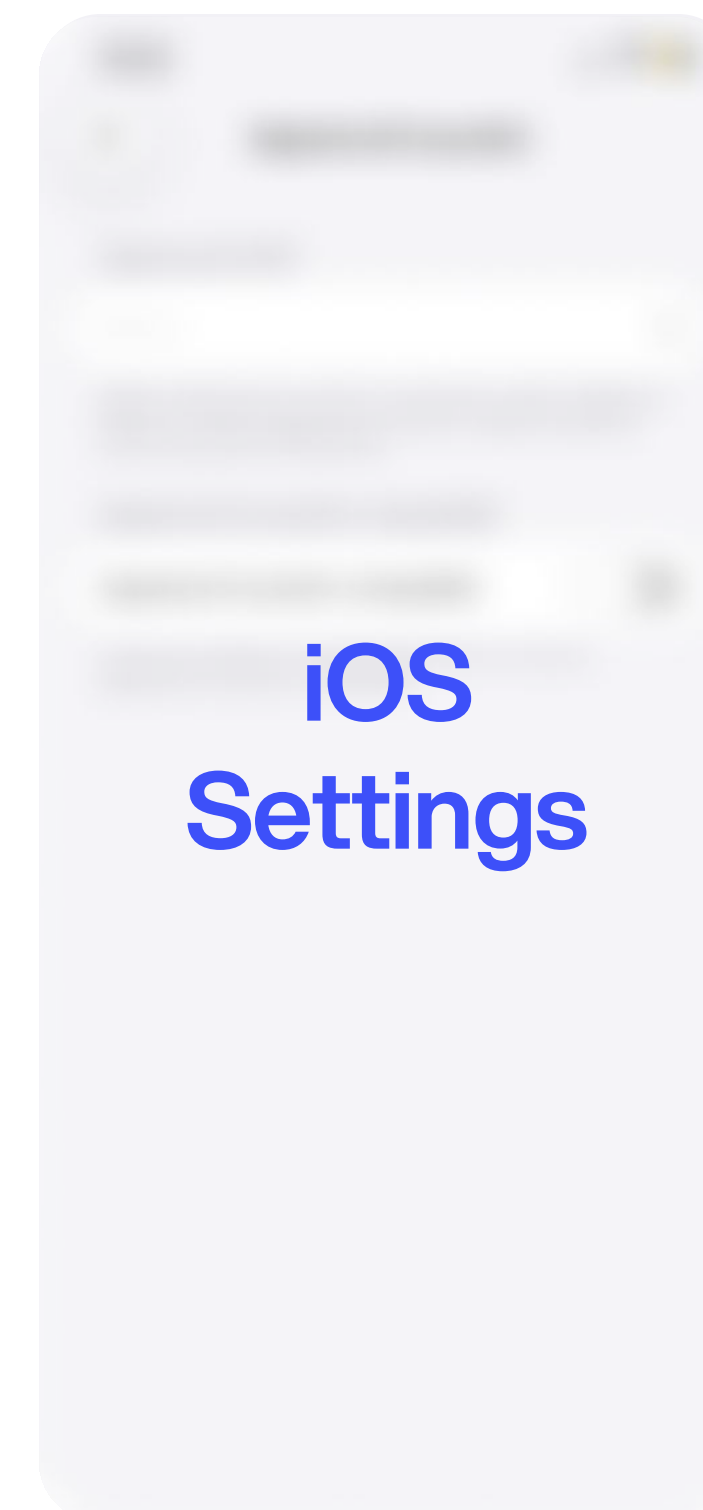


Go to **Accessibility**
> **Hearing Devices**




Step 1 — iOS Setup (required)


To set up phone calls on iPhone (pt.2)

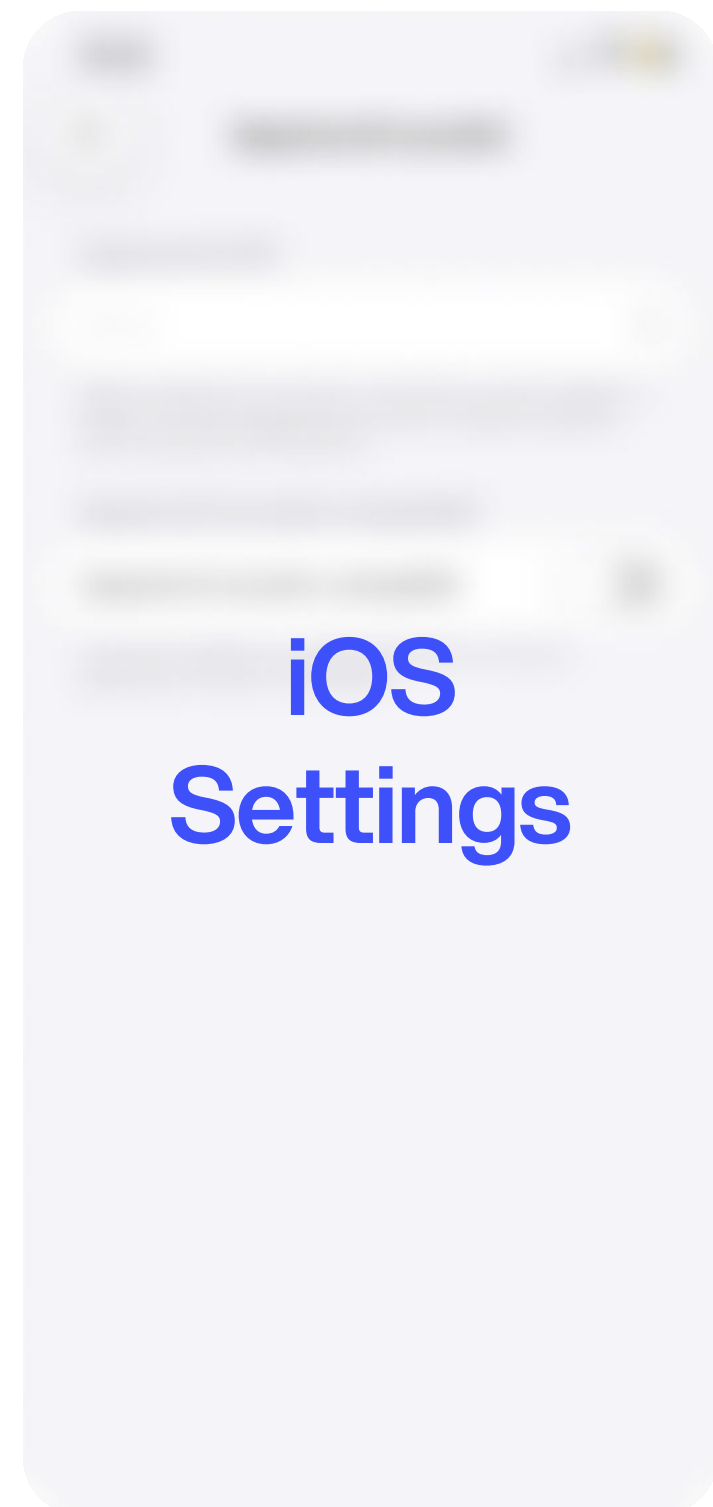

Select **Nuance Audio Plus** as your MFi hearing device.

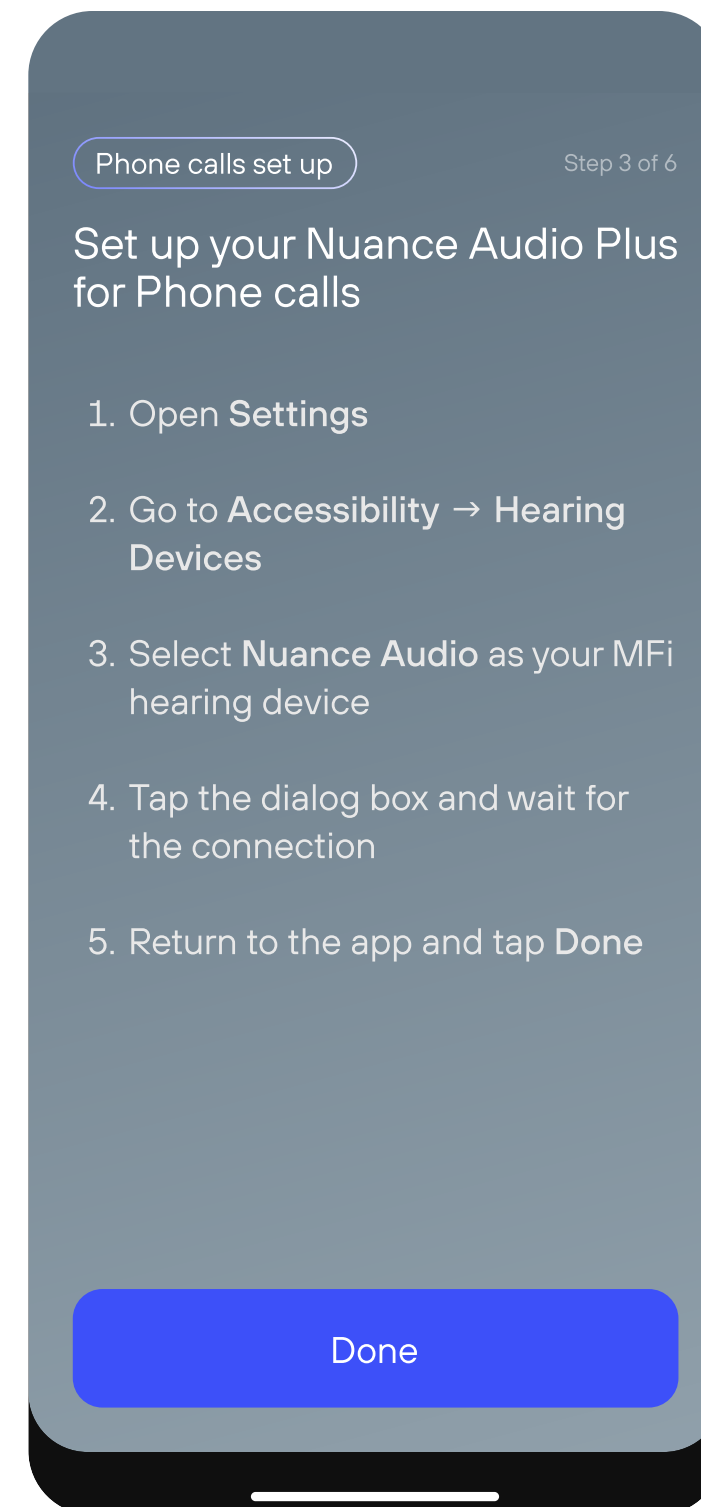

Tap the dialog box and wait for the connection to be completed.

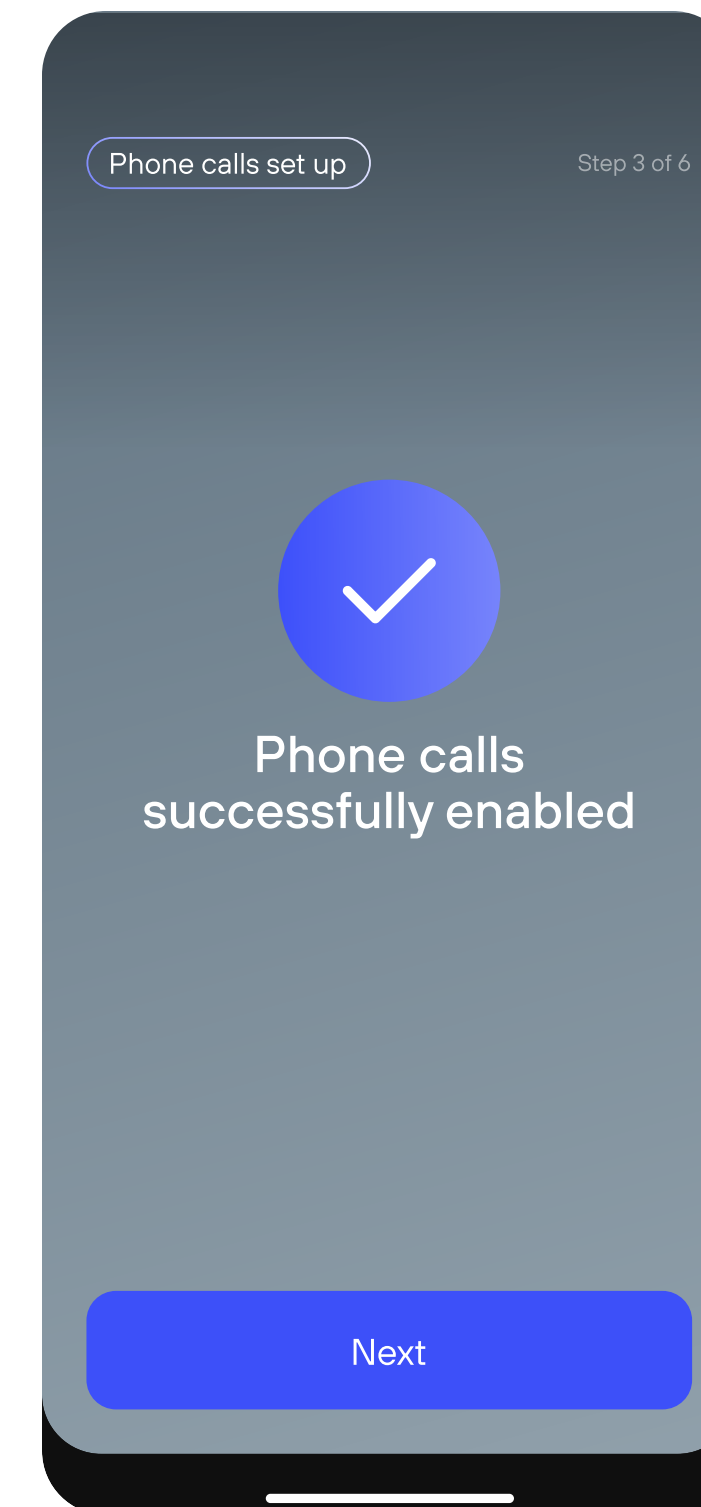


Return to the App and tap "**Done**".




Step 2 — Answer and manage calls

You can manage calls either from your iPhone or directly from the Glasses

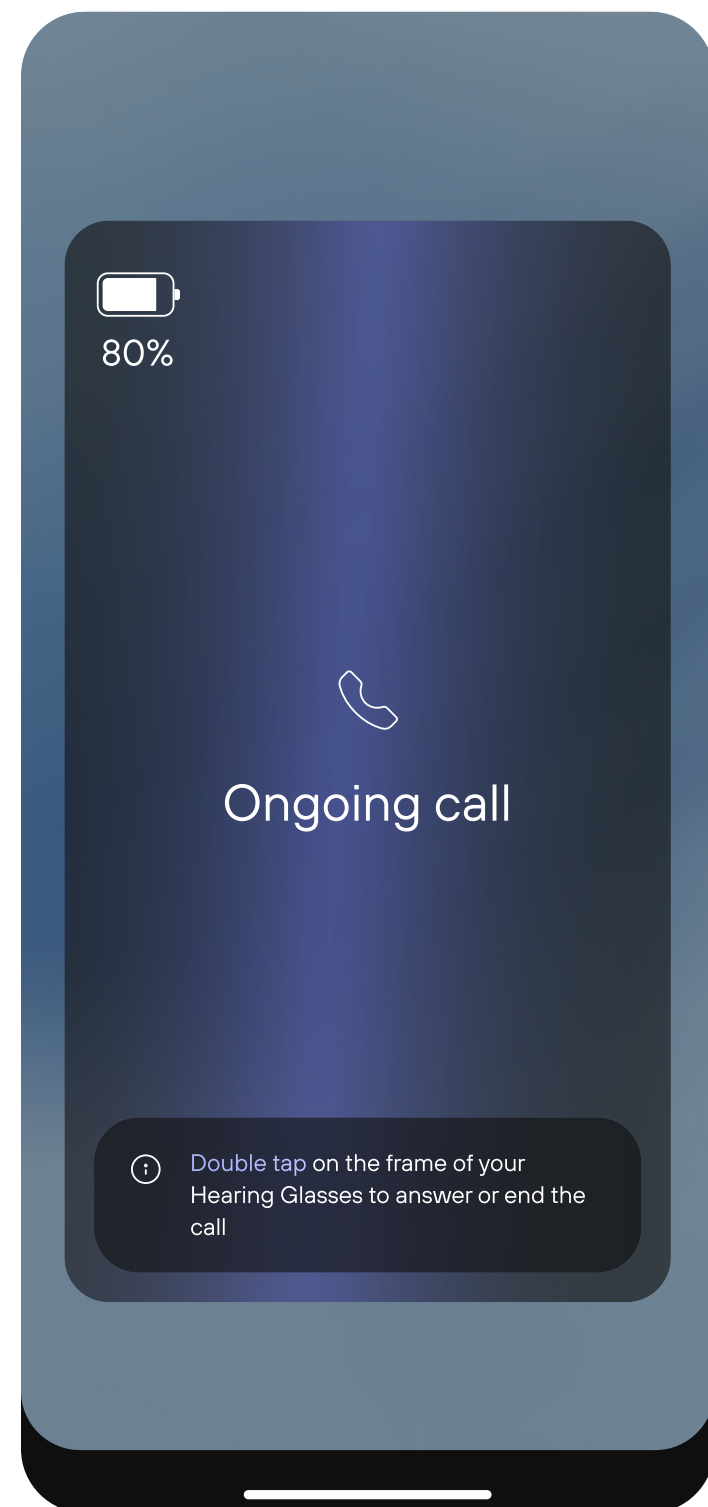

Option A — From your iPhone

Answer or end calls directly from the screen.

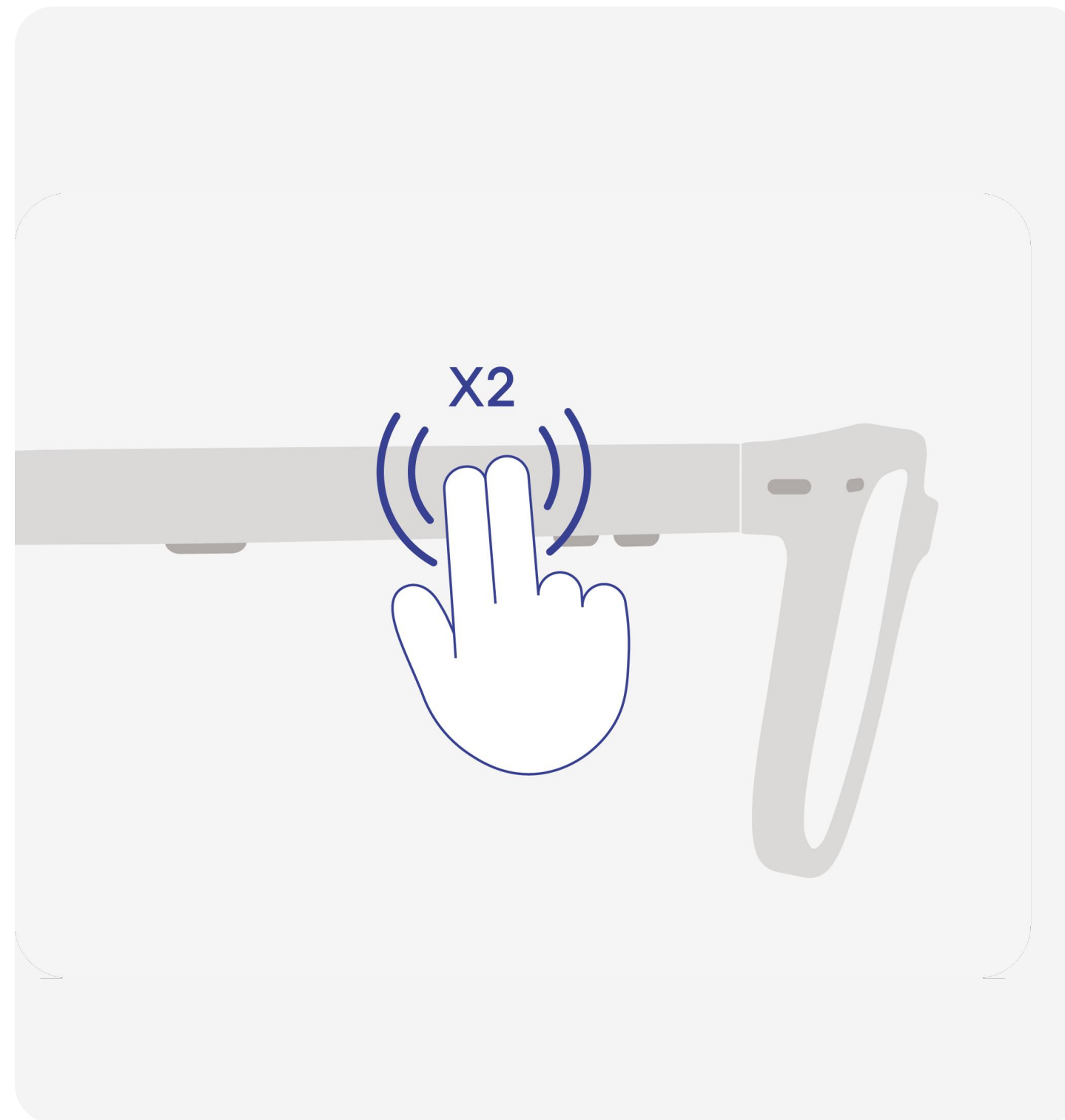

Option B — From the Glasses (Hands-Free)

First enable shortcuts:
Go to **Menu > Settings > Shortcuts**
> Turn on **"Double Tap"**

Then:

- Double tap the right temple to answer a call
- Double tap again to end the call



Additional information

Glasses can be paired with up to 5 phones.

Only **1 phone at a time** can be actively connected.



For the most updated Instructions
for Use please follow this link

[↗ View instruction for use](#)