Nuance Audio Plus

Instructions For Use Safety & Warranty

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FOR USA ONLY

Nuance Audio™ Hearing Aid Software

WARNING: If you are younger than 18, do not use this.

You should go to a doctor, preferably an ear-nose-throat doctor (an ENT), because your condition needs specialized care. Over-the-counter hearing aids are only for users who are age 18 or older.

The Nuance Audio™ Hearing Aid Software is for use by adults who have perceived mild to moderate hearing loss.

How do you know if you have a perceived mild to moderate hearing loss?

- You have trouble hearing speech in noisy places
- You find it hard to follow speech in groups
- You have trouble hearing on the phone
- Listening makes you tired
- You need to turn up the volume on the TV or radio, and other people complain that it is too loud

Some people with hearing loss may need help from a hearing healthcare professional even after using the Nuance Audio™ Hearing Aid Software. How do you know if you need to see a hearing healthcare professional?

- You can't hear speech even if the room is quiet
- You don't hear loud sounds well, for example, you don't hear loud music, power tools, engines, or other very noisy things
- If this hearing aid does not help you enough, ask for help from a hearing healthcare professional
- If your hearing loss makes it hard to hear loud noises, this hearing aid may not be your best choice without help from a professional

WARNING: When to See a Doctor

If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus, or fluid coming out of your ear in the past 6 months
- Your ear feels painful or uncomfortable
- You have a lot of ear wax, or you think something could be in your ear
- You get really dizzy or have a feeling of spinning or swaying (called vertigo)
- Your hearing changed suddenly in the past 6 months
- Your hearing changes: it gets worse then gets better again
- You have worse hearing in one ear
- You hear ringing or buzzing in only one ear

Note: the Nuance Audio™ Hearing Aid Software is compatible with the Nuance Audio™ Plus (smart glasses) that allows you to configure the amplification of the speakers to meet your hearing needs. IT IS RECOMMENDED THAT YOU SEE A DOCTOR, PREFERABLY AN EAR-NOSE-THROAT DOCTOR IF YOU HAVE WORSE HEARING IN ONE EAR BEFORE USING AN OTC HEARING AID.

Caution: The sound output should not be uncomfortable or painful.

You should turn down the volume or remove the device if the sound output is uncomfortably loud or painful. If you constantly need to turn the volume down, you may need to further adjust your Nuance Audio™ Plus preset or volume settings.

Caution: This is not hearing protection.

You should remove this device if you experience overly loud sounds, whether short or long-lasting. If you're in a loud place, you should use the right kind of hearing protection instead of wearing this device. In general, if you would use ear plugs in a loud place, you should remove this device and use ear plugs.

Note: If you remain concerned, consult a professional.

If you try this device and continue to struggle with or remain concerned about your hearing, you should consult with a hearing healthcare professional.

Note: What you might expect when you start using a hearing aid

A hearing aid can benefit many people with hearing loss. However, you should know it will not restore normal hearing, and you may still have some difficulty hearing over noise. Further, a hearing aid will not prevent or improve a medical condition that causes hearing loss. People who start using hearing aids sometimes need a few weeks to get used to them. Similarly, many people find that training or counseling can help them get more out of their devices. If you have hearing loss in both ears, you might get more out of using hearing aids in both, especially in situations that make you tired from listening—for example, noisy environments.

PLEASE REFER TO SECTION 3.3 OF THIS DOCUMENT FOR A MORE COMPLETE LIST OF WARNINGS AND PRECAUTIONS.

Note: Tell the FDA about injuries, malfunctions, or other adverse events associated with your use of the device.

To report a problem involving your hearing aid, you should submit information to FDA as soon as possible after the problem. The FDA calls them “adverse events”, and they might include: skin irritation in your ear, injury from the device (like cuts or scratches, or burns from an overheated battery), suddenly worsening hearing loss from using the device, etc. Instructions for reporting are available at www.fda.gov/Safety/MedWatch, or call 1- 800-FDA-1088. You can also download a form to mail to the FDA.

Table of contents

1.	INDICATIONS	8
	1.1. Intended Use	8
	1.2. Indications For Use	8
	1.3. Important Health and Safety Information	8
	1.4. Intended Use Environment	8
2.	CONTRAINDICATIONS	8
3.	WARNINGS, PRECAUTIONS, AND SAFETY CONSIDERATIONS	8
	3.1. Warnings	9
	3.2. Battery	9
	3.3. Precautions	10
	3.4. Preventing damage and malfunctions to your Nuance Audio™ Hearing Aid and the Nuance Audio™ Charging Pad	10
	3.5. Recommended Use and Storage	11
	3.6. Cybersecurity	11

4.	WHAT'S IN THE BOX	11
5.	OPENING THE BOX	12
6.	YOUR NUANCE AUDIO™ HEARING AID	12
7.	NUANCE AUDIO™ APP SETUP AND BLUETOOTH® PAIRING	12
	7.1. Review Required Labelling	13
	7.2. Onboarding the Nuance Audio™ App	13
	7.3. Application main page	14
	7.4. Menu	15
	7.5. Settings	16
	7.6. Phone Streaming	16
	7.7. About	16
	7.8. Support	17
8.	NUANCE AUDIO™ PLUS CARE AND MAINTENANCE	23
	8.1. Battery Information	23

	8.2. Charging the battery	24
	8.3. The Nuance Audio™ Charging Pad works thanks to a USB-C Charging Cable	24
	8.4. Caring for Nuance Audio™ Plus	24
	8.5. General Maintenance	25
	8.6. General Information	25
9.	PRIVACY	25
10.	TROUBLESHOOTING	25
	10.1. Common Solutions for the App	25
	10.2. Other Troubleshooting Tips	26
11.	HEARING AID TECHNICAL SPECIFICATIONS	27
12.	INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS	30
13.	REGULATORY COMPLIANCE INFORMATION	30
	13.1. FCC compliance statement	30
14.	SYMBOLS AND DESCRIPTIONS	31

WARNING To reduce the risk of personal injury, discomfort, or property damage, please read and follow the health and safety information below before using the Nuance Audio™ Hearing Aid.

1. INDICATIONS

1.1. INTENDED USE

The Nuance Audio™ Hearing Aid is a software-only mobile medical application that is intended to be used with a compatible wearable electronic computing platform. The Nuance Audio™ Hearing Aid Software is intended to amplify sounds for individuals 18 years of age or older with perceived mild to moderate hearing loss. The Nuance Audio™ Hearing Aid Software also allows for user customization to suit their perceived hearing needs through utilizing 4 pre-set fitting options based on typical hearing loss profiles of a mild to moderate hearing loss (Urbanski et al, 2021). The presets are selected subjectively by the user to meet their hearing needs without the assistance of a hearing healthcare professional. The Nuance Audio™ Hearing Aid Software device is intended for over-the-counter (OTC) use.

1.2. INDICATIONS FOR USE

The Nuance Audio™ Hearing Aid Software is a software-only mobile medical application that is intended to be used with compatible wearable electronic products. The feature is intended to amplify sound for individuals 18 years of age or older with perceived mild to moderate hearing loss. The software utilizes a preset fitting strategy and is adjusted by the user to meet their hearing needs without the assistance of a hearing healthcare professional. The device is intended for Over-the-Counter (OTC) use.

NOTE: Nuance Audio™ Plus in these instructions refers to the HW platform together with the hearing aid functionality enabled software.

1.3. IMPORTANT HEALTH AND SAFETY INFORMATION

It is very important to read and follow this Instructions for Use, the User Guide, and all the safety, warnings and regulatory information thoroughly and completely before using the Nuance Audio™ Plus, for safe and effective operation follow safety instructions to avoid damage or injury. Follow also all instructions provided through the Nuance Audio™ App.

The Nuance Audio™ Plus are designed for adult patients, without cognitive disability preventing the use of them, both lay users and professional users.

These Nuance Audio™ Plus are may help you hear better if you strain to follow conversations when others don't, especially in noisy places.

1.4. Intended Use Environment.

The Nuance Audio™ Plus is intended to be used for listening to the sounds in a variety of environments (e.g., home, office, transit, social settings, outdoors, etc.). The device should not be used while bathing or swimming (i.e., hearing aids should not be submerged in water).

2. CONTRAINDICATIONS

The Nuance Audio™ Plus may not be right for you if you: Consistently experience feedback (whistling) of the hearing aid when you turn up the Volume to a level that is comfortable for you, even after you have followed the suggestions in these Instructions for Use to make sure your hearing aid fits properly. Feel the hearing aid is not providing enough amplification even at the highest Volume setting.

WARNING: When to See a Doctor

If you have any of the problems listed below, please see a doctor, preferably an ear-nose- throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident
- You saw blood, pus, or fluid coming out of your ear in the past 6 months
- Your ear feels painful or uncomfortable
- You have a lot of ear wax, or you think something could be in your ear
- You get really dizzy or have a feeling of spinning or swaying (called vertigo)
- Your hearing changed suddenly in the past 6 months
- Your hearing changes: it gets worse then gets better again
- You have worse hearing in one ear
- You hear ringing or buzzing in only one ear

3. Warnings, Precautions, and Safety Considerations

For safe and effective operation of the Nuance Audio™ Plus, please familiarize yourself with the following information. Please read and keep all safety and use instructions.

NOTE: the Nuance Audio™ Plus are designed for personal use

3.1. Warnings

Over-the-counter hearing aids are only for users who are age 18 or older.

- Remove the Nuance Audio™ Plus if you encounter overly loud sounds or if they become uncomfortable.
- The Nuance Audio™ Plus and the Charging Pad contain magnetic material and components that emit radio waves, which could affect the operation of medical devices such as but not limited to pacemakers or insulin pumps, or other implantable devices, consult your doctor or the medical device manufacturer before using the Nuance Audio™ Plus and the Charging Pad.
- If you have an active or inactive implant device (e.g. defibrillator, pacemaker), follow the implant manufacturer's instructions regarding the use of mobile devices. The Nuance Audio™ Plus should be kept at least 6 inches from implantable pacemakers. In the event of malfunctions, stop using your Nuance Audio™ Glasses and contact your doctor or the implant manufacturer. If you have an active brain implant, ask your doctor or the implant manufacturer for a risk analysis.
-  The Nuance Audio™ Plus are MR Unsafe and could present as a projectile hazard.
-  Check before using the Nuance Audio™ Plus and the Charging Pad in areas where electronics or wireless devices are restricted. This may cause interference.
- **Do NOT** resell the Nuance Audio™ Plus or share it with others. It is intended for use by a single user.
- Keep the Nuance Audio™ Plus and the Charging Pad away from fire, heat sources and excessive heat (e.g., from storage in direct sunlight, near candles, or in a hot car) to avoid burns or damage to The Nuance Audio™ Plus and/or the Charging Pad.
-  Do NOT submerge in water or liquids, wear in the shower, or wear while participating in water sports (e.g., swimming). Prolonged exposure to water or liquids may damage the Nuance Audio™ Plus. The Nuance Audio™ Plus are not designed to resist extended exposure to water or other liquids. In case of water exposure, dry the Nuance Audio™ Plus thoroughly.
- Nuance Audio™ Plus level of protection against dust and water is IP54. IP54 rating offers protection against dust in quantities not sufficient to compromise the operation of the equipment and IPX4 rating offers protection against splashes of water.
- Do not submerge the Charging Pad in water and keep it away from liquids. Exposure to water or

other liquids may damage the Charging Pad and lead to injury. It is not water resistant and is not designed to resist submersion or exposure to other liquids. In case of water exposure, dry the Charging Pad thoroughly.

- When receiving a phone call, the hearing aid's primary amplification function is temporarily disabled.
- In this mode, the hearing aid streams the phone call audio directly, but it does not amplify surrounding environmental sounds.
- This may reduce your awareness of ambient sounds.
- Once the call ends, the amplification function will be automatically restored.
- For use in critical environments or situations requiring full auditory awareness, avoid using the call streaming mode.

When to see an Ear, Nose, and Throat doctor (ENT)

- If you have any of the problems listed below, please see a healthcare professional.
- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus or fluid coming out of your ear within the past 6 months.
- Your ear feels painful or uncomfortable
- You have a lot of earwax or you think something could be in your ear
- You get dizzy or have a feeling of spinning or swaying (called vertigo)
- Your hearing changed suddenly in the past 6 months
- Your hearing changes: it gets worse then gets better again
- You have worse hearing in one ear
- You hear ringing or buzzing in only one ear
- Avoid unauthorized modifications to the Nuance Audio™ Plus and the Charging pad.
- Do not use the Nuance Audio™ Glasses and the Charging Pad in environments where there is a risk of explosion (e.g. in mines or industrial plants subject to such risk). Do not store or carry flammable liquids, gases, or explosive materials in the same compartment as your Nuance Audio™ Plus, its parts, and/or components.

3.2. Battery

- The Nuance Audio™ Plus should not be exposed to temperatures below 14°F (-10°C) or above 95°F

(35°C) or when conditions are expected to exceed these temperatures. Using the Nuance Audio™ Plus above 95°F could result in overheating, fire, or injury.

- If abused or misused, the lithium-ion battery may leak and may cause chemical burns.
- Keep battery away from excessive heat such as heaters, fires, and solar radiation as batteries may rupture and release corrosive materials.
- Store batteries in a dry, moisture-free environment at room temperature. Do not keep battery in a refrigerator, in excessively warm areas or in a dehumidifying device. Appropriate battery care reduces the risk of leakage which is harmful to you and your hearing aids, and it saves battery life.

3.3. Precautions

- When the Nuance Audio™ Plus has mounted prescription lenses, it is intended only for use by the prescribed user. Use of Nuance Audio™ Plus with prescription lenses by other users may provide inadequate vision correction or vision that is worse than normal.
- We recommend that you remove the Nuance Audio™ Plus before sleeping. While not harmful, wearing your Nuance Audio™ Plus while sleeping can be uncomfortable, disrupt sleep, and drain the battery.
- Exposure to loud sound may damage your hearing and could distract you. You can reduce risks by avoiding loud environments and using appropriate hearing protection devices.
- The sound output should not be uncomfortable or painful. You should turn down the volume or remove the Nuance Audio™ Hearing Aid if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your Nuance Audio™ Plus. See section 'Nuance Audio™ App Setup and Bluetooth® Pairing' for instructions.
- The Nuance Audio™ Plus and the Charging Pad shouldn't overheat or cause thermal discomfort during normal use. If they do, stop using them immediately and contact support.
- Check applicable aviation security restrictions on personal electronics before flying with the Nuance Audio™ Plus and the Charging Pad. The Nuance Audio™ Plus contains lithium-ion batteries.
- Use the Nuance Audio™ Plus and the Charging pad in compliance with all the applicable local laws, rules and regulations. Use common sense and respect third parties' rights, privacy, and confidentiality.
- Interacting with the Nuance Audio™ Hearing Aid may distract you or increase the time it takes to

react to hazards. Always pay attention to your surroundings.

- This Nuance Audio™ Plus must not be used as safety or protection against mechanical impact.
- The Nuance Audio™ Plus can be only be charged with a Nuance Audio™ Charging Pad. If the Nuance Audio™ Hearing Aid are placed on generic wireless charge pads, it will not charge.
- Use caution when operating a vehicle while wearing Nuance Audio™ Plus and keep your focus on driving. Nuance Audio™ Plus are an air-conduction product intended for individuals aged 18 or older with perceived mild to moderate hearing loss. Follow local laws and requirements applicable to mobile phone operation and use of other devices while driving, as well as the rules governing any requirement to use hearing devices while driving, as established by the relevant legislation and the authorities responsible for its enforcement. Adjust all relevant settings of your Nuance Audio™ Plus by selecting from the available options so that you can adequately perceive surrounding sounds. If you notice that wearing Nuance Audio™ Plus may distract you or affect your attention, do not wear them while driving. This warning does not concern visual correction: wear any corrective lenses you may need to drive safely. Adjust all relevant Nuance Audio™ Plus settings before driving, and do not interact with them or any of the accessories when the vehicle is in motion. Please note that when the "phone call" function is active, the sound amplification features, both frontal and 360°, cannot be activated or remain active.

Detailed system requirements, software terms of license and terms of service are available at www.nuanceaudio.com.

Note: If you try this device and continue to struggle with or remain concerned about your hearing, you should consult with a healthcare professional.

3.4. Preventing damage and malfunctions to the Nuance Audio™ Hearing Aid and the Nuance Audio™ Charging Pad

- For safe operation do not submerge the Nuance Audio™ Plus in liquid. Liquids entering the housing of the Nuance Audio™ Plus will damage the electronics.
- Avoid exposure of The Nuance Audio™ Plus and the Nuance Audio™ Charging pad to extreme temperatures and intense heat sources (hairdryer, heater, etc).

- Avoid exposure in direct sunlight.
- Avoid exposure to water and salt water.
- Avoid exposure to sand and fine particulates.
- Do not use the devices if there are visible sharp edges, cracks, or damage to the housing.

3.5. Recommended Use and Storage

-  Operating conditions: 14°F (-10°C) to 95°F (+35°C); 101.3kPa to 61.9kPa; humidity between 20% and 60% under 95°F (+35°C) for 1000 hours; maximum altitude of 13,000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C).
- Transport and Storage conditions: Hearing Aid shall remain functional after recovery from minimum -22°F (-30°C) or maximum +167°F (+75°C) storage temperatures; high humidity 90% and temperature 95°F (+35°C) for 288 hours; exposure to altitude of 40,000 ft (18,8kPa) for 12 hours at both 14°F (-10°C) and 95°F (+35°C).

3.6. Cybersecurity

EssilorLuxottica has assessed the Nuance Audio™ Hearing Aid Software from a Cybersecurity perspective, in accordance with FDA guidance. A full risk assessment of the device in regard to cybersecurity revealed no vulnerabilities impacting safety or efficacy with the product. Based on this assessment, the only action required by the user to ensure the product remains secure is to keep the product updated with the latest product updates. For information on how to keep your product up to date, please see "Updating the Nuance Audio™ Hearing Aid" in the mobile App. In the unlikely event that your Nuance Audio™ Hearing Aid have been compromised from a cybersecurity perspective, please reach out to us immediately at csirt@essilorluxottica.com for assistance.

In the event of a system compromise, EssilorLuxottica will issue official communications through its institutional channels.

Proper operation of the Nuance Audio™ Hearing Aid requires a safe, private, and secure operating environment. To ensure your security, we recommend following best practices for securing your home network and mobile device. For more information, refer to the US Federal Trade Commission's guidance on online security. Visit: consumer.ftc.gov/topics/online-security.

4. WHAT'S IN THE BOX

1 - 1 each Nuance Audio™ Plus

2 - 1 each Nuance Audio™ Folded Case

3 - 1 each Cleaning Cloth


4 - 1 each Nuance Audio™ Charging Pad

5 - 1 each USB-C Cable

The Box also includes:


6 - 1 each Instructions for Use

7 - 1 each Quick Start Guide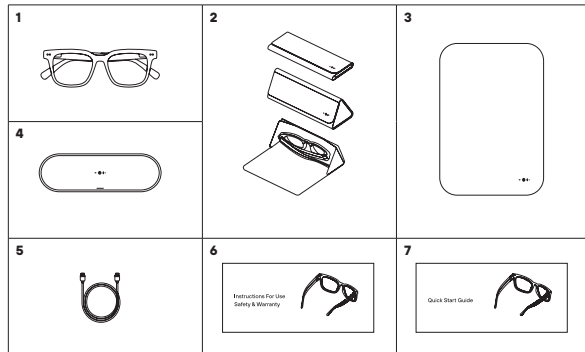

5. OPENING THE BOX

- Ensure the components mentioned above are included in the package.
- Scan the QR Code printed on the inside lid of the box and download the Nuance Audio™ App.
- Extract the User Guide from the envelope inside the box's drawer and follow the instructions.
- Do not use the Nuance Audio™ Hearing Aid until the App has instructed you to do so and until you have read the User Guide indications.

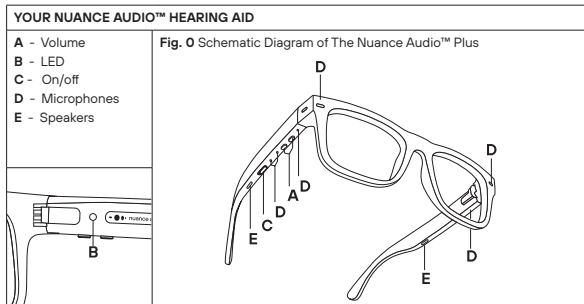6. THE NUANCE AUDIO™ HEARING AID

The Nuance Audio™ Hearing Aid is considered an air conduction hearing aid software within a smart glasses frame. Nuance Audio™ Plus in these instructions refers to the HW platform together with the HA functionality enabled SW. The Nuance Audio™ Plus components are:

- Nuance Audio™ Plus : Smart glasses compatible with Nuance Audio™ Hearing Aid Software.
- Nuance Audio™ Charging Pad: intended to enable the air conduction hearing aid functionality in terms of charging the hearing aid. The Charging Pad has a LED on top view, to give info to the user regarding the charging status of the Nuance Audio™ Plus. Please refer to the User Guide or 'Charging the Battery' section for images and usage explanation of the Charging Pad.
- USB-C Charging Cable: to be connected to the Nuance Audio™ Charging Pad.
- Folded Case: intended to be used to carry the Nuance Audio™ Plus during the day, with the main functionality to be protective and easily carried by the user. The Folded Case does not contain any electronics.
- Cleaning Cloth: to be used for cleaning the lenses.
- Nuance Audio™ App: the Nuance Audio™ Plus are required to be used in conjunction with the Nuance Audio™ App, via Bluetooth® wireless connection with a mobile phone. The App requires an iPhone with iOS operating systems (iOS 16.6.1+) or a smartphone with an Android (Android 12 or above) operating system. It is possible to use the Nuance Audio™ Hearing Aid main functions without the App, using the button and the tap area on the Nuance Audio™.

7. Nuance Audio™ APP setup and Bluetooth® pairing

The Nuance Audio™ Plus are a wireless Bluetooth® hearing aid intended to be used for wirelessly listening to the world

around you. Download the Nuance Audio™ App by scanning the QR Code located on the inside lid of the box or on the first page of the User Guide, or by accessing the App via the Apple® App Store or Google Play® Store. Once installed, open the App to begin the setup process. All setup instructions will be in the App to guide you. Follow the Nuance Audio™ App registration and set up instructions prior

For Nuance Audio™ Plus models and sizes, please check <https://www.nuanceaudio.com>.

to using The Nuance Audio™ Plus. Android®, Google Play® and the Google Play® logo are registered trademarks of Google LLC. The Apple® wordmark, the Apple® logo, App store and iPhone® is a trademark of Apple® Inc., registered in the U.S. and other countries. The trademark iPhone® is used with a license from Apple® Inc. Apple® Inc. is not responsible for the operation of this device or its compliance with safety and regulatory standards. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth® SIG, Inc. and any use of such marks by is under license. is a registered trademark of Essilor International SAS.

Note: if your already use the Nuance Audio™ App for your previous Nuance Audio™ hearing aid, you can use the same account and app for Nuance Audio™ Plus (skip to 'First pairing' instructions)

7.1. Review Required Labelling

The first time you log into the Nuance Audio™ App, the first screen to appear will contain information pertaining to over-the-counter hearing aid controls and associated labelling. You must acknowledge that you have read and understand the information before you will be allowed to proceed. Refer to **Fig. 3**. Overall, clinical data showed that people with mild to moderate hearing impairment perceived a hearing benefit when using the Nuance Audio™ Hearing Aid Software in conjunction with the Nuance Audio™ Plus.

7.2. Onboarding the Nuance Audio™ Plus

Onboarding is performed the first time you access the Nuance Audio™ App. The primary purpose of the onboarding section of the App is to systematically walk you through the setup and configuration of the Nuance Audio™ Plus.

Creation of the Nuance Audio™ App user account

User authentication is mandatory to use the Nuance Audio™ App, the first action is to complete the sign-in process starting from the authentication/sign-in page. **Fig. 1 Authentication page** The user is prompted to input an email address to be used as login ID and password, together with some personal information. **Fig. 2 Sign-in form** Email address is verified through a one-time password (OTP) sent via email do the email address used in the account creation process. The user must type in the code to complete the sign-in. **Fig. 4 One-time password input and verification** If the onetime

verification code is not received, user may ask to send a new one directly from the page of **Fig. 4** below.

First pairing

Once the sign-in process is completed and the user executes the login to the app, all is ready for the first pairing procedure. The Nuance Audio™ Plus must be set in pairing mode before starting the scanning from the mobile app tapping on the "Pair my Hearing Aid" button and selecting the hearing aid type (3 buttons for Nuance Audio™ Plus). **Fig. 5A Pair my Hearing Aid, Fig. 5B select your hearing aid type**. The mobile phone will ask permission to the user to use Bluetooth® before starting the scanning of Nuance Audio™ Plus. **Fig. 5C** All the found devices are listed, the user must select the one she/he is interested to pair with. **Fig. 5D** The last step is to accept the Bluetooth® pairing pop-up displayed by the operating system of the mobile phone. **Fig. 5E** A successful paring is represented by the page here below. **Fig. 6 Pairing completed with success**

Personal Calibration

Personal calibration is the procedure that considers the user's head shape to minimize acoustic feedback. The device incorporates a feedback control algorithm to reduce the likelihood of the user experiencing acoustic feedback. Please follow the Nuance Audio™ App instructions to perform the required steps to complete personal calibration It is best to be in a quiet environment to set up the device and perform the personal calibration set up steps. The user will hear a short burst of sound and should remain quiet during the initial personal calibration sound measurement, then the user will be asked to speak per the prompts in the Nuance Audio™ App personal calibration procedure to finalize the measurements. The personalization calibrating procedure can be repeated at any time from the mobile app Settings menu.

Once you have completed the onboarding and set up, then you can begin using the Nuance Audio™ Plus. The Nuance Audio™ Plus will start with a default setting of preset A, Flat mild hearing loss profile, and volume setting 3. You can fine tune your device using the main App screen choices (see section 7.3). Select a hearing preset (A, B, C, and D) that is best suited to meet your listening needs.

Nuance Audio™ App and Nuance Audio™ Plus tutorial

The tutorial on how to use the Nuance Audio™ Hearing Aid is displayed right after the personal

calibration. If skipped, the same tutorial is always available in the Support session of the mobile app.

Fig. 7A, 7B

App tutorial

Once the Personal calibration is completed the user may access the mobile app tutorial that is also available for later access in the Support page of the mobile app. **Fig. 9**

7.3. Application main page

The application main page, also called home page, is displayed in the app when the Nuance Audio™ Plus are turned on and correctly paired with the Nuance Audio™ App in your mobile phone. **Fig. 10A**

The following configuration options are directly available from the Home page.

- Select the preferred Preset configuration (A, B, C, D)
- Background noise adjustment
- Volume Control
- Battery charge level.
- Environmental noise measurement
- Audio Mode (Frontal or All Around)

Preset Fitting Settings:

Note: Default setting is Preset A and Volume 3

- **Preset A:** Mild flat hearing loss. Slightly boosts all frequencies
- **Preset B:** Mild sloping hearing loss. Slightly enhances higher frequencies
- **Preset C:** Moderate sloping hearing loss. Boosts all frequencies with more emphasis on higher frequencies
- **Preset D:** Moderate flat hearing loss. Boosts all frequencies

NOTE: You cannot choose a different preset for each ear. The preset chosen will be the same amplification provided to both ears. Therefore, if you have worse hearing in one ear, then please choose a profile that serves your needs for the worse ear and use the volume adjustment to adjust to meet your hearing needs.

Battery level

A battery level indicator is displayed in the top left angle of the home page (**Fig. 10A**). When the indicator

turns red it is time to charge the Nuance Audio™ Hearing Aid. Tapping on the component - the Hearing Aid name and FW version is displayed.

Noise tracking

Noise tracking measures (**Fig. 11**) the environmental noise using the microphones of the Nuance Audio™ Hearing Aid. To access the noise tracking measurement, touch the Noise tracking icon on the top right of the main App screen. The background noise tracking is for informational purposes only.

NOTE: If the Noise tracking shows that you are in a "Very High" noise environment, it is recommended that the user turn off the Nuance Audio™ Plus.

- Exposure to loud sound may damage your hearing and could distract you. You can reduce risks by avoiding loud environments and using appropriate hearing protection devices.
- The sound output should not be uncomfortable or painful. You should turn down the volume or remove the Nuance Audio™ Plus if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your Nuance Audio™ App preset selection settings.

Background noise

From the background noise command in the main page the user can select how much background noise to hear while using the hearing aid. Options are: Lowered, Standard and Heightened. **Fig. 12**

Tapping one of the Background noise settings allows you to adjust your desired background noise level. Choosing "Lowered" will soften sounds to the maximum noise reduction setting, "Standard" will allow the user to hear more sounds amplified in the environment, and "Heightened" will allow a more transparent amplification setting with minimal noise reduction with all sounds amplified.

Volume

Volume setting, 5 loudness levels, are available on the main Nuance Audio™ App menu. Tapping on the plus or minus icons allows the user to adjust volume loudness. Volume 1 is the lowest level and softest loudness setting. Volume 5 is the highest level and highest loudness setting. Volume 3 is the default loudness setting.

Volume level can be also controlled via the Hearing Aid by using volume buttons: '+' to increase volume

level, '-' to decrease volume level.

- By default, pressing and holding for minimum 1 sec will adjust volume level rapidly
- Change presets configuration (see shortcuts) allows only regular volume adjustment

Presets

Four different Presets are designed to personalize user's hearing experience, making speech perception clearer and improving listening comfort tailored to his preferences.

Presets can be also changed via the Hearing Aid by using volume buttons: '+' for preset up, '-' for preset down.

- Change Presets configuration (see Shortcuts) allows preset change by pressing and holding for minimum 2 sec

Audio Mode

The audio mode selector gives to the user the additional option to configure the Nuance Audio™ Plus 'hearing mode' by selecting frontal beamforming or all-around mode.

- Frontal mode: Provides directional microphone beamforming to provide amplification to sounds coming from the front of the user and minimizes any amplified sounds from the side or behind the user for improved hearing in noisy environments.
- All-around mode: Provides 360-degree amplification of all sounds from the back, side and front of the user for awareness and improved hearing in quiet environments or when hearing all-around is important for awareness of sounds.

The audio mode can be also changed via the Hearing Aid by using double tap gesture on any of the temples

- By default, this option is disabled (see personalization)
- The change is followed by Audio prompt (if enabled, see personalization)

7.4. Menu

Nuance Audio™ Plus selection and add a new device

This menu item gives the user the opportunity to switch between multiple Nuance Audio™ Plus that are associated with the user profile and paired with the mobile phone. This feature allows up to 4 hearing

aids that can be associated to a single profile. This is done by adding a new device (maximum of 4) to the user profile by initiating a new pairing process. **Fig. 13**

My Account

The personal information section contains profile information such as full name, email, phone number, date of birth, and gender. **Fig. 14**

Password

In the password section (**Fig. 15**) allows the user to change their password by entering the current password and then replacing it with a new one. Password accepted format is:

- 8 characters
- 1 upper case
- 1 lower case
- 1 number
- 1 special character

Preferences

Language

The application language is automatically set to the same language of the mobile phone if supported, otherwise default is English. User can select a new language from this configuration page.

Push notifications

User can enable push notification from this session and can also flag their categories of interest. Available 'Notifications' categories are firmware (FW) upgrades or suggestions on how to better configure the Nuance Audio™ Plus. **Fig. 16**

Preferences

In the commercial email section, the user can choose whether to enable commercial emails and/or whether to enable tracing of activities and configurations applied to the Nuance Audio™ Hearing Aid.

Data Management

In the legal documents section, all the documents regarding privacy and data processing can be found.

Delete account

User can decide to permanently delete the account and all the associated data. If engaging this operation, the user can no longer log-in to any to the Nuance Audio™ App. **Fig. 17**

Log-out

Clicking the log out button disconnects the account from the Nuance Audio™ App.

7.5. Settings

My own voice

User can select how to perceive and monitor own voice. Settings are: No, Yes (**Fig. 19**)

Personal calibration

From this Settings menu item, the user can select from the user's most recent Personal Calibration measurement or select the standard default setting (**Fig. 20A**)

My Hearing Aid

This section of the Settings menu includes all the Nuance Audio™ Hearing Aid configuration activities the user can execute.

My Glasses: Edit name

Provides the user the ability to customize the Nuance Audio™ Plus name overwriting the default name. **Fig. 20B**

My Hearing Aid: Configuration and Firmware (FW) upgrade

This section contains information on the FW version of the Nuance Audio™ Hearing Aid Software and prompts the user if a new FW version is available. FW upgrade can be executed only if the charging level of the battery is above 20%. **Fig. 21A**

At the end of the upgrade the app will wait for the Nuance Audio™ Plus to reboot with the new firmware. **Fig. 22A, Fig. 22B**

My Hearing Aid: Detailed information

This section of the My Hearing Aid page contains all the detailed information on The Nuance Audio™ Plus such as the serial number, model, color, etc. **Fig. 23**

My Hearing Aid: Forget these Glasses

This function removes the link between the user account profile and the paired Glasses. Hearing

Aid configuration is not erased because of the Forget operation. This operation is mandatory if the very same pair of Glasses needs to be paired and associated with a different user's phone. No undo is available with this operation, but user is prompted for a confirmation. Mobile phone Bluetooth® association with the Nuance Audio™ Plus must be also deleted from the mobile phone Bluetooth® setting page.

My Hearing Aid: Factory reset

This returns the Nuance Audio™ Plus and Nuance Audio™ App to the factory configuration. All data stored in the hearing aid and in association with the user account is cleared.

There is also an option to reset to factory configuration via the Hearing Aid: press and hold both volume buttons for 10 sec until purple LED color is seen

Note: The factory reset is permanent, there is no ability to undo this operation. The user is prompted for a confirmation prior to factory reset completion.

Settings

My own voice – Personal Calibration – Phone streaming– Shortcuts – Audio Mode default turn on **Fig. 25**

7.6. Phone Streaming

There is an option to receive phone calls via the Glasses:

- This option is available only for iPhones with iOS operating systems
- Configurable via the operating system settings, by default this option is enabled
- If this option is enabled, while the glasses are on and connected to the cell phone, phone calls will be streamed via the Glasses (incoming call sound notification and the calling itself)
- Answer/End calls is possible via double tap if this option enabled
- Other phone sounds (media, notifications, etc.) are streamed if this option is enabled via the operating system settings **Fig. 2**

7.7. About

The device also might support media streaming functionality for the playback of multimedia content. This page contains all the information on the application such as the App version number and other product codes. For App label, please check App details page. **Fig. 24**


Clinical and non-clinical performance testing

Overall, clinical data showed that people with mild to moderate hearing impairment perceived a hearing benefit when using the Nuance Audio™ Hearing Aid Software in conjunction with The Nuance Audio™ Plus. The Nuance Audio™ Hearing Aid Software passed all relevant non-clinical performance testing and biological endpoints with the representative, compatible computing platform hardware (smart glasses). Software verification and validation, bench performance testing against established consensus standards for electrical safety, EMC and battery safety were conducted with the representative, compatible hardware (smart glasses) to verify the amplified acoustic signal output by the hardware platform. Please visit www.nuanceaudio.com for more information.

7.8. Support


The Support section of the mobile app includes: **Fig. 26**

- Access to the Nuance Audio™ Hearing Aid tutorial
- Access to the mobile app tutorial
- Access to the Nuance Audio™ Remote Plus tutorial
- Link to the FAQ pages of the www.nuanceaudio.com website
- Link to the Troubleshooting pages on the www.nuanceaudio.com website
- Link to the Educational resources on the www.nuanceaudio.com website
- Link to the Contact is section on the www.nuanceaudio.com website
- Link to the Quick Start Guide on the www.nuanceaudio.com website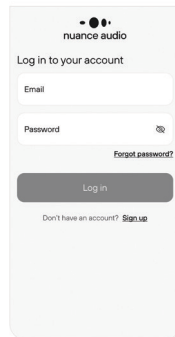


The authentication page features the Nuance Audio logo at the top. Below it, the text "Log in to your account" is displayed. There are two input fields: "Email" and "Password". A "Forgot password?" link is located next to the password field. A large "Log in" button is positioned below the input fields. At the bottom, there is a link that says "Don't have an account? Sign up".

Fig. 1. Authentication page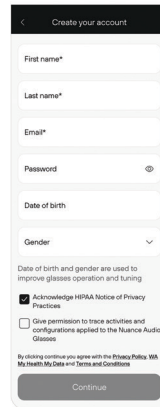

The sign-in form is titled "Create your account". It includes several input fields: "First name*", "Last name*", "Email*", "Password", "Date of birth", and "Gender". Below these fields, a note states: "Date of birth and gender are used to improve glasses operation and tuning". There are two checkboxes: "Acknowledge HIPAA Notice of Privacy Practices" (which is checked) and "Give permission to trace activities and configurations applied to the Nuance Audio Glasses". At the bottom, there is a link: "By clicking continue you agree with the Privacy Policy, My Health My Data and Terms and Conditions". A large "Continue" button is at the very bottom.

Fig. 2. Sign-in form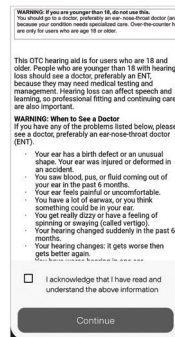

The "Review Required Labelling" screen contains a warning at the top: "WARNING: If you are younger than 18, do not use this. You should go to a doctor, preferably an ear-nose-throat doctor (an ENT) because your condition needs specialized care. Over-the-counter hearing aids are only for users who are age 18 or older." Below this, a paragraph explains: "This OTC hearing aid is for users who are 18 and older. People who are younger than 18 with hearing loss should see a doctor, preferably an ENT, because they may need medical testing and management. Hearing loss can affect speech and learning, so professional fitting and continuing care are also important." Another warning follows: "WARNING: When to See a Doctor. If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (ENT).". A list of symptoms is provided, including birth defects, ear injury, discharge, pain, dizziness, and sudden hearing changes. At the bottom, there is a checkbox for "I acknowledge that I have read and understand the above information" and a "Continue" button.

Fig. 3. Review Required Labelling

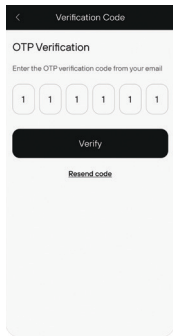
Fig. 4.
Onetime code input and
verification

Fig.5A
Pair my Hearing Aid

Fig. 5B
Select your hearing
aid type

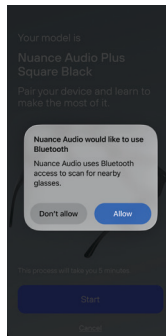
Fig. 5C

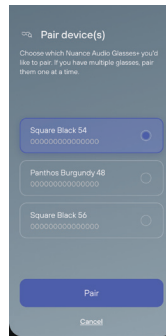
Fig. 5D

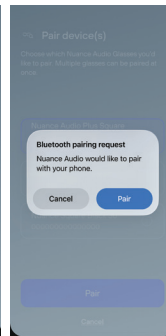
Fig. 5E

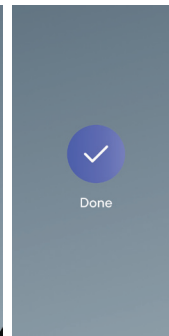
Fig. 6
Pairing completed with
success

Fig. 7A
Nuance Audio™ Hearing Aid tutorial. Start page and a sample of one of the tutorial's steps.

Fig. 7B

Fig. 8A
Calibration

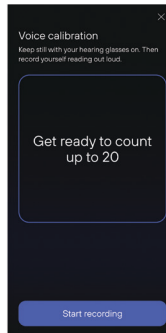
Fig. 8B

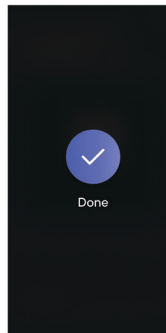
Fig. 8C

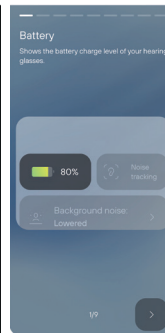
Fig. 9A
Mobile app tutorial

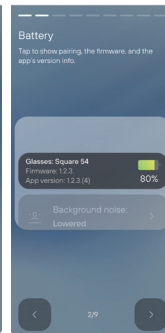
Fig. 9B

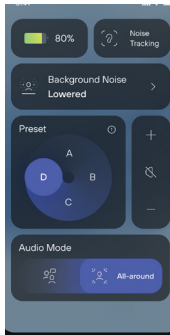
Fig. 10A Home page
The following configuration options are directly available from the home page

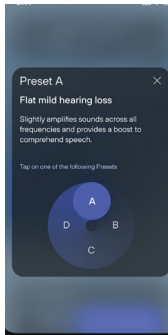
Fig. 10B

Fig. 11 Noise tracking

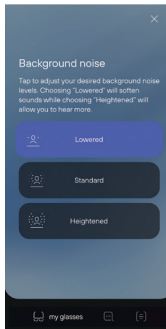
Fig. 12 Background noise configuration

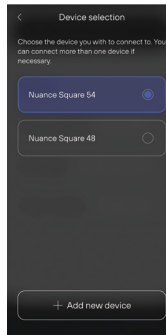
Fig. 13 Hearing Aid selection

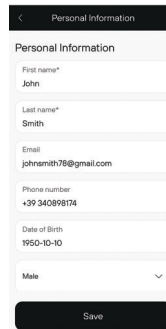
Fig. 14 Personal information

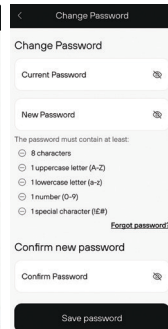
Fig. 15 Change password information

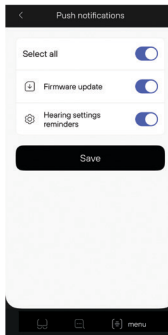
Fig. 16 Push Notifications

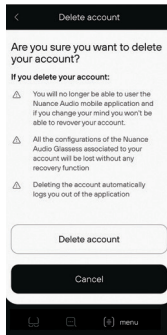
Fig. 17 Delete account, warning page

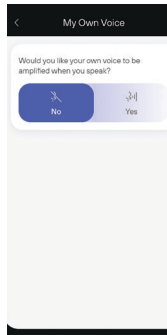
Fig. 19 Own voice

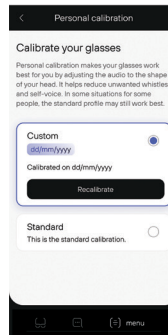
Fig. 20A Personal calibration

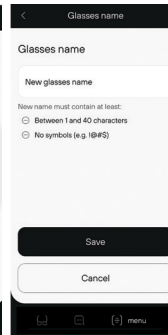
Fig. 20B

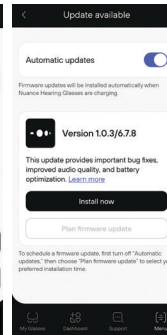
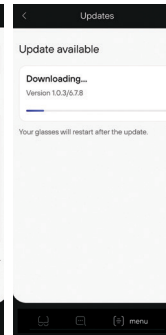
Fig. 21A
Firmware update steps

Fig. 21B

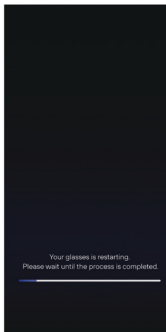
Fig. 22A
Waiting for reboot and
successful execution

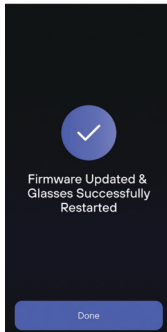
Fig. 22B

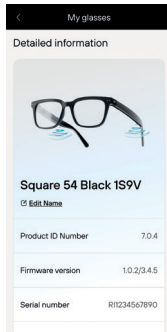
Fig. 23
Detailed information

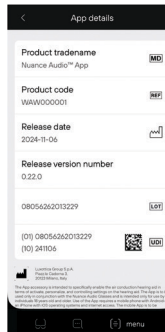
Fig. 24
About

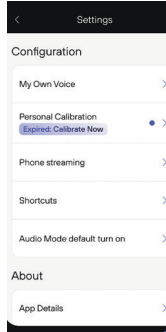
Fig. 25
Settings

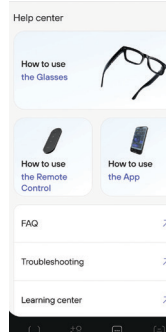
Fig. 26
Support page

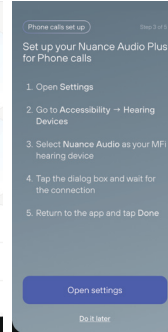
Fig. 27
Phone calls pairing
(iOS)

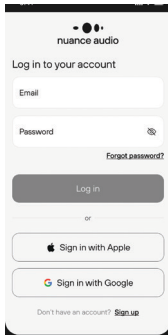
Fig. 28
Social login

8. Nuance Audio™ Care and Maintenance

8.1 Battery information

The Nuance Audio™ Plus contain a lithium-ion battery that is not user-replaceable. Do not attempt to remove or replace the battery. The battery may present a fire or chemical burn hazard if misused. Do not disassemble, crush or puncture the Nuance Audio™ Plus. Do not leave or store the Nuance Audio™ Plus under direct sunlight or expose to excessive heat or incinerate. Dispose of the Nuance Audio™ Plus in line with instructions and local waste requirements.

The battery contained in the Nuance Audio™ Plus is expected to perform as follows, provided the Nuance Audio™ Plus are always used under Normal Use Conditions (see below) (the “Expected Battery Performance”):

- at original capacity, and once fully charged, the battery is expected to provide 10 hours of continuous use under average conditions (being where there is moderate surrounding noise and the standard amplification settings are being used).
- Assuming that you do, approximately, 1 full charge per day, following 500 full charging cycles (corresponding to approximately 16 months of use) the battery is expected to perform at or above 85% of its original capacity, meaning the battery will allow for 8 hours and 30 minutes of continuous use under average conditions.

“Normal Use Conditions” means:

- Operating the product in line with this user manual, in particular Section 3 (Warnings, Precautions and Safety Considerations), which includes operating the product within the recommended temperature range.
- Not exposing the product to extreme heat, cold or humidity, as indicated in Section 12 (Nuance Audio™ Glasses technical specifications key features).
- Using the product only for its intended use, as described in Section 1 (Intended Use).
- Using only the provided or certified charging accessories as indicated in Section 8.3 (The Charging Pad works thanks to a USB-C charging cable).
- Ensuring the battery is charged and discharged in accordance with the guidelines provided at

Section 8.2 below.

- Always making sure that Nuance Audio™ Plus is using the latest firmware version.
- Charging the battery at room temperature instead of a lower or higher temperature.
- Charging the battery only when necessary.
- Making sure to charge the battery fully before unplugging it.

Luxottica does not guarantee the above battery performance beyond the term of the Limited Warranty Period. Please refer to the "Warranty" section.


Battery performance may be extended if the hearing aid is turned off when amplification is not needed, or when the surrounding environment requires less amplification and/or noise reduction.

8.2. Charging the battery

IMPORTANT! If the Nuance Audio™ Plus battery is not recharged for a prolonged period (6 months from last use) then this could lead to permanent damage to the battery, which may render the Nuance Audio™ Plus inoperable. When the Nuance Audio™ App displays a battery level of 20% or less, or when the low battery alert sounds in your ear, the Nuance Audio™ Plus should be charged. For charging, place the Charging Pad flat on a non-metal table or desktop to ensure it will not shake and move. Place the Nuance Audio™ Plus (as shown below) on the Charging Pad. Ensure that the Nuance Audio™ Plus are aligned with the magnetic components on the Charging Pad. If the alignment is not accurate, you can try adjusting the positioning of the Nuance Audio™ Plus. During the charging process, do not move or disturb the Nuance Audio™ Plus to avoid interfering the charging operation.

Battery level is indicated via the Glasses LED once charging is stopped by removing the Nuance Audio™ Plus from the Charging Pad. (Fig. 29)

- Red color – battery level is below 20%
- Orange color – battery level is above 20% and below 89%
- Green color – battery level is above 90%

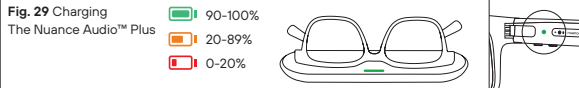8.3. The Charging Pad works thanks to a USB-C Charging Cable

Use only the provided Charging Pad to charge the Nuance Audio™ Plus. Use only the supplied USB-C Charging Cable with USB-C wall adapter with a minimum output level of DC5V/1A (sold separately). (Fig. 29)

- The Charging Pad may become warm to the touch while charging.
- If the Charging Pad becomes hot to touch, immediately remove the Nuance Audio™ Plus and disconnect the USB-C charging cable from adapter and discontinue use. Contact customer support at www.nuanceaudio.com
- If the Charging Pad cannot charge properly or experiences any malfunctions, please stop using it immediately.
- Charging Pad possible errors indications:
 1. Charging Error – the LED on the charging pad is blinking red.
 2. Startup Failure – the LED on the charging pad is blinking red for 3 sec
- Do not clean the Nuance Audio™ Plus and the Charging Pad while charging.
- Periodically inspect the Nuance Audio™ Plus, the Charging Pad and the Charging Cable for damage or signs of wear. Do not use if any part is damaged, cracked, or if any internal component is exposed.
- Do not try to repair the Nuance Audio™ Plus or its components, or replace parts yourself, instead contact support at www.nuanceaudio.com.

8.4. Caring for the Nuance Audio™ Plus

Important Note: We do not recommend using alcohol wipes or liquid cleaning agents when caring for your Nuance Audio™ Hearing Aid.

8.5. Hearing Aids General Maintenance

- Handle with care & avoid damage. Follow the steps under the General Information section below to help maintain quality and performances of your Nuance Audio™ Plus.
- Inspect the Nuance Audio™ Plus regularly for particulate build up and follow cleaning procedures.
- Only wipe the Nuance Audio™ Plus with a soft cloth. Don't use any other cleaning tool (such as a hearing aid brush, toothpick, or pressurized air) to clean the Nuance Audio™ Plus. Do not use dirty or abrasive cloths which could negatively affect the lenses' surfaces. Moisture can damage the Nuance Audio™ Plus and device failure. To clean the lenses, make sure to only spray lens cleaner on a clean, non-abrasive lens cloth. Do not spray lens cleaner directly on the lenses.
- To clean the Nuance Audio™ Plus, wipe with a dry, clean cloth.

8.6. General Information

- The Nuance Audio™ Plus and Charging Pad service life is about 3 years.
- The Nuance Audio™ Plus and the Charging Pad are sensitive electronic equipment and should be handled with care. Do not heat or bend the Nuance Audio™ Plus and the Charging Pad, including for fitting, as they may become damaged and may void the warranty.
- Take care when folding the temples (earpieces or arms) in or out.
- Do not drop, strike, or shake the Nuance Audio™ Plus and the Charging Pad.
- Do not use the Nuance Audio™ Plus or the Charging Pad if it is damaged.
- Do not attempt to open, disassemble, or tamper with the Nuance Audio™ Plus or the Charging Pad.
- Protect your Nuance Audio™ Plus against moisture (e.g. showers, rain), and and remove your Nuance Audio™ Plus when using hairspray.
- It is recommended to carry out daily care on Nuance Audio™ Plus.
- Keep the Nuance Audio™ Plus dry. Never wear the Nuance Audio™ Plus in the shower, bath, pool, or while swimming. Protect the Nuance Audio™ Plus from prolonged exposure to rain. If the Nuance Audio™ Plus become wet, thoroughly dry.
- Should your Nuance Audio™ Plus not function correctly despite regular care, contact for customer support at www.nuanceaudio.com.

- Do not try under any circumstances to repair your Nuance Audio™ Plus by yourself as this will void your warranty.
- The serial number of the Nuance Audio™ Plus may be found marked on the inner part of the left temple or in the Nuance Audio™ App.

Only for the Charging Pad:

- Do not soak in water or stay in a humid environment for a long time.
- Do not be exposed to the sun.
- Do not place it around high-power electrical fields audio, do not be in a strong magnetic environment, do not place it on a metal surface for charging.
- To clean the Charging Pad, use a dampened cloth and mild soap avoiding all types of sprays, solvents, chemicals, or cleaning solutions containing alcohol or ammonia.
- Do not use dirty or abrasive cloths which could damage the Charging Pad.
- Keep the Charging pad dry. If the Charging pad becomes excessively wet, there is a risk of damaging the charging function, thoroughly dry it, store them in a container with desiccants.
- Should your Nuance Audio™ Charging Pad not function correctly despite regular care, contact for customer support at www.nuanceaudio.com.

9. PRIVACY - In developing Nuance Audio™ Plus, EssilorLuxottica followed the principle of privacy by design. A full description of how Nuance Audio™ Plus and the mobile app linked process personal data can be found in the Privacy Notice within the App. To learn more about EssilorLuxottica Governance on personal data, you can visit the section "Governance" on www.essilorluxottica.com and click on "Compliance/Data Privacy".

10. TROUBLESHOOTING

10.1. Common Solutions for the App

THE APP DOES NOT RESPOND TO COMMANDS. WHAT SHOULD I DO?

Close the Nuance Audio™ App and reopen it again. This will refresh the connection with the Nuance Audio™ Plus and should resolve the issue. **If the issue is still present:**

- Uninstall the app and install it again

- Restart your mobile phone and control that Bluetooth is enabled
- If the issue is present, please check if the OS version of your mobile phone is listed as a supported version
- Turn off and on again your Nuance Audio™ Plus pressing and holding the multifunction button on the frame for about 2 secs.
- Press and hold for 2 secs to turn on the Nuance Audio™ Plus and test again if the mobile app functions properly.

I RECEIVED A CONNECTIVITY ERROR IN THE APP. WHAT SHOULD I DO?

You must have network connectivity (Cellular or Wi-Fi). In case you get this error please check if your Wi-Fi or mobile data connectivity are correctly configured and working. Sometimes connectivity errors are generated because of an error in the communication between the mobile app and the Nuance Audio™ Plus. Close the Nuance Audio™ App and reopen it again. This will refresh the connection with the Nuance Audio™ Plus and should resolve the issue.

I HAVE ISSUES DOWNLOADING THE APP. WHAT SHOULD I DO?

- Check your Wi-Fi or mobile data connection.
- Check available storage space on your mobile phone.
- Check for latest iOS or Android software upgrades available for your mobile device.
- Close and re-open the app store where you are attempting to download the Nuance Audio™ App and retry.
- If none of these steps solve the issue, then shutdown and restart your mobile phone.

10.2. Other Troubleshooting Tips

MY Nuance Audio™ Plus ARE NOT CHARGING. WHAT SHOULD I DO?

If your device isn't charging, make sure that:

- The charging area is facing up and the Nuance Audio™ Plus are placed on Charging Pad properly
- The Charging Pad is connected correctly to the provided USB-C charging cable, and that the charging cable is connected properly to a USB-C power source with a minimum output level of DC5V/1A (sold separately).
- The power source is in proper condition and not damaged.

- If all 3 conditions above are met, try to gently adjust the position of the hearing aid lying on the charging pad.

THE BATTERY IN MY HEARING AID DRAINS VERY QUICKLY. WHAT SHOULD I DO?

If your Nuance Audio™ Plus is new and its battery lasts less than 7 hours under average use conditions. To maximize battery life, it is recommended to turn off the hearing aid function when it is not being used. See section 8.2 for battery charging instructions. Contact for customer support at www.nuanceaudio.com

HOW CAN I FORCE RESTART MY NUANCE AUDIO™ PLUS ?

Step 1: Slide the switch to the OFF position. **Step 2:** Press the Volume Minus (-) button for 12 seconds
Step 3: Slide the switch to the ON position

AMPLIFIED SOUNDS ARE METALLIC AND UNNATURAL. WHAT SHOULD I DO?

Here are some tips for improving sound quality:

- Try changing the volume loudness level.
- Select a different presets profile (A, B, C or D) to provide a more comfortable listening setting.
- Try to select a different the Background noise setting (see section 9.3.3).
- Repeat Personal Calibration process (see section 9.2.4).

WHAT DO I DO IF I DON'T FIND BENEFIT AFTER USING DEVICE?

If you have a problem with your Product, please visit us at www.nuanceaudio.com to get support. You may return the product to the retailer that you purchased it from based on that retailers return policy.

WHAT DO I DO IF I RETURN THE DEVICE AND HAVE PURCHASED THE OPTIONAL PRESCRIPTION VISION LENSES?

If you have a problem with your Product, please visit us at www.nuanceaudio.com to get support. If you have purchased the device and chose to purchase the optional prescription vision lenses, you must return to the retailer where you purchased the optional prescription vision lenses based on their prescription lenses warranty, return and/or replacement policies.

IS THE NUANCE AUDIO™ HEARING AID SUITABLE FOR ME IF I ONLY HAVE HEARING LOSS IN ONE EAR OR I PERCEIVE WORSE HEARING IN ONE EAR VERSUS THE OTHER EAR?

The Nuance Audio™ Hearing Aid Software is compatible with the Nuance Audio™ Plus , and delivers

the same amplification to each ear. Once you have chosen a preset based on your listening preference (A, B, C or D), you may also adjust the volume setting to meet your hearing needs.

NOTE: IT IS RECOMMENDED THAT YOU SEE A DOCTOR, PREFERABLY AN EAR-NOSE-THROAT DOCTOR IF YOU HAVE PERCEIVED WORSE HEARING IN ONE EAR BEFORE USING AN OTC HEARING AID.

11. HEARING AID TECHNICAL SPECIFICATIONS	
KEY FEATURES	
Processing Channels	16
WDRC	Yes
Feedback Management	Yes
Battery	Rechargeable, Li-ion
Programs	4 selectable 'presets'
iOS App	Yes
Android App	Yes
Volume Control	Yes via on device buttons or app
Microphone Pattern	Focused and Omni options
Self-Setup & Personalisation	Yes via app
Low Battery Alarm	Yes via LED, App & audio prompt
App Compatibility	Android 12 and above, iOS 16.6.1 and above
Hands-free Calling	Yes
WIRELESS CONNECTIVITY	
Bluetooth® Compatible	Yes, BLE v5 and above

Bluetooth® Phone Call Compatible	Yes, Bluetooth/MFI
Operation frequency	2402-2480 MHz
Max radiated power	Free space max EIRP is 3,17 dBm
Modulation	GFSK
Channel spacing	2MHz
Channel bandwidth	2MHz
Number of channels	40
WIRELESS CHARGING	
Transmitting frequency	13,56 MHz
Receiving frequency	13,56 MHz
Max radiated power	69,72 dBuV/m at 3m
Modulation type transmitting	ASK 100% modified Miller
Modulation type receiving	Load Modulation subcarrier fc/16) 848kbit/s) OOK Manchester
FITTING AND PROGRAMMING	Not intended to be a fitted device or programmed
Nuance Audio™ App	Yes, for set up
TECHNICAL SPECIFICATIONS	
Microphones	6 Microphones
TRANSPORT & STORAGE CONDITIONS	
Temperature	From -22°F (-30°C) to +167°F (+75°C)

Humidity	From 20% to high humidity 90% and temperature 95°F (+35°C) for 288 hours
Altitude & atmospheric pressure	Altitude of 40,000 ft (18,8kPa) for 12 hours at both 14°F (-10°C) and 95°F (+35°C)
OPERATING CONDITIONS	
Temperature	From 14°F (-10°C) to 95°F (+35°C)
Humidity	From 20% to humidity 60% under 95°F (35°C) for 1000 hours
Altitude & atmospheric pressure	Altitude of 13,000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C); 101.3kPa to 61.9kPa
Electroacoustic Characteristics (ANSI S3.22:2014 & CTA-2051:2017)	
OTC Hearing Aids	
Frequency Response Bandwidth	200 – 6300 Hz
Equivalent Input Noise	32 dB
Total Harmonic Distortion	<1%
Maximum OSPL90	111.9 dB
Latency	13.9 ms
Data for all parameters measured per ANSI S3.22:2014 / CTA-2051:2017 can be found at www.nuanceaudio.com/en-us/c/technical-reports	

Guidance and manufactures declaration for electromagnetic emissions		
Emission standard	Type	Compliance to standard
CISPR 11	Radiated and conducted RF emissions	Yes, Group 1, Class B
Guidance for application Environment		
Nuance Audio™ Hearing Aid is suitable for use in both professional as well as domestic environments. The device must emit electromagnetic energy in order to perform its intended function. Nearby electronic equipment may be affected.		
Features per ANSI/CTA 2051-2017		
Feature Category	The Hearing Aids	
Fixed or Level Dependent Frequency Equalization	Tone Control Available ANSI/CTA-2051-2017 clause 4.10	
Level Dependent Gain / Compression	Multiband Wide Dynamic Range Compression ANSI/CTA-2051-2017 clause 4.11	
Feedback Control	Available with the Nuance Audio™ Hearing Aid ANSI/CTA-2051-2017 clause 4.14	
SNR Enhancement	Available with the Nuance Audio™ Hearing Aid ANSI/CTA-2051-2017 clause 4.13	
Guidance and manufactures declaration for electromagnetic immunity		
Emission standard	Type	Compliance level
EN/IEC61000-4-2	Electrostatic discharge (ESD)	± 4kV, ± 8kV contact ±2kV, ±4kV, ±8kV, ± 15kV air

Specific guidance for application environment		
Suitable for use on wood, concrete , or ceramic floor materials. Keep relative humidity below 30% when used on floors of syntitic materials e.g. carpets.		
Emission standard	Type	Compliance level
EN/IEC61000-4-8	Power frequency magnetic field	30 A/m 50/60 Hz
Specific guidance for application environment		
Suitable for in proximity to equipment mains connected to public low-voltage network supplying domestic environment. Portable and mobile radio devices, including their wires, should not be used closer to the unit than the recommended safe distance or 30 cm. The level for Power frequency magnetic fields corresponds to the levels typical for professional or domestic Environment.		
Emission standard	Type	Compliance level
EN/IEC61000- 4- 3	RF electromagnetic fields. Proximity fields from RF wireless communication equipment.	80 MHz to 2.7 GHz, 80% AM at 1 kHz, 10 V/m
		Test freq. MHz V/m
		385 27
		450 28
		710, 745, 780 9
		810, 870, 930 28
		1720, 1845, 1970 28
		2450 28
		5240, 5500, 5785 9

Specific guidance for application environment			
Portable and mobile radio devices, including their wires, should not be used closer to the unit than the recommended safe distance or 30 cm.			
Emission standard	Type	Compliance level	
EN/IEC61000- 4- 39	IMMUNITY to proximity magnetic	Test frequency	A/m
		30 kHz	8
		134.2 kHz	65
		13,56 MHz	7.5
Specific guidance for application environment			
The device is tested to function in close proximity to powerful magnetic fields, e.g. from RFID equipment. The test levels ensure a safe distance of 15 cm.			

The Nuance Audio Plus was tested according to recommendations of IEC TS 60601-4-2:2024 Medical electrical equipment-Part 4-2: Guidance and interpretation – Electromagnetic immunity: performance of medical electrical equipment and medical electrical systems.

Phenomenon	Basic EMC standard	Immunity Test Level	Performance criteria
ELECTROSTATIC DISCHARGE	IEC 61000-4-2	±4KV contact ±2kV,±4kV,±8kV air	Criterion B
Radiated RF EM field	IEC 61000-4-3	3V/m 80MHz to 2,7GHz 80%AM at 1kHz	Criterion A

Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Test freq. MHz	V/m	Criterion A
		385	6	
		450	9	
		710, 745, 780	3	
		810, 870, 930	9	
		1720, 1845, 1970	9	
		2400	9	
		5240, 5500, 5785	6	
RATED power frequency magnetic fields	IEC 61000-4-8	3A/m,50Hz or 60Hz		Criterion A

NOTE:

a) Criterion A: The ME EQUIPMENT or ME SYSTEM should continue to meet the performance criteria during and after the following tests, without the need for OPERAT or intervention tore store operation. This also applies to tests for other continuous phenomena that might be applicable to the ME EQUIPMENT or ME SYSTEM.

b) Criterion B: The ME EQUIPMENT or ME SYSTEM should continue to meet the performance criteria after the following tests, without the need for OPERAT or intervention to restore operation. The MAN-UFACTURER should specify in the instructions for use the maximum recovery time. While operation as intended during the test is desirable, some degradation is acceptable. This also applies to tests for other TRANSIENT phenomena that might be applicable to the ME EQUIPMENT or ME SYSTEM.

12. INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS

Luxottica is committed to the proper reuse, recycling and recovery of batteries and Waste Electrical and Electronic Equipment ("WEEE").

Batteries and WEEE should not be discarded in normal domestic waste bins.

Discard the Product and the battery according to local regulations. Do not discard as household waste.

Bring them to designated collection points or in alternative to an affiliated store.

For more info on proper collection, please visit www.nuanceaudio.com.

INFORMATION ON DISCARDING	
The crossed-out wheeled bin symbols below mean that, according to local laws and regulations, The Nuance Audio™ Products, shall be discarded separately from household waste. When The Nuance Audio™ Products, reaches its end of life, bring them to a collection point designated by local authorities. In alternative, bring the Product and the battery to a affiliated store. The separate collection and recycling of the Nuance Audio™ Products, will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.	
	Crossed-out wheeled bin (Batteries): Do not discard of batteries with household waste. Collect separately for proper recycling (Battery Regulation (EU) 2023/1542).
	Crossed-out wheeled bin (WEEE): Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).

13. REGULATORY COMPLIANCE INFORMATION

13.1. FCC compliance statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy

and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio technician for help.

Note: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Radiation Exposure Statement. This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

Country	Type Approval	Regulatory Mark/Logo
Global	The Nuance Audio™ Plus comply with the following domestic and international regulations for medical devices: IEC 60601-1:2005 Ed3+C1, C2, A1, IEC 60601-1-6, IEC 60601-1-11, IEC 60601-1-2, 4th Edition, FCC 47 Part 15.247 Classification: Type B Applied Part, IEC 60601-2-66, this Nuance Audio™ Plus have no essential performance. The testing performed above has been in accordance with IEC 60601-1-2:2014 (4th Ed). The values identified during this evaluation were found to be within acceptable specified ranges.	

14. SYMBOLS AND DESCRIPTIONS

	Federal Communications Commission (U.S.)		Manufacturer
	Bluetooth®		Type B applied part
	Review operating instructions prior to use		Operating Instructions
	Warning		Made in China
	Serial Number		Patient Information Website
	Medical Device		Keep Dry
	MR Unsafe		Temperature Limit
	Single patient, multiple use		Atmospheric pressure limitation
	Date of manufacture / Manufacturer		Humidity limitation
	Model Number	IP54	IP54 rating
	Do NOT use if package is damaged		

	Magnetic field warning. Possible interference with cardiac pacemakers, implanted defibrillators (ICD's) and other implants
	Do not heat or bend the Nuance Audio™ Plus
	To ensure battery health, recharge before the date on the packaging
	Contains small parts which may be choking hazard. Not suitable for children under the age of 3
	Crossed-out wheeled bin (Batteries): Do not discard of batteries with household waste. Collect separately for recycling (Battery Regulation (EU) 2023/1542).
	Crossed-out wheeled bin (WEEE): Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).

Use of the Made for iPhone badge means that an accessory has been designed to connect specifically to iPhone models identified in the badge and has been certified by the developer

to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

Compatible with iOS smartphones running iOS 16 or later, starting from version 0.3.7 and above. Specifically, for: iPhone 17 Pro Max, iPhone 17 Pro, iPhone 17, iPhone 16 Pro Max, iPhone 16 Pro, iPhone 16 Plus, iPhone 16, iPhone 16e, iPhone 15 Pro Max, iPhone 15 Pro, iPhone 15 Plus, iPhone 15, iPhone 14 Pro Max, iPhone 14 Pro, iPhone 14 Plus, iPhone 14, iPhone 13 Pro Max, iPhone 13 Pro, iPhone 13 mini, iPhone 13, iPhone 12 Pro Max, iPhone 12 Pro, iPhone 12 mini, iPhone 12, iPhone 11 Pro Max, iPhone 11 Pro, iPhone 11, iPhone XS Max, iPhone XS, iPhone XR, iPhone X, iPhone 8 Plus, iPhone 8, iPhone SE (3rd and 2nd generation).

	No part of this work may be reproduced, modified, distributed or otherwise used without prior written permission. Nuance Audio™ is a registered trademark of Essilor International SAS. 1. Nuance Audio™ Hearing Aid Software FCC ID: 2AYOA-4008 2. Nuance Audio™ Charging Pad FCC ID: 2AYOA-4005 Common Name: Air Conduction Hearing Aid Software		
 Luxottica Group S.p.A. Piazzale Cadorna 3, 20123 Milan, Italy	 Made in China Fabriqué en Chine	 www.nuanceaudio.com	
This information and other labeling, including the user instructional brochure, are available on the internet at: nuanceaudio.com/support/manual-guide. You may also call 844-844-0919 or write to customerservice@nuanceaudio.com or to Customer service, 4000 Luxottica Place, Mason, OH 45040 to request a paper copy of this information and other labeling.			
	For the latest and up-to-date Instructions For Use - Safety and Warranty, please refer to the Instructions in the Support section of the Nuance Audio™ App and to the following website page: nuanceaudio.com/support/manual-guide		

Instructions For Use & Safety
NOT FOR CANADA

Nuance Audio™ Plus & Nuance Audio™ Charging Pad

* Product availability, regulatory requirements, and warranty terms may differ from country to country. Please note that the local market product category of Nuance Audio™ Plus may vary depending on the market in which the product is sold, and some countries do not adopt a dedicated local category (such as the UK, Australia, and New Zealand). As a result, the terminology used to denote the product's category may vary across the language versions of these Instructions for Use – Safety & Warranty (e.g., "assistant d'écoute" in France vs. "hearing aid" elsewhere). As this document is organised by language rather than by country, the terminology used to address the product in a given language version does not necessarily reflect the category applicable in all countries where that language is used. For example, although French is France's official language, the terminology used in the French version does not apply to the French market; for France, the correct terminology is "assistant d'écoute".

WARNING: When to See a Doctor

If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus, or fluid coming out of your ear in the past 6 months.
- Your ear feels painful or uncomfortable.
- You have a lot of ear wax, or you think something could be in your ear.
- You get really dizzy or have a feeling of spinning or swaying (called vertigo).
- Your hearing changed suddenly in the past 6 months.
- Your hearing changes: it gets worse then gets better again.

- You have worse hearing in one ear.
- You hear ringing or buzzing in only one ear

Caution: This is not hearing protection.

You should remove this device if you experience overly loud sounds, whether short or long-lasting. If you're in a loud place, you should use the right kind of hearing protection instead of wearing this device. In general, if you would use ear plugs in a loud place, you should remove this device and use ear plugs.

Caution: The sound output should not be uncomfortable or painful.

You should turn down the volume or remove the device if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your device.

Note: If you remain concerned, consult a professional.

If you try this device and continue to struggle with or remain concerned about your hearing, you should consult with a hearing healthcare professional.

Note: What you might expect when you start using Nuance Audio™ Plus

Nuance Audio™ Plus can benefit many people with hearing loss. However, you should know that they will not restore normal hearing, and you may still have some difficulty hearing over noise. Furthermore, the device will not prevent or improve a medical condition that causes hearing loss. People who start using medical devices sometimes need a few weeks to get used to them. Similarly, many people find that training or counselling can help them get more out of their device. If you have hearing loss in both ears, you might get more out of using medical devices in both, especially in situations that make you tired from listening—for example, noisy environments. Nuance Audio™ Plus provides hearing assistance to both ears.

Table of contents

1.	INDICATIONS	35
	1.1. Intended Use	35
	1.2. Indications For Use	35
	1.3. Important Health and Safety Information	35
	1.4. Intended Use Environment	35
2.	CONTRAINDICATIONS	36
3.	WARNINGS, PRECAUTIONS, AND SAFETY CONSIDERATIONS	36
	3.1. Warnings	36
	3.2. Battery	36
	3.3. Precautions	36
	3.4. Preventing damage and malfunctions to your Nuance Audio™ Plus and the Nuance Audio™ Charging Pad	38
	3.5. Recommended Use and Storage	38
	3.6. Cyber Security	39

4.	WHAT'S IN THE BOX	39
5.	OPENING THE BOX	39
6.	YOUR NUANCE AUDIO™ PLUS	39
7.	NUANCE AUDIO™ APP SETUP AND BLUETOOTH® PAIRING	40
	7.1. Onboarding the Nuance Audio™ App	40
	7.2. Application main page	41
	7.3. Menu	41
	7.4. Settings	42
	7.5. Phone Streaming	42
	7.6. About	42
	7.7. Support	43
8.	NUANCE AUDIO™ PLUS CARE AND MAINTENANCE	43
	8.1. Battery Information	43
	8.2. Charging the battery	43

	8.3. The Nuance Audio™ Charging Pad works thanks to a USB-C Charging Cable	44
	8.4. Caring for Nuance Audio™ Plus	44
	8.5. General Maintenance	44
	8.6. General Information	44
9.	PRIVACY	45
10.	TROUBLESHOOTING	45
11.	NUANCE AUDIO™ PLUS TECHNICAL SPECIFICATIONS	45
12.	CLINICAL BENEFIT	49
13.	INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS	49
14.	REGULATORY COMPLIANCE INFORMATION	49
	14.1. EU compliance statement	49
15.	SYMBOLS AND DESCRIPTIONS	50

WARNING To reduce the risk of personal injury, discomfort, or property damage, please read and follow the health and safety information below before using the Nuance Audio™ Plus.

1. INDICATIONS

1.1. Intended Use

The air conduction hearing aid* is intended to amplify sound for users with hearing impairment. The air conduction hearing aid* is intended to be used in association with prescription spectacle lenses for the correction of visual defects. The product is intended only for use by individuals 18 years old and older with perceived mild to moderate hearing impairment. It is adjusted by the user to meet the user's hearing needs through the selection of presets. No pre-programming or pre-hearing test is necessary.

1.2. Indications For Use

The product is intended only for use by individuals 18 years old and older with perceived mild to moderate hearing impairment. It is adjusted by the user to meet the user's hearing needs through the selection of presets. No pre-programming or pre-hearing test is necessary. When the Nuance Audio™ Plus have mounted prescription lenses, they are intended only for use by the prescribed user.

1.3. Important Health and Safety Information

Read the Instructions.

Read the Instructions for Use for the correct functioning of the Nuance Audio™ Plus.

It is very important to read and follow this Instructions for Use, the User Guide, and all the safety, warnings and regulatory information thoroughly and completely before using the Nuance Audio™ Plus, for safe and effective operation. Follow safety instructions to avoid damage or injury. Follow also all instructions provided through the Nuance Audio™ App.

The device is designed for adult patients, without cognitive disability preventing the use of the device, both lay users and professional users. Nuance Audio™ Plus may help you hear better if you strain to follow conversations when others don't, especially in noisy places. The end user is required to report any serious incident that occurs in relation to the use of the device to the competent authority of the Member State where the incident occurred and to the manufacturer of the device.

1.4. Intended Use Environment.

Nuance Audio™ Plus are intended to be used for listening to sounds in a variety of environments (e.g., home, office, transit, social settings, outdoors, etc.), and they are suitable for home and healthcare environments. The device should not be used while bathing or swimming (i.e., the device should not be submerged in water).

2. CONTRAINDICATIONS

Nuance Audio™ Plus may not be right for you if you:

- Consistently experience feedback (whistling) when you turn up the volume to a level that is comfortable for you, even after you have followed the suggestions in these Instructions for Use to make sure your Nuance Audio™ Plus fit properly.
- Feel Nuance Audio™ Plus are not providing enough amplification even at their highest volume setting.
- Nuance Audio™ Plus and the Nuance Audio™ Charging Pad contain magnetic material and components that emit radio waves, which could affect the operation of medical devices such as but not limited to pacemakers or insulin pumps, or other implantable devices. Consult your doctor or the medical device manufacturer before using Nuance Audio™ Plus and the Nuance Audio™ Charging Pad.
- If you have an active or inactive implant device (e.g. defibrillator, pacemaker), follow the implant manufacturer's instructions regarding the use of mobile devices. Nuance Audio™ Plus should be kept at least 6 inches / 20 cm from implantable pacemakers. In the event of malfunctions, stop using your Nuance Audio™ Plus and contact your doctor or the implant manufacturer. If you have an active brain implant, ask your doctor or the implant manufacturer for a risk evaluation.
-  Nuance Audio™ Plus are MR Unsafe and could present as a projectile hazard.
- Do not use Nuance Audio™ Plus and the Nuance Audio™ Charging Pad in environments where there is a risk of explosion (e.g. in mines or industrial plants subject to such risk). Do not store or carry flammable liquids, gases, or explosive materials in the same compartment as your Nuance Audio™ Plus, its parts, and/or accessories.
- Nuance Audio™ Plus must not be used as eye safety or protection against mechanical impact. They are not to be used as protection against UV rays.
- Use of Nuance Audio™ Plus with prescription lenses by other users may provide inadequate vision correction or vision that is worse than normal.

3. WARNINGS, PRECAUTIONS, AND SAFETY CONSIDERATIONS

3.1. Warnings

Warning: Don't go near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of Nuance Audio™ Plus adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, Nuance Audio™ Plus and the other equipment should be observed to verify that they are operating normally.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm (12 inches) to any part of Nuance Audio™ Plus, including cables specified by the manufacturer. Otherwise, degradation of the performance of Nuance Audio™ Plus could occur.

RF: Radio Frequency | EM: Electro Magnetic | ME: Medical Electrical | HF: High-Frequency

3.2. Battery

- Nuance Audio™ Plus should not be exposed to temperatures below -10°C or above 35°C or when conditions are expected to exceed these temperatures. Using Nuance Audio™ Plus above 35°C could result in overheating, fire, or injury.
- If abused or misused, the lithium-ion battery may leak and may cause chemical burns.
- Keep the battery away from excessive heat such as heaters, fires, and solar radiation as the battery may rupture and release corrosive materials.
- Store the battery in a dry, moisture-free environment at room temperature. Do not keep the battery in a refrigerator, in excessively warm areas or in a dehumidifying device. Appropriate battery care reduces the risk of leakage which is harmful to you and your Nuance Audio™ Plus, and it saves battery life.

3.3. Precautions

When to see an Ear, Nose, and Throat doctor (ENT)

If you have any of the problems listed below, please see a healthcare professional

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.

- You saw blood, pus or fluid coming out of your ear within the past 6 months.
- Your ear feels painful or uncomfortable.
- You have a lot of earwax or you think something could be in your ear,
- You get dizzy or have a feeling of spinning or swaying (called vertigo).
- Your hearing changed suddenly in the past 6 months.
- Your hearing changes: it gets worse then gets better again.
- You have worse hearing in one ear.
- You hear ringing or buzzing in only one ear.

You should see a hearing health care professional if you:

- Have a visible deformity of the ear.
- Have a current ear infection or a history of active discharge from one or both ears within the past 90 days.
- Have sudden or rapid progression of hearing loss within the past 90 days in one or both ears.
- Have acute or chronic dizziness, poor dexterity, poor vision, or significant dementia.
- Suspect that you have significant ear wax accumulation or a foreign object in the ear canal. Symptoms of significant ear wax accumulation can include itching in your ear, a feeling of fullness in your ear, and/or reduced hearing.
- Experience pain or discomfort in the ear.
- Have a noticeable difference in hearing between ears.
- Have sudden onset or rapid worsening of tinnitus (ringing in the ear) in one or both ears within the past 90 days.

It is good health practice for a person with hearing loss to have a medical evaluation by a licensed physician, preferably a physician who specialises in ear, nose, and throat (ENT). Licensed physicians who specialise in diseases of the ear are often referred to as otolaryngologists, otologists, or otorhinolaryngologists. The purpose of a medical evaluation is to assure that all medically treatable conditions that may affect hearing are identified and treated.

- Avoid unauthorised modifications to your Nuance Audio™ Plus and the Nuance Audio™ Charging Pad.
- Remove Nuance Audio™ Plus if you encounter overly loud sounds or if they become uncomfortable.

-  Check before using Nuance Audio™ Plus and the Nuance Audio™ Charging Pad in areas where electronics or wireless devices are restricted. This may cause interference.
- Do NOT resell your Nuance Audio™ Plus or share them with others. They are intended for use by a single user.
- Keep the Nuance Audio™ Plus and the Nuance Audio™ Charging Pad away from fire, heat sources and excessive heat (e.g., from storage in direct sunlight, near candles, or in a hot car) to avoid burns or damage to the Nuance Audio™ Plus and/or the Nuance Audio™ Charging Pad.
-  Do NOT submerge the Nuance Audio™ Plus and the Nuance Audio™ Charging Pad in water or liquids. Do NOT wear the Nuance Audio™ Plus in the shower, or wear while participating in water sports (e.g., swimming, water-skiing, surfing, etc.). Exposure to water or liquids may damage the devices and may pose a risk of injury. In case of water exposure/contact, dry the Nuance Audio™ Plus and/or the Nuance Audio™ Charging Pad thoroughly before further use.
- Against dust and water, Nuance Audio™ Plus have a protection level of IP54. The IP5 rating signifies protection against dust in quantities not sufficient to compromise the operation of the equipment and the IPX4 rating signifies protection against splashes of water. The Nuance Audio™ Charging Pad is not water resistant.
- The sound output should not be uncomfortable or painful. You should turn down the volume or remove your Nuance Audio™ Plus if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your Nuance Audio™ Plus. See section 'Nuance Audio™ App Setup and Bluetooth® Pairing' for instructions.
- Nuance Audio™ Plus and the Nuance Audio™ Charging Pad shouldn't overheat or cause thermal discomfort during normal use. If they do, stop using them immediately and contact support.
- Check applicable aviation security restrictions on personal electronics before flying with Nuance Audio™ Plus and the Nuance Audio™ Charging Pad. Nuance Audio™ Plus contains lithium-ion batteries and should not be placed in checked luggage.
- Use Nuance Audio™ Plus and the Nuance Audio™ Charging Pad in compliance with all the applicable local laws, rules and regulations. Use common sense and respect third parties' rights,

privacy, and confidentiality.

- Interacting with Nuance Audio™ Plus may distract you or increase the time it takes to react to hazards. Always pay attention to your surroundings, including signage and obstacles.
- Use caution when operating a vehicle while wearing Nuance Audio™ Plus and keep your focus on driving. Nuance Audio™ Plus are an air-conduction product intended for individuals aged 18 or older with perceived mild to moderate hearing loss. Follow local laws and requirements applicable to mobile phone operation and use of other devices while driving, as well as the rules governing any requirement to use hearing devices while driving, as established by the relevant legislation and the authorities responsible for its enforcement. Adjust all relevant settings of your Nuance Audio™ Plus by selecting from the available options so that you can adequately perceive surrounding sounds. If you notice that wearing Nuance Audio™ Plus may distract you or affect your attention, do not wear them while driving. This warning does not concern visual correction: wear any corrective lenses you may need to drive safely. Adjust all relevant Nuance Audio™ Plus settings before driving, and do not interact with them or any of the accessories when the vehicle is in motion. Please note that when the "phone call" function is active, the sound amplification features, both frontal and 360°, cannot be activated or remain active.
- Nuance Audio™ Plus are intended to be worn for extended periods. If you feel any discomfort (for example, headaches, nausea or other abnormalities, etc), take a break from using the electronic functionalities and the Nuance Audio™ App for as many minutes as needed. If discomfort persists, stop using your Nuance Audio™ Plus and consult a healthcare professional.
- Nuance Audio™ Plus can be charged only with a Nuance Audio™ Charging Pad.
- Use only with original accessories and with the compatible USB-C cables and wall plugs / adaptor indicated below. Using non approved accessories may permanently damage Nuance Audio™ Plus, void the Luxottica Limited Warranty, and increase the risk of injury or damage to other property.
- Use only with the authorised Nuance Audio™ App. Update the Nuance Audio™ App and the Nuance Audio™ Plus firmware when prompted. Do not attempt to modify the software provided. Software is provided as is and is subject to terms and conditions.
- Features, functionality, and content of your Nuance Audio™ Plus are subject to change or removal

at any time, may be protected by digital rights management technology, may be unavailable or restricted in some areas, may depend on wireless service plan or internet service provider, and may require subscriptions or fees. Nuance Audio™ Plus may require software updates to be installed from time to time, including prior to first use. Detailed system requirements, software terms of license and terms of service are available at www.nuanceaudio.com.

- When receiving a phone call, the hearing aid's primary amplification function is temporarily disabled.
- In this mode, the hearing aid streams the phone call audio directly, but it does not amplify surrounding environmental sounds.
- This may reduce your awareness of ambient sounds.
- Once the call ends, the amplification function will be automatically restored.
- For use in critical environments or situations requiring full auditory awareness, avoid using the call streaming mode.

Note: If you try this device and continue to struggle with or remain concerned about your hearing, you should consult with a healthcare professional.

3.4. Preventing damage and malfunctions to your Nuance Audio™ Plus and the Nuance Audio™ Charging Pad

- Avoid exposure of your Nuance Audio™ Plus and the Nuance Audio™ Charging Pad to extreme temperatures and intense heat sources (hairdryer, heater, etc).
- Avoid exposure in direct sunlight.
- Avoid exposure to sand and fine particles.
- Do not use the devices if there are visible sharp edges, cracks, or damage to the housing.

3.5. Recommended Use and Storage

-  Operating conditions: -10°C to 35°C, 101.3kPa to 61.9kPa; humidity between 20% and 60% under 35°C for 1000 hours; maximum altitude of 13,000 ft for 1 hour at 0°C and 35°C.
- Transport and Storage conditions: Nuance Audio™ Plus shall remain functional after recovery from minimum -30°C or maximum 75°C storage temperatures; high humidity 90% and temperature 35°C for 288 hours; exposure to altitude of 40,000 ft (18,8kPa) for 12 hours at both -10°C and 35°C.

3.6. Cyber Security

EssilorLuxottica has assessed Nuance Audio™ Plus from a Cyber Security perspective, in accordance with applicable regulation. A full risk assessment of the device in regard to cyber security revealed no vulnerabilities impacting safety or efficacy with the product. Based on this assessment, the only action required by the user to ensure the product remains secure is to keep the product updated with the latest product updates. For information on how to keep your product up to date, please see "Updating your Nuance Audio™ Plus" in the mobile App. In the unlikely event that your Nuance Audio™ Plus have been compromised from a cyber security perspective, please reach out to us immediately at csirt@essilorluxottica.com for assistance. **In the event of a system compromise, EssilorLuxottica will issue official communications through its institutional channels.**

Proper operation of Nuance Audio™ Plus requires a safe, private, and secure operating environment. To ensure your security, we recommend following best practices for securing your home network and mobile device.

4. WHAT'S IN THE BOX

- 1 Nuance Audio™ Plus
- 1 Nuance Audio™ Foldable Case
- 1 Cleaning Cloth
- 1 Nuance Audio™ Charging Pad
- 1 USB-C Cable

The Box also includes:

- 1 Quick Start Guide
- 1 Instructions For Use - Safety & Warranty

5. OPENING THE BOX

- Ensure the accessories mentioned above are included in the package.
- Scan the QR Code printed on the inside lid of the box and download the Nuance Audio™ App.
- Extract the User Guide from the envelope inside the box's drawer and follow the instructions.
- Do not use the Nuance Audio™ Plus until the Nuance Audio™ App has instructed you to do so and

until you have read the User Guide indications.

6. YOUR NUANCE AUDIO™ PLUS

A - Volume


B - LED

C - On/off

D - Microphones

E - Speakers

Fig. 1 Schematic Diagram of The Nuance Audio™ Plus

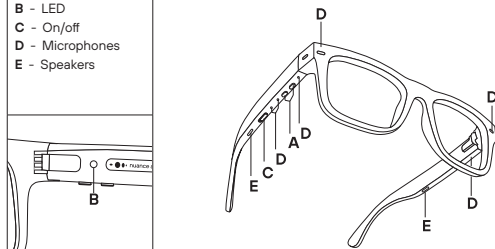Nuance Audio™ Plus are considered an air conduction medical device within a prescription spectacle frame. The Nuance Audio™ Plus ' accessories are:

- Nuance Audio™ Charging Pad: intended to enable the air conduction Nuance Audio™ Plus functionality in terms of charging the Nuance Audio™ Plus. The Nuance Audio™ Charging Pad has a LED on top view, to give info to the user regarding the charging status of the Nuance Audio™ Plus, and it has no battery. Please refer to the 'Charging the Battery' section of the User Guide for images and usage of the Nuance Audio™ Charging Pad.
- USB-C Charging Cable: to be connected to the Nuance Audio™ Charging Pad.
- Nuance Audio™ Foldable Case: intended to be used to carry the Nuance Audio™ Plus during the

day, with the main functionality to be protective and easily carried by the user. The Nuance Audio™ Foldable Case does not contain any electronics.

- Cleaning Cloth: to be used for cleaning the lenses.
- Nuance Audio™ App: intended to specifically enable the air conduction Nuance Audio™ Plus in terms of activating, personalising, and controlling settings on the Nuance Audio™ Plus. The Nuance Audio™ App is to be used only in conjunction with the Nuance Audio™ Plus, via Bluetooth® wireless connection with a mobile phone. Use of the Nuance Audio™ App requires a mobile phone with Android (12 and above) or an iPhone with iOS (16.6.1 and above) operating systems and internet access. It is possible to use the Nuance Audio™ Plus ' main functions without the Nuance Audio™ App, using the button and the tap area on the Nuance Audio™ Plus themselves. The Nuance Audio™ App is considered mandatory for audio calibration and firmware updates, and it can also serve as a digital remote for the device.

7. NUANCE AUDIO™ APP SETUP AND BLUETOOTH® PAIRING

Nuance Audio™ Plus are a wireless Bluetooth® medical device intended to be used for wirelessly listening to the world around you.

Download the Nuance Audio™ App by scanning the QR Code located on the inside lid of the box or on the first page of the User Guide, or by accessing the Nuance Audio™ App via the Apple® App Store or Google Play® Store. Once installed, open the Nuance Audio™ App to begin the setup process.

All setup instructions will be in the Nuance Audio™ App to guide you. Do not start using your Nuance Audio™ Plus until the Nuance Audio™ App has instructed you to do so. Proceed with pairing your Nuance Audio™ Plus and Bluetooth® set up using the Nuance Audio™ App.

Android®, Google Play® and the Google Play® logo are registered trademarks of Google LLC.

The Apple® wordmark, the Apple® logo, App store and iPhone® is a trademark of Apple Inc., registered in the U.S. and other countries. The trademark iPhone® is used with a license from Apple Inc. Apple Inc. is not Bluetooth® for the operation of this device or its compliance with safety and regulatory standards. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks is under license. Nuance Audio™ is a registered trademark of Essilor International SAS.

Note: if you already use the Nuance Audio™ App for your previous Nuance Audio™ Plus, you can use the same account and app for Nuance Audio™ Plus (skip to 'First pairing' instructions).

7.1. Onboarding the Nuance Audio™ Plus

Onboarding is performed the first time you access the Nuance Audio™ App. The primary purpose of the onboarding section of the Nuance Audio™ App is to systematically walk you through the setup and configuration of the Nuance Audio™ Plus.

Creation of the Nuance Audio™ user account

User authentication is mandatory to use the Nuance Audio™ mobile app. The first action is to complete the sign-in process starting from the authentication / sign-in page. The user is prompted to input an email address to be used as login ID and password, together with some personal information. Email address is verified through a onetime code sent via email to the email address used in the account creation process. The user must type in the code to complete the sign-in. If the onetime verification code is not received, the user may ask to send a new one.

First pairing

Once the sign-in process is completed and the user executes the login on the Nuance Audio™ App, everything is ready for the first pairing procedure. Nuance Audio™ Plus must be set in pairing mode before starting the scanning from the mobile app, tapping on the "Pair my Nuance Audio™ Plus " button and selecting the glasses type (3 buttons for Nuance Audio™ Plus) .

Calibration

Personal calibration is the procedure needed to customise the Nuance Audio™ Plus performance to the user. It is a three-step process lasting no more than a few minutes:

- **Step 1:** Measurement of the noise level of the room where the tuning is executed
- **Step 2:** Feedback tuning
- **Step 3:** User's voice calibration.

The calibration procedure can be repeated any time from the mobile app Settings menu.

Nuance Audio™ Plus tutorial

The tutorial on how to use the Nuance Audio™ Plus is displayed right after the personal calibration. If

skipped, the same tutorial is always available in the Support session of the mobile app.

App Calibration

Once the Personal calibration is completed, the user may access the mobile app tutorial that is also available for later access in the Support page of the mobile app.

7.2. Application Main Page

The application main page, also called home page, is displayed in the Nuance Audio™ App when Nuance Audio™ Plus are turned on and correctly paired with the mobile phone.

- Set volume level of the background noise (lowered, standard, heightened)
- Set the volume of the speakers, 5 levels available;
- Check the battery charge level.
- Measure the environmental noise
- Select the preferred Preset configuration (A, B, C, D)
- Select the audio mode between Frontal or All Around

Battery Level

A battery level indicator is displayed in the top left side of the home page. When the indicator turns red, it is time to charge the Nuance Audio™ Plus.

Noise Tracking

Noise tracking measures the room noise using microphones of the Nuance Audio™ Hearing Aid.

Background noise

From the background noise command in the main page the user can select the level of background noise filtering from Lowered, Standard or Heightened.

Volume

Volume setting, 5 levels, is also available in the home page controls. By tapping on the plus or minus icons the user can adjust volume.

Presets

Four different Presets are designed to personalise user's hearing experience, making speech perception clearer and improving listening comfort tailored to their preferences.

Audio Mode

The audio mode allows the user to choose their desired listening focus based on their environment. Frontal mode enhances sound coming from the front, making it easier to focus on conversations with someone directly in front. All-around mode amplifies sounds and speech from all directions.

7.3. Menu

Nuance Audio™ Plus selection and add new Nuance Audio™ Plus

This menu item gives the user the ability to switch between all Nuance Audio™ Plus devices linked to their user profile and paired with the mobile phone. Up to a maximum of 4 Nuance Audio™ Plus can be associated to a single profile. It is also possible to add new Nuance Audio™ Plus to the profile by initiating a new pairing process.

My Account Personal Information

The personal information section contains profile information such as full name, email, phone number, date of birth, and gender.

Password

In the password section it's possible to change the password by entering the current password and the new one. Accepted password format is: 8 digits, 1 upper case, 1 lower case, 1 number, 1 special digit.

Language Preferences

The application language is automatically set to the same language of the mobile phone if supported, otherwise the default is English. The user can select a new language from this configuration page.

Push notifications

The user can enable push notification from this session and can also flag which category the user is interested in. Available notifications categories are FW upgrades or suggestions on how to better configure the Nuance Audio™ Plus.

Newsletter

In the newsletter section, the user can choose whether to enable commercial emails and/or whether to enable tracing of activities and configurations applied to the Nuance Audio™ Plus.

Legal documents

All documents regarding privacy and data processing can be found in the legal documents section.

Delete account

The user can decide to permanently delete the account and all the associated data. After this operation the user will no longer be able to log into the Nuance Audio™ Plus mobile application.

Log out

Clicking the log out button disconnects the account from the application.

7.4. Settings:

My own voice

The user can select how much they want to hear their own voice. Possible choices are: No, Yes.

Personal calibration

From this Settings menu item, the user can execute a new Personal Calibration operation or switch back to the default one which is the same for all Nuance Audio™ Plus users.

Nuance Audio™ Plus - My Glasses

This section of the Settings menu includes all configuration activities for Nuance Audio™ Plus that the user can execute.

My Glasses: Edit name

The user can customise the Nuance Audio™ Plus name by overwriting the default name.

My Glasses: Configuration and Firmware (FW) upgrade

This section contains information on the FW version of the Nuance Audio™ Plus and prompt the user if a new FW version is available.

FW upgrade can be executed only if the charging level of the battery is above 20%. During the FW upgrade operation, it is mandatory to keep the mobile phone nearby the Nuance Audio™ Plus. At the end of the upgrade the Nuance Audio™ App will wait for the Nuance Audio™ Plus to reboot with the new firmware.

My Glasses: Detailed information

This section of the My Glasses page contains all the detailed information on the Nuance Audio™ Plus such as the Serial Number and the Model.

My Glasses: Forget these Glasses

This function removes the link between the user account profile and the paired Nuance Audio™ Plus. This operation is mandatory if the very same pair of Nuance Audio™ Plus needs to be paired and associated with a different user profile of phone.

No undo is available with this operation, but the user is prompted for confirmation. Mobile phone Bluetooth association with the Nuance Audio™ Plus must be also deleted from the mobile phone Bluetooth setting page.

My Glasses: Factory reset

Return the Nuance Audio™ Plus to the factory configuration. All data stored in the Nuance Audio™ Plus and the association with the user account is cleared.

No undo is available with this operation, but the user is prompted for a confirmation. Mobile phone Bluetooth association with the Nuance Audio™ Plus must be also deleted from the mobile phone Bluetooth setting page.

Shortcuts

List of configurations allows the user to personalise his glasses according to his preferences.

7.5. Phone Streaming

There is an option to receive phone calls via the Glasses:

- This option is available only for iPhones with iOS operating systems
- Configurable via the operating system settings, by default this option is enabled
- If this option is enabled, while the glasses are on and connected to the mobile phone, phone calls will be streamed via the Glasses (incoming call sound notification and the calling itself)
- Answer/End calls is possible via double tap if this option enabled
- Other phone sounds (media, notifications, etc.) are streamed if this option is enabled via the operating system settings

The device also might support media streaming functionality for the playback of multimedia content.

Default audio mode

Configuration of the preferred Audio Mode when power on or wake the glasses

7.6. About

This page contains all the information on the application such as the Nuance Audio™ App version

number and other product codes. For the Nuance Audio™ App label, please check the Nuance Audio™ App details page.

7.7. Support

The Support section of the mobile app includes:

- Access to the Nuance Audio™ Plus tutorial
- Access to the mobile app tutorial
- Access to the Nuance Audio™ Remote Plus tutorial
- Link to the FAQ pages on www.nuanceaudio.com
- Link to the Troubleshooting pages on www.nuanceaudio.com
- Link to the Educational resources on www.nuanceaudio.com
- Link to the Contact Us section of www.nuanceaudio.com
- Link to the Quick Start Guide on the www.nuanceaudio.com website

8. NUANCE AUDIO™ PLUS CARE AND MAINTENANCE

8.1 Battery information

Nuance Audio™ Plus contains a lithium-ion battery that is not user-replaceable. Do not attempt to remove or replace the battery. The battery may present a fire or chemical burn hazard if misused. Do not disassemble, crush or puncture Nuance Audio™ Plus. Do not leave or store Nuance Audio™ Plus under direct sunlight or expose to excessive heat or incinerate. Dispose of Nuance Audio™ Plus in line with instructions and local waste requirements.

The battery contained in Nuance Audio™ Plus is expected to perform as follows, provided the Nuance Audio™ Plus are always used under Normal Use Conditions (see below) (the "Expected Battery Performance"):

- at original capacity, and once fully charged, the battery is expected to provide 10 hours of continuous use under average conditions (where there is moderate surrounding noise and the standard amplification settings are being used).
- Assuming that you do, approximately, 1 full charge per day: following 500 full charging cycles (corresponding to approximately 16 months of use) the battery is expected to perform at or above 85% of its original capacity, meaning the battery will allow for 8 hours and 30 minutes of continuous

use under average conditions.

- following 1,000 full charging cycles (corresponding to approximately 32 months of use), the battery is expected to perform at or above 80% of its original capacity, meaning the battery will allow for 8 hours of continuous use under average conditions.

"Normal Use Conditions" means:

- Operating the product in line with this user manual, in particular Section 3 (Warnings, Precautions and Safety Considerations), which includes operating the product within the recommended temperature range.
- Not exposing the product to extreme heat, cold or humidity, as indicated in Section 12 (Nuance Audio™ Plus technical specifications key features).
- Using the product only for its intended use, as described in Section 1.1 (Intended Use).
- Using only the provided or certified charging accessories as indicated in Section 8.3 (The Charging Pad works thanks to a USB-C charging cable).
- Ensuring the battery is charged and discharged in accordance with the guidelines provided at Section 8.2 below.
- Always making sure that Nuance Audio™ Plus is using the latest firmware version.
- Charging the battery at room temperature instead of a lower or higher temperature.
- Charging the battery only when necessary.
- Making sure to charge the battery fully before unplugging it.

Luxottica guarantees the above battery performance for the applicable Limited Warranty Period. Please refer to the "Warranty" section.

Battery performance may be extended if Nuance Audio™ Plus are turned off when amplification is not needed, or when the surrounding environment requires less amplification and/or noise reduction.

8.2. Charging the battery


IMPORTANT! If battery is not fully recharged every 6 months from last use, permanent damage to the battery may occur, which could render the Nuance Audio™ Plus inoperable.

When the Nuance Audio™ App displays a battery level of 20% or less, or when the low battery alert

sounds in your ear, the Nuance Audio™ Plus should be charged. For charging, place the Nuance Audio™ Charging Pad flat on a desktop to ensure it does not shake and place the Nuance Audio™ Plus flat on the Nuance Audio™ Charging Pad. Ensure that the Nuance Audio™ Plus are aligned with the magnetic components on the Nuance Audio™ Charging Pad. If the alignment is not accurate, you can try adjusting the angle of the Nuance Audio™ Plus. During the charging process, do not move or vibrate the Nuance Audio™ Plus, as this may impact the charging process.

Battery level is indicated via the Glasses LED once charging is stopped by removing the Nuance Audio™ Plus from the Charging Pad **(Fig. 2)**

- Red color – battery level is below 20%
- Orange color – battery level is above 20% and below 89%
- Green color – battery level is above 90%

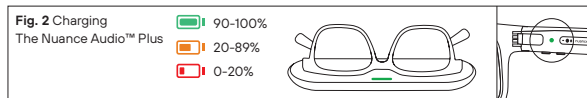8.3. The Nuance Audio™ Charging Pad works thanks to a USB-C Charging Cable

 Use only the provided Nuance Audio™ Charging Pad to charge Nuance Audio™ Plus. Use the supplied USB-C Charging Cable or other USB-C type cable, with a USB wall plug/adaptor (not included). It is strongly discouraged to connect the Nuance Audio™ Charging Pad to a wall plug/adaptor that is not certified for the market in which the Nuance Audio™ Charging Pad is marketed, and that it has not been tested and approved according to national and local standards. The wall plug/adaptor must have a minimum output level of DC5V/1A.

- The Nuance Audio™ Charging Pad may become warm to the touch while charging. If the Nuance Audio™ Charging Pad becomes hot to touch, discontinue use.
- If the Nuance Audio™ Charging Pad cannot charge properly or experiences any malfunctions,

please stop using it immediately.

- Charging Pad possible errors indications:
 1. Charging Error – the LED on the charging pad is blinking red.
 2. Startup Failure – the LED on the charging pad is blinking red for 3 sec.
- Do not clean the Nuance Audio™ Plus and the Nuance Audio™ Charging Pad while you are charging them.
- Periodically inspect the Nuance Audio™ Plus, the Nuance Audio™ Charging Pad and the Charging Cable for damage or signs of wear. Do not use if any part is damaged, cracked, or if any internal component is exposed.
- Do not try to repair Nuance Audio™ Plus or its accessories, or replace parts yourself. Instead contact support at www.nuanceaudio.com

8.4. Caring for Nuance Audio™ Plus

Important Note: We do not recommend using alcohol wipes when caring for your Nuance Audio™ Plus.

8.5. Nuance Audio™ Plus General Maintenance

- **Handle with care & avoid damage.** Follow the steps under the General Information section below to help maintain quality and performance of your Nuance Audio™ Plus.
- Inspect Nuance Audio™ Plus regularly for particle build up and follow cleaning procedures.
- Only wipe Nuance Audio™ Plus with a soft cloth. Don't use any other cleaning tool (such as a hearing aid brush, toothpick, or air pressure) to clean Nuance Audio™ Glasses. Do not use dirty or abrasive cloths which could alter the lenses' features. Moisture can damage Nuance Audio™ Plus or cause them to fail. To clean the lenses, use a lens spray, available in stores. Make sure to spray on a cloth and not directly on the lenses.
- To clean Nuance Audio™ Plus, wipe with a dry, clean cloth. Avoid all types of sprays, solvents, chemicals, or cleaning solutions containing alcohol or ammonia.

8.6. General Information

- Nuance Audio™ Plus and the Nuance Audio™ Charging Pad are sensitive electronic equipment and should be handled with care. Do not heat or bend Nuance Audio™ Plus and the Nuance

Audio™ Charging Pad, including for fitting, as they may become damaged and even catch fire.

- Do not drop, strike, or shake Nuance Audio™ Plus and the Nuance Audio™ Charging Pad excessively.
- Do not use Nuance Audio™ Plus or the Nuance Audio™ Charging Pad if it is damaged.
- Do not attempt to open, disassemble, or tamper with Nuance Audio™ Plus or the Nuance Audio™ Charging Pad.
- Protect your Nuance Audio™ Plus against moisture (e.g. showers, rain), and take your Nuance Audio™ Plus out before using hairspray or similar cosmetic product.
- It is recommended to carry out daily care on Nuance Audio™ Plus.
- Keep Nuance Audio™ Plus dry.
- Should your Nuance Audio™ Plus still not function correctly despite regular care, contact us at www.nuanceaudio.com.
- Do not try under any circumstances to repair your Nuance Audio™ Plus by yourself. The serial number of the Nuance Audio™ Plus may be found marked on the inner part of the left temple.

General Maintenance for the Nuance Audio™ Charging Pad:

- Do not place it around high-power electrical fields audio, do not be in a strong magnetic environment.
- To clean the Nuance Audio™ Charging Pad, use a dampened cloth and mild soap avoiding all types of sprays, solvents, chemicals, or cleaning solutions containing alcohol or ammonia.
- Do not use dirty or abrasive cloths which could damage it.
- Keep the Nuance Audio™ Charging Pad dry. If the Nuance Audio™ Charging Pad becomes excessively wet, store it in a container with desiccants or in a drying case.

9. PRIVACY- In developing Nuance Audio™ Plus, EssilorLuxottica followed the principle of privacy by design. A full description of how Nuance Audio™ Plus and the mobile app linked process personal data can be found in the Privacy Notice within the App. To learn more about EssilorLuxottica Governance on personal data, you can visit the section "Governance" on www.essilorluxottica.com and click on "Compliance/Data Privacy".

10. TROUBLESHOOTING - For troubleshooting please check [/www.nuanceaudio.com/en-gb/c/support/troubleshooting](http://www.nuanceaudio.com/en-gb/c/support/troubleshooting)

11. NUANCE AUDIO™ PLUS TECHNICAL SPECIFICATIONS

KEY FEATURES

Processing Channels	16
WDRC	Yes
Feedback Management	Yes
Battery	Rechargeable, Li-ion
Programs	4 selectable 'presets'
iOS App	Yes
Android App	Yes
Volume Control	Yes via on device buttons or app
Microphone Pattern	Focused and Omni options
Self-Setup & Personalisation	Yes via app
Low Battery Alarm	Yes via LED, App & audio prompt
App Compatibility	iOS 16.6.1 and above (minimum memory requirement is 128GB), Android 12 and above (minimum memory requirement is 32GB)
App lifetime	The lifetime of Nuance Audio™ App has no upper limit as long as the user keeps his or her Android/iOS operating system up-to-date, the operating system version is one of those validated and declared as compatible by Luxottica (iOS 16.6.1 and later and Android 12 and later), and the operating system version used is a version still being maintained by the operating system vendor
Hands-free Calling	Yes

BATTERY AND CHARGING	
Integrated battery	3,87V/165mAh lithium-ion
Battery usage time	10 hours (with Nuance Audio™ Plus on, depending on conditions)
Battery cycles	Up to 500 full cycles, battery performance expected above 85% of its original capacity; up to 1000 cycles, above 80%
TECHNICAL SPECIFICATIONS	
Hearing Assistance	"Own Voice" reduction Background noise reduction
Microphone array	6 microphones
Lifetime	At least 3 years
Shelf life	800 days
Waterproofing	IPX4 rated
Dustproofing	IP5 rated
OPERATING CONDITIONS	
Temperature	From -10°C to 35°C
Humidity	From 20% to humidity 60% under 35°C for 1000 hours
Altitude & atmospheric pressure	Altitude of 13,000 ft for 1 hour at 0°C and 35°C; 101.3kPa to 61.9kPa

TRANSPORT AND STORAGE CONDITIONS	
Temperature	From -30°C to 75°C
Humidity	From 20% to high humidity 90% and temperature 35°C for 288 hours
Altitude & atmospheric pressure	Altitude of 40,000 ft (12,192 m) (18,8kPa) for 12 hours at both -10°C and 35°C
Time to warm from minimum storage temperature until it is ready for intended use	Approximately 15 minutes
Time to cool from the maximum storage temperature until it is ready for intended use	Approximately 10 minutes
WIRELESS CONNECTIVITY	
Bluetooth® Compatible	Yes, BLE v5 and above
Bluetooth® Phone Call Compatible	Yes, Bluetooth/MFI
Operation frequency	2402-2480 MHz (RF)
Max radiated power	Free space max EIRP is 2,21 dBm
Modulation	Gaussian frequency shift keying (GFSK)
Channel spacing	2MHz
Channel bandwidth	2MHz
Number of channels	40

WIRELESS CHARGING	Yes
Transmitting frequency	13,56 MHz (RF)
Receiving frequency	13,56 MHz
Max radiated power	-8.88 dBuA/m at 10m
Modulation type transmitting	Amplitude shift keying (ASK) 100% modified Miller
Modulation type receiving	Load Modulation subcarrier fc/16 (848kbit/s) on-off keying OOK Manchester
FITTING AND PROGRAMMING	Not intended to be a fitted device or programmed
Nuance Audio™ App	Yes, for set up
Electroacoustic Characteristics (ANSI S3.22:2014 & CTA-2051:2017)	
Medical Device Hearing Aids	
Frequency Band	Standard Band Device
Frequency Response Bandwidth	200Hz – 6300 Hz
Equivalent Input Noise	32 dB
Total Harmonic Distortion	<1%
Maximum OSPL90	111.9 dB
HFA OSPL90	101.9 dB
Maximum Full-On-Gain	22.2 dB
HFA Full-On-Gain	12.1 dB
Input Distortion at 80 dB SPL	0.8 %
Latency	13.9 ms

Guidance and manufactures declaration for electromagnetic immunity		
Emission standard	Type	Compliance level
EN/IEC61000-4-2	Electrostatic discharge (ESD)	± 4kV, ± 8kV contact ±2kV, ±4kV, ±8kV, ± 15kV air
Specific guidance for application environment		
Suitable for use on wood, concrete, or ceramic floor materials. Keep relative humidity below 30% when used on floors of syntitic materials e.g. carpets.		
Emission standard	Type	Compliance level
EN/IEC61000-4-8	Power frequency magnetic field	30 A/m 50/60 Hz
Specific guidance for application environment		
Suitable for in proximity to equipment mains connected to public low-voltage network supplying domestic environment. Portable and mobile radio devices, including their wires, should not be used closer to the unit than the recommended safe distance or 30 cm. The level for Power frequency magnetic fields corresponds to the levels typical for professional or domestic Environment.		

Emission standard	Type	Compliance level	
EN/IEC61000- 4- 3	RF electromagnetic fields. Proximity fields from RF wireless communication equipment.	80 MHz to 2.7 GHz, 80% AM at 1 kHz, 10 V/m	
		Test freq. MHz	V/m
		385	27
		450	28
		710, 745, 780	9
		810, 870, 930	28
		1720, 1845, 1970	28
		2450	28
		5240, 5500, 5785	9
Specific guidance for application environment			
Portable and mobile radio devices, including their wires, should not be used closer to the unit than the recommended safe distance or 30 cm.			
Emission standard	Type	Compliance level	
EN/IEC61000- 4- 39	IMMUNITY to proximity magnetic	Test frequency	A/m
		30 kHz	8
		134.2 kHz	65
		13,56 MHz	7.5
Specific guidance for application environment			
The device is tested to function in close proximity to powerful magnetic fields, e.g. from RFID equipment. The test levels ensure a safe distance of 15 cm.			

The Nuance Audio Plus was tested according to recommendations of IEC TS 60601-4-2:2024 Medical electrical equipment-Part 4-2: Guidance and interpretation – Electromagnetic immunity: performance of medical electrical equipment and medical electrical systems.

Phenomenon	Basic EMC standard	Immunity Test Level	Performance criteria
ELECTROSTATIC DISCHARGE	IEC 61000-4-2	±4kV contact ±2kV,±4kV,±8kV air	Criterion B
Radiated RF EM field	IEC 61000-4-3	3V/m 80MHz to 2,7GHz 80%AM at 1kHz	Criterion A
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Test freq. MHz	Criterion A
		V/m	
		385	
		450	
		710, 745, 780	
		810, 870, 930	
		1720, 1845, 1970	
		2400	
		5240, 5500, 5785	
RATED power frequency magnetic fields	IEC 61000-4-8	3A/m,50Hz or 60Hz	Criterion A
NOTE: a) Criterion A: The ME EQUIPMENT or ME SYSTEM should continue to meet the performance criteria during and after the following tests, without the need for OPERAT or intervention to restore operation. This also applies to tests for other continuous phenomena that might be applicable to the ME EQUIPMENT or ME SYSTEM.			

b) Criterion B: The ME EQUIPMENT or ME SYSTEM should continue to meet the performance criteria after the following tests, without the need for OPERAT or intervention to restore operation. The MANUFACTURER should specify in the instructions for use the maximum recovery time. While operation as intended during the test is desirable, some degradation is acceptable. This also applies to tests for other TRANSIENT phenomena that might be applicable to the ME EQUIPMENT or ME SYSTEM.

12. CLINICAL BENEFIT

The clinical benefit of Nuance Audio™ Plus is the improvement in speech intelligibility when compared to no intervention.

13. INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS

Luxottica is committed to the proper reuse, recycling and recovery of batteries and Waste Electrical and Electronic Equipment ("WEEE").

Batteries and WEEE should not be discarded in normal domestic waste bins.

Discard the Product and the battery according to local regulations. Do not discard as household waste.

Bring them to designated collection points or in alternative to an affiliated store.

For more info on proper collection, please visit www.nuanceaudio.com.

INFORMATION ON DISCARDING

The crossed-out wheeled bin symbols below mean that, according to local laws and regulations, The Nuance Audio™ Products, shall be discarded separately from household waste. When The Nuance Audio™ Products, reaches its end of life, bring them to a collection point designated by local authorities. In alternative, bring the Product and the battery to a affiliated store. The separate collection and recycling of the Nuance Audio™ Products, will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Crossed-out wheeled bin (Batteries):

Do not discard of batteries with household waste. Collect separately for proper recycling (Battery Regulation (EU) 2023/1542).

Crossed-out wheeled bin (WEEE):

Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).

Discarding Information for France only

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14. REGULATORY COMPLIANCE INFORMATION

14.1. EU compliance statement

Luxottica Group S.p.A hereby declares that these devices are in compliance with Radio equipment Directive 2014/53/EU and EU medical device Regulation 2017/745 on medical devices, and Directive 2011/65/EU on RoHS. The full text of the EU Declaration of Conformity is available at the following internet address: www.essilorluxottica.com/en/brands/conformity-declaration

Radio Frequency and Power:

BLE of the Product: 2402–2480 MHz < 20 dBm;

Note: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. Keep the product away from the womb of pregnant women. Keep the product away from the lower abdomen of adolescents. Use the product in good reception conditions to reduce the amount of electromagnetic radiation received. Information on energy

consumption in accordance with Regulation (EU) 2023/826 is available online at www.nuanceaudio.com in the FAQ page or in the mobile app under Support, instructions link.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

Country	Type Approval	Regulatory Mark/Logo
Global	The Nuance Audio™ Plus comply with the medical devices: IEC 60601-1:2005 Ed3+C1, C2, A1, IEC 60601-1-6, IEC 60601-1-11, IEC 60601-1-2, 4th Edition, FCC 47 Part 15.247 Classification: Type B Applied Part, IEC 60601-2-66, this Nuance Audio™ Plus have no essential performance. The testing performed above has been in accordance with IEC 60601-1-2:2014 (4th Ed). The values identified during this evaluation were found to be within acceptable specified ranges.	

15. SYMBOLS AND DESCRIPTIONS

	Federal Communications Commission (U.S.)		Manufacturer
	Bluetooth®		Type B applied part
	Review operating instructions prior to use		Operating Instructions

	Warning		Made in China
	Serial Number		Patient Information Website
	Medical Device		Keep Dry
	MR Unsafe		Temperature Limit
	Single patient, multiple use		Atmospheric pressure limitation
	Date of manufacture / Manufacturer		Humidity limitation
	Model Number	IP54	IP54 rating
	Do not heat or bend the Nuance Audio™ Plus		Do NOT use if package is damaged
	Contains small parts which may be choking hazard. Not suitable for children under the age of 3		To ensure battery health, recharge before the date on the packaging
	Magnetic field warning. Possible interference with cardiac pacemakers, implanted defibrillators (ICD's) and other implants		
	Crossed-out wheeled bin (Batteries): Do not discard of batteries with household waste. Collect separately for recycling (Battery Regulation (EU) 2023/1542).		
	Crossed-out wheeled bin (WEEE): Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).		

Use of the Made for iPhone badge means that an accessory has been designed to connect specifically to iPhone models identified in the badge and has been certified by the developer

to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

Compatible with iOS smartphones running iOS 16 or later, starting from version 0.3.7 and above. Specifically, for: iPhone 17 Pro Max, iPhone 17 Pro, iPhone 17, iPhone 16 Pro Max, iPhone 16 Pro, iPhone 16 Plus, iPhone 16, iPhone 16e, iPhone 15 Pro Max, iPhone 15 Pro, iPhone 15 Plus, iPhone 15, iPhone 14 Pro Max, iPhone 14 Pro, iPhone 14 Plus, iPhone 14, iPhone 13 Pro Max, iPhone 13 Pro, iPhone 13 mini, iPhone 13, iPhone 12 Pro Max, iPhone 12 Pro, iPhone 12 mini, iPhone 12, iPhone 11 Pro Max, iPhone 11 Pro, iPhone 11, iPhone XS Max, iPhone XS, iPhone XR, iPhone X, iPhone 8 Plus, iPhone 8, iPhone SE (3rd and 2nd generation).

Luxottica Group S.p.A.

Piazzale Cadorna 3, 20123 Milan, Italy

Made in China

Fabriqu  en Chine

www.nuanceaudio.com

No part of this work may be reproduced, modified, distributed or otherwise used without prior written permission.

For the latest and up-to-date Instructions For Use - Safety and Warranty, please refer to the Instructions in the Support section of the Nuance Audio™ App and to the following website page:
nuanceaudio.com/support/manual-guide

Nuance Audio™ Plus e Nuance Audio™ Charging Pad

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LUXOTTICA LIMITED WARRANTY

NOT FOR CANADA

This limited consumer warranty (the "Warranty") is provided by Luxottica Group S.p.A., Piazzale Cadorna 3, 20123 Milano, Italy ("Luxottica"). This Warranty covers defects and malfunctions in the Luxottica Product(s) it accompanies (the "Product"). The Warranty continues for one (1) year from the date of purchase of the Product (the "Limited Warranty Period"). However, if you purchase the Product from within the EU, UK or EFTA, or Australia or New Zealand, the Limited Warranty Period shall be two (2) years. This Warranty applies only to those countries where Nuance is officially marketed and supported, meaning countries where the product is certified and Luxottica is authorized to distribute it ("Warranty Coverage Territory"). We warrant that the Product will, under normal and intended use, function in accordance with the technical specifications or accompanying Product documentation (the "Warranted Functionality") during the Limited Warranty Period. For the avoidance of doubt, any software or services that are required to achieve the Warranted Functionality may be updated, modified, or limited, so long as it continues to maintain (or exceed) the Warranted Functionality. This Warranty is not available to Products that were purchased from any source other than Luxottica or an authorised dealer. This Warranty is valid only with proof of purchase that clearly shows the purchase date. If you send us a Product without valid proof of purchase, then we reserve the right to return the Product to you at your cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up. This Warranty gives you specific legal rights, and you may also have other rights under consumer protection laws or regulations in the country where the product was purchased or, if different, where you are located, provided that this product is officially marketed there at the time the warranty service is needed. Such rights vary from state to state and country to country. The benefits conferred by this Warranty are in addition to and do not affect any rights and remedies conveyed by such consumer protection laws and regulations (including, without limitation, national laws implementing Directive 1999/44/EC and Directive 2019/771). You are entitled to remedies

free of charge in the event that the Product does not conform with the requirements set out under the laws in the Warranty Coverage Territory. The other remedies available to you according to the laws in the Warranty Coverage Territory are not affected by this Warranty. For full information on your rights according to the laws in the Warranty Coverage Territory, please see below and/or contact your local consumer rights organisation/citizens advice bureau. Please visit us at www.nuanceaudio.com for information about the countries Luxottica supports and to get helpful service and contact information. The scope of technical support offered by Luxottica consists of helping to diagnose and resolve problems with defects in the Product. Please note that when contacting Luxottica via telephone, long distance charges may apply, depending on your calling area.

REGISTRATION

Product registration is not required as a condition to coverage under this Warranty, but some Luxottica Products require periodic connection to an online account to ensure full functionality.

HOW TO MAKE A WARRANTY CLAIM

If you have a problem with your Product, unless specifically stated elsewhere in this Warranty and except as provided by applicable state law, to make a warranty claim, you need to return your Product to the authorised dealer from which the Product was purchased, together with your proof of purchase. Please visit www.nuanceaudio.com to get helpful service and contact information. If your warranty claim is deemed to be valid, and we confirm a defect or malfunction covered by this Warranty, the appropriate remedy will be determined. Depending on the circumstances, we may take one of the following actions: i) Repair the Product, only where repair is technically feasible (given the highly technological content of the Product, the only repair available is lens replacement); or (ii) Replace the Product; or (iii) Refund the purchase price paid for the Product, if neither repair nor replacement is possible or appropriate. Subject to your rights under applicable local laws of the Warranty Coverage Territory, the choice of remedy rests with the Company. Subject to your rights under any local laws, if we determine that a Product should be replaced, the replacement may be a new, refurbished, or a remanufactured Product. The Limited Warranty Period is extended by a duration equal to the time during which we had the Product in our possession for performance of the Warranty, as described above. Save for your rights and remedies available under

your local laws, the provision of a repaired or replacement Product does not restart or otherwise extend the Limited Warranty Period. We may not return the original Product to you. If your warranty claim is not deemed to be valid, then we reserve the right to return the Product to you at your cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up before it is disposed of. Except to the extent that the law requires otherwise, any expenses incurred by you in returning the Product for warranty assessment will be borne by Luxottica. If you need to return your product to Luxottica, you will be provided with a prepaid label. If the returned Product is deemed (at Luxottica's discretion) to be eligible for a valid warranty claim, then Luxottica will automatically incur the costs of any further freight charges required to return the repaired or replacement Product to you. If the warranty claim is deemed to be invalid and we do not find a defect or malfunction covered by this Warranty, we will contact you to see if you want us to conduct repairs at your cost and, in any event, you will be charged for the outbound freight to return your Product based on Luxottica's standard shipping rates.

LIMITATIONS AND EXCLUSIONS

THIS WARRANTY DOES NOT COVER, AND LUXOTTICA IS NOT RESPONSIBLE FOR: DELIVERY OR INSTALLATION, OR LABOR CHARGES FOR SETUP OF THE PRODUCT AND/OR ADJUSTMENT OF CUSTOMER CONTROLS ON THE PRODUCT. DAMAGES CAUSED BY MISUSE, ABUSE, ACCIDENTS, FIRE, THEFT, DISAPPEARANCE, MISPLACEMENT, FLUCTUATIONS AND POWER SURGES, CONNECTIONS TO IMPROPER VOLTAGE OR INCORRECT ELECTRICAL LINE VOLTAGE, VIRUSES, MALWARE, RECKLESS, WILLFUL, OR INTENTIONAL CONDUCT OR NEGLIGENCE. DAMAGES CAUSED BY SERVICING OF THE PRODUCT WHICH HAS NOT BEEN PRE- AUTHORISED BY LUXOTTICA. DAMAGES CAUSED BY USAGE THAT IS NOT IN ACCORDANCE WITH THE PRODUCT INSTRUCTIONS/ DOCUMENTATION. DAMAGES CAUSED BY FAILURE TO FOLLOW THE PRODUCT INSTRUCTIONS/ DOCUMENTATION OR FAILURE TO PERFORM CLEANING OR PREVENTIVE MAINTENANCE. DAMAGES CAUSED BY THE COMBINATION OF THE PRODUCT WITH OTHER NON-LUXOTTICA BRANDED PRODUCTS, ACCESSORIES, PARTS OR COMPONENTS OR USE OF PRODUCTS, EQUIPMENT, SYSTEMS, UTILITIES, SERVICES, SOFTWARE, PARTS SUPPLIES, ACCESSORIES, APPLICATIONS, INSTALLATIONS, REPAIRS, EXTERNAL WIRING OR CONNECTORS NOT SUPPLIED OR AUTHORISED

BY LUXOTTICA THAT DAMAGE THE PRODUCT. SIGNAL ISSUES, RECEPTION PROBLEMS AND DISTORTION RELATED TO NOISE, ECHO, INTERFERENCE OR OTHER SIGNAL TRANSMISSION AND DELIVERY PROBLEMS. ANY EQUIPMENT OR COMPONENTS THAT WERE NOT INCLUDED IN, OR WITH YOUR PRODUCT AS ORIGINALLY SOLD. NORMAL WEAR AND TEAR, INCLUDING LENS SCRATCHES. DAMAGE CAUSED AS A RESULT OF IMPROPER TRANSPORTATION OR PACKING/PACKAGING WHEN RETURNING THE PRODUCT TO LUXOTTICA OR AN AUTHORIZED DEALER. DAMAGES CAUSED BY MODIFICATION OR ADAPTATION THAT MAY BE REQUIRED TO ENABLE A PRODUCT TO OPERATE IN ANY COUNTRY OTHER THAN THE COUNTRY FOR WHICH IT WAS DESIGNED, MANUFACTURED, APPROVED AND/OR AUTHORISED, OR REPAIR OF THE PRODUCT WHICH RESULTS IN DAMAGE AS A RESULT OF THESE MODIFICATIONS. DAMAGE TO PRODUCTS WHERE THE ANTI-COUNTERFEITING CODE (OR EQUIVALENT) HAS BEEN REMOVED, ERASED, DEFACED, ALTERED OR MADE ILLEGIBLE. NON ACHIEVEMENT OF THE EXPECTED BATTERY PERFORMANCE, AS REFERRED TO IN SECTION 8.1, CAUSED BY USAGE THAT IS NOT IN ACCORDANCE WITH THE DEFINITION OF "NORMAL USE CONDITIONS" SET OUT IN THAT SECTION.

This Warranty is limited and not applicable to any lens scratches. The installation or replacement of any lenses** should be conducted by Luxottica or a Luxottica authorised dealer for Nuance Audio™. Any damage caused by the unauthorised installation, replacement or alteration of the lenses is not covered by this Warranty.

This Warranty does not include any specific guarantees that the Product will be error-free, or regarding uptime or continued availability of data security features of software or online accounts, that any software, firmware or online sites will function uninterrupted or error-free. Except in the US and to the extent prohibited by any applicable law in any other country, this Warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover).

IMPLIED WARRANTIES

EXCEPT IN THE EU, EFTA AND UK AND EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW IN ANY OTHER COUNTRY, ANY IMPLIED WARRANTY OF MERCHANTABILITY, FITNESS FOR A

PARTICULAR PURPOSE AND NON-INFRINGEMENT IS DISCLAIMED.

FOR THE US ONLY, ANY IMPLIED WARRANTIES THAT CANNOT BE DISCLAIMED ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTY PERIOD AS SET OUT IN THIS WARRANTY. IF YOU ARE A CONSUMER SUBJECT TO THE QUEBEC CONSUMER PROTECTION ACT, THE WARRANTY DISCLAIMER CONTAINED IN THIS SECTION DOES NOT LIMIT YOUR RIGHTS AND REMEDIES UNDER SUCH ACT, INCLUDING THE RIGHT TO MAKE A CLAIM UNDER ANY OF THE STATUTORY WARRANTIES PROVIDED UNDER SECTIONS 34 TO 54 OF SUCH ACT.

LIMITATION OF LIABILITY

EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW, LUXOTTICA SHALL NOT, UNDER THIS WARRANTY, BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES FOR BREACH OF ANY EXPRESS OR IMPLIED WARRANTY. LUXOTTICA'S MAXIMUM LIABILITY FOR ANY CLAIM ARISING OUT OF OR RELATED TO THIS WARRANTY IS LIMITED TO THE PURCHASE PRICE PAID BY YOU FOR THE PRODUCT PLUS ANY INTEREST ALLOWED BY LAW REGARDLESS OF THE FORM OF CLAIM. TO THE EXTENT PERMITTED BY LAW, LUXOTTICA IS NOT LIABLE FOR EVENTS BEYOND ITS CONTROL, SUCH AS ACTS OF GOD, VIRUSES, PROPERTY DAMAGE, LOSS OF USE, INTERRUPTION OF BUSINESS, LOST PROFITS, LOST DATA OR OTHER CONSEQUENTIAL, PUNITIVE OR SPECIAL DAMAGES, HOWSOEVER CAUSED, FOR BREACH OF THIS WARRANTY, CONTRACT OR TORT (INCLUDING NEGLIGENCE). This Warranty does not affect your legal rights under any applicable state and national law governing the sale of consumer goods. Some countries, states, provinces, and territories (including the EU, EFTA and UK) do not allow the exclusion or limitation of incidental or consequential damages, the exclusion or limitation of certain damages or types of loss or allow limitations on how long an implied warranty or condition may last, so the limitations or exclusions described in this Warranty may not apply to you.

**Lenses can be mounted only by Luxottica or a Luxottica authorised dealer in order to ensure the proper fitting and functionality of the Product.

FOR CONSUMERS IN AUSTRALIA

For purchases made from a retail store operated by Luxottica in Australia, the Supplier is Luxottica Retail

Australia Pty Ltd (ABN 26 000 025 758) of level 34, 1 Denison Street, North Sydney, NSW, 2060. For purchases made through the wholesale channel: the Supplier is Luxottica South Pacific Holdings Pty Ltd (ABN 58 104 629 772) of level 34, 1 Denison Street, North Sydney, NSW, 2060. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. Your rights and the benefits given to you under this Warranty are in addition to other statutory rights and remedies you have under the Australian Consumer Law and other laws.

FOR CONSUMERS IN ITALY

This Warranty is a voluntary warranty provided by Luxottica in accordance with Article 135-quinquies of Legislative Decree No. 206 of 6 September 2005 (the "Consumer Code"). In addition to this Warranty, consumers are entitled to the rights granted under Articles 128-135-septies of the Consumer Code, which apply for two (2) years from the date the Product is delivered. Accordingly, this Warranty does not affect the consumer's statutory rights relating to the legal guarantee of conformity. For more details on your rights under Articles 128-135-septies of the Consumer Code, please visit www.nuanceaudio.com.

FOR CONSUMERS IN FRANCE

The consumer has a period of two years from the delivery of the goods to obtain the implementation of the legal guarantee of conformity in the event of a lack of conformity. During this period, the consumer is only required to establish the existence of the lack of conformity and not the date on which it occurred. Where the contract for the sale of goods provides for the continuous supply of digital content or a digital service for a period exceeding two years, the legal guarantee shall apply to that digital content or digital service throughout the specified supply period. During this period, the consumer is only required to establish the existence of the lack of conformity affecting the digital content or digital service, and not the date on which it occurred.

The legal guarantee of conformity requires the professional, where applicable, to provide all updates necessary to maintain the conformity of the goods.

The legal guarantee of conformity entitles the consumer to have the goods repaired or replaced within thirty days of their request, at no cost and without significant inconvenience to them. If the goods are repaired under the legal guarantee of conformity, the consumer benefits from a six-month extension of the initial guarantee.

If the consumer requests repair of the goods, but the seller insists on replacement, the legal guarantee of conformity is renewed for a period of two years from the date of replacement of the goods.

The consumer may obtain a reduction in the purchase price by keeping the goods or terminate the contract and receive a full refund in exchange for returning the goods, if:

1° The professional refuses to repair or replace the goods;

2° The repair or replacement of the goods takes place after a period of thirty days;

3° The repair or replacement of the goods causes a major inconvenience to the consumer, in particular when the consumer definitively bears the costs of returning or removing the non-compliant goods, or if they bear the costs of installing the repaired or replacement goods;

4° The non-conformity of the goods persists despite the seller's unsuccessful attempt to ensure conformity. The consumer is also entitled to a price reduction or termination of the contract when the lack of conformity is so serious that it justifies an immediate price reduction or termination of the contract. In this case, the consumer is not required to request repair or replacement of the goods beforehand. The consumer is not entitled to cancel the sale if the lack of conformity is minor. Any period during which the goods are out of service for repair or replacement suspends the warranty that remained until the repaired goods were delivered.

The above rights result from the application of Articles L. 217-1 to L. 217-32 of the Consumer Code.

A seller who acts in bad faith to obstruct the implementation of the legal guarantee of conformity is liable to a civil fine of up to €300,000, which may be increased to 10% of average annual turnover (Article L. 241-5 of the Consumer Code).

The consumer also benefits from the legal guarantee against hidden defects pursuant to Articles 1641 to 1649 of the Civil Code, for a period of two years from the discovery of the defect. This guarantee entitles the consumer to a price reduction if the goods are retained, or a full refund upon return of the goods.

This commercial warranty does not grant any additional rights beyond the legal guarantee of conformity provided for in Articles L. 217-1 to L. 217-32 of the Consumer Code. This commercial warranty does not affect consumers' rights and recourse under the legal guarantee of conformity. Finally, the limitation of liability clause does not apply to French consumers in the event of a breach of any of Luxottica's obligations.

FOR CONSUMERS IN GERMANY

Your statutory rights and claims under German law are not affected by this Warranty. You are entitled to exercise those rights and claims free of charge. This Warranty applies in addition to any applicable contractual and/or statutory warranty and liability provisions and does not limit any of your statutory rights and claims against Luxottica, the retailer, or any other mandatory rights and claims under German or EU law.

FOR CONSUMERS IN THE U.S.

FOR THE US ONLY, ANY IMPLIED WARRANTIES THAT CANNOT BE DISCLAIMED ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTY PERIOD AS SET OUT IN THIS LIMITED CONSUMER WARRANTY (THE "WARRANTY").

FOR PRODUCTS PURCHASED IN CALIFORNIA:

This Nuance Audio™ hearing Aid and its accessories are warranted to be specifically fit for the particular needs of you, the buyer. If the hearing aid is not initially fit for your particular needs, it may be returned to the seller within 45 days of the initial date of delivery to you. If you return the hearing aid, the seller will either adjust or replace the hearing aid or promptly refund the total amount paid. This warranty does not affect the protections and remedies you have under other laws.

RETURNS AND EXCHANGES:

If you bought Nuance Audio™ products from an EssilorLuxottica banner, you may return them within 30 days from the date of purchase, or any longer period expressly provided by applicable law or the specific EssilorLuxottica banner. Nuance Audio™ products purchased through non-EssilorLuxottica retailers must be returned to those retailers in accordance with their return policies. For further information on EssilorLuxottica returns, please visit:

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