
Nuance Audio Charging Pad

Instructions For Use Safety & Warranty

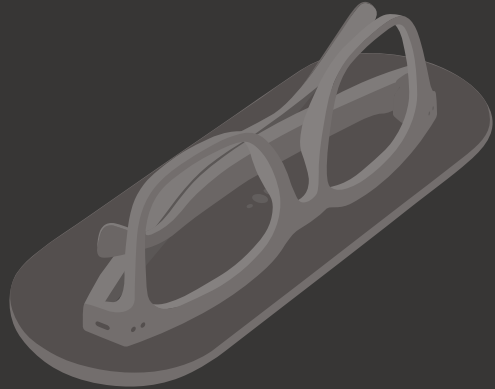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Nuance Audio™ Charging Pad

Nuance Audio™ Charging Pad is referred to within this Instructions For Use as the "Product". The Charging Pad accessory is intended to enable the air conduction hearing aid functionality in terms of charging the hearing aid Nuance Audio™ Glasses and Nuance Audio™ Plus.

The Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses or the Nuance Audio™ Plus. The Charging Pad is used to charge the Nuance Audio™ Glasses or the Nuance Audio™ Plus wirelessly. The charge pad has an LED on the top surface which provides the user information regarding charging status of the Nuance Audio™ Glasses or the Nuance Audio™ Plus. The charge pad is powered by USB-C and does not contain a battery.

Read instructions. Read and follow all instructions included with the Charging Pad before using it. Please refer to the User Guide for images and usage explanation of the Charging Pad.

Restrictions to combinations

The Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses and Nuance Audio™ Plus. To charge them, use the supplied USB-C Charging Cable or other USB-C type cable with a USB wall plug / adapter (not included).

Cautions

1. Do not use the Charging Pad if any part of it is damaged or overheating, or if it emits any loud unusual noise. If this happens, carefully unplug the Charging Pad from the power source immediately, then please visit us at www.nuanceaudio.com to get helpful service and contact information.
2. Do not make unauthorized alterations to the Charging Pad.

IMPORTANT! If Nuance Audio™ Glasses or the Nuance Audio™ Plus battery is not fully recharged every 6 months from last use this could lead to permanent damage to the battery, which could render Nuance Audio™ Glasses or Nuance Audio™ Plus inoperable.

HEALTH & SAFETY INFORMATION

WARNING To reduce the risk of personal injury, discomfort, or property damage, please read and follow the health and safety information below before using the Charging Pad.

Intended uses & users

Intended use. The Charging Pad accessory is intended to enable the air conduction hearing aid functionality in terms of charging the hearing aid. The Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses or the Nuance Audio™ Plus and is intended only for use by individuals 18 years old and older. The Charging Pad is used to charge the Nuance Audio™ Glasses or the Nuance Audio™ Plus wirelessly. It has a LED, on top view, in order to give info to the user regarding the charging status of the Nuance Audio™ Glasses or the Nuance Audio™ Plus and it is battery less, it works thanks to a USB-C cable.

For ages 18 and up. The Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses or the Nuance Audio™ Plus and is intended only for use by individuals 18 years old and older. Do not allow anyone under 18 to use the Charging Pad. Keep the Charging Pad and the USB-C Charging Cable away from children.

Interference with personal medical devices

The Charging Pad contains magnetic material and components that emit radio waves, which could affect the operation of medical devices such as but not limited to pacemakers or other implantable devices, or insulin pumps, consult your doctor or the medical device manufacturer before using the Charging Pad. Unplug the Charging Pad from the power if you notice any interference with your medical device.

Interference

Check before using the Charging Pad in areas where electronics or wireless devices are restricted. This may cause interference.

Interactions & environments

Limitations of Use and Functionality

1: Place the Charging Pad on a flat surface to ensure it does not wobble.

2: Place the Nuance Audio™ Glasses or the the Nuance Audio™ Plus folded and upside down on the Charging Pad. Refers to User Guide for pictures.

3: Ensure that the Glasses are aligned with the magnetic components on the Charging Pad, Charging Pad's LED will light up in orange or green when properly aligned. If the alignment is not accurate, you can try adjusting the angle or position of the Glasses.

4: During the charging process, do not move or vibrate the Glasses significantly to avoid affecting the charging effect.

5: If the Charging Pad cannot charge properly or experiences any malfunctions, please stop using it immediately. Detailed system requirements, and terms of service are available at www.nuanceaudio.com.

Flying

Check applicable aviation security restrictions on personal electronics before flying with the Charging Pad. Use the Charging Pad in compliance with all the applicable local laws, rules and regulations. Use common sense and respect third parties' rights, privacy, and confidentiality.

Intended handling & maintenance

Handle with care & avoid damage. The Charging Pad is sensitive electronic equipment and should be handled with care. Do not heat or bend the Charging Pad, as it may become damaged and even catch fire. Do not drop, strike, or shake the Charging Pad excessively. Do not use the Charging Pad if it is damaged. Do not attempt to open, disassemble or tamper with the Charging Pad. Do not expose to the sun. Do not place it around high-power electrical products. Do not use the charging pad in a strong magnetic environment. Do not place it on a metal surface for charging. Product could be affected in its use by near magnetic field (such as microwave oven). If the product stops working, its normal functioning can be restored by moving it away from the magnetic field.

Operating conditions.

14°F (-10°C) to 95°F (+35°C); 101.3kPa to 61.9kPa; humidity between 20% and 60% under 95°F (+35°C) for 1000 hours; maximum altitude of 13,000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C).

Transport and Storage.

Charging Pad shall remain functional after recovery from minimum -22°F (-30°C) or maximum +167°F (+75°C) storage temperatures; high humidity 90% and temperature 95°F (+35°C) for 288 hours; exposure to altitude of 40,000 ft (18,8kPa) for 12 hours at both 14°F (-10°C) and 95°F (+35°C).

Cleaning and maintenance.

To clean the Charging Pad, use a dampened cloth and neutral soap avoiding all types of sprays, solvents, chemicals or cleaning solutions containing alcohol or ammonia. Do not use dirty or abrasive cloths which could damage the Charging Pad. Keep the Charging Pad dry. Excessive moisture can damage

the charging function. If the Product becomes wet, thoroughly dry it, and store it in a container with desiccants or in a drying case until completely dry.

WARNING Potentially explosive atmospheres.

Do not store or carry flammable liquids, gases, or explosive materials in the same compartment as the Charging Pad. Do not use the Charging Pad when you are in an area with potentially explosive atmospheres.

Water & liquids. Do not submerge in water and keep it away from liquids. Exposure to water or other liquids may damage the Charging Pad and lead to injury. It is not water resistant and is not designed to resist submersion or exposure to other liquids. In case of water exposure, dry the Charging Pad thoroughly. Do not soak in water or stay in a humid environment for a long time.

Do not use if broken or damaged.

Periodically inspect the Charging Pad for damage or signs of wear. Do not use it if any part is damaged, cracked, or if any internal component is exposed. Do not try to repair the Charging Pad yourself, instead contact support at www.nuanceaudio.com.

Charging, electrical & heat

Charging. Use the Charging Pad only to charge the Nuance Audio™ Glasses. Use the supplied USB-C Charging Cable or other USB-C type cable with a USB wall plug/adaptor (not included). It is prohibited to connect the Charging Pad to a wall plug/ adaptor that is not certified for the market in which the Charging Pad is marketed, and that it has not been tested and approved according to national and local standards. The wall plug/adaptor must have a minimum output level of DC5V/1A. Do not clean the Charging Pad while you are charging it. Do not charge the Nuance Audio™ Glasses or the Nuance Audio™ Plus on metal surfaces or in strong electric or magnetic field environments. Periodically inspect the Charging Pad and the USB-C Charging Cable for damage or signs of wear. Do not use if any part is damaged, cracked, or if any internal component is exposed. Do not try to repair the Charging Pad or replace parts yourself, instead contact support at www.nuanceaudio.com.

Overheating. Keep the Charging Pad away from fire and excessive heat (e.g., from storage in direct sunlight, near candles, or in a hot car) to avoid burns or damage to the Charging Pad. The Charging Pad shouldn't overheat during normal use. If it does, stop using it immediately and contact support. Keep the Charging Pad away from fire and heat sources.

Accessories. Use only with the compatible USB-C cables and wall plugs/adapters indicated above. Using non-approved accessories may permanently damage the Charging Pad or the Glasses, and increase the risk of injury or damage to other property. The Luxottica Limited Warranty does not cover damage caused by the use of non-approved accessories.

KEY FEATURES	
WIRELESS CHARGING	
Transmitting frequency	13,56 MHz
Receiving frequency	13,56 MHz
Max radiated power	69,72 dBuV/m at 3m
Modulation type transmitting	ASK 100% modified Miller
Modulation type receiving	Load Modulation subcarrier fc/16 (848kbit/s) OOK Manchester
Charging time	4 hours (0 > 100%)
Fast charging time	30 minutes (0 > 12%)
TECHNICAL SPECIFICATIONS	
Battery	Battery less
Lifetime	3 years
Shelf life	5 years
TRANSPORT AND STORAGE CONDITIONS	
Temperature	From -22°F (-30°C) to +167°F (+75°C)
Humidity	From 20% to high humidity 90% and temperature 95°F (+35°C) for 288 hours

KEY FEATURES	
OPERATING CONDITIONS	
Temperature	From 14°F (-10°C) to 95°F (+35°C)
Humidity	From 20% to humidity 60% under 95°F (+35°C) for 1000 hours
Altitude & atmospheric pressure	Altitude of 13.000 ft for 1 hour at 32°F (0°C) and 95°F (+35°C); 101.3kPa to 61.9kPa

FCC compliance statement

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio technician for help.

Note: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

No part of this work may be reproduced, modified, distributed or otherwise used without prior written permission.

Keep the Charging Pad away from the womb of pregnant women. Keep the Charging Pad away from the lower abdomen of adolescents.

Use the Charging Pad in good reception conditions to reduce the amount of electromagnetic radiation received.

FCC Radiation Exposure Statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS

Luxottica is committed to the proper reuse, recycling and recovery of batteries and Waste Electrical and Electronic Equipment ("WEEE").

Batteries and WEEE should not be discarded in normal domestic waste bins.

Discard the Product and the battery according to local regulations. Do not discard as household waste. Bring them to designated collection points or in alternative to an affiliated store.

For more info on proper collection, please visit www.nuanceaudio.com.

INFORMATION ON DISCARDING

The crossed-out wheeled bin symbols below mean that, according to local laws and regulations, The Nuance Audio™ Products, shall be discarded separately from household waste. When The Nuance Audio™ Products, reaches its end of life, bring them to a collection point designated by local authorities. In alternative, bring the Product and the battery to a affiliated store. The separate collection and recycling of the Nuance Audio™ Products, will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Crossed-out wheeled bin (WEEE):

Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).

Nuance Audio™ is a registered trademark of Essilor International SAS. FCC ID: 2AYOA-4005

Luxottica Group S.p.A.
Piazzale Cadorna 3,
20123 Milan, Italy

Made in China
Fabriqué en Chine

www.nuanceaudio.com

For the latest and up-to-date Instructions For Use - Safety and Warranty, please refer to the Instructions in the Support section of the Nuance Audio™ App and to the following website page: nuanceaudio.com/support/manual-guide

Instructions For Use & Safety - NOT FOR CANADA

Nuance Audio™ Charging Pad

* Product availability, regulatory requirements, and warranty terms may differ from country to country. Please note that the local market product category of Nuance Audio™ Plus and Nuance Audio™ Glasses may vary depending on the market in which the product is sold, and some countries do not adopt a dedicated local category (such as the UK, Australia, and New Zealand). As a result, the terminology used to denote the product's category may vary across the language versions of these Instructions for Use – Safety & Warranty (e.g., "assistant d'écoute" in France vs. "hearing aid" elsewhere). As this document is organized by language rather than by country, the terminology used to address the product in a given language version does not necessarily reflect the category applicable in all countries where that language is used. For example, although French is France's official language, the terminology used in the French version does not apply to the French market; for France, the correct terminology is "assistant d'écoute".

"Nuance Audio™ Charging Pad is referred to within these Instructions For Use also as "Product", "Device" or "Equipment".

INTENDED USES & USERS

Intended use. The Nuance Audio™ Charging Pad accessory is intended to enable the Nuance Audio™ Plus functionality in terms of charging the hearing aid*. The Nuance Audio™ Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses or the Nuance Audio™ Plus and is intended only for use by individuals 18 years old and older. The Nuance Audio™ Charging Pad is used to charge the Nuance Audio™ Glasses or the Nuance Audio™ Plus wirelessly. It has a LED, on top view, in order to give info to the user regarding the charging status of the Nuance Audio™ Glasses or the Nuance Audio™ Plus and it is battery less, it works thanks to a USB-C cable.

For ages 18 and up. The Nuance Audio™ Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses or the Nuance Audio™ Plus and is intended only for use by individuals 18 years

old and older. Do not allow anyone under 18 to use the Nuance Audio™ Charging Pad. Children can choke or suffocate in case of swallowing the Nuance Audio™ Charging Pad. Keep the Nuance Audio™ Charging Pad away from children. Please refer to the Quick Guide for images and usage explanation of the Nuance Audio™ Charging Pad.

Restrictions to combinations. The Nuance Audio™ Charging Pad is to be used only in conjunction with the Nuance Audio™ Glasses. To charge the Nuance Audio™ Glasses, use the supplied USB-C Charging Cable or other USB-C type cable, with a USB wall plug/adaptor (not included). It is strongly discouraged to connect the Nuance Audio™ Charging Pad to a wall plug/adaptor that is not certified for the market in which the Nuance Audio™ Charging Pad is marketed, and that it has not been tested and approved according to national and local standards. The wall plug/adaptor must have a minimum output level of DC5V/1A.

Read instructions. Read and follow all instructions included with the Nuance Audio™ Charging Pad before using it.

Cautions.

1. Do not use the Nuance Audio™ Charging Pad if any part of it is damaged or overheating, if they emit any loud unusual noise, if this happens, unplug the Nuance Audio™ Charging Pad from the power source immediately, then please visit us at www.nuanceaudio.com to get helpful service and contact information.
2. Do not make unauthorised alterations to the Nuance Audio™ Charging Pad.

IMPORTANT! If the Nuance Audio™ Glasses' battery or the Nuance Audio™ Plus' battery is not fully recharged every 6 months from last use this could lead to permanent damage to the battery, which could render Nuance Audio™ Glasses or the Nuance Audio™ Plus inoperable

HEALTH & SAFETY INFORMATION

WARNING To reduce the risk of personal injury, discomfort, or property damage, please read and follow the health and safety information below before using the Nuance Audio™ Charging Pad.

Interference with personal medical devices. The Nuance Audio™ Charging Pad contains magnetic material and components that emit radio waves, which could affect the operation of medical devices such as but not limited to pacemakers or other implantable devices, or insulin pumps, consult your doctor or the medical device manufacturer before using the Nuance Audio™ Charging Pad. Unplug the Nuance Audio™ Charging Pad from the power and move it away from your other medical device if you notice any interference with your medical device.

Interference. Check before using the Nuance Audio™ Charging Pad in areas where electronics or wireless devices are restricted. This may cause interference.

Water & liquids. Do not submerge in water and keep it away from liquids. Exposure to water or other liquids may damage the Nuance Audio™ Charging Pad and lead to injury. It is not water resistant and is not designed to resist submersion or exposure to other liquids. In case of water exposure, dry the Nuance Audio™ Charging Pad thoroughly. Do not soak in water or stay in a humid environment for a long time.

WARNING Potentially explosive atmospheres

Do not store or carry flammable liquids, gases, or explosive materials in the same compartment as the Nuance Audio™ Charging Pad. Do not use the Nuance Audio™ Charging Pad when you are in an area with potentially explosive atmospheres.

Do not use if broken or damaged.

Overheating. Keep the Nuance Audio™ Charging Pad away from fire and excessive heat (e.g., from storage in direct sunlight, near candles, or in a hot car) to avoid burns or damage to the Nuance Audio™ Charging Pad. The Nuance Audio™ Charging Pad shouldn't overheat during normal use. If it does, stop using it immediately and contact support. Keep the Nuance Audio™ Charging Pad away from fire and heat sources.

Handle with care & avoid damage. The Nuance Audio™ Charging Pad is sensitive electronic Equipment and should be handled with care. Do not heat or bend the Nuance Audio™ Charging Pad, as it may become damaged and even catch fire. Do not drop, strike, or shake the Nuance Audio™ Charging Pad excessively. Do not use the Nuance Audio™ Charging Pad if it is damaged. Do not attempt to open,

disassemble or tamper with the Nuance Audio™ Charging Pad. Do not expose to the sun. Do not place it around high-power electrical products. Do not be in a strong magnetic environment. If the Product stops working, its normal functioning can be restored by moving it away from the magnetic field.

Interactions & environments

Limitations of Use and Functionality

1: Place the Nuance Audio™ Charging Pad flat on the desktop to ensure it does not wobble.

2: Place the Nuance Audio™ Glasses or the Nuance Audio™ Plus folded and upside down on the Nuance Audio™ Charging Pad. Refer to Quick Start Guide for pictures.

3: Ensure that the Nuance Audio™ Glasses or the Nuance Audio™ Plus are aligned with the magnetic components on the Nuance Audio™ Charging Pad, Nuance Audio™ Charging Pad's LED will light up in orange or green when properly aligned. If the alignment is not accurate, you can try adjusting the angle or position of the Nuance Audio™ Glasses or the Nuance Audio™ Plus.

4: During the charging process, do not move or vibrate the Nuance Audio™ Glasses or the Nuance Audio™ Plus significantly to avoid affecting the charging effect.

5: If the Nuance Audio™ Charging Pad cannot charge properly or experiences any malfunctions, please stop using it immediately.

Detailed system requirements, and terms of service are available at - www.nuanceaudio.com.

Flying. Check applicable aviation security restrictions on personal electronics before flying with the Nuance Audio™ Charging Pad. Use the Nuance Audio™ Charging Pad in compliance with all the applicable local laws, rules and regulations. Use common sense and respect third parties' rights, privacy, and confidentiality.

Operating conditions. -10°C to 35°C; 101.3kPa to 61.9kPa; humidity between 20% and 60% under 35°C for 1000 hours; maximum altitude of 13,000 ft for 1 hour at 0°C and 35°C.

Transport and Storage.

Nuance Audio™ Charging Pad shall remain functional after recovery from minimum -30°C or maximum 75°C storage temperatures; high humidity 90% and temperature 35°C for 288 hours; exposure to altitude of 40,000 ft (12,192 m) (18,8kPa) for 12 hours at both -10°C and 35°C.

Cleaning and maintenance. To clean the Nuance Audio™ Charging Pad, use a dampened cloth and neutral soap avoiding all types of sprays, solvents, chemicals or cleaning solutions containing alcohol or ammonia. Do not use dirty or abrasive cloths which could damage the Nuance Audio™ Charging Pad. Keep the Nuance Audio™ Charging Pad dry. If the Product becomes excessively wet, there is a risk of damaging the charging function, thoroughly dry it, store them in a container with desiccants or in a drying case.

Charging, electrical & heat

Charging. Use only the Nuance Audio™ Charging Pad to charge the Nuance Audio™ Glasses. Use the supplied USB-C Charging Cable or other cables USB-C type, C to C, with a USB wall plug/adaptor (not included). It is strongly discouraged to connect the Nuance Audio™ Charging Pad to a wall plug/adaptor that is not certified for the market in which the Nuance Audio™ Charging Pad is marketed, and that it has not been tested and approved according to national and local standards. The wall plug/adaptor must have a minimum output level of DC5V/1A.

Do not clean the Nuance Audio™ Charging Pad while you are charging it. Do not charge the Nuance Audio™ Glasses or the Nuance Audio™ Plus on metal surfaces or in strong electric or magnetic field environments. Periodically inspect the Nuance Audio™ Charging Pad and the USB-C Charging Cable for damage or signs of wear. Do not use if any part is damaged, cracked, or if any internal component is exposed. Do not try to repair the Nuance Audio™ Charging Pad or replace parts yourself, instead contact support at www.nuanceaudio.com.

Accessories. Use only with the compatible USB-C cables and wall plugs/adapters indicated above. Using non-approved accessories may permanently damage the Nuance Audio™ Charging Pad or the Nuance Audio™ Glasses, void the Luxottica Limited Warranty, and increase the risk of injury or damage to other property.

KEY FEATURES	
WIRELESS CHARGING	
Transmitting frequency	13,56 MHz (RF)

Receiving frequency	13,56 MHz
Max radiated power	-8.88 dBuA/m at 10m
Modulation type transmitting	ASK 100% modified Miller
Modulation type receiving	Load Modulation subcarrier fc/16 (848kbit/s) OOK Manchester
Charging time	4 hours (0 > 100%)
Fast charging time	30 minutes (0 > 12%)

KEY FEATURES

TECHNICAL SPECIFICATIONS

Battery	Battery less
Lifetime	3 years
Shelf life	5 years

TRANSPORT AND STORAGE CONDITIONS

Temperature	From -30°C to 75°C
Humidity	From 20% to high humidity 90% and temperature 35°C for 288 hours
Altitude & atmospheric pressure	Altitude of 40,000 ft (12,192 m) (18,8kPa) for 12 hours at both -10°C and 35°C

OPERATING CONDITIONS

Temperature	From -10°C to 35°C
Humidity	From 20% to humidity 60% under 35°C for 1000 hours

Altitude & atmospheric pressure	Altitude of 13.000 ft for 1 hour at 0°C and 35°C; 101.3kPa to 61.9kPa
---------------------------------	--

Note: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the Equipment.

No part of this work may be reproduced, modified, distributed or otherwise used without prior written permission.

EU compliance statement

Luxottica Group S.p.A hereby declares that this Device is in compliance with Radio Equipment Directive 2014/53/EU, EU Medical Device Regulation 2017/745 on Medical Devices and RoHS II Directive 2011/65/EU. The full text of the EU Declaration of Conformity is available at the following internet address: www.essilorluxottica.com/en/brands/conformity-declaration

Keep the Nuance Audio™ Charging Pad away from the womb of pregnant women. Keep the Nuance Audio™ Charging Pad away from the lower abdomen of adolescents.

Use the Nuance Audio™ Charging Pad in good reception conditions to reduce the amount of electromagnetic radiation received.

The end user is required to report any serious incident that occurs in relation to the use of the Device to the competent authority of the Member State where the incident occurred and to the manufacturer of the Device.

INFORMATION ON DISCARDING OF NUANCE AUDIO™ PRODUCTS

Luxottica is committed to the proper reuse, recycling and recovery of batteries and Waste Electrical and Electronic Equipment ("WEEE").

Batteries and WEEE should not be discarded in normal domestic waste bins.

Discard the Product and the battery according to local regulations. Do not discard as household waste. Bring them to designated collection points or in alternative to an affiliated store.

For more info on proper collection, please visit www.nuanceaudio.com.

INFORMATION ON DISCARDING

The crossed-out wheeled bin symbols below mean that, according to local laws and regulations, The Nuance Audio™ Products, shall be discarded separately from household waste. When The Nuance Audio™ Products, reaches its end of life, bring them to a collection point designated by local authorities. In alternative, bring the Product and the battery to a affiliated store. The separate collection and recycling of the Nuance Audio™ Products, will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Crossed-out wheeled bin (WEEE):

Do not discard of electrical and electronic equipment with household waste. Collect separately for proper recycling (Directive 2012/19/EU).

Discarding Information for France only

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LUXOTTICA LIMITED WARRANTY

NOT FOR CANADA

This limited consumer warranty (the "Warranty") is provided by Luxottica Group S.p.A., Piazzale Cadorna 3, 20123 Milano, Italy ("Luxottica"). This Warranty covers defects and malfunctions in the Luxottica Product it accompanies. The Warranty continues for one (1) year from the date of purchase of the Product (the "Limited Warranty Period"). However, if you purchase the Product from within the EU, UK or EFTA, the Limited Warranty Period shall be two (2) years. This Warranty applies only to those countries where Nuance is officially marketed and supported, meaning countries where the product is certified and Luxottica is authorized to distribute it ("Warranty Coverage Territory"). We warrant that the Product will, under normal and intended use, function in accordance with the technical specifications or accompanying Product documentation (the "Warranted Functionality") during the Limited Warranty Period. For the avoidance of doubt, any services that are required to achieve the Warranted Functionality may be updated, modified, or limited, so long as it continues to maintain (or exceed) the Warranted Functionality. This Warranty is not available to products that were purchased from any source other than Luxottica or an authorised dealer. This Warranty is valid only with proof of purchase that clearly shows the purchase date. If you send us a Product without valid proof of purchase, then we reserve the right to return the Product to you at your cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up. This Warranty gives you specific legal rights, and you may also have other rights under consumer protection laws or regulations in the country where the product was purchased or, if different, where you are located, provided that this product is officially marketed there at the time the warranty service is needed. Such rights vary from state to state and country to country. The benefits conferred by this Warranty are in addition to and do not affect any rights and remedies conveyed by such consumer protection laws and regulations (including, without limitation, national laws implementing Directive 1999/44/EC and Directive 2019/771). You are entitled to remedies free of charge in the event that the Product does not conform with the requirements set out under the laws in the Warranty Coverage Territory. The other remedies available to you according

to the laws in the Warranty Coverage Territory are not affected by this Warranty. For full information on your rights according to the laws in the Warranty Coverage Territory, please see below and/or contact your local consumer rights organisation/citizens advice bureau. Please visit us at www.nuanceaudio.com for information about the countries Luxottica supports and to get helpful service and contact information. The scope of technical support offered by Luxottica consists of helping to diagnose and resolve problems with defects in the Product. Please note that when contacting Luxottica via telephone, long distance charges may apply, depending on your calling area.

REGISTRATION

Product registration is not required as a condition to coverage under this Warranty, but some Luxottica products require periodic connection to an online account to ensure full functionality.

HOW TO MAKE A WARRANTY CLAIM

If you have a problem with your Product, unless specifically stated elsewhere in this Warranty and except as provided by applicable state law, to make a warranty claim, you need to return your Product to the authorised dealer from which the Product was purchased, together with your proof of purchase. Please visit www.nuanceaudio.com to get helpful service and contact information. If your warranty claim is deemed to be valid, and we confirm a defect or malfunction covered by this Warranty, the appropriate remedy will be determined. Depending on the circumstances, we may take one of the following actions: i) Repair the Product, only where repair is technically feasible (given the highly technological content of the Product, the only repair available is lens replacement); or (ii) Replace the Product; or (iii) Refund the purchase price paid for the Product, if neither repair nor replacement is possible or appropriate. Subject to your rights under applicable local laws of the Warranty Coverage Territory, the choice of remedy rests with the Company. Subject to your rights under any local laws, if we determine that a Product should be replaced, the replacement may be a new, refurbished, or a re-manufactured Product. The Limited Warranty Period is extended by a duration equal to the time during which we had the Product in our possession for performance of the Warranty, as described above. Save for your rights and remedies available under your local laws, the provision of a repaired or replacement Product does not restart or otherwise extend the Limited Warranty Period. We may not return the original Product to you. If your warranty claim is not deemed to be valid, then we reserve the right to return the Product to you at your

cost, subject to prepayment, or, if such costs are not prepaid, we will keep the Product for thirty (30) days for your pick-up before it is disposed of. Except to the extent that the law requires otherwise, any expenses incurred by you in returning the Product for warranty assessment will be borne by Luxottica. If you need to return your product to Luxottica, you will be provided with a prepaid label. If the warranty claim is deemed to be invalid and we do not find a defect or malfunction covered by this Warranty, we will contact you to see if you want us to conduct repairs at your cost and, in any event, you will be charged for the outbound freight to return your Product based on Luxottica's standard shipping rates.

LIMITATIONS AND EXCLUSIONS

THIS WARRANTY DOES NOT COVER, AND LUXOTTICA IS NOT RESPONSIBLE FOR: DELIVERY OR INSTALLATION, OR LABOR CHARGES FOR SETUP OF THE PRODUCT AND/OR ADJUSTMENT OF CUSTOMER CONTROLS ON THE PRODUCT. DAMAGES CAUSED BY MISUSE, ABUSE, ACCIDENTS, FIRE, THEFT, DISAPPEARANCE, MISPLACEMENT, FLUCTUATIONS AND POWER SURGES, CONNECTIONS TO IMPROPER VOLTAGE OR INCORRECT ELECTRICAL LINE VOLTAGE, VIRUSES, MALWARE, RECKLESS, WILLFUL, OR INTENTIONAL CONDUCT OR NEGLIGENCE. DAMAGES CAUSED BY SERVICING THE PRODUCT WHICH HAS NOT BEEN PRE-AUTHORISED BY LUXOTTICA. DAMAGES CAUSED BY USAGE THAT IS NOT IN ACCORDANCE WITH THE PRODUCT INSTRUCTIONS/ DOCUMENTATION. DAMAGES CAUSED BY FAILURE TO FOLLOW THE PRODUCT INSTRUCTIONS/ DOCUMENTATION OR FAILURE TO PERFORM CLEANING OR PREVENTIVE MAINTENANCE. DAMAGES CAUSED BY THE COMBINATION OF THE PRODUCT WITH OTHER NON-LUXOTTICA BRANDED PRODUCTS, ACCESSORIES, PARTS OR COMPONENTS OR USE OF PRODUCTS, EQUIPMENT, SYSTEMS, UTILITIES, SERVICES, SOFTWARE, PARTS SUPPLIES, ACCESSORIES, APPLICATIONS, INSTALLATIONS, REPAIRS, EXTERNAL WIRING OR CONNECTORS NOT SUPPLIED OR AUTHORISED BY LUXOTTICA THAT DAMAGE THE PRODUCT. SIGNAL ISSUES, RECEPTION PROBLEMS AND DISTORTION RELATED TO NOISE, ECHO, INTERFERENCE OR OTHER SIGNAL TRANSMISSION AND DELIVERY PROBLEMS. ANY EQUIPMENT OR COMPONENTS THAT WERE NOT INCLUDED IN, OR WITH YOUR PRODUCT AS ORIGINALLY SOLD. NORMAL WEAR AND TEAR. DAMAGE CAUSED AS A RESULT OF IMPROPER TRANSPORTATION OR PACKING/PACKAGING WHEN RETURNING THE PRODUCT TO LUXOTTICA OR A COLLECTION DEALER. DAMAGES CAUSED BY MODIFICATION OR

ADAPTATION THAT MAY BE REQUIRED TO ENABLE A PRODUCT TO OPERATE IN ANY COUNTRY OTHER THAN THE COUNTRY FOR WHICH IT WAS DESIGNED, MANUFACTURED, APPROVED AND/OR AUTHORISED, OR REPAIR OF THE PRODUCT WHICH RESULTS IN DAMAGE AS A RESULT OF THESE MODIFICATIONS. DAMAGE TO PRODUCTS WHERE THE ANTI-COUNTERFEITING CODE (OR EQUIVALENT) HAS BEEN REMOVED, ERASED, DEFACED, ALTERED OR MADE ILLEGIBLE.

This Warranty does not include any specific guarantees that the Product will be error-free. Except in the US and to the extent prohibited by any applicable law in any other country, this Warranty is void if a Product is returned with removed, damaged or tampered labels or any alterations (including the unauthorized removal of any component or external cover).

IMPLIED WARRANTIES

EXCEPT IN THE EU, EFTA AND UK AND EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW IN ANY OTHER COUNTRY, ANY IMPLIED WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NON-INFRINGEMENT IS DISCLAIMED. FOR THE US ONLY, ANY IMPLIED WARRANTIES THAT CANNOT BE DISCLAIMED ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTY PERIOD AS SET OUT IN THIS WARRANTY. IF YOU ARE A CONSUMER SUBJECT TO THE QUEBEC CONSUMER PROTECTION ACT, THE WARRANTY DISCLAIMER CONTAINED IN THIS SECTION DOES NOT LIMIT YOUR RIGHTS AND REMEDIES UNDER SUCH ACT, INCLUDING THE RIGHT TO MAKE A CLAIM UNDER ANY OF THE STATUTORY WARRANTIES PROVIDED UNDER SECTIONS 34 TO 54 OF SUCH ACT.

LIMITATION OF LIABILITY

EXCEPT TO THE EXTENT PROHIBITED BY ANY APPLICABLE LAW, LUXOTTICA SHALL NOT, UNDER THIS WARRANTY, BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES FOR BREACH OF ANY EXPRESS OR IMPLIED WARRANTY. EXCEPT TO THE EXTENT PROHIBITED BY LAW, LUXOTTICA'S MAXIMUM LIABILITY FOR ANY CLAIM ARISING OUT OF OR RELATED TO THIS WARRANTY IS LIMITED TO THE PURCHASE PRICE PAID BY YOU FOR THE PRODUCT PLUS ANY INTEREST ALLOWED BY LAW REGARDLESS OF THE FORM OF CLAIM. TO THE EXTENT PERMITTED BY LAW, LUXOTTICA IS NOT LIABLE FOR EVENTS BEYOND ITS CONTROL, SUCH AS ACTS OF GOD, VIRUSES, PROPERTY DAMAGE, LOSS OF USE, INTERRUPTION OF BUSINESS, LOST PROFITS, LOST DATA OR OTHER

CONSEQUENTIAL, PUNITIVE OR SPECIAL DAMAGES, HOWSOEVER CAUSED, FOR BREACH OF THIS WARRANTY, CONTRACT OR TORT (INCLUDING NEGLIGENCE).

This Warranty does not affect your legal rights under any applicable state and national law governing the sale of consumer goods. Some countries, states, provinces, and territories (including the EU, EFTA and UK) do not allow the exclusion or limitation of incidental or consequential damages, the exclusion or limitation of certain damages or types of loss or allow limitations on how long an implied warranty or condition may last, so the limitations or exclusions described in this Warranty may not apply to you.

FOR CONSUMERS IN AUSTRALIA

For purchases made from a retail store operated by Luxottica in Australia, the Supplier is Luxottica Retail Australia Pty Ltd (ABN 26 000 025 758) of level 34, 1 Denison Street, North Sydney, NSW, 2060. For purchases made through the wholesale channel: the Supplier is Luxottica South Pacific Holdings Pty Ltd (ABN 58 104 629 772) of level 34, 1 Denison Street, North Sydney, NSW, 2060. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. Your rights and the benefits given to you under this Warranty are in addition to other statutory rights and remedies you have under the Australian Consumer Law and other laws.

FOR CONSUMERS IN ITALY

This Warranty is a voluntary warranty provided by Luxottica in accordance with Article 135–quinquies of Legislative Decree No. 206 of 6 September 2005 (the “Consumer Code”). In addition to this Warranty, consumers are entitled to the rights set out in Articles 128–135–septies of the Consumer Code, which apply for two (2) years from the date the Product is delivered. Accordingly, this Warranty does not affect the consumer’s statutory rights under the legal guarantee of conformity. For more information on your rights under Articles 128–135–septies of the Consumer Code, please visit www.nuanceaudio.com.

FOR CONSUMERS IN FRANCE

The consumer has a period of two years from the delivery of the goods to obtain the implementation of the legal guarantee of conformity in the event of a lack of conformity. During this period, the consumer is

only required to establish the existence of the lack of conformity and not the date on which it occurred. Where the contract for the sale of goods provides for the continuous supply of digital content or a digital service for a period exceeding two years, the legal guarantee shall apply to that digital content or digital service throughout the specified supply period. During this period, the consumer is only required to establish the existence of the lack of conformity affecting the digital content or digital service, and not the date on which it occurred. The legal guarantee of conformity requires the professional, where applicable, to provide all updates necessary to maintain the conformity of the goods. The legal guarantee of conformity entitles the consumer to have the goods repaired or replaced within thirty days of their request, at no cost and without significant inconvenience to them. If the goods are repaired under the legal guarantee of conformity, the consumer benefits from a six-month extension of the initial guarantee. If the consumer requests repair of the goods, but the seller insists on replacement, the legal guarantee of conformity is renewed for a period of two years from the date of replacement of the goods.

The consumer may obtain a reduction in the purchase price by keeping the goods or terminate the contract and receive a full refund in exchange for returning the goods, if:

- 1° The professional refuses to repair or replace the goods;
- 2° The repair or replacement of the goods takes place after a period of thirty days;
- 3° The repair or replacement of the goods causes a major inconvenience to the consumer, in particular when the consumer definitively bears the costs of returning or removing the non-compliant goods, or if they bear the costs of installing the repaired or replacement goods;
- 4° The non-conformity of the goods persists despite the seller’s unsuccessful attempt to ensure conformity.

The consumer is also entitled to a price reduction or termination of the contract when the lack of conformity is so serious that it justifies an immediate price reduction or termination of the contract. In this case, the consumer is not required to request repair or replacement of the goods beforehand. The consumer is not entitled to cancel the sale if the lack of conformity is minor. Any period during which the goods are out of service for repair or replacement suspends the warranty that remained until the repaired goods were delivered. The above rights result from the application of Articles L. 217-1 to L. 217-32 of the Consumer Code. A seller who acts in bad faith to obstruct the implementation of

the legal guarantee of conformity is liable to a civil fine of up to €300,000, which may be increased to 10% of average annual turnover (Article L. 241-5 of the Consumer Code) The consumer also benefits from the legal guarantee against hidden defects pursuant to Articles 1641 to 1649 of the Civil Code, for a period of two years from the discovery of the defect. This guarantee entitles the consumer to a price reduction if the goods are retained, or a full refund upon return of the goods. This commercial warranty does not grant any additional rights beyond the legal guarantee of conformity provided for in Articles L. 217-1 to L. 217-32 of the Consumer Code. This commercial warranty does not affect consumers' rights and recourse under the legal guarantee of conformity. Finally, the limitation of liability clause does not apply to French consumers in the event of a breach of any of Luxottica's obligations.

FOR CONSUMERS IN GERMANY

Your statutory rights and remedies under German law are not affected by this Warranty. You are entitled to exercise those statutory rights and remedies free of charge. This Warranty applies in addition to any applicable contractual and/or statutory warranty and liability provisions and does not restrict any of your statutory rights and remedies against Luxottica, the retailer, or any other mandatory rights and remedies under German or EU law.

FOR CONSUMERS IN THE U.S.

FOR THE US ONLY, ANY IMPLIED WARRANTIES THAT CANNOT BE DISCLAIMED ARE EXPRESSLY LIMITED TO THE DURATION OF THE LIMITED WARRANTY PERIOD AS SET OUT IN THIS LIMITED CONSUMER WARRANTY (THE "WARRANTY").

FOR PRODUCTS PURCHASED IN CALIFORNIA:

This Nuance Audio™ hearing Aid and its accessories is warranted to be specifically fit for the particular needs of you, the buyer. If the hearing aid is not initially fit for your particular needs, it may be returned to the seller within 45 days of the initial date of delivery to you. If you return the hearing aid, the seller will either adjust or replace the hearing aid or promptly refund the total amount paid. This warranty does not affect the protections and remedies you have under other laws.

RETURNS AND EXCHANGES:

If you bought Nuance Audio™ products from an EssilorLuxottica banner, you may return them within 30 days from the date of purchase, or any longer period expressly provided by applicable law or the

specific EssilorLuxottica banner. Nuance Audio™ products purchased through non-EssilorLuxottica retailers must be returned to those retailers in accordance with their return policies. For further information on EssilorLuxottica returns, please visit: <http://www.nuanceaudio.com/en-us/c/support/shipping-and-returns>.

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